

Aptinka šarmines nuosėdas ant instrumentų ir paviršių



- Aptinka ploviklių likučius ir kieto vandens nuosėdas
- Paruoštas naudoti testas, kurio rezultatas matomas nedelsiant
- Neužteršiami instrumentai
- Spalvos pokytis į raudoną rodo likučių buvimą

Funkcija:

pHCheck testo indikatorinis tirpalas keičia spalvą į raudoną sąlytyje su šarminėmis nuosėdomis.

Šarminių ploviklių likučiuose ir kieto vandens nuosėdose yra medžiagų, keičiančių indikatoriaus spalvą į raudoną.

Naudojimas:

Tamponu paimkite mėginį nuo tiriamo ploto ir įmeskite jį į testo indikatorinį tirpalą. Įvertinkite ar įvyko spalvos pokytis.

Vertinimas:

Raudonas taškas ant tampono reiškia šarminių likučių buvimą ant tiriamo ploto. Didelis šarminių likučių kiekis gali pakeisti viso tirpalo spalvą į raudoną.

PASTABA: užteršti instrumentai yra pavojingi ir turi būti nenaudojami. Ant instrumentų neturi būti jokių nuosėdų!

MAKE IT VISIBLE!

Laikymas:

- Laikykite uždarytoje dėžutėje 2 - 25°C temperatūroje
- Laikykite atokiai nuo šviesos, karščio ir cheminių medžiagų
- Galiojimo laikas: 18 mėnesių

Detects alkaline residue on instruments and surfaces



- Detects detergent residue and hard water deposits
- A pre-prepared test easy to use with immediate result
- No contamination to instruments
- Colour change to red indicates residue

Function:

pHCheck contains an indicator solution showing a colour change when in contact to alkaline residue

Alkaline detergent and water hardness contain substances triggering the colour change of the indicator.

Application:

Swab the area which is to be tested then drop the swab into the pre-prepared test and check for a colour change. to red.

Evaluation:

A red spot on the swab indicates alkaline residue on the tested surface. Large amount of alkaline residue can change the whole solution to red.

NOTE: Contaminated instruments are a hygiene risk and need to be rejected. Instruments need to be free of any residue!

MAKE IT VISIBLE!

Storage:

- Store in closed box at 2 - 25°C
- Keep away from light, heat and chemicals
- Shelf Life: 18 months