5 WORKING WITH THE DEVICE

5.1 MAIN MENU

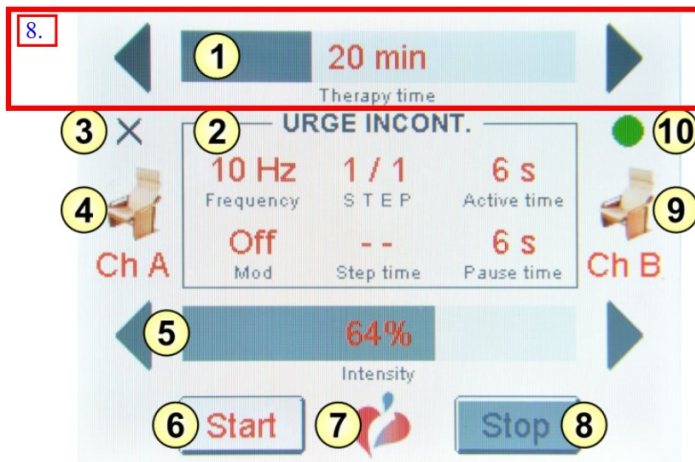
Fig. 4: Main menu

1. Set the therapy duration - To increase or decrease, press the left or right arrow. To quickly set the value, slide your finger along the rectangle.
2. Shows the selected program and its parameters.
3. Status of applicator on Channel A (X indicates the applicator is not connected).
4. Type of applicator on channel A
5. Set applicator intensity
6. START button – starts therapy
7. Press logo to enter the info menu
8. STOP button – stops therapy
9. Type of applicator on channel B
10. Status of applicator on Channel B (Green dot indicates the applicator is connected).

5.1.1 MAIN MENU - DETAILED DESCRIPTION

Fig. 5: Main menu - program and selected parameters

1. Selected program
2. Selected frequency
3. Step display
4. Active time in individual step
5. Selected modulation
6. Duration of an individual step
7. Pause time in individual step

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5.2 PROGRAM SETTINGS

2.4.

To change the parameters, use the left arrow (◀) to decrease and the right arrow (▶) to increase.

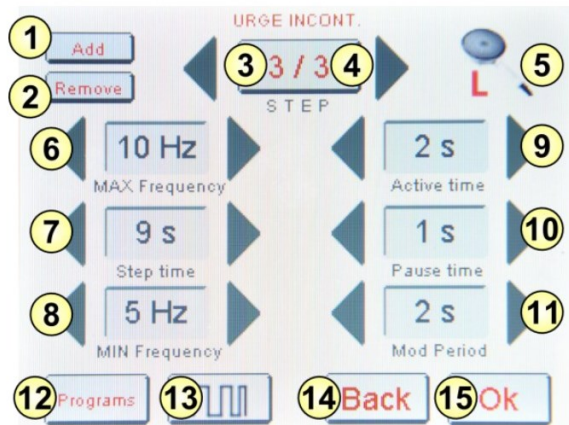
Fig. 6: Program settings

1. Add a step
2. Remove a step
3. Currently selected step
4. Total number of steps
5. Select the applicator or channel.
6. Set the frequency
7. Duration of individual step, the sum of active and pause time with repetitions
8. Modulation intensity (frequency: 10–50 Hz, amplitude 10-30%)
9. Active time of individual period, multiple of period (1–10 s)
10. Pause time of individual step
11. Modulation period is adjustable (1-5 s)
12. Program (Preset, Manually adjustable)
13. Select the modulation (waveform - amplitude modulation sine, amplitude modulation rectified sine, frequency modulation)
14. BACK - Cancel the settings
15. OK - Confirm (Save) the settings

5.3 PROGRAMS (manual view)

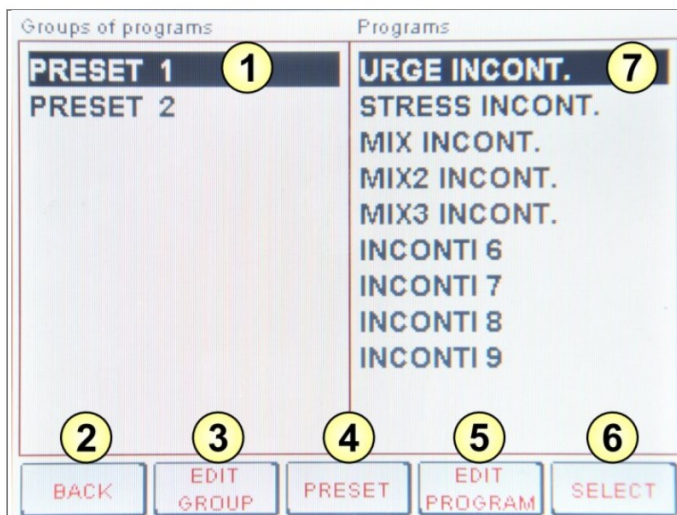
Fig. 7: Programs menu (manual view)

1. Name of program group (groups can be added, removed or renamed)
2. BACK – Return to main menu
3. EDIT GROUP opens edit group menu
4. Switches programs menu to preset view
5. EDIT PROGRAM opens edit program menu
6. SELECT opens currently selected program
7. Currently selected program

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