



## Incidin® active

Active oxygen based concentrate

## SPORICIDAL. SAFE.

Powerful surface disinfection in combination with excellent cleaning properties are characteristics for Incidin® active. It's peracetic acid generating system "PerOxyBalance®" has the power to also destroy all kinds of viruses. It meets a wide range of international test standards making this a trustworthy product for your high risk areas.



# Incidin<sup>®</sup> active

PerOxyBalance<sup>®</sup>

Our Offer	Your Benefit
Extensive efficacy bactericidal, yeasticidal, virucidal, and sporicidal incl. Clostridium difficile	Suitable for intensive care units and other high risk areas
Neutral pH of application solution	Good material compatibility, can be used on medical devices, floors, and walls
Excellent cleaning and disinfecting performance in one step	Time saving for user



**Sporicidal disinfectant for medical devices** C€0297



**Sporicidal disinfectant for all types of surfaces**

## INSTRUCTIONS FOR USE

**Preparation of a 2% solution:** Add 20 g of powder (1 dosing spoon) to 1 L of water or empty 160 g pot into 8 L of water. Stir the solution several times. After 15 minutes the solution is ready to use. Wet surfaces entirely with suitable procedure.

## INGREDIENTS

**Active ingredient of the solution 20 g/L (2 %):** >1000 mg/L (ppm) peracetic acid. The active ingredient peracetic acid is formed when the application solution is produced.

## MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

	Concentration		Contact time
	%	g/L	
EN 13727 Bactericidal	1.0	10	5 min
EN 13624 Yeasticidal	1.0	10	5 min
Disinfection of surfaces with mechanical action acc. to DGMH (2001) (bactericidal, yeasticidal, tuberculocidal)	1.0	10	60 min
EN 14348 Tuberculocidal	1.0	10	5 min
EN 13704 Sporicidal	2.0	20	15 min
EN 14476 Virucidal	2.0	20	10 min

## PACKAGING AND ORDERING INFORMATION

**Incidin<sup>®</sup> active** 3030120  
160 g pot 24 × 160 g

**Incidin<sup>®</sup> active** 3051850  
1,5 kg bucket 4 × 1,5 kg

**For professional use only.** Use disinfectants safely. Always read the description and product information before use.



*Incidin<sup>®</sup> active*

**PerOxyBalance<sup>®</sup>**  
The active substance with "Oxy Power"

**PerOxyBalance<sup>®</sup>** is setting new standards in terms of high level disinfection, combining powerful cleaning properties with optimum material compatibility.

**Ecolab Healthcare**  
Clean. Safe. Healthy.

**Ecolab Inc.**  
370 Wabasha Street N St. Paul, MN 55102  
www.ecolab.com

**Ecolab Europe GmbH**  
Richtstr. 7  
CH-8304 Wallisellen

**ECOLAB<sup>®</sup>**