

Seku® Zyme



Enzymatic detergent



- Excellent cleaning performance
- Formulated for protein breakdown
- Compatible with Sekusept® easy



Seku® Zyme

Enzymatic detergent



Why use Seku® Zyme?

- Excellent cleaning performance
- Contains non-ionic surfactants and enzymes to help break down protein
- CE marked for use on medical devices
- Low-foaming formulation for easy rinsing
- Confirmed compatibility with Sekusept® easy
- Unperfumed making Seku® Zyme pleasant for manual use
- Accurate dosing with measuring beaker

Areas of use

For manual pre-cleaning of surgical instruments and medical devices including flexible endoscopes

Recommended Use

For manual pre-cleaning:
Prepare Seku® Zyme solution with cold water,
5 – 20 ml/l, depending on degree of soiling

Immerse device in Seku® Zyme solution
for 5 – 15 mins

Clean device according to manufacturer's
instructions

Remove any remaining dirt mechanically to
achieve visual cleanliness

Rinse thoroughly

Product Specification

Seku® Zyme – Contains non-ionic surfactants
and enzymes



Package Size

| Description | Unit Size | Outer Size | Ecolab Pickcode |
|-------------|----------------|------------|-----------------|
| Seku® Zyme | 2 litres | 4 | P301832 |
| IW2 basin | 2 litre basin | 1 | P301501 |
| IW8 basin | 8 litre basin | 1 | P301502 |
| IW10 basin | 10 litre basin | 1 | P301503 |
| IW30 basin | 30 litre basin | 1 | P301504 |



We offer complete service and systems to cover all of
your institutional, industrial, and hospital cleaning and
sanitation needs. Ask for more of our products and
services in Europe and worldwide.

Ecolab — Your partner for Total Hygiene Solutions!



Ecolab GmbH & Co. OHG
P.O. Box 13 04 06
D-40554 Düsseldorf
Tel: 0049-211-9893-0
Fax: 0049-211-9893-384

Ecolab Ltd.
Lotherton Way, Garforth,
Leeds LS25 2JY, (GB)
Tel: 0044-113-232-0066
Fax: 0044-113-287-1317

www.ecolab.com