Higiiena

Citrosteril

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Literatūra

1. Solbach W, Ohgke H: Verification of the sporicidal efficacy of the product Puristeril®plus at 37°C in 4008 H(F) – 4008 B(S) Dialysis Systems (Fresenius Medical Care); Institute for Hygiene and Med. Microbiology, Medical University Lübeck, 7.3.2000
2. Solbach W, Ohgke H: Hygienic Evaluation of the Disinfectability of the Port of the Dialysis System 4008 (E)/(H)– 4008 B (S) using the disinfectant Puristeril®plus (Fresenius Medical Care); Institute for Hygiene and Med. Microbiology, Medical University Lübeck, 31.3.2000
3. Jentsch G: Peracetic Acid – a Disinfectant Agent; Hyg. Med. 3 (1978) 230-233
4. Wallhäuser KH: Praxis der Sterilisation – Desinfektion – Konservierung; Georg Thieme Verlag Stuttgart – New York 1995:511
5. Werner HP (Institute for Hygiene, Johannes – Gutenberg – University, Mainz, Germany): Disinfectants in Dialysis: Dangers, Drawbacks and Disinformation, Nephron 49; (1988), 2
6. Solbach W, Ohgke H, Senkpiel K: Test for disinfectant residues after rinsing with reverse osmosis water in 4008 F (H) – 4008 B (S) dialysis systems (Fresenius Medical Care); Institute for Hygiene and Med. Microbiology, Medical University Lübeck, 13.3.2000

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Hygiene

Citrosteril

Heat Disinfectant for Haemodialysis Machines
with Recirculation

**FRESENIUS
MEDICAL CARE**

Citrosteril

41.1 Pirkimo dalis

Heat Disinfectant for Haemodialysis Machines with Recirculation

Disinfectant for chemo-thermal disinfection of haemodialysis machines¹.

- pH value 1.7 to 2.0
- excellent removal of limescale
- disinfection and decalcification in one process
- active ingredients composed of natural substances
- biodegradable
- odourless
- free from colouring additives

Action

The synergistic effect of its components makes Citrosteril a potent disinfectant solution.

Citrosteril at 84°C has a broad spectrum of micro-biocidal activity and works bactericidal and virucidal² including HBV/HCV/HIV.

Specification

100 g Citrosteril contains:

21 g citric acid 1-hydrate; lactic acid, malic acid

Ordering information

Unit	Language combination	Art. No.
1 x 5 L (single canister)	multilingual	F00005157
2 x 5 L (carton)	multilingual	F00005158
6 x 2 L (carton)	multilingual	508 536 1

Literature:

1. Solbach W, Universität zu Lübeck: Verification of the sporicidal efficacy of the product Citrosteril at 85°C in the Fresenius 5008 dialysis machine, 20.12.2002
2. Labor Dr. Merk & Kollegen, Ochsenhausen: Antiviral efficacy of Citrosteril against bovine Parvovirus, 9.9.2005

Further information is available on request.