

LOFTEX®-SERVETĖLĖS

SERVETĖLĖS VALYMUĮ IR DEZINFEKCIJAI

Saugiam ir higieniniam daiktų bei paviršių valymui / dezinfekcijai ligoninėse, slaugos namuose, medicinos praktikoje, laboratorijose, virtuvėse, sanitarinėse patalpose, farmacijos ir maisto pramonėje.

Naudojimo vietos: vietos, kuriose yra ir nėra infekcijos rizikos, ar yra tikėtina infekcijos rizika. Impregnuotos paviršių dezinfekcijos tirpalu servetėlės LOFTEX -Wipes, gali būti naudojamos iki 28 dienų.

Naudojimas:

1. Dėvėkite pirštines ir dezinfekuokite darbo paviršių.
2. Atidarykite dangtelį ir lėtai įpilkite apie 2,0 - 2,5l paruošto naudoti dezinfekcinio tirpalo iš viršaus nuo servetelių centro link kraštų.
3. Ištraukite servetėlę iš centro ir iškiškite pro dangtelio viduryje esančią angą.
4. Saugiai uždėkite dangtelį, palikdami šiek tiek iškištos servetėlės per angą, kad būtų patogų ištraukti.
5. Paženklinkite etiketę markeriu. Maždaug po 20 min. LOFTEX –Wipes servetėlės bus pilnai sudrėkusios ir paruoštos naudoti.
6. Po paruošimo ir atidarymo LOFTEX servetėlės reikia sunaudoti per 28 dienas. Neskirta naudoti invazinių medicinos prietaisų dezinfekcijai. Perskaitykite pasirinkto dezinfektanto naudojimo instrukciją. Nenaudokite išdžiūvusių/išsausėjusių servetelių.
7. Išmesti pasibaigus naudojimui arba kibirėlį dezinfekuoti ir naudoti toliau.

115 servetelių,
dydis: 25x30cm

Importuotojas: UAB „Goodpoint“
Birželio 23-iosios g. 23F,
LT-50220, Kaunas, Lietuva
El.p.: info@medsauga.lt / www.gpoint.lt



LOFTEX GmbH

manufacturing, development and distribution
of cleaning, care and sanitary products

LOFTEX GmbH
Gottlieb-Daimler-Str. 16
28237 Bremen / Germany

Tel.: +49 (0) 421 38650 0
Fax: +49 (0) 421 38650 99
eMail: info@loftex.de

PRODUCT SPECIFICATION

LOFTEX®-Wipes DESi-Box

Disposable wipe dispenser system for surface disinfection
Non-woven wipe roll - Ready to fill (RTF)



Article number: 1250 115 (Dispenser Bucket including non-woven wipe roll)


Non-Woven wipe roll: (item no. 1240 115)


Material: 100% PET
Surface: perforated
Format: 300 x 250 mm
Folded to approx. 155 x 250 mm
Content: 115 wipes per Box/roll
Weight: approx. 425 g / roll



Dispenser bucket:

Height: 201 mm without lid
D1: 199 x 199 mm
D2: 186 x 186 mm
D3: 167 x 167 mm
Volume: 6,00 l (to the edge)
Weight: approx. 145 g
Material: Polypropylen (PP)

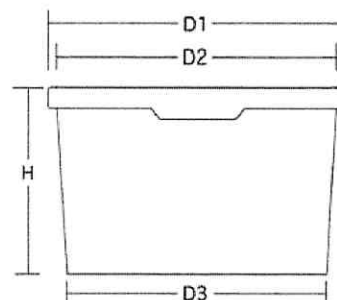

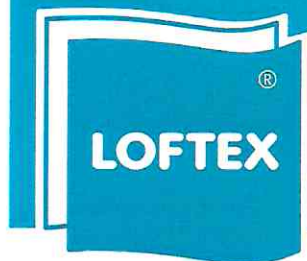

LOFTEX GmbH

manufacturing, development and distribution
of cleaning, care and sanitary products

LOFTEX GmbH
Gottlieb-Daimler-Str. 16
28237 Bremen / Germany

Tel.: +49 (0) 421 38650 0
Fax: +49 (0) 421 38650 99
eMail: info@loftex.de

Lid:

Size: 194,8 x 194,8 mm
Height: 19,1 mm
Weight: approx. 36,00 g
Material: Polypropylen (PP)

Wet Wipe Removal:

Ø Wipe-Dispenser, Lid: approx. 103 mm
Ø Wipe-Dispenser, inside: approx. 75 mm
Depth: approx. 23,6 mm
Material: Polypropylen (PP)

Packing:

Unit: 8 boxes, each with 115 wipes
Unit-Inner dimensions: 790 x 395 x 205 mm
Pallet: 192 boxes
(24 units á 8 boxes)

Weight:


1 DESi-Box (incl. roll) = approx. 630 g
1 Carton (8 DESi-boxes) = approx. 5,7 kg
1 Pallet (192 boxes) = approx. 150 kg

Applications:

In areas without / with risk of infection and in areas with special risk of infection. After impregnation with surface disinfectants, LOFTEX®-Wipes DESi-Box can be used for 28 days.

Logistics / Pallet Scheme:

Euro pallet 800 mm x 1.200 mm

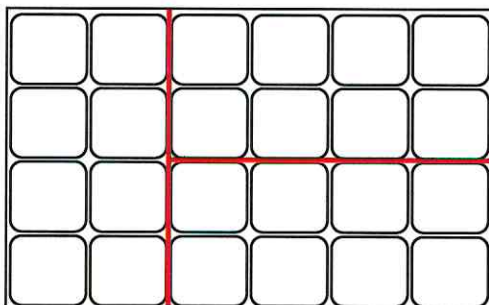

3 units per layer

8 layers per pallet

Pallet height with 8 layers = 1,90 m