Metapaste

Calcium Hydroxide
with Barium Sulfate

Water Soluble Temporary Root Canal Filling Material

APPLICATIONS

- Exposed pulp in pulp capping and pulpotomy
- Lapage canal
- Apicalization
- Formation of hard tissue barrier
- Root canal filling material

DIRECTIONS FOR USE

- Clean and dry it after preparing the root canal
- Set up a disposable tip in syringe and insert the ring provided for an easy re-position of the tip
- Direct the disposable tip with hand and fill the canal completely by pushing the syringe plunger while withdrawing the tip slowly
- Open any excess paste with a sterilized cotton pellet
- Remove the used up tip, a new one and cover up

How to use Rotary Ring

- As contact with air, its content could be hardened, so it is necessary to cover a cap on the needle after use
- Putting a Syringe into a Rotary Ring is more convenient for use to handle & Use Rotary Ring as it wishes to tighten or loosen a tip

ADVANTAGES

- Easy cleaning and removal
- Excellent antitoxigenic effect and radiopacity
- Prepared paste in a convenient syringe
- Excellent accessibility to the root canal and prevention of cross-contamination

CONTENTS

- 22g paste in 1 syringe
- Disposable tip
- One ring (rotator for the direction control of the tip)

STORAGE

- Do not fill beyond the apex
- For dental use only
- Do not use in infected person
- Keep the syringe closed at all times, when not in use
- Store in a dry and cool area to avoid cracks or separation of the paste

SHELF LIFE

- 2 years from the manufacturing date

SYMBOLS

See Me

Best Care

CE Mark and manufacturer name, finished date, product contents, lot, serial number, name of the Medical Device (metapaste) and ISO 13485

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Gamintojas: META BIOMED CO.LTD