



# Foliodrape® Protect Single Drapes

## General Product Description/Intended Purpose

Foliodrape® Protect surgical drapes and covers consist of a variety of products and materials: different layers of nonwoven material laminated with Polyethylene-film (PE) that are available in different designs, sizes and for different applications. They are single-use sterile patient drapes and covers for short-term use for prevention of infection on humans in clinical environment.

Foliodrape® patient drapes and covers are intended to be used as sterile draping of the patient and dedicated inventory in operating theatres and all other surgical environments (e.g. ambulance, doctor's surgery). Product is intended for use by healthcare professionals.

The main task of drapes consists of the prevention of pathogen entry into the surgical wound. Consequently, the risks of infections and blood-borne diseases are reduced. Product is intended for use by healthcare professionals. Foliodrape® self-adhesive 75 x 75 cm drape can be used as additional reinforcement for the front of a gown, which is added after putting on the gown.

The Foliodrape® products are placed on the market sterile and are in 'Medical Device Class' Is.

## Application/Indication

Draping is the perioperative process of covering a patient and surrounding areas in the operating theater with sterile drapes and covers to create and maintain a barrier, which protects the wound from the entry of microorganisms. The microorganisms can originate from the patient's skin, the OR, or the healthcare personnel. The aim of draping is to prevent the passage of pathogens between non-sterile and sterile areas. The different layers of hydrophilic nonwoven material laminated with Polyethylene-film act together as a fluid and bacteria barrier and minimize the transmission of micro-organisms. Depending on the procedure for which they are used, the Foliodrape® products usage time will vary from few minutes to several hours of continuous usage.

## Catalogue Numbers

Name of product	REF number	size	Attributes
Foliodrape® Protect Arthroscopy drape	938 820/2	245 x 320 cm	With fenestration, hook and loop fastener, pouch, and elastic cuff; wrapped in crepe paper
Foliodrape® Protect Back table cover	936 151/1	100 x 150 cm	-
	250 224/5	150 x 200 cm	-
	936 354/1	245 x 295 cm	Wrapped in crepe paper
Foliodrape® Protect Epidural drape, adhesive	936 276/1	75 x 60 cm	Self-adhesive, with fenestration
	938 809/2	90 x 60 cm	Self-adhesive, with fenestration
Foliodrape® Protect Extremity drape	936 651/0	130 x 150 cm	With fenestrated elastic cuff
Foliodrape® Protect Fenestrated drape	277 549/2	50 x 60 cm	With fenestration
	277 503/2	45 x 75 cm	With fenestration
	277 511/2	75 x 90 cm	With fenestration
	936 200/1	150 x 100 cm	With fenestration
	277 537/2	50 x 60 cm	Self-adhesive, with fenestration



## Foliodrape® Protect Single Drapes

Name of product	REF number	size	Attributes
<b>Foliodrape® Protect Fenestrated drape, adhesive</b>	936 184/0	50 x 60 cm	Self-adhesive, with fenestration
	277 517/2	90 x 75 cm	Self-adhesive, with fenestration
	277 555/2	90 x 75 cm	Self-adhesive, with fenestration
	936 177/0	100 x 150 cm	Self-adhesive, with fenestration
	936 186/0	150 x 120 cm	Self-adhesive, with fenestration
	936 176/1	150 x 170 cm	Self-adhesive, with fenestration
<b>Foliodrape® Protect Fenestrated drape, adhesive</b>	936 183/0	150 x 180 cm	Self-adhesive, with fenestration, wrapped in crepe paper
	936 154/0	170 x 250 cm	Self-adhesive, with fenestration, wrapped in crepe paper
	936 185/0	200 x 280 cm	Self-adhesive, with fenestration, wrapped in crepe paper
<b>Foliodrape® Protect Instrument collection drape</b>	936 335/0	100 x 150 cm	Self-adhesive
<b>Foliodrape® Protect Patient underlay</b>	936 653/0	100 x 120 cm	-
	936 652/0	100 x 120 cm	With Bag
	936 252/0	100 x 120 cm	With 2 Bags
<b>Foliodrape® Protect Split sheet, adhesive</b>	936 225/1	110 x 75 cm	Self-adhesive
	938 815/3	200 x 150 cm	Self-adhesive, wrapped in crepe paper
	938 817/2	225 x 280 cm	Self-adhesive, wrapped in crepe paper
<b>Foliodrape® Protect Surgical drape</b>	277 516/3	45 x 37,5 cm	-
	277 500/3	45 x 75 cm	-
	277 547/4	50 x 50 cm	-
	936 155/0	75 x 75 cm	-
	277 502/3	75 x 90 cm	-
	277 504/3	90 x 100 cm	-
	277 506/3	90 x 150 cm	-
	277 505/4	95 x 120 cm	-
	936 152/1	150 x 120 cm	-
	277 548/5	150 x 175 cm	-



## Foliodrape® Protect Single Drapes

Name of product	REF number	size	Attributes
	938 810/2	150 x 240 cm	-
<b>Foliodrape® Protect Surgical drape, adhesive</b>	277 507/3	45 x 75 cm	Self-adhesive
	277 546/4	50 x 50 cm	Self-adhesive
	936 277/1	75 x 75 cm	Self-adhesive
	277 508/3	75 x 90 cm	Self-adhesive
	936 275/1	75 x 90 cm	Self-adhesive
	277 542/4	90 x 100 cm	Self-adhesive
	277 551/5	240 x 150 cm	Self-adhesive
	277 543/5	150 x 175 cm	Self-adhesive, wrapped in crepe paper
	277 544/6	170 x 175 cm	Self-adhesive, wrapped in crepe paper
<b>Foliodrape® Protect Surgical drape, adhesive</b>	938 812/2	175 x 200 cm	Self-adhesive, wrapped in crepe paper
	936 280/0	240 x 200 cm	Self-adhesive, wrapped in crepe paper
<b>Foliodrape® Protect Turban drape</b>	277 538/5	300 x 170 cm	Self-adhesive, wrapped in crepe paper
	938 828/2	100 x 65 cm & 90 x 75 cm	Self-adhesive
		936 331/0	175 x 150 cm & 100 x 110 cm
<b>Foliodrape® Protect 2 parts variable fenestrated drape, adhesive</b>	277 509/2	45 x 75 cm	Self-adhesive, 2 pieces variable
	277 510/2	75 x 90 cm	Self-adhesive, 2 pieces variable

### Residual Risks, Contra-Indications and Undesirable Side Effects, Warnings

No contraindications known.

Skin and allergic reactions are rare and they occur only in single cases, not systematically. Due to different types of skin, disinfectant and different conditions during usage, the adhesive strength of the glue can differ. If the patient has known allergies, it needs to be checked before. All products and packaging belonging to the Foliodrape® assortment, including the raw materials as well as the finished goods, are not intentionally made with natural rubber latex. Furthermore, to the best of our knowledge, no natural rubber latex is used in the production processes. Thus, the Foliodrape® assortment fulfills the requirements set out in IMPLICATIONS OF THE MEDICAL DEVICES DIRECTIVES (93/42/EEC) IN



## Foliodrape® Protect Single Drapes

RELATION TO MEDICAL DEVICES CONTAINING NATURAL RUBBER LATEX: MEDDEV 2.5/9 rev.1, and are therefore not required to have a latex-free symbol on packaging.

### Preparatory Treatment

Sterile surgical drapes are used during surgery to prevent contact with unprepared surfaces and to maintain the sterility of environmental surfaces, equipment and the patient's surroundings.

**Skin and mucosis disinfection prior drape application:** Prepare the surface with antiseptic solution, the solution should remain in contact with the skin as per the instruction on the disinfectant label. Do not make contact with any other area of the patient. Lay the drapes on the patient as per their intended purpose recommended by the product name. Leave uncovered only the operative field and those areas necessary for the maintenance of anesthesia.

### Sterile Device

Do not use when sterile packaging or the product are damaged or were unintentionally opened before use.

### Single-use device

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

### Product disposal

In order to minimise the risk of potential infection hazards, or environmental pollution, disposable components of Foliodrape® products should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards. All infected waste is recommended to be disposed of by incineration. Non-contaminated material such as waste paper, packaging and non-sterile but not biologically contaminated materials can be disposed of separately. Separate disposal containers should be available where waste is created, so that staff can sort the waste as it is being discarded.

### Incident reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

### Product Performance Characteristics

Foliodrape® Protect drapes correspond to all requirements of European standard EN 13795-1:2019 for the performance level "High performance, critical area".

# Foliodrape® Protect Single Drapes

Foliodrape® Protect drapes were tested according to STANDARD FOR THE FLAMMABILITY OF CLOTHING TEXTILES (16 CFR 1610) and belong to class 1.

Foliodrape® Protect, base material, testing results according to EN 13795-1:2019					
Property	Test method	Unit	Product requirement (PR) equivalent EN 13795-1:2019 for level "High Performance"	Test result	
Resistance to microbial penetration – Dry	EN ISO 22612	CFU	Less critical product area: $\leq 300$ Critical product area: Not required <sup>2</sup>	Not required <sup>2</sup>	
Resistance to microbial penetration – Wet	EN ISO 22610	l <sub>B</sub>	Less critical product area: Not required Critical product area: 6.0	6.0 (no penetration)	
Cleanliness – microbial	EN ISO 11737-1	CFU/cm <sup>2</sup>	Less critical product area: $\leq 300$ Critical product area: $\leq 300$	Not required <sup>3</sup>	
Particle release	EN ISO 9073-10	log <sub>10</sub> (lint count)	Less critical product area: $\leq 4,0$ Critical product area: $\leq 4,0$	M <sub>d</sub> = 2.2 (side A) and 2.2 (side B) U <sub>q</sub> = 2.6 (side A) and 2.2 (side B)	
Resistance to liquid penetration <sup>4</sup>	EN 20811	cm (H <sub>2</sub> O)	Less critical product area: $\geq 10$ Critical product area: $\geq 100$	M <sub>d</sub> = 158 L <sub>q</sub> = 153	
Bursting strength – Dry <sup>5</sup> (both sides)	EN ISO 13938-1	kPa	Less critical product area: $\geq 40$ Critical product area: $\geq 40$	M <sub>d</sub> = 145 (outside) and 137 (inside) L <sub>q</sub> = 137 (outside) and 129 (inside)	
Bursting strength – Wet <sup>5</sup> (both sides)	EN ISO 13938-1	kPa	Less critical product area: Not required Critical product area: $\geq 40$	M <sub>d</sub> = 134 (outside) and 128 (inside) L <sub>q</sub> = 128 (outside) and 122 (inside)	
Tensile strength – Dry	EN 29073-3	N/50 mm	Less critical product area: $\geq 20$ Critical product area: $\geq 20$	Longitudinal direction: M <sub>d</sub> = 78,5 L <sub>q</sub> = 75.1 Lateral direction : M <sub>d</sub> = 46,3 L <sub>q</sub> = 45.6	
Tensile strength – Wet	EN 29073-3	N/50 mm	Less critical product area: Not required Critical product area: $\geq 20$	Longitudinal direction: M <sub>d</sub> = 82,2 L <sub>q</sub> = 81.5 Lateral direction: M <sub>d</sub> = 46,8 L <sub>q</sub> = 46.2	
Absorbency (laminate)	EN ISO 9073-6	%	No normative requirement	285 ( $\cong$ approx. 156 ml/m <sup>2</sup> )	

# Foliodrape® Protect Single Drapes

<sup>1</sup> Example REF 938 707 – Lot 400114277, unreinforced part, Test report from independent Testing-Institute is available on request. Abbreviations: L<sub>q</sub> = lower quartile value, U<sub>q</sub> = upper quartile value and M<sub>d</sub> = median value. Product conforms when: L<sub>q</sub> ≥ PRmin resp. U<sub>q</sub> ≤ PRmax.

<sup>2</sup> Not required when the product fulfill the criteria for high performance in critical product area for resistance to microbial penetration wet (see Annex B7 in EN 13795:2019)

<sup>3</sup> see clause A1.2 in EN 13795-1:2019, Foliodrape® articles are sterile according to EN 556

<sup>4</sup> measured without use of supporting tissue (according to EN 20811),

<sup>5</sup> values after membrane correction (according to EN ISO 13938-1)

## Material Characteristics

Composition	
<b>Nonwoven Material:</b>	Hydrophilic PP-spunbond, (30g/m <sup>2</sup> ) color: aqua
<b>Film:</b>	Pore-free polyethylene-film, (25g/m <sup>2</sup> ) color: aqua
<b>Total Laminate:</b>	Thermobonded,(55g/m <sup>2</sup> ) latex-free
<b>Sterilization:</b>	With ethylene oxide gas according to EN 556-1 and EN 11135-1

- 
1. Hydrophilic PP spunbond
  2. PE-Film (impermeable for moisture and bacteria)

## Product Characteristics

Name of Product	REF number	size of drape	size of self-adhesive area	size of fenestration or split
<b>Foliodrape® Protect Arthroscopy drape</b>	938 820/2	245 x 320 cm	---	Ø 5 cm (round) 5 x 7 cm (oval)
<b>Foliodrape® Protect Back table cover</b>	936 151/1	100 x 150 cm	---	---
	250 224/5	150 x 200 cm	---	---
	936 354/1	245 x 295 cm	---	---
<b>Foliodrape® Protect Epidural drape, adhesive</b>	936 276/1	75 x 60 cm	60 x 4,5 cm & 16 x 18 cm	10 x 10 cm (rhomboid)
	938 809/2	90 x 60 cm	60 x 4,5 cm	10 x 12 cm (rhomboid)
<b>Foliodrape® Protect Extremity drape</b>	936 651/0	130 x 150 cm	---	Ø 3 cm
<b>Foliodrape® Protect Fenestrated drape</b>	277 549/2	50 x 60 cm	---	Ø 5 cm
	277 503/2	45 x 75 cm	---	Ø 7 cm
	277 511/2	75 x 90 cm	---	Ø 10 cm
	936 200/1	150 x 100 cm	---	7 x 10 cm (oval)
<b>Foliodrape® Protect Fenestrated drape, adhesive</b>	277 537/2	60 x 50 cm	17 x 18 cm	Ø 7 cm (round)
	936 184/0	50 x 60 cm	2,5 cm wide around fenestration	Ø 7 cm (round)
	277 517/2	90 x 75 cm	18 x 17 cm	Ø 7 cm (round)



## Foliodrape® Protect Single Drapes

Name of Product	REF number	size of drape	size of self-adhesive area	size of fenestration or split
	277 555/2	90 x 75 cm	2,5 cm wide around fenestration	Ø 10 cm (round)
	936 177/0	100 x 150 cm	14 x 14 cm	Ø 10 cm (round)
	936 186/0	150 x 120 cm	18 x 17 cm	Ø 7 cm (round)
	936 176/1	150 x 170 cm	18 x 18 cm	Ø 12 cm (round)
	936 183/0	150 x 180 cm	20,5 x 25 cm	5 x 15 cm (oval)
	936 154/0	170 x 250 cm	20,5 x 17 cm	Ø 10 cm (round)
	936 185/0	200 x 280 cm	22 x 25 cm	Ø 15 cm (round)
<b>Foliodrape® Protect Instrument collection drape</b>	936 335/0	100 x 150 cm	96 x 6 cm	---
<b>Foliodrape® Protect Patient underlay</b>	936 653/0	100 x 120 cm	---	---
	936 252/0	100 x 120 cm	---	---
	936 652/0	100 x 120 cm	---	---
<b>Foliodrape® Protect Split sheet, adhesive</b>	936 225/1	110 x 75 cm	3 cm around split	10 x 12 cm
	938 815/3	200 x 150 cm	5 cm around split	10 x 60 cm
	938 817/2	225 x 280 cm	5 cm around split	10 x 95 cm
<b>Foliodrape® Protect Surgical drape</b>	277 516/3	45 x 37,5 cm	---	---
	277 500/3	45 x 75 cm	---	---
	277 547/4	50 x 50 cm	---	---
	936 155/0	75 x 75 cm	---	---
	277 502/3	75 x 90 cm	---	---
	277 504/3	90 x 100 cm	---	---
	277 506/3	90 x 150 cm	---	---
	277 505/4	95 x 120 cm	---	---
	936 152/1	150 x 120 cm	---	---
	277 548/5	150 x 175 cm	---	---
	938 810/2	150 x 240 cm	---	---
<b>Foliodrape® Protect Surgical drape, adhesive</b>	277 507/3	45 x 75 cm	71 x 4,5 cm	---
	277 546/4	50 x 50 cm	48 x 4,5 cm	---
	936 277/1	75 x 75 cm	71 x 4,5 cm	---
	277 508/3	75 x 90 cm	86 x 4,5 cm	---
	936 275/1	75 x 90 cm	86 x 2,5 cm	---
<b>Foliodrape® Protect Surgical drape, adhesive</b>	277 542/4	90 x 100 cm	96 x 4,5 cm	---
	277 551/5	240 x 150 cm	80 x 5 cm	---
	277 543/5	150 x 175 cm	90 x 5 cm	---
	277 544/6	170 x 175 cm	90 x 5 cm	---



# Foliodrape® Protect Single Drapes

Name of Product	REF number	size of drape	size of self-adhesive area	size of fenestration or split
	938 812/2	175 x 200 cm	90 x 5 cm	---
	936 280/0	240 x 200 cm	240 x 6,25 cm	---
	277 538/5	300 x 170 cm	102 x 6,5 cm	---
<b>Foliodrape® Protect Turban drape</b>	938 828/2	100 x 65 cm & 90 x 75 cm	6 x 76 cm 6 x 15 cm x 2	---
	936 331/0	175 x 150 cm & 100 x 110 cm	---	75 cm slit
<b>Foliodrape® Protect 2 parts variable fenestrated drape, adhesive</b>	277 509/2	45 x 75 cm	18 x 33 cm	16,5 x 22 cm
	277 510/2	75 x 90 cm	28 x 34 cm	21,5 x 25 cm

## Labelling

The Foliodrape® labeling describes the intended purpose of the device as supported by clinical evaluation. Safety information supplied by the legal manufacturer on the labeling are based on the risk management documentation and elaborately present all warnings and precautions according to the new requirements of the standards concerning the product.

### Lot-No. with 9-Digit Code

e.g.:



0

year

12

week  
of production

XXXXXX

for internal purposes only

### Date of manufacture

e.g.:



2022

year

04

month

07

day

### Use-by Date

e.g.:



2027

year

04

month

07

day

Shelf Life: 5 years

### Medical Device



### Unique Device Identifier



### Single Sterile Barrier System



Single sterile barrier system

# Foliodrape® Protect Single Drapes

Do not use if the product packaging is compromised



Single use device



Keep dry



Sterilized using ethylene oxide



Legal manufacturer



European conformity



Dispose of your waste in an appropriate manner



Recycling symbols – Corrugated cardboard and Low-density polyethylene



Producer contributes to recycling of packaging



Indication arrow where to open the packaging




Name and intended purpose of the product

Lochtuch, selbstklebend  
Fenestrated drape, adhesive

Reference / catalogue number



Pictogram with product look and dimensions

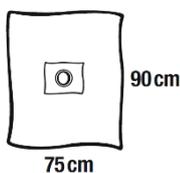



# Foliodrape® Protect Single Drapes

## Logistic Data

Name of Product	REF number	Peel Pouch size [mm]	Products per Peel Pouch	Peel Pouches per dispenser	Dispenser size	EAN of dispenser	Dispensers per carton
<b>Foliodrape® Protect Arthroscopy drape</b>	938 820/2	Size: XL 345 x 510	1	6	Size: XL 374 x 545 x 234	4052199212302	1
<b>Foliodrape® Protect Back table cover</b>	936 151/1	Size: M 266,5 x 360	1	35	Size: L 274 x 368 x 296	4052199501581	2
	250 224/5	Size: M 266,5 x 360	1	18	Size: L 274 x 368 x 296	4052199501994	2
	936 354/1	Size: M 266,5 x 360	1	7	Size: L 274 x 368 x 296	4052199574660	2
<b>Foliodrape® Protect Epidural drape, adhesive</b>	936 276/1	Size: S 133,5 x 290	1	36	Size: M 140 x 300 x 370	4052199501758	4
	938 809/2	Size: S 133,5 x 290	1	50	Size: M 140 x 300 x 370	4049500113151	4
<b>Foliodrape® Protect Extremity drape</b>	936 651/0	Size: M 266,5 x 360	1	23	Size: L 274 x 368 x 296	4052199574172	2
<b>Foliodrape® Protect Fenestrated drape</b>	277 549/2	Size: S 133,5 x 290	1	70	Size: M 140 x 300 x 370	4049500639224	4
	277 503/2	Size: S 133,5 x 290	1	65	Size: M 140 x 300 x 370	4049500497831	4
	277 511/2	Size: S 133,5 x 290	1	40	Size: M 140 x 300 x 370	4049500562324	4
	936 200/1	Size: M 266,5 x 360	1	40	Size: L 274 x 368 x 296	4052199501918	2
<b>Foliodrape® Protect Fenestrated drape, adhesive</b>	277 537/2	Size: S 133,5 x 290	1	70	Size: M 140 x 300 x 370	4049500627276	4
	936 184/0	Size: M 266,5 x 360	1	80	Size: L 274 x 368 x 296	4052199577463	2
	277 517/2	Size: S 133,5 x 290	1	40	Size: M 140 x 300 x 370	4052199501383	4
	277 555/2	Size: S 133,5 x 290	1	35	Size: M 140 x 300 x 370	4049500647472	4
	936 177/0	Size: M 266,5 x 360	1	26	Size: L 274 x 368 x 296	4052199509587	2
	936 186/0	Size: M 266,5 x 360	1	25	Size: L 274 x 368 x 296	4052199577616	2
	936 176/1	Size: M 266,5 x 360	1	22	Size: L 274 x 368 x 296	4052199501543	2
	936 183/0	Size: M	1	17	Size: L	4052199570587	2



## Foliodrape® Protect Single Drapes

		266,5 x 360			274 x 368 x 296		
	936 154/0	Size: M 266,5 x 360	1	11	Size: L 274 x 368 x 296	4052199530413	2
	936 185/0	Size: L 360 x 580	1	15	Size: XL 368 x 568 x 300	4052199577425	1
<b>Foliodrape® Protect Instrument collection drape</b>	936 335/0	Size: M 266,5 x 360	1	30	Size: L 274 x 368 x 296	4052199521916	2
<b>Foliodrape® Protect Patient underlay</b>	936 653/0	Size: M 266,5 x 360	1	30	Size: L 274 x 368 x 296	4052199600437	2
	936 652/0	Size: M 266,5 x 360	1	25	Size: L 274 x 368 x 296	4052199577579	2
	936 252/0	Size: M 266,5 x 360	1	15	Size: L 274 x 368 x 296	4052199532493	2
<b>Foliodrape® Protect Split sheet, adhesive</b>	936 225/1	Size: S 133,5 x 290	2	15	Size: M 140 x 300 x 370	4052199570488	4
	938 815/3	Size: M 266,5 x 360	1	14	Size: L 274 x 368 x 296	4052199509747	2
	938 817/2	Size: L 360 x 580	1	13	Size: XL 368 x 568 x 300	4052199509266	1
<b>Foliodrape® Protect Surgical drape</b>	277 516/3	Size: S 133,5 x 290	1	125	Size: M 140 x 300 x 370	4049500570015	4
	277 500/3	Size: S 133,5 x 290	1	65	Size: M 140 x 300 x 370	4049500497558	4
	277 547/4	Size: S 133,5 x 290	1	100	Size: M 140 x 300 x 370	4052199028972	4
	936 155/0	Size: S 133,5 x 290	1	50	Size: M 140 x 300 x 370	4052199600475	4
	277 502/3	Size: S 133,5 x 290	1	35	Size: M 140 x 300 x 370	4049500497794	4
	277 504/3	Size: S 133,5 x 290	1	25	Size: M 140 x 300 x 370	4049500497879	4
	277 506/3	Size: S 133,5 x 290	1	20	Size: M 140 x 300 x 370	4049500562126	4
	277 505/4	Size: S 133,5 x 290	1	25	Size: M 140 x 300 x 370	4049500564953	4
	936 152/1	Size: M 266,5 x 360	1	30	Size: L 274 x 368 x 296	4052199501345	2
	277 548/5	Size: M 266,5 x 360	1	25	Size: L 274 x 368 x 296	4052199502199	2
	938 810/2	Size: M 266,5 x 360	1	16	Size: L 274 x 368 x 296	4052199500836	2
<b>Foliodrape® Protect Surgical drape, adhesive</b>	277 507/3	Size: S 133,5 x 290	1	60	Size: M 140 x 300 x 370	4049500562164	4
	277 546/4	Size: S	1	90	Size: M	4052199029016	4



## Foliodrape® Protect Single Drapes

	133,5 x 290			140 x 300 x 370		
936 277/1	Size: S 133,5 x 290	1	45	Size: M 140 x 300 x 370	4052199501284	4
277 508/3	Size: S 133,5 x 290	1	40	Size: M 140 x 300 x 370	4049500562201	4
936 275/1	Size: S 133,5 x 290	1	40	Size: M 140 x 300 x 370	4052199501505	4
277 542/4	Size: S 133,5 x 290	1	25	Size: M 140 x 300 x 370	4052199502038	4
277 551/5	Size: M 266,5 x 360	1	15	Size: L 274 x 368 x 296	4052199502076	2
277 543/5	Size: M 266,5 x 360	1	19	Size: L 274 x 368 x 296	4052199502151	2
277 544/6	Size: M 266,5 x 360	1	15	Size: L 274 x 368 x 296	4052199502854	2
938 812/2	Size: M 266,5 x 360	1	17	Size: L 274 x 368 x 296	4052199500546	2
936 280/0	Size: M 266,5 x 360	1	11	Size: L 274 x 368 x 296	4052199522968	2
277 538/5	Size: M 266,5 x 360	1	11	Size: L 274 x 368 x 296	4052199509761	2
<b>Foliodrape® Protect Turban drape</b>	938 828/2 Size: L 265 x 365	1	22	Size: L 275 x 374 x 234	4052199211718	2
	936 331/0 Size: M 266,5 x 360	1	13	Size: L 274 x 368 x 296	4052199523163	2
<b>Foliodrape® Protect 2 parts variable fenestrated drape, adhesive</b>	277 509/2 Size: S 133,5 x 290	2	65	Size: M 140 x 300 x 370	4052199533377	4
	277 510/2 Size: S 133,5 x 290	2	40	Size: M 140 x 300 x 370	4052199533339	4

Date of revision of the text: 2022-03-04