

## SURFACE DISINFECTION AND CLEANING



## Sani-Cloth® Active

Alcohol-free wipes

## VIRUCIDAL. RELIABLE.

Ready-to-use QAC-based cleaning and disinfection wipes with a unique efficacy - including TB and viruses like Noro and Polio - and an excellent material compatibility. The performance of Sani-Cloth® Active is proven in tests with solution released from the wipe. Thereby the efficacy under real working conditions is ensured.



## SURFACE DISINFECTION AND CLEANING

## Alcohol-free wipes

## VIRUCIDAL

Our Offer	Your Benefit
Efficacy against bacteria (incl. TB), fungi and viruses	Also suitable for high risk areas (e. g. operating room, intensive care units)
Excellent material compatibility	Suitable for all kind of surfaces including ultrasound probes and transducers
Combines cleaning and disinfection	Time saving for user
Pre-impregnated wipes	No risk of misdosing
Convenient and safe dispensing of single wipes	Reduced risk of drying out

Disinfection of medical devices and all kind of surfaces in hospital, nursing and dental - including alcohol sensitive surfaces, e. g. ultrasound probes and transducers, acryl and plexiglass.

Take single wipe. Wipe surface, let it dry and do not rinse. Ensure contact with complete surface. Close canister after use.

## MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

<b>Efficacy undiluted</b> Tested with solution released from the wipe	<b>Contact time</b>
Disinfection of surfaces acc. to DGHM Method 2001 with mechanical action: bactericidal, yeasticidal (clean & dirty)	<b>5 min</b>
EN 13727 bactericidal (clean & dirty)	<b>1 min</b>
EN 13624 yeasticidal (clean & dirty)	<b>1 min</b>
EN 13624 fungicidal (clean & dirty)	<b>15 min</b>
EN 14348 tuberculocidal (clean & dirty)	<b>15 min</b>
Virucidal efficacy against enveloped viruses (incl. HBV, HCV, HIV) acc. to RKI recommendation 01/2004	<b>30 sec</b>
Noro virus (tested with murine noro virus)	<b>1 min</b>
Virucidal efficacy acc. RKI recommendation 01/2004	<b>30 min</b>
EN 14476 virucidal (clean)	<b>30 min</b>

100 g of soaking solution contain the following active ingredient:  
0.45g Didecyldimethylammonium chloride

Sani-Cloth® Active is been broadly tested. Please ask for our compilation of expertises. VAH certification submitted.

**For professional use only.**  
Use disinfectants safely. Always read product description and information before use.

**Sani-Cloth® Active** is available in various wipe and packaging sizes.

**Sani-Cloth® Active** 3051080  
200 wipe canister 6 x 200 wipes  
wipe size: 200 x 220 mm

<b>Sani-Cloth® Active</b>	<b>3051100</b>
125 wipe canister	6 x 125 wipes
wipe size: 130 x 220 mm	

**Sani-Cloth® Active** 3051120  
125 wipe refill bag 9 x 125 wipes  
wipe size: 130 x 220 mm

<b>Sani-Cloth® Active</b>	<b>3051140</b>
225 wipe bucket	1 x 225 wipes
wipe size: 245 x 300 mm	

11/13/09/EII



**Ecolab Healthcare Europe**  
Clean. Safe. Healthy.

**Ecolab Inc.**  
370 Wabasha Street N St. Paul, MN 55102  
www.ecolab.com  
© 2009 Ecolab Inc. All rights reserved.

**Ecolab Europe GmbH**  
Richtistr. 7  
CH-8304 Wallisellen

