254047 20 plates Salmonella Shigella Agar (SS Agar)

A differential selective medium for the isolation of pathogenic enteric bacilli, especially those belonging to the genus *Salmonella*. This medium is not recommended for the primary isolation of *Shigella*.

254042 20 plates Schaedler Agar with Vitamin K1 and 5 % Sheep Blood

A highly nutritious medium for the isolation and cultivation of fastidious anaerobic microorganisms. It is especially useful for the recovery of the fastidious anaerobic bacteria such as *Bacteroides*, *Prevotella* and *Porphyromonas* species.

Antimicrobial Susceptibility Testing (AST) Media

CAT. NO. SIZE DESCRIPTION

254030 20 plates Mueller Hinton Agar with 5 % Sheep Blood

Recommended by CLSI (Clinical and Laboratory Standards Institute) for disc diffusion susceptibility testing of *Streptococcus pneumoniae* and other streptococci.

254032 20 plates Mueller Hinton II Agar

Recommended by CLSI (Clinical and Laboratory Standards Institute) for disc diffusion susceptibility testing of rapidly-growing aerobic organisms.

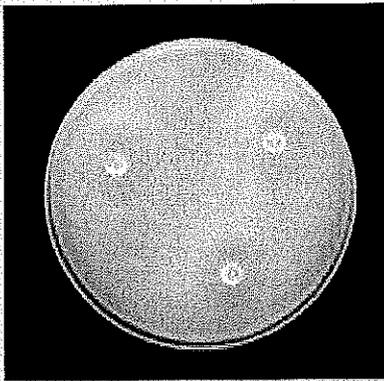
✓ 254060 20 plates Chocolate Agar (GC II Agar with BD IsoVitalX™)

#140

Recommended by CLSI (Clinical and Laboratory Standards Institute) for disc diffusion susceptibility testing of *Neisseria gonorrhoeae*. This medium is prepared with a chemically defined supplement developed specially to aid the growth of gonococci.

254058 20 plates Haemophilus Test Medium Agar

Recommended by CLSI (Clinical and Laboratory Standards Institute) for disc diffusion susceptibility testing of *Haemophilus influenzae* and related species.

BD CHROMagar™ Media

An advanced microbiological testing media that enable, in a single step, the isolation and differentiation of pathogenic micro-organisms by colony colors. These media include a chromogen mix that consists of substrates (chromogens) that release differently colored compounds upon degradation by specific microbial enzymes, thus assuring the direct differentiation of certain species or the detection of certain groups of organisms with only a minimum of confirmatory tests.

CAT. NO. SIZE DESCRIPTION

✓ 257480 20 plates BD CHROMagar™ Candida

#135

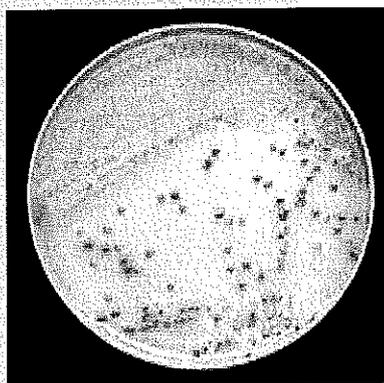
For the isolation and differentiation of *Candida albicans*, *C. tropicalis* and *C. krusei*. Due to the differences in morphology and colors of the yeast colonies, this medium facilitates the detection of mixed yeast cultures in specimens. It may also be used as a selective isolation medium for other yeasts and for filamentous fungi.

257434 20 plates BD CHROMagar™ MRSA II

A selective and differential medium for the direct detection of methicillin-resistant *Staphylococcus aureus* (MRSA) from clinical specimens. The test can be performed on respiratory (e.g., nares, throat and sputum), lower gastrointestinal (GI) (e.g., rectal and stool), skin (e.g., groin/axilla and perineum/perianal), and wound specimens, and positive blood culture bottles containing grampositive cocci.

257481 20 plates BD CHROMagar™ Orientation

A non-selective medium for the direct identification, differentiation and enumeration of urinary tract pathogens.

INSTRUCTIONS FOR USE --
READY-TO-USE PLATED
MEDIA

Rev.: Sep 2011

PA-255509.05

BD™ Brucella Blood Agar with Hemin and Vitamin K1

#136,

INTENDED USE

BD Brucella Blood Agar with Hemin and Vitamin K1 is a highly nutritious medium for the isolation and cultivation of strict anaerobes from clinical specimens.

PRINCIPLES AND EXPLANATION OF THE PROCEDURE

Microbiological method.

BD Brucella Blood Agar with Hemin and Vitamin K1 is a modification of Brucella Agar that has been supplemented with hemin and vitamin K1 to support the growth of fastidious anaerobes, especially *Bacteroides*, *Prevotella*, and *Porphyromonas* when incubated anaerobically.^{1,3} It is used for the isolation of strict anaerobes from clinical specimens. It is also used for susceptibility testing of anaerobes with the E test[®] method.^{4,5} In BD Brucella Blood Agar with Hemin and Vitamin K1, peptones and yeast extract, together with glucose, supply nutrients. Sodium bisulfite lowers the redox potential to a range suitable for strict anaerobes. Hemin and vitamin K1 have been shown to be necessary for supporting the growth of certain strict anaerobes.⁷ Sheep blood provides additional nutrients and is used to detect hemolytic reactions.

REAGENTS

BD Brucella Blood Agar with Hemin and Vitamin K1

Formula* Per Liter Purified Water

| | |
|--------------------------------|--------|
| Pancreatic Digest of Casein | 10.0 g |
| Peptic Digest of Animal Tissue | 10.0 |
| Yeast Extract | 2.0 |
| Glucose | 1.0 |
| Sodium Chloride | 5.0 |
| Sodium Bisulfite | 0.1 |
| Hemin | 0.005 |
| Vitamin K1 | 0.01 |
| Agar | 15.0 |
| Sheep Blood, defibrinated | 5% |

pH 7.2 +/- 0.2

*Adjusted and/or supplemented as required to meet performance criteria.

PRECAUTIONS

[VD] For professional use only. ☹

Do not use plates if they show evidence of microbial contamination, discoloration, drying, cracking or other signs of deterioration. Consult GENERAL INSTRUCTIONS FOR USE document for aseptic handling procedures, biohazards, and disposal of used product.

STORAGE AND SHELF LIFE

On receipt, store plates in the dark at 2 to 8° C, in their original sleeve wrapping until just prior to use. Avoid freezing and overheating. The plates may be inoculated up to the expiration date (see package label) and incubated for the recommended incubation times. Plates from opened stacks of 10 plates can be used for one week when stored in a clean area at 2 to 8° C.

PA-255509.05

- 1 -

USER QUALITY CONTROL

Inoculate representative samples with the following strains (for details, see GENERAL INSTRUCTIONS FOR USE document). Incubate for 48 to 72 hours in an anaerobic atmosphere (e.g. BD GasPak™ Anaerobic System) at 35 to 37° C.

| Strains | Growth Results |
|---|--|
| <i>Bacteroides fragilis</i> ATCC™ 25285 | Growth good to excellent; grey colonies |
| <i>Clostridium perfringens</i> ATCC 13124 | Growth good to excellent; large, lobate, grey-white colonies; beta (double-zone) hemolysis |
| <i>Fusobacterium nucleatum</i> ATCC-25586 | Growth good to excellent; grey-white colonies, surrounded by dark-grey zones |
| <i>Peptostreptococcus anaerobius</i> ATCC 27337 | Growth good to excellent; whitish colonies |
| <i>Porphyromonas levii</i> ATCC 29147 | Growth fair to good; dirty whitish to grey-brown colonies |
| Uninoculated | Red to dark red (blood color) |

PROCEDURE

Materials Provided

BD Brucella Blood Agar with Hemin and Vitamin K1 (90 mm Stacker™ plates). Microbiologically controlled.

Materials Not Provided

Ancillary culture media, reagents and laboratory equipment as required.

Specimen Types

BD Brucella Blood Agar with Hemin and Vitamin K1 is a universal medium for the isolation and cultivation of strict anaerobes from all types of specimens (see also PERFORMANCE CHARACTERISTICS AND LIMITATIONS OF THE PROCEDURE). Observe approved techniques for collection and transport of anaerobic specimens.⁸⁻¹⁰ Suitable transport media, e.g., BD Port-A-Cult™, must be used.

BD Brucella Blood Agar with Hemin and Vitamin K1 is also used for susceptibility testing of strict anaerobes with the E test which requires the use of pure cultures.^{4,5}

Test Procedure

Streak the specimen directly after arrival, using an approved streaking technique onto BD Brucella Blood Agar with Hemin and Vitamin K1. Directly after streaking, place the plates into anaerobic jars supplied with an anaerobic atmosphere. It is recommended to use BD GasPak jars and BD GasPak H₂/CO₂ envelopes together with a catalyst. Incubate for 2 to 3 days or longer, if necessary, at 35 to 37° C. Regardless of the anaerobic system used, it is important to include an indicator of anaerobiosis such as the BD GasPak disposable anaerobic indicator. If specimens containing mixed flora are streaked onto the medium, it is recommended to also include selective media such as BD Schaedler Kanamycin-Vancomycin Agar with 5% Sheep Blood and/or BD Wilkins-Chalgren Agar with Amikacin and 7% Sheep Blood. Also, facultative anaerobes might be present in the specimen. Therefore, it is recommended to always include an aerobic medium (such as BD Columbia Agar with 5% Sheep Blood) when the primary cultures are set up. This plate is incubated aerobically enriched with carbon dioxide together with the anaerobic cultures.⁹ It allows the detection of facultative organisms in the specimen.

For use of BD Brucella Blood Agar with Hemin and Vitamin K1 in anaerobic susceptibility testing with the E test, consult the references or the instructions of the manufacturer.^{4,5}

Results

After incubation, the plates are inspected for growth. Colonies which appear on this medium are suspected to be strict anaerobes if they do not grow on aerobically incubated blood agar plates. Also, growth on BD Schaedler Kanamycin-Vancomycin Agar with 5% Sheep Blood is compared to the growth on the other media. If mixed cultures of strict and facultative

PA-255509.05

- 2 -

anaerobes are present, appropriate subcultures on non-selective media, incubated aerobically and anaerobically, must be made from the anaerobic media to confirm that the isolate is a strict anaerobe.

Further microscopic and biochemical examination is necessary for the identification of the genera and species of the strict anaerobes. Consult appropriate texts for further information, including identification procedures.^{8, 9, 11} For reading the E test results obtained on this medium, consult the references or the instructions of the manufacturer.^{4, 6}

PERFORMANCE CHARACTERISTICS AND LIMITATIONS OF THE PROCEDURE

On BD Brucella Blood Agar with Hemin and Vitamin K1 which is one of the nonselective standard media for the isolation of strict anaerobes, *Bacteroides*, *Prevotella*, *Porphyromonas*, *Fusobacterium*, *Clostridium*, *Peptostreptococcus*, strictly anaerobic non-sporforming rods (e.g., the former genus *Eubacterium*), *Mobiluncus*, *Actinomyces* and many others will grow.^{8, 9, 11} Additionally, it is used for susceptibility testing with the E test method.^{4, 6}

Performance Results¹²

BD Brucella Blood Agar with Hemin and Vitamin K1 was evaluated internally with clinical isolates and collection strains of the following strictly anaerobic species and compared to BD Schaedler Agar with Vitamin K1 and 5% Sheep Blood (= reference medium): *Bacteroides fragilis*, *B. distasonis*, *Prevotella bivia*, *Fusobacterium nucleatum*, *Porphyromonas levii*, *Porphyromonas gingivalis*, *Campylobacter (Bacteroides) gracilis*, *Peptostreptococcus anaerobius*, *Clostridium perfringens*, *Mobiluncus mulieris*, *Eggerthella lenta (Eubacterium lentum)*. Both media were inoculated with 10⁷ to 10¹⁰ cfu per plate and were incubated anaerobically for 3 days. The number of colonies on the test medium was equal to or higher than that on test reference medium for all of the strains tested.

Limitations of the Procedure

Note that the growth rates of strict anaerobes vary considerably: While *Bacteroides fragilis* will grow well after 24 hours, *Mobiluncus* or strains of *Porphyromonas* need 4 to 5 days, and *Actinomyces* may need 1 to 3 weeks to produce well visible colonies. If cultures are negative after 2 or 3 days of incubation, re-incubate anaerobically for an additional 2 to 3 days. If *Actinomyces* is suspected, dedicated culture plates of this and other media (e.g., BD Columbia Agar with 5% Sheep Blood) should be inoculated that are inspected after 1, 2 and eventually 3 weeks of incubation.

This medium is not specifically selective for strict anaerobes; facultative organisms will also grow. Therefore, it is important to compare the result of the anaerobic culture with that of an aerobically incubated plate if mixed cultures are obtained.

REFERENCES

1. MacFaddin, J.F. 1985. Media for isolation cultivation-identification-maintenance of medical bacteria, vol. 1. Williams & Wilkins, Baltimore.
2. Gibbons, R.J., and J.B. MacDonald. 1960. Hemin and vitamin K compounds as required factors for the cultivation of certain strains of *Bacteroides melaninogenicus*. J. Bacteriol. 80:164-170.
3. Finegold, S.M., V.L. Sutter, H.R. Attebery, and J.E. Rosenblatt. 1974. Isolation of anaerobic bacteria, p. 368. In E.H. Lennette, E.H. Spaulding, and J.P. Truant (ed.), Manual of clinical microbiology, 2nd ed. American Society for Microbiology, Washington, D.C.
4. Citron, D.M., et al. 1991. Evaluation of the E test for susceptibility testing of anaerobic bacteria. J. Clin. Microbiol. 29: 2197-2203.
5. Church, D.L., et al. 1996. Metronidazole susceptibility and the presence of hydrogenase in pathogenic bacteria.
6. Citron, D.A., and D.W. Hecht. 2003. Susceptibility test methods: anaerobic bacteria. In: Murray, P. R., E. J. Baron, J.H. Tenover, M. A. Tenover, and R. H. Tenover (ed.), Manual of clinical microbiology, 8th ed. American Society for Microbiology, Washington, D.C.

7. Gibbons, R.J., and J.B. MacDonald. 1960. Hemin and vitamin K compounds as required factors for the cultivation of certain strains of *Bacteroides melaninogenicus*. J. Bacteriol. 80:164-170.
8. Engelkirk, P. G., J. Duben-Engelkirk, and V.R. Dowell, Jr. 1992. Principles and practice of clinical anaerobic bacteriology. Star Publishing Co., Belmont, Calif.
9. Summanen, P., E. J. Baron, D.M. Citron, C.A. Strong, H.M. Wexler, and S.M. Finegold. 1993. Wadsworth anaerobic bacteriology manual, 5th ed. Star Publishing Co., Belmont, Calif.
10. Murray, P. R., and D.M. Citron. 1991. General processing of specimens for anaerobic bacteria. In: A. Balows, W.J. Hausler, Jr., K.L. Herrmann, H.D. Isenberg, and H.J. Shadomy (ed.), Manual of clinical microbiology, 5th ed. American Society for Microbiology, Washington, D.C.
11. Murray, P.R., E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.). 1995. Manual of clinical microbiology, 6th ed. American Society for Microbiology, Washington, D.C.
12. Data on file. BD Diagnostic Systems Europe. Heidelberg, Germany.

PACKAGING/AVAILABILITY

BD Brucella Blood Agar With Hemin And Vitamin K1
Cat. No. 255509  Ready-to-use Plated Media, cpu 20 #136.

FURTHER INFORMATION

For further information please contact your local BD representative.

Becton Dickinson GmbH

Tullastrasse 8 - 12
D-69126 Heidelberg/Germany
Phone: +49-62 21-30 50 Fax: +49-62 21-30 52 16
Reception_Germany@europe.bd.com

<http://www.bd.com>

<http://www.bd.com/europe/regulatory/>

E test is a registered trademark of AB Biodisk, Solna, Sweden
ATCC is a trademark of the American Type Culture Collection

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. © 2011 BD

**INSTRUCTIONS FOR USE -
READY-TO-USE PLATED
MEDIA**

PA-254005.06

Rev.: Apr 2013

BD™ Columbia Agar with 5% Sheep Blood

#139

INTENDED USE

BD Columbia Agar with 5% Sheep Blood is a highly nutritious general purpose medium for the isolation and cultivation of nonfastidious and fastidious microorganisms from clinical specimens.

PRINCIPLES AND EXPLANATION OF THE PROCEDURE

Microbiological method.

Ellner et al.¹ in 1968 reported the development of a new blood agar formulation, which has been designated as Columbia Agar. BD Columbia Agar with 5% Sheep Blood derives its superior growth-supporting properties from the combination of two peptones, and yeast extract as a supplier of the B complex vitamins. Corn starch is included to absorb toxic by-products contained in the specimen and serves as an energy source for organisms possessing alpha-amylases. Sheep blood allows detection of hemolytic reactions and supplies the X factor (heme) necessary for the growth of many pathogenic species.

On this medium, colonies tend to be larger, and growth is more luxuriant than on media containing other blood agar bases. Columbia Blood Agar is recommended as a primary isolation medium in the MIQ standards and in other diagnostic manuals.^{2,3} In many European countries, this medium has become the most frequently used primary isolation medium for clinical specimens.

REAGENTS

BD Columbia Agar with 5% Sheep Blood

| Formula* Per Liter Purified Water | |
|-----------------------------------|--------|
| Pancreatic Digest of Casein | 12.0 g |
| Peptic Digest of Animal Tissue | 5.0 |
| Yeast Extract | 3.0 |
| Beef Extract | 3.0 |
| Corn Starch | 1.0 |
| Sodium Chloride | 5.0 |
| Agar | 13.5 |
| Sheep Blood, Defibrinated | 5% |

pH 7.3 ± 0.2

*Adjusted and/or supplemented as required to meet performance criteria.

PRECAUTIONS

IVD. For professional use only.

Do not use plates if they show evidence of microbial contamination, discoloration, drying, cracking or other signs of deterioration. Consult **GENERAL INSTRUCTIONS FOR USE** document for aseptic handling procedures, biohazards, and disposal of used product.

STORAGE AND SHELF LIFE

On receipt, store plates in the dark at 2 to 8° C, in their original sleeve wrapping until just prior to use. Avoid freezing and overheating. The plates may be inoculated up to the expiration date (see package label) and incubated for the recommended incubation times. Plates from opened stacks of 10 plates can be used for one week when stored in a clean area at 2 to 8° C.

USER QUALITY CONTROL

Inoculate representative samples with the following strains (for details, see **GENERAL INSTRUCTIONS FOR USE** document). Incubate the inoculated plates at 35 ± 2°C in an aerobic atmosphere supplemented with carbon dioxide. Examine plates after 18 to 24 h for amount of growth, colony size and hemolytic reactions.

| Strains | Growth Results |
|---|--|
| <i>Streptococcus pyogenes</i> ATCC™ 19615 | Growth good to excellent; beta hemolysis weak to good. |
| <i>Streptococcus pneumoniae</i> ATCC 6305 | Growth good to excellent; alpha hemolysis |
| <i>Staphylococcus aureus</i> ATCC 25923 | Growth good to excellent; may or may not be beta hemolytic |
| <i>Escherichia coli</i> ATCC 25922 | Growth good to excellent; may or may not be beta hemolytic |
| Uninoculated | Red (blood color) |

PROCEDURE

Materials Provided

BD Columbia Agar with 5% Sheep Blood (90 mm Stacker™ plates). Microbiologically controlled.

Materials Not Provided

Ancillary culture media, reagents and laboratory equipment as described.

Specimen Types

This is a universal isolation medium and can be used for all types of aerobically incubated clinical specimens (see also **PERFORMANCE CHARACTERISTICS AND LIMITATIONS OF THE PROCEDURE**).

Test Procedure

Streak the specimen as soon as possible after it is received in the laboratory. The streak plate is used primarily to isolate pure cultures from specimens containing mixed flora. Alternatively, if material is being cultured directly from a swab, roll the swab over a small area of the surface at the edge; then streak for isolation from this inoculated area. Appropriate selective media for detection of specific pathogens, e.g., BD MacConkey II Agar for the isolation of Enterobacteriaceae, should be included.

Since many pathogens require carbon dioxide on primary isolation, BD Columbia Agar with 5% Sheep Blood plates should be incubated in an aerobic atmosphere containing approximately 3 to 10% CO₂. Incubate plates at 35 ± 2°C for 18 to 72 h. Read for the first time after 18 to 24 hours and re-incubate if necessary.

Results

After incubation, most plates will show an area of confluent growth. Because the streaking procedure is, in effect, a "dilution" technique, diminishing numbers of micro-organisms are deposited on the streaked areas. Consequently, one or more of these areas should exhibit isolated colonies of the organisms contained in the specimen. Further, growth of each organism may be semi-quantitatively scored on the basis of growth in each of the streaked areas.

The number of species growing on this medium is very large. Therefore, no comprehensive information on the appearance of the organisms on this medium can be given here. Consult the appropriate references for information on appearance and further differential tests of the organisms isolated.^{2,4}

Typical colonial morphology of frequently isolated organisms on BD Columbia Agar with 5% Sheep Blood is as follows:

| | |
|----------------------------|---|
| Streptococci (non-group D) | Small, white to grayish. Beta or alpha hemolysis |
| Enterococci (Group D) | Small, but larger than group A streptococci, grayish. Alpha (rarely beta) hemolysis |
| Staphylococci | Large, white to gray or cream to yellow, with or without hemolysis |
| Corynebacteria | Small to large, white to gray or yellow, with or without hemolysis |
| Listeria monocytogenes | Small to medium-sized, grayish, with weak beta hemolysis |
| Enterobacteriaceae | Medium-sized to large, grey colonies, with or without hemolysis |
| Candida spp. | Small, white |

PERFORMANCE CHARACTERISTICS AND LIMITATIONS OF THE PROCEDURE

BD Columbia Agar with 5% Sheep Blood is a primary isolation medium on which most microorganisms, such as Enterobacteriaceae, Pseudomonas and other non-fermenting Gram negative rods, streptococci, enterococci, staphylococci, coryneforms, Candida species, and many others will grow.^{2,3,8}

The medium lacks V factor (nicotinamide adenine dinucleotide, NAD) since sheep blood contains NADase which destroys the NAD. For this reason, Haemophilus influenzae which requires both the X and V factors, will not grow on this medium.

Neisseria gonorrhoeae does not grow well on this medium. Instead, BD Chocolate Agar (GC II Agar with IsoVitalex) should be used for its recovery.

Also, the medium is not suitable for the isolation and growth of Mycobacterium, Legionella, Bordetella and other organisms with highly specific nutritive requirements.

The number and types of bacterial species occurring as infectious agents is very large.

Therefore, before the medium is routinely used for rarely isolated or newly described microorganisms, its suitability must first be tested by the user by cultivating pure cultures of the organism in question.

Due to the rather high carbohydrate (starch) content of Columbia Agar Base, beta-hemolytic streptococci may exhibit alpha rather than beta hemolytic reactions or may exhibit weak hemolytic reactions on media based on this formulation.

Although certain diagnostic tests may be performed directly on this medium, biochemical and, if indicated, immunological testing using pure cultures is recommended for complete identification. Consult appropriate references for further information.^{3,5,8}

REFERENCES

1. Ellner, P. D., C. J. Stessel, E. Drakeford, and F. Vasi. 1966. A new culture medium for medical bacteriology. Am. J. Clin. Pathol. 45: 502-504.
2. MIQ - Qualitätsstandards in der mikrobiologisch-infektiologischen Diagnostik, edited by Mauch, H., R. Lütken, and S. Gatermann for the Deutsche Gesellschaft für Hygiene und Mikrobiologie (DGHM). Volumes 3, 6, and 7. Urban & Fischer, Munich, Germany.
3. Murray, P. R., E. J. Baron, J. H. Tenover, M. A. Tenover, and R. H. Tenover (ed.). 2003. Manual of clinical microbiology, 8th ed. American Society for Microbiology, Washington, D.C.
4. Chapin, K.C., and T.-L. Lauderdale. 2003. Reagents, stains, and media. In: Murray, P. R., E. J. Baron, J. H. Tenover, M. A. Tenover, and R. H. Tenover (ed.). Manual of clinical microbiology, 8th ed. American Society for Microbiology, Washington, D.C.
5. Isenberg, H. D. (ed.). 1992. Interpretation of aerobic bacterial growth on primary culture media. Clinical microbiology procedures handbook, vol. 1, p. 1.6.1-1.6.7. American Society for Microbiology, Washington, D.C.
6. Baron, E. J., L. R. Peterson, and S. M. Finegold. 1994. Bailey & Scott's diagnostic microbiology 9th ed, p. 415. Mosby-Year Book, Inc. St. Louis, MO.
7. MacFaddin, J. F. 1985. Media for isolation-cultivation-identification-maintenance of medical bacteria, vol. 1, p. 86-92. Williams & Wilkins, Baltimore, MD.

PACKAGING/AVAILABILITY

BD Columbia Agar with 5 % Sheep Blood
 Cat. No. 254005 Ready-to-use Plated Media, cpu 20
 Cat. No. 254071 Ready-to-use Plated Media, cpu 120

#139.

FURTHER INFORMATION

For further information please contact your local BD representative.

Becton Dickinson GmbH

Tullastraße 8 – 12

D-69126 Heidelberg/Germany

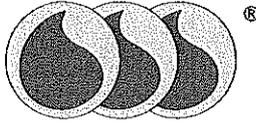
Phone: +49-62 21-30 50 Fax: +49-62 21-30 52 16

Reception_Germany@europe.bd.com

http://www.bd.com

http://www.bd.com/europe/regulatory/

ATCC is a trademark of the American Type Culture Collection
 BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. © 2013 BD

Pandy's Reagent

for determinations of liquor cerebrospinalis (CSF)

REF

Cont.

006698

(EDMA 11 90 01 90 00)

100ml • 250ml

Usable for Pandy's reaction. Other proceedings according to literature are possible.

Principle

Pandy's Reagent is a phenol solution in water which precipitates globulines and also albumines, albumoses and proteides. For further differentiation it is possible to use the additional proceeding with Nonne-Apelt's Reagent which is precepitating nearly selective only globulines and fibrin globulines.

Reagent

The reagent is ready for use and original closed and stored at +10...25 °C durable until the imprinted expiry date. After opening hold Pandy's reagent free from contaminations, well closed and use within 6 month but not longer than the expiry date.

Risks and Safety

Please maintain the necessary precautions for use laboratory reagents. Applications should occur only from expertly personnels.

R: 24/25-34 • S: 26-36/37/39-45

Toxic in contact with skin and if swallowed. Causes burns. • In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wear suitable protective clothing, gloves and eye/face protection. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

Content / main components

- 006698-0100 1x 100ml Pandy's Reagent
- 006698-0250 1x 250ml Pandy's Reagent
- Phenol 800mmol/l, non-reactive components, Aqua z.A.

#242

Sample Specimen

fresh liquor cerebrospinalis.

Range of reference

Assessments:

normal: slight turbidity and a very slight cloud pass for normaly.

Preparation

Device:

Watch glass slides or block slide, dropping pipettes.

Procedure

Give 4 drops Pandy's Reagent on the glass slide. To that flow slowly from the reagent border 1 drop CSF (cerebrospinal fluid). Control the reaction against a black background.

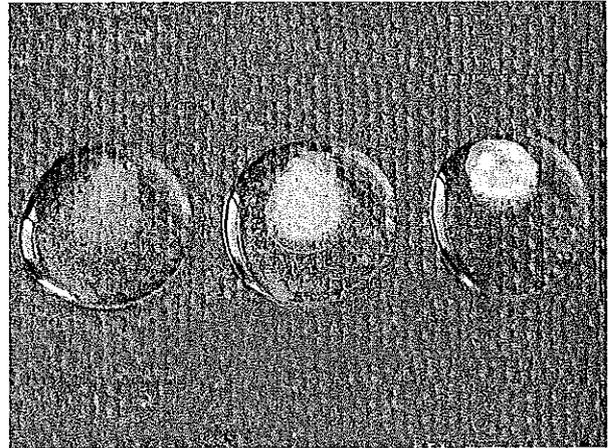
Valuation

Each strong turbidity implies pathological reaction of CSF.

The ranking of turbidity as follow:

- very slight cloud: (-) negative
- slight turbidity: (+) positive
- strong turbidity: (++) strong positive
- Precipitation: (+++) very strong positive

Positive: > ~30mg/dl total protein resp. > ~0,30g/l.

Picture:

From left to right:

- + positive
- ++ strong positive
- +++ very strong positive

Specificity

Beside the reaction to pathologically concentrations of globulines, the Pandy's reaction shows reactions also with albumines and other proteins. The Nonne Apelt's reaction determines nearly only globulines [2].

On positive Pandy's reactions it is recommended to check the reaction against Nonne Apelt's Reagent (bioanalytic REF 006697). Reagent and instructions on request.

Quality Controls

For internal quality controls and for training and visual evaluation it is recommended to use a control CSF (liquor cerebrospinal) fluid, e.g. *Rapitrol*® CSF or *Rapitrol*® LSQ.

Information

Support / Infoservice

Methodical and technical support by email support@bioanalytic.de or by fax (German, English).

Waste management

Please see regulations by law of your country.

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© Copyright by bioanalytic GmbH

Product information Pandy's Reagent

2010-05-25

(en)

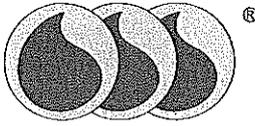
006698-PR01

bioanalytic GmbH

• biomed. + analytic-chemical reagents • medical laboratory diagnostics (IVD) • biomedical analysis technology Waldmatten 10-13 • D-79224 Umkirch/Freiburg • Germany

Telephone: +49 (0)7665 - 5951
Fax: +49 (0)7665 - 5683
eMail: office@bioanalytic.de
Internet: www.bioanalytic.de

Samson's Concentrate

Special reagent for WBC counting out of CSF

REF

Cont.

006688

(EDMA 13 01 90 90 00)

20ml, 100ml

Product information for the microscopic counting of WBC (white blood cells) in Liquor cerebrospinalis (CSF, cerebrospinal fluid).

Importantly:

Under the name „Samson's Reagent" or similar, different recipes and operating procedures are known. The below described instruction is valid exclusively for this Samson's Concentrate from bioanalytic! Only this in combination are CE conformable!

Principle

Microscopic WBC count in the counting chamber.

Reagents

The solution is ready for use. Store at ambient temperature (+15... 25 °C) up to the printed expiry date.

The reagent is free of azide and contains no mercury compounds (Thimerosal or similar).

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Please maintain the necessary precautions for use laboratory reagents. Applications should occur only from expertly personal.

R: 34 • S: 26-36/37/39

Causes burns. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wear suitable protective clothing, gloves and eye/face protection.

Content / main components

006688-0020 Samson's Concentrate 20mL for 200... 400 determinations

006688-0100 Samson's Concentrate 100mL for 1000...2000 determinations

Acetic acid 30%, Phenol-Fuchsin-Ethanol-Solution 2%, non-reactive components.

Additional are needed

Microscope for med. laboratory use, pipettes, Fuchs-Rosenthal counting chamber.

Sample

Fresh taken Liquor cerebrospinalis (CSF). Test process immediately (within 1 hour otherwise cell disintegrating).

Disturbances: Fibrin clots can lead to lower values.

Range of reference

| | Zellen / μ L |
|---------|------------------|
| Normal: | 0 ... 3 |

Sample Preparation

Because the leukocyte-mixing pipettes were classified by the World Health Organization WHO already in 1988 as obsolete (inexactly, outdated), we recommend to apply the more exact diluting procedure B.

With leukocyte-mixing pipette (Diluting procedure A):

Pre-fill a leukocyte-mixing pipette Samson's concentrate up to the mark 1.0. Afterwards from a block dished plate postfill the Liquor to mark 11 freely of any air bubble (This method corresponds just to the reverse method compared with the white cell count in blood). Close pipette ends and mix well the contents approx. 1 minute.

Mixture 9 + 1 accordingly diluting 9 : 10 = 1 : 1.111.

Diluting by laboratory pipettes (Diluting procedure B):

With a (Eppendorf) laboratory pipette give 100 μ L Samson's concentrate in a 1.5ml Eppendorf reaction vial*1. In addition give also 1000 μ L Liquor (CSF). Close vial and and mix well the contents approx. 1 minute. For small quantity of test material the mixing proportion can be also reduced (50 μ L Samson's concentrate + 500 μ L CSF). Further reduction of the volumes is not to be recommended!

Mixture 10 + 1 accordingly diluting 10 : 11 = 1 : 1.10.

Procedure

Prepare the counting chamber for filling. Mix sample directly before use (cells resuspension).

Diluting procedure A:

Dispose the CSF in the capillary (ca. 5...8 first drops). Charge the counting chamber by means of capillary effect.

Diluting procedure B:

With the Pipette aspirate a sample from the tube and charge the counting chamber.

Evaluation/Calculation

The following information is valid for the Fuchs-Rosenthal counting chamber. Count with the phase contrast or in the bright field (of lowered condenser) with 100-fold enlargement.

Count in the counting chamber the leukocytes in all 16 big squares of 1mm² surface, counted out consisting of 4 x 4 single squares

Calculation for dilution A (1 : 1.111):

The value of the counted cells from 16 squares, multiplies by the following factor results the value of cells per μ L Liquor (CSF).

$$\text{counted cells} \times 0,35 = \text{cells}/\mu\text{L (cells per } \mu\text{L)}$$

Calculation for dilution B (1 : 1.10):

The value of the counted cells from 16 squares, multiplies by the following factor results the value of cells per μ L Liquor (CSF).

$$\text{counted cells} \times 0,34 = \text{cells}/\mu\text{L (cells per } \mu\text{L)}$$

Information

For in-vitro diagnostic use.

The concentration of the reagent corresponds not to the references [1], but it is in coordination with this instruction. The leukocytes nucleus are released and fixed. Any erythrocytes (mammalian) will be lysed. Only hereby it is guaranteed to capture the mostly very low cell values in Liquor (CSF).

For historical reasons cell values are given partially also in other units, e. g., n/3 cells. Besides, it concerns the cell number in 3 μ L Liquor. Such information must be looked from today's view as obsolete.

Stains crystals ordinarily do not appear in the microscopic picture. Nevertheless, they cannot be excluded, e. g., by storage and strong cooling. If these interfere, the Samsons Concentrate can be centrifuged before use, e. g., in a reaction tube for some seconds at high rpm.

Support / Infoservice

Methodical and technical support by email (support@bioanalytic.de) or by fax (German, English).

Waste management

Please see regulations by law of your country.

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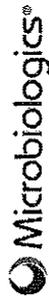
(en)

006688-PR01

Lyophilized Biological Material Preparations

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Prekybos ženklas: Taxo Ethyl Hydrocuprein

(Puslapio 6 tęsinys)

EINECS: European Inventory of Existing Commercial Chemical Substances
ELINCS: European List of Notified Chemical Substances
CAS: Chemical Abstracts Service (Division of the American Chemical Society)

MATERIAL SAFETY DATA SHEET

1) IDENTIFICATION OF PRODUCT AND SUPPLIER

- Product Name : FUNGFAST (30) #101.1
Ref: 44430
- Product use: In Vitro Diagnostic Medical Device for purpose of medical analysis
- Supplier : ELITech MICROBIO
19 allées d'Athènes
Parc d'activité
83870 SIGNES
FRANCE
Tel: 33 (0)4 94 88 55 00
Fax: 33 (0)4 94 88 55 22
- Appel d'urgence : Contact the nearest poison center.

2) HAZARDS IDENTIFICATION

- Main hazard :
- Effect of acute exposure
 - . Skin contact: May be harmful if absorbed through the skin. May be irritating
 - . Eye Contact: May be irritating
 - . Inhalation: May be harmful if inhaled. Material may be irritating to respiratory system.
 - . Ingestion: May be harmful if swallowed.
 - . Infectious :
- Four l'environnement : The preparation contain antimicrobial agents. These should not be discarded in the environment to prevent a development of microorganism resistance.
- After reconstitution : The reagents, after reconstitution, are potentially infectious and must be handled with caution, in observance of hygiene rules and the current regulations for this type of product in the country of use.
- Classification :
- Risks and security :

3) COMPOSITION / INFORMATION OF INGREDIENTS

- This product contain :
- 1 Vial of liquid medium FUNGI TC - ref 2202
 - 35 vials of liquid medium Fungi Suspension - ref 2203
 - 30 vials of liquid medium FUNGI MES - ref 2207
 - 30 Trays FUNGFAST - ref 2208

4) FIRST AID MEASURES

General advice: Consult a physician. Show this safety data sheet to the doctor in attendance.

- Eyes contact : Flush eyes with plenty of water . Seek medical advice.
- Skin contact : Flush area with soap and plenty of water and with a disinfectant.
- Inhalation : Remove to fresh air. If not breathing, give artificial respiration. Call a physician.
- Ingestion: Wash out mouth with water provided person is conscious. Never give anything by mouth to an unconscious person. Seek medical advice.
- Infection : Consult a physician

5) FIRE-FIGHTING MEASURES

- Extinguishing media : Use extinguishing measures that are appropriate to local circumstances and surrounding environment.
- Special risks : If devices are stored in large quantity Mask for respiratory and eye protection ; special protective equipment and gloves for skin protection. Inform help about localization and cautions to be taken.
- Infectious hazards :

6) ACCIDENTAL RELEASE MEASURES

- Individual protection: Gloves, Mask, Lab coat
- Measure for environmental protection: Do not let product enter drains, surface and ground water. Limit the spreading by using absorbent material. Avoid airstream.
- Measure for cleaning/collecting: Place in a suitable-closed container, and hold for biological waste disposal. Avoid raising dust. Ventilate area, wash and disinfect spill site after material pickup is complete.
Disposal procedures have to be respected (see § 13).

7) HANDLING AND STORAGE

The product are potentially infectious and must be handled with caution, in observance of hygiene rules and the current regulations for this type of product in the country of use

- Handling : Usual precautions to prevent vial breakages.
Open and handle vial with due care. Not breath dust. Wash hands after use.
The usual precautions taken when handling microbiological products should be observed. Refer to §8
- Storage : Store at 2-8°C. Keep tightly closed in a dry and well ventilated area. Stocker à 2-8°C.

8) EXPOSURE CONTROLS/PERSONAL PROTECTION

- Engineering controls: Provide appropriate exhaust ventilation at places where dust is formed to maintain air-born concentrations low. Provide a shower and a eyewash fountain
- Personal protective equipment:
 - . Respiratory protection: Not required. Where risk assessment shows air-purifying respirators are appropriate, use a dust mask type N99 (US) or type P2 (EN143) to protect from dusts. Use material tested and approved under appropriate government standards such as NIOSH (US) and CEN (EU).
 - . Eye protection: Wear chemical safety goggles.
 - . Hand protection: Wear chemical-resistant protective gloves satisfying to Directive EU 89/689/CEE and to deriving standard EN 374.
 - . Skin protection: Wear appropriated protective clothes.
- General hygiene measures: Wash thoroughly after handling and before break. Wash contaminated clothes before reuse.

9) PHYSICAL AND CHEMICAL PROPERTIES

| | FUNGI TC 2202 | FUNGI Suspension 2203 | FUNGI MES 2207 | FUNGI FAST 2208 |
|-------------------------|------------------|--------------------------|-------------------|--------------------|
| Appearance | Liquid | Liquid | Liquid | Solid |
| Color | Opaque | Transparent | Purple | |
| Propriétés | | | | |
| Melting point: | | No data available | | |
| pH | | No data available | | |
| Boiling Point | | No data available | | |
| Flash Point | | No data available | | |
| Ignition temperature | | No data available | | |
| Explosion limits | | No data available | | |
| Density | | No data available | | |
| Solubility in the water | Soluble | Soluble | Soluble | Soluble |

10) STABILITY AND REACTIVITY

- Stability: Stable until the expiry date stated on the label.

11) TOXICOLOGICAL INFORMATION

- Route of exposure: See point 2).
- Acute toxicity: no data available
- Irritation and corrosion: no data available
- Sensitization: no data available
- Chronic exposure: no data available
- Signs and symptoms of exposure: no data available

12) ECOLOGICAL INFORMATION

- No data available for elimination (persistence and degradability) and for ecotoxicity effects.
- Disposal procedures have to be respected (see § 13).

13) DISPOSAL CONSIDERATIONS

- Wastes evacuation procedure: Contact a licensed professional waste disposal service to dispose of this material. Observe all federal, state, and local environmental regulations.
- Contaminated packaging: Dispose of as unused product.

14) TRANSPORT INFORMATION

International regulation: Non hazardous goods

15) REGULATORY INFORMATION

Device registered according to : 98/79/EC directive (in vitro diagnostic medical devices Classification and labeling according to: 1999/45/EC directive. Exempt labeling according to Art. 12 .b)

Hazard symbol : -

16) OTHER INFORMATION

The material safety data sheet is established according to the Regulation (EC) n° 1907/2006.

The reagents in this kit are intended solely for in vitro use and must be handled by authorized personnel
The information herein is based on the current extent of our knowledge and is believed to be correct but is provided without warranty of any kind. The recipient of our product is responsible for observing all applicable laws and regulations

MATERIAL SAFETY DATA SHEET

1) IDENTIFICATION OF PRODUCT AND SUPPLIER

- Product Name : FUNGIPAST AFG (12) # 101.2.
Ref : 44412 ✓
- Product use: In Vitro Diagnostic Medical Device for purpose of medical analysis
- Supplier : ELITech MICROBIO
19 allées d'Athènes
Parc d'activité
83870 SIGNES
FRANCE
Tel: 33 (0)4.94.88.55.00
Fax: 33 (0)4.94.88.55.22
Mail : elitechmicrobio.msds@elitechgroup.com
- Emergency call : Contact the nearest poison center.

2) HAZARDS IDENTIFICATION

- Main hazard :
- Effect of acute exposure :
 - . Skin contact : May be harmful if absorbed through the skin. May be irritating
 - . Eye Contact : May be irritating
 - . Inhalation: May be harmful if inhaled. Material may be irritating to respiratory system.
 - . Ingestion: May be harmful if swallowed.
 - . Infectious :
- For environmentment : The preparation contains antimicrobial agents. These should not be discarded in the environment to prevent a development of microorganism resistance.
- After reconstitution : The reagents, after reconstitution, are potentially infectious and must be handled with caution, in observance of hygiene rules and the current regulations for this type of product in the country of use.
- Classification: non dangerous according to 1999/45/EEC directive
- Risks and security : S36/S37/39 ; S26

3) COMPOSITION / INFORMATION OF INGREDIENTS

- This product contains :
- 1 Vial of liquid medium FUNGI TC- ref 2202
 - 14 vials of liquid medium Fungi Suspension - ref 2203
 - 12 vials of liquid medium FUNGI MES - ref 2207
 - 12 Trays FUNGIPAST AFG - ref 2210

4) FIRST AID MEASURES

- General advice: Consult a physician. Show this safety data sheet to the doctor in attendance.
- Eyes contact : Flush eyes with plenty of water . Seek medical advice.
 - Skin contact : Flush area with soap and plenty of water and with a disinfectant.
 - Inhalation : Remove to fresh air. If not breathing, give artificial respiration. Call a physician.
 - Ingestion: Wash out mouth with water provided person is conscious. Never give anything by mouth to an unconscious person. Seek medical advice...
 - Infection : Consult a physician

5) FIRE-FIGHTING MEASURES

- Extinguishing media : Use extinguishing measures that are appropriate to local circumstances and surrounding environment..
- Special risks : If devices are stored in large quantity :Mask for respiratory and eye protection ; special protective equipment and gloves for skin protection. ; Inform help about localization and cautions to be taken.
- Infectious hazards :

6) ACCIDENTAL RELEASE MEASURES

- Individual protection: Gloves, Mask, Lab coat
Avoid breathing vapours, mist or gas. Remove all sources of ignition. Beware of vapours accumulating to form explosive concentrations. Vapours can accumulate in low areas
- Measure for environmental protection: Do not let product enter drains, surface and ground water. Limit the spreading by using absorbent material. Avoid airstream.
- Measure for cleaning/collecting: Place in a suitable-closed container, and hold for biological waste disposal. Avoid raising dust. Ventilate area, wash and disinfect spill site after material pickup is complete.
Disposal procedures have to be respected (see § 13) . .

7) HANDLING AND STORAGE

- The product are potentially infectious and must be handled with caution, in observance of hygiene rules and the current regulations for this type of product in the country of use
- Handling : Usual precautions to prevent vial breakages.
Open and handle vial with due care. Not breath dust. Wash hands after use.
The usual precautions taken when handling microbiological products should be observed. Refer to §§
- Storage : Store at 2-8°C. Keep tightly closed in a dry and well ventilated area Stocker à 2-8°C.

8) EXPOSURE CONTROLS/PERSONAL PROTECTION

- Engineering controls: Provide appropriate exhaust ventilation at places where dust is formed to maintain air-born concentrations low. Provide a shower and an eyewash fountain
- Personal protective equipment: Not required. Where risk assessment shows air-purifying respirators are appropriate, use a dust mask type N99 (US) or type P2 (EN143) to protect from dusts. Use material tested and approved under appropriate government standards such as NIOSH (US) and CEN (EU).
Wear chemical safety goggles.
Wear chemical-resistant protective gloves satisfying to Directive EU 89/689/CEE and to deriving standard EN 374.
Wear appropriated protective clothes.
- General hygiene measures: Wash thoroughly after handling and before break. Wash contaminated clothes before reuse.

9) PHYSICAL AND CHEMICAL PROPERTIES

| | FUNGI TC 2202 | FUNGI Suspension 2203 | FUNGI MDS 2207 | FUNGIFAST AFG 2210 |
|----------------------|------------------|--------------------------|-------------------|-----------------------|
| Appearance | Liquid | Liquid | Liquid | Solid |
| Color | Opaque | Transparent | Purple | |
| Properties | | | | |
| Melting point: | | No data available | | |
| pH | | No data available | | |
| Boiling Point | | No data available | | |
| Flash Point | | No data available | | |
| Ignition temperature | | No data available | | |
| Explosion limits | | No data available | | |
| Density | | No data available | | |
| Solubility in water | Soluble | Soluble | Soluble | Soluble |

10) STABILITY AND REACTIVITY

- Stability: Stable until the expiry date stated on the label.

11) TOXICOLOGICAL INFORMATION

- Route of exposure: See point 2).
- Acute toxicity: no data available
- Irritation and corrosion: no data available
- Sensitization: no data available
- Chronic exposure: no data available
- Signs and symptoms of exposure: no data available

12) ECOLOGICAL INFORMATION

- No data available for elimination (persistence and degradability) and for ecotoxicity effects.
- Disposal procedures have to be respected (see § 13).

13) DISPOSAL CONSIDERATIONS

- Wastes evacuation procedure: Contact a licensed professional waste disposal service to dispose of this material. Observe all federal, state, and local environmental regulations.
- Contaminated packaging: Dispose of as unused product.

14) TRANSPORT INFORMATION

International regulation: Non hazardous goods

15) REGULATORY INFORMATION

Device registered according to : 98/79/EC directive [in vitro diagnostic medical devices
Classification and labelling according to: 1999/45/EC directive.

Hazard symbol : -

16) OTHER INFORMATION

The material safety data sheet is established according to the Regulation (EC) n° 1907/2006.

- Risk sentences, §3 :
- Safety sentences, §3 : S36/37/39 : Wear suitable protective clothing, gloves and eye/face protection
S26 : In case of contact with eyes, rinse immediately with plenty of water and seek medical advice

The reagents in this kit are intended solely for in vitro use and must be handled by authorized personnel
The information herein is based on the current extent of our knowledge and is believed to be correct but is provided without warranty of any kind. The recipient of our product is responsible for observing all applicable laws and regulations

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IATA-DGR: Dangerous Goods Regulations by the "International Air Transport Association" (IATA)

ICAO: International Civil Aviation Organisation

ICAO-TI: Technical Instructions by the "International Civil Aviation Organisation" (ICAO)

ADR: Accord européen sur le transport des marchandises dangereuses par Route (European Agreement concerning the International Carriage of Dangerous Goods by Road)

IMDG: International Maritime Code for Dangerous Goods

IATA: International Air Transport Association

GHS: Globally Harmonised System of Classification and Labeling of Chemicals

EINECS: European Inventory of Existing Commercial Chemical Substances

ELINCS: European List of Notified Chemical Substances

CAS: Chemical Abstracts Service (division of the American Chemical Society)

LC50: Lethal concentration, 50 percent

LD50: Lethal dose, 50 percent

Acute Tox. 4: Acute toxicity, Hazard Category 4

Skin Corr. 1A: Skin corrosion/irritation, Hazard Category 1A

Eye Dam. 1: Serious eye damage/eye irritation, Hazard Category 1

LT

**SAFETY DATA SHEET
READY TO USE MEDIA**

Date of issue - 04/07/2012

Section 1. Chemical Product and Company Identification

Product No. 1xxx, 3xxx, 6xxx; all ready to use media
Trade name Poured plates, Ready to use media in bottles and tubes
Manufacturer Graso Zenon Sobiecki
Krag 4A
83-200 Starogard Gdański
Poland
Tel: + 48 58 562 30 21
Fax: + 48 58 562 79 87

115,
120-126,

Section 2. Hazards Identification

Most Important Hazards Not classified as hazardous.

Section 3. Composition, Information on Ingredients

Hazardous ingredients This preparation does not contain any substances presenting a health hazard within the meaning of the Dangerous Substances Directive 67/548/EEC.

Section 4. First Aid Measures

First Aid - Eyes First Wash out eye with plenty of water.
Aid - Skin First Aid - Wash skin with soap and water.
Ingestion Avoid hand to mouth contact. If ingested, seek medical advice. Have victim drink 1-3 glasses of water to dilute stomach contents.

Section 5. Fire Fighting Measures

Extinguishing Media - Suitable Not combustible.

Section 6. Accidental Release Measures

Personal Precautions No specific measures necessary.
Spill Transfer into suitable containers for recovery or disposal. Finally flush area with plenty of water.

Section 7. Handling and Storage

Handling Avoid contact with eyes, skin and clothing.
Storage Store as directed on container.

SAFETY DATA SHEET READY TO USE MEDIA

Date of issue - 11/01/2009

Section 8. Exposure Controls, Personal Protection

Protective Measures - Hands Disposable vinyl gloves.

Section 9. Physical and Chemical Properties

Physical state Solid or liquid
Colour Varies with product

Section 10. Stability and Reactivity

Stability Conditions Stable under normal conditions.
to Avoid Exposure to direct sunlight.

Section 11. Toxicological Information

Acute toxicity Low order of acute toxicity. Oral LD50 (rat) > 2000 mg/kg, Dermal LD50 (rabbit) > 2000 mg/kg, Inhalation LC50 (rat) > 20 mg/litre/4h.

Section 12. Ecological Information

Ecotoxicity No relevant studies identified.

Section 13. Disposal Considerations

Disposal considerations Sterilize used containers by autoclaving before final disposal. Dispose of in accordance with all applicable local and national regulations. Containers should be cleaned by appropriate methods and then re-used or disposed of by landfill or incineration as appropriate.

Container information

Section 14. Transport Information

UN ; UN number Not classified.

Section 15. Regulatory Information

Risk Phrases EC Not applicable.
Classification This product is not classified according to the EU regulations.

Section 16. Other Information

MSDS first issued MSDS 11/01/2009
data revised Revisions 04/07/2012
Highlighted Notes Date of issue
This MSDS is for the following non-hazardous products : - All ready to use media produced by Graso Zenon Sobiecki

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Prekybos ženklas: CHROMagar Candida

(Puslapis 6 tęsiasi)

- Asmuo kontaktams: techninis aptarnavimas
- Santurpmos ir akronimai:
 - RID: Règlement International concernant le transport des marchandises dangereuses par chemin de fer (Regulations Concerning the International Transport of Dangerous Goods by Rail)
 - IATA-DGR: Dangerous Goods Regulations by the "International Air Transport Association" (IATA)
 - ICAO: International Civil Aviation Organisation
 - ICAO-TI: Technical Instructions by the "International Civil Aviation Organisation" (ICAO)
 - ADR: Accord européen sur le transport des marchandises dangereuses par Route (European Agreement concerning the International Carriage of Dangerous Goods by Road)
 - IMDG: International Maritime Code for Dangerous Goods
 - IATA: International Air Transport Association
 - GHS: Globally Harmonised System of Classification and Labeling of Chemicals
 - EINECS: European Inventory of Existing Commercial Chemical Substances
 - ELINCS: European List of Notified Chemical Substances
 - CAS: Chemical Abstracts Service (division of the American Chemical Society)

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TO WHOM IT MAY CONCERN

AUTHORIZATION LETTER

We, the undersigned OPERON S.A. located at Camino del Plano 19, 50410 Cuarte de Huerva, Zaragoza, Spain.

hereby certify that:

Interlux Co. Ltd.
Avieciu str. 16, LT-08418, Vilnius, Lithuania

Is our distributor for the promotion, distribution, provision of sales and services in the territory of Lithuania.

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We, OPERON, S.A., hereby certify, that our above mentioned product for in vitro diagnostic and commercialised with Operon s.a., comply with the Essential Requirements (annex I) of the European Directive 98/79/EC related to in vitro diagnostic products and the RD 1662/2000 its Spanish transposition. Likewise, above product comply with the requests imposed in annex III (2 to 5) from the mentioned Directive. This Declaration of Conformity is signed below, certifying these requirements have been met and documented.

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recognised by an antibody specific for that antigen and bound to the membrane, serving as the control band test.

To run the test, the sample is first treated with a sample diluent buffer (provided in the kit) that extracts the toxins from the stool matrix. Following extraction, an aliquot of the supernatant needs to be added to the test strip followed by a 15 minute wait.

When the extract flows through the test membrane, the coloured particles begin to migrate. In the event of a positive sample, the specific antibodies on the membrane will capture antigen-covered coloured particles.

The pattern of lines obtained after 15 minutes of incubation at room temperature are used to interpret the result (see Fig. 1).

TEST FORMATS

The 2A-Bdfff test is available in two formats:

- **Stick format:** uses the reactive strip itself packaged individually within an aluminium wrap or included in a 25 units plastic tube. An additional test tube (included in the kit) or an 96-well microplate with flat-bottom is required to deposit the extracted sample and run the test.
- **Simple format:** uses the reactive strip inside a plastic cassette. The extractions are added directly to the sample window that is marked with an arrow on the casing.

Both formats display the same features, however there are some differences in the operations layout (see the relevant "Procedures" section below).

MATERIALS INCLUDED IN THE KIT

- Reaction devices (Stick or Simple format)
- Sample diluent buffer
- Disposable graduated plastic pipettes
- Disposable non-graduated plastic pipettes (yellow)
- Wooden applicators for taking formed stool samples.
- 1.5 ml capped microtubes.
- Test tubes.
- Stands to hold the previous test tubes in a stable upright position.

MATERIAL NOT INCLUDED IN THE KIT

- Centrifuge adapted to 1.5 ml microtubes
- Vortex apparatus
- Timer

PRECAUTIONS

1. Patient samples (faeces) should be handled with care as they may contain infectious agents. All the necessary protections should be used throughout handling (disposable gloves, goggles, laboratory coat and others).

C.difficile description

Clostridium difficile is an anaerobic, gram-positive, spore-forming bacteria that is carried by approximately 5% of the healthy population. Following hospitalisation, the carrier rate climbs to approximately 30%. Children are colonised by *C.difficile* early after birth, but they do not appear to suffer any clinical symptoms despite intensive investigations. The current discussion suggests a lack of enteric receptors for the toxins or differences in faecal pH in children that prevent toxin activity.

As previously mentioned, *C.difficile* can produce different toxins:

Toxin A (TcdA) (308 kDa) is called an enterotoxin given that it can induce full symptoms in the hamster animal model. TcdA also displays high cytotoxicity but only on specific *TcdA* sensitive cells such as HT-29^s. Toxin B (TcdB) (279 kDa) is classified as a cytotoxin. In the majority of cells cultured in laboratories (e.g. Vero, CHO or HeLa) it is about 1,000 times more potent than toxin A. The TcdB amino acid sequence varies between different strains.

TcdA/TcdB detection patterns

The following *C.difficile* strains can be identified based on the production of toxins:

- Non-toxic strains are non-pathogenic and lack TcdA and TcdB production as well as production of the binary toxin.
- TcdA+ / TcdB+ strains are the most common CDI inducing strains of which ribotypes 001, 014 and 078 are the most prevalent in Europe.
- TcdA+ / TcdB+ strains were first identified by Depierre et al. in Belgium⁸. They are considered pathogenic despite not producing toxin A⁹. Ribotype 017 strains belong to this group and were responsible for various endemic outbreaks in North America.
- TcdA+ / TcdB- strains can be identified directly in stool samples with the 2A-Bdfff test due to provides a simple, fast and separate detection of toxin A and toxin B in a single test. Very few isolates of these strains have been found until now. Recent results show that TcdB mutant *C.difficile* strains still induce CDI in hamsters owing to the production of toxin A². These results appear to indicate the presence of these strains in clinical samples.

BASIC PRINCIPLES OF THE TEST

The 2A-Bdfff test employs a combination of:

- 1) Red latex particles conjugated to a specific toxin A antibody that cooperates with another antibody specific for toxin A and that is located on the membrane, below the control band.
- 2) Other red latex particles conjugated to a specific toxin B antibody that cooperates with another specific toxin B antibody that is located on the membrane, above the control band.
- 3) Blue latex particles conjugated to an antigen

in tubes immediately once the reaction strip has been removed, since high levels of humidity may damage the unused strips inside the tube.

STORAGE

The product can be stored at any temperature between 2 and 30°C.

Its expiry date is printed on the tube or on the aluminium wrap.

SAMPLES

This test is designed to analyse liquid or semi-liquid stool samples. Solid samples may be analysed, but it is unusual given that the leading symptom of *C. difficile* infection is diarrhoea.

- Do not use samples that have been collected in means of conveyance or to which enrichment media / preserving agents have been added (e.g. formalin, SAF, PVA or similar) as they may interfere with the test.
- The analysis of untreated fresh samples is recommended. If preservation is needed, they should not be stored in the refrigerator (+2-8°C) for longer than 1 to 2 days. For longer storage, samples must be frozen at -20°C, however please bear in mind that some samples turn negative when they have been frozen.
- Pay special attention when analyzing hemorrhagic samples as they often give false positive results when the blood content is high. An indicator of this destabilization of the test is usually the alteration of the colour in the control band (tends to show a purple or dark blue colour).
- Ensure that frozen samples have completely defrosted and reached room temperature prior to proceeding with their analysis.
- Avoid repeatedly freezing and thawing the stool samples as the integrity of the toxins may suffer.

STOOL SAMPLES PREPARATION

General remarks: all the necessary precautions should be used throughout the test procedure due to the handling of infectious samples. Once finished, do not forget to comply with the hygiene procedures detailed in point 4 of the "Precautions" section. Whatever format of test is used (Simple or Stick), the protocol for the preparation of stool samples is as follows:

- For liquid or semi-liquid samples add 3 drops (or 75 µl) of sample using the plastic non-graduated pipette (yellow) to a 1.5 ml microtube. If the sample is solid, take a portion of approximately 75 mg (a small ball of 3 mm in diameter) with the wooden applicator and add it to a 1.5 ml labelled microtube. Important: ensure the sample is homogeneous by taking a portion from three different sample areas in order to obtain the most representative

sample possible.

- Add 1 ml of the sample diluent to the previous 1.5 ml microtube containing the sample (or the appropriate volume to maintain a ratio of 75 µl -or approximately 75 mg- of sample for 1 ml of diluent buffer).
- Vortex the microtube thoroughly for 30 seconds to ensure the total resuspension of the sample within the buffer.
- Centrifuge the 1.5 ml microtubes for 5 minutes at 700xg (approximately 3000 rpm) in a small benchtop centrifuge to settle solid particles. If a centrifuge is not available, wait 3-5 minutes for the solid particles to settle at the bottom of the tube. In any case, optimum test performance is achieved with a clear solution of a sample extracted following centrifugation.

SAMPLE 2A-Bdiff PROCEDURE

Once the samples have been prepared as described above, proceed as follows:

- Take the reaction device out of its aluminium pouch. Discard the desiccant as it only functions as to preserve the test from any excess of humidity.
- Following centrifugation and using the yellow plastic pipette supplied with the kit, transfer 4 drops (or 100 µl) of the sample supernatant to the sample area of the reaction device (round window marked with an arrow)
- Wait for exactly 15 minutes to read and interpret the result.

STICK 2A-Bdiff PROCEDURE

Once the samples have been prepared as described above, proceed as follows:

- Take the reaction strip out of its tube or aluminium pouch (recap it immediately to avoid damages due to the humidity getting enclosed).
- If a test tube included in the kit is used, insert this tube into the tube-stand also included in the kit. Take an aliquot of the centrifuged sample supernatant: the appropriate volume is approximately 265 µl (fourth mark of the plastic graduated pipette). Transfer this volume to the test tube.
- If a 265-µl microplate with flat-bottom is used, approximately 150 µl of the centrifuged sample supernatant is enough (third mark of the plastic graduated pipette).
- Dip the reaction strip with the arrow heads pointing to the microplate.
- Incubate the test at room temperature for 15 minutes and read the test result in the white area (see Fig. 1)

RESULTS READING

The five strips shown in Fig. 1 exemplify the various results that can be obtained using the 2A-Bdiff test.

There are three different coloured bands:

Blue band: the control band that indicates a correct performance of the test.

Upper red band: TcdB positive sample

Lower red band: TcdA positive sample

The blue band (control) should always appear. The additional appearance of any red band in the strip indicates the presence of *C. difficile* that produces TcdA and/or TcdB in the analysed sample.

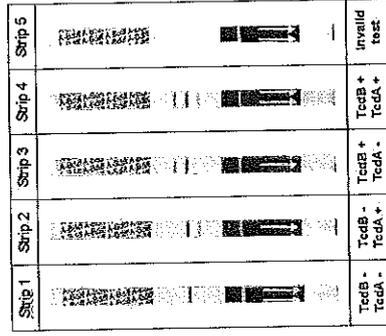
Fig. 1: Pattern of possible results. Result as indicated below the strips.

Strip 1. NEGATIVE result: the sample does not contain *C. difficile* or it contains a strain that does not produce TcdA/TcdB. A single BLUE horizontal band appears within the central area of the reactive device (in the Simple format it is aligned with the letter "C" marked on the cassette). This is the control band and it should always appear as an indication of the chromatography running smoothly.

Strips 2-4: POSITIVE results:

Strip 2. Detection of TcdA: a BLUE (control band) and a RED band appear just below the control band (in the Simple format it is aligned with the letter "T") marked on the cassette). The intensity depends on the concentration of toxin A in the sample.

Strip 3. Detection of TcdB: a BLUE (control band) and a RED band appear just above the control band (in the Simple format it is aligned with the letter "T2" marked on the cassette). The intensity depends on the concentration of toxin B in the sample.

Strip 4. Detection of both TcdA and TcdB: a BLUE (control band) and two RED bands (one above [TcdB] and one below [TcdA] the control band).

- The sample diluent buffer contains Sodium Azide as an antimicrobial agent. Avoid direct contact with the skin and mucous membranes. Dispose of appropriately. The buffer should not be used if there are signs of contamination or precipitation.

- Do not eat, drink, smoke, store or prepare food in areas where the reagents and the samples are handled.

- Once the work has been concluded, remove the gloves and first disinfect your hands with alcoholic disinfectant. Secondly, wash your hands with soap. Finally, the sink that has been used needs to be decontaminated with sporocidal disinfectants, as the *C. difficile* spores are not eliminated with alcohol.

- Do not exchange components between kits with different lot numbers.

- Allow kit components and stool samples to reach room temperature before use, as cold reagents and/or samples may reduce test performances. About 20-30 minutes are usually sufficient for reaching room temperature.

- All reagents are for in vitro use only.

- Do not use kit components beyond their expiry date.

- If the package is broken, the product may still be used providing none of its components have been damaged.

- In the case of the Simple format, it is very important to add the correct volume of extracted sample to the reactive device. If the volume is lower than indicated, chromatography may not occur because the sample may not reach the reaction area. If higher volumes are used, brown lines may appear instead of red or blue ones.

- All products are for single use only and should be discarded according to current legislation.

- Do not use the test if any coloured lines appear in the result area prior to performing the test.

- It is critical to collect the correct sample quantity: approximately 75 mg of a solid sample (a small ball of 3 mm in diameter) or 75 µl of a liquid or semi-liquid sample. These quantities are extracted in 1 ml of the sample diluent supplied with the kit. If a larger sample is taken, maintaining a ratio of approximately 75 mg (or 75 µl) of sample in 1 ml of sample diluent is sufficient.

An excess of sample in relation to the amount of buffer added prevents the chromatography from running correctly; this is especially critical in the case of solid samples, since it is harder to obtain an appropriate quantity. Please bear in mind that the 2A-Bdiff test was designed to analyse liquid and semi-liquid samples.

- In order to ensure an adequate chromatography, it is very important to centrifuge the 1.5 ml microtubes prior to extracting the specific quantity of supernatant. Correct results cannot be guaranteed if the solid particles are left to settle rather than being centrifuged. This is particularly true in the case of solid stool samples, as the greater number of suspended particles can interfere with the chromatography.

- It is very important to recap product that is packaged

Strip 5. INVALID result: the blue control band does not appear or the blue colour of this band is clearly altered (very dark blue or a purple colour), also nonspecific colours in the positive bands (different from red) are invalid results. In general, any colour combination different from those indicated in 1-4 strips indicates an anomalous test performance. Possible reasons are:

- some reagents have got damaged or the test has expired.
- the sample was not prepared according to the instructions of use.
- high blood content in the sample.

In the event of an invalid result it is recommended that another test is run, strictly following the protocol described in this manual. For blood samples, it is advisable the use of an alternative technique because the problem of destabilization does not usually depend on the strip but of the sample matrix.

Any line appearing after the standard 15 minutes reaction time is of NO diagnostic value.

NOTE: The final and definitive diagnosis CDI/PMC is established by the clinician. This test only detects TcdA/TcdB in a sample, but does not constitute a case to confirm whether a person has CDI.

LIMITATIONS OF THE PROCEDURE

1. The 2A-Bdiff test analyses liquid or semi-liquid human stool samples; solid samples may be used, however the test has not been optimised for their use since, in rare occasions, toxins sequestration phenomena have been observed with such solid matrices.

2. This test is qualitative and not quantitative, although the intensity of the positive bands is associated with the quantity of toxins that are detected in the stool sample.

3. Over 200 stool samples were evaluated to ensure the correct performance of the test. The correlation of the results with other techniques (ELISA and Cytotoxicity) was good. However, this study does not exclude interferences in the performance of the tests with other stool samples.

4. Weak signals may be due to excessively low amounts of sample. In the event of this occurring, the test should be repeated with a greater amount of sample whilst maintaining the recommended sample diluent ratio (see "Sample preparation" section).

5. An excess of sample can significantly slow down the development of the test or even prevent the test from running (control band remains invisible). In the event of this occurring, the test should be repeated with a reduced sample amount. This is particularly relevant in the analysis of solid samples.

6. A negative result does not fully exclude the possibility of infection with a *C.difficile* (CDI). The test result must be interpreted in relation to the clinical symptoms of the patient. In addition, it is important to keep in mind that toxins are fragile molecules that can be easily degraded, for example owing to inappropriate storage of the sample

or the presence of inhibitors. Under these conditions, toxin concentration may be below the test detection limit (see "Analytical Sensitivity" section).

7. A positive result obtained from solid samples must be interpreted with a great amount of caution. In principle, diarrhoea is the leading symptom of *C.difficile* (CDI) infection (CD), and a solid stool implies an absence of diarrhoea. The person performing the test must provide the clinician with information on the nature of the sample. This, in combination with the patient's medical history, will allow the clinician to establish the most accurate diagnosis possible.

8. Possible cross-reaction with two other micro-organisms has been detected:

- *Clostridium sordellii* produces a lethal toxin (Tcs1) that is homologous to TcdB of *C.difficile*¹⁹. To evaluate possible cross-reaction with Tcs1, two *C.sordellii* strains (IP82 and RE1522) that produce Tcs1 were grown in the appropriate culture media. The supernatant of a 3-day culture was used to run the 2A-Bdiff test. No positive bands appeared. We also assume that cross-reaction with TcdA/TcdB will not be observed in 2A-Bdiff tests on other *C. sordellii* strains that produce Tcs1.

- *Enterobacter histolytica*: a certain degree of cross-reaction has been observed with stool samples that are strongly positive for this parasite. The reaction was negative when using a preparation of isolated *E.histolytica* (without stool matrix). Other ELISA and quick tests that are available on the market yielded similar results for these samples. It has been proven that all such cross-reactive stool samples display the same positive pattern in the 2A-Bdiff test: toxin A is negative whilst toxin B is weak positive. Therefore we recommend using the *E.histolytica* test to confirm or exclude the presence of this parasite in the stool sample.

9. It is observed that faecal samples with a high blood content may interfere negatively with the test. In this cases, specificity problems may appear with *C.difficile* negative samples. This destabilization of the test is often accompanied by an alteration in the colour of the control band; a purple or dark blue colour appears instead of the expected light blue (see images in "Results Reading" section).

10. *C. difficile* colonisation rates of up to 50% have been reported in infants. A high rate has also been reported in cystic fibrosis patients. Normally, these two groups of patients remain asymptomatic and do not require specific treatment. These positive results that are of no clinical significance should be treated with caution as affected patients do not require treatment for *C.difficile* infection.

ANALYTICAL SENSITIVITY

In order to determine the sensitivity of the 2A-Bdiff test, toxins A and B from different sources were used diluted in the sample diluent buffer of this test:

- List Biological Laboratories: an internal standard at a concentration of 1 µg/ml was prepared by mixing equal quantities of toxin A (ref. 152) and toxin B (ref. 155). All batches of the 2A-Bdiff test must detect this internal standard, at least, at a concentration of 12.5 ng/ml for toxin A and at a concentration of 3 ng/ml for toxin B, although most of batches are capable of detecting toxin A up to 6 ng/ml and toxin B below 1 ng/ml.

- tgc BIOMICS: the 2A-Bdiff test was also tested against toxin A (ref. 01A01-1) and toxin B (ref. 01A02-1) from tgc BIOMICS. Most manufactured batches detect a concentration of 0.75 ng/ml of both toxins A and B.

- The Native Antigen: the 2A-Bdiff test was also tested against toxin A (ref. CDA-TNL) and toxin B (ref. CDB-TNL) from this company. Most manufactured batches detect a concentration of 3 ng/ml for toxin A and a concentration of 1.2 ng/ml for toxin B.

DIAGNOSTIC SENSITIVITY AND SPECIFICITY

A.- External Evaluation

The 2A-Bdiff test from Operon was evaluated at a Spanish hospital by measuring a total of 150 negative samples and 44 positive samples according to the reference technique, the Cytotoxicity at 48h. All samples were fresh.

The evaluation included the analysis of the same samples with a commercial rapid test based on immunochromatography (IC), very similar to Operon test. The results obtained with both tests considering as reference the Cytotoxicity at 24h were:

| | Sensitivity | Specificity |
|---------------|-------------|-------------|
| 2A-Bdiff | 92.5% | 95.5% |
| Commercial IC | 87.5% | 99.3% |

If the Cytotoxicity at 48h is considered:

| | Sensitivity | Specificity |
|---------------|-------------|-------------|
| 2A-Bdiff | 84.1% | 95.3% |
| Commercial IC | 79.5% | 99.3% |

It is very important to keep in mind that samples characterized as negative by Cytotoxicity at 24h but positive by Cytotoxicity at 48h (four samples in this evaluation) often correlate with low toxin content in the sample, that is, it is very likely that the concentration of toxins in those samples is below the detection limit of a rapid test; in fact, these four samples are detected as negative by both rapid tests of this evaluation (2A-Bdiff and commercial IC), hence, the sensitivity decreases.

B.- Internal Evaluation

The 2A-Bdiff test was evaluated internally by measuring

242 negative samples and 105 positive samples according to the Cytotoxicity developed at the place of origin of the samples (different Spanish hospitals). All samples analysed were frozen.

The results obtained were the follows:

Sensitivity: 99.0%
Specificity: 96.3%

The 2A-Bdiff test shows a good performance in this internal evaluation with sensitivity and specificity values clearly above 90%.

REPEATABILITY

Purified toxins A and B from Listlabs were used to design a sensitivity curve to measure the sensitivity of the 2A-Bdiff test in different conditions. Dilutions were two-fold and the samples were assayed by the same person in triplicate during a single session. The results differed by less than a factor of 2 indicating a high accuracy of the test.

REPRODUCIBILITY

INTER-DAY PRECISION

A sensitivity curve was measured with the same lot of strips on four different days. The results were very reproducible (the same level of sensitivity is obtained for both TcdA and TcdB over the four measurement days).

INTER-OPERATOR PRECISION

Five people with no prior training measured a sensitivity curve in duplicate. Differences were observed in the stronger curve dilutions (weaker signals), never exceeded a factor of 2.

INTER-LOT PRECISION

Three different lots of the 2A-Bdiff test were used to measure the sensitivity curve in duplicate. The analysis was performed by a single person on the same day. Differences of a dilution factor of 2 were appreciated, which are acceptable and tolerable for the test that was carried out.

The differences found in the "Reproducibility" sections are acceptable for this qualitative immunochromatographic test due to the inherent variability associated to this technique.

HOOK EFFECT

Several publications indicate that in the case of severe *C.difficile* infections, the highest concentration of toxins found in feces is around 112 ng/ml¹¹. To evaluate the effect of a very high concentration of toxins, values well above those that can be found among the population affected by CDI, concentrations of toxin A and toxin B up to 5000 ng/ml were measured with the 2A-Bdiff test. This concentration is about 400 times the LPC test for toxin A (12.5 ng/ml), about 1500 times the LPC test for

toxin B (3 ng/ml) and about 40 times the highest concentration of toxins found in faeces from patients. No decrease in the intensity of the positive signals was observed.

INTERFERING SUBSTANCES

The substances indicated in the below table, at the specified concentration, did not interfere with the results of the test when they were added to stool samples (positive and negative ones):

| | | | |
|---------------|-----------|----------------------|-----------|
| Racemorini | 5% (p/v) | Ibuprofen | 20% (p/v) |
| Cimetidine | 10% (p/v) | Acetylsalicylic acid | 30% (p/v) |
| Loperamide | 5% (p/v) | Edulcorant | 5% (p/v) |
| Metronidazole | 10% (p/v) | Palmitic acid | 40% (p/v) |
| Omeprazole | 3% (p/v) | Borium Sulfate | 5% (p/v) |
| Ampicillin | 15% (p/v) | Mucin | 5% (p/v) |

CROSS-REACTIVITY WITH OTHER MICRO-ORGANISMS

This study was developed in two different places:

- **OPERON**: the 2A-Bdiff test was evaluated against strongly positive samples for the following micro-organisms:

Adenovirus - Rotavirus - Norovirus - Astrovirus -
Helicobacter pylori - *Entamoeba histolytica* - *Giardia lamblia* - *Cryptosporidium parvum*

A certain degree of cross-reaction was observed with strongly positive samples for *Entamoeba histolytica* (see point 8 in the "Limitations of the Procedure" section).

- **HOSPITAL de BELMOTGE** (Barcelona -Spain-): the 2A-Bdiff test was evaluated against different micro-organisms likely to be present in the intestinal tract at any time in a sufficiently high concentration. Tests were carried out on bacterial suspensions at a concentration of 10⁸ cfu/ml. The 2A-Bdiff test showed no cross reactivity with any of the micro-organisms listed below:

Aeromonas caviae, *Aeromonas hydrophila*, *Bacillus cereus*, *Bacillus spp*, *Bacteroides nordii*, *Campylobacter jejuni*, *Candida albicans*, *Clostridium butyricum*, *Clostridium cadaveris*, *Clostridium perfringens*, *Clostridium sordeletii*, *Enterobacter aerogenes*, *Enterobacter cloacae*, *Enterococcus faecalis*, *Enterococcus faecium*, *Escherichia coli* 1, *Escherichia coli* 2, *Klebsiella pneumoniae*, *Lactobacillus gossyeri*, *Listeria monocytogenes*, *Plesiomonas shigelloides*, *Proteus mirabilis*, *Pseudomonas aeruginosa*, *Salmonella enteritidis*, *Salmonella typhi*, *Salmonella typhimurium*, *Serratia marcescens*, *Shigella dysenteriae*, *Shigella flexneri*, *Shigella sonnei*, *Staphylococcus aureus*, *Staphylococcus epidermidis*, *Vibrio cholerae*, *Vibrio parahaemolyticus*, *Yersinia enterocolitica*.

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LITERATURA

1. Iyres D, O'Connor JR, Howarth PM, Sombol SP, Carter GP, Phumoonna T, Poon R, Adams V, Vedamam G, Johnson S, Gerding DN and Rood JI. Toxin B is essential for virulence of *Clostridium difficile*. Nature. Apr. 2009. Vol. 458(7242). pp: 1176-1179.
2. Kuehn SA, Carman ST, Heep JT, Kelly ML, Cockayne A and Minton NP. The role of toxin A and toxin B in *Clostridium difficile* infection. Nature. Oct. 2010. Vol. 467(7314). pp: 711-713.
3. Vonberg RP, Reichardt C, Behnke M, Schwab F, Zindler S and Gostmeier P. Costs of nosocomial *Clostridium difficile* associated diarrhoea. J Hosp Infect. Sep. 2008. Vol. 70(1). pp: 15-20.
4. Rupnik M, Dupuy B, Fairweather NF, Gerding DN, Johnson S, Just I, Iyery DM, Popoff MR, Rood JI, Sonestein AL, Thelestam M, Wren BW, Wilkins TD and von Eichel-Streiber C. Revised nomenclature of *Clostridium difficile* toxins and associated genes. J Med Microbiol. Feb. 2005. Vol. 54(Pt 2). pp: 113-117.
5. Tucker KD, Carrig PE and Wilkins TD. Toxin A of *Clostridium difficile* is a potent cytotoxin. J Clin Microbiol. May. 1990. Vol. 28(5). pp: 869-871.
6. Seehn F, Wagenlecht-Wiesner A, Leukel P, Kohl M, Weidmann M, von Eichel-Streiber C and Braun V. Genetic rearrangements in the pathogenicity locus of *Clostridium difficile* strain 8864—implications for transcription, expression and enzymatic activity of toxins A and B. Mol Gen Genet. May. 1998. Vol. 258(3). pp: 222-232.
7. Borbut F, Mastrantonio P, Delmée M, Brazier J, Kuijper E and Poxton I. European Study Group on *Clostridium difficile* (ESGCD). Prospective study of *Clostridium difficile* infections in Europe with phenotypic and genotypic characterisation of the isolates. Clin Microbiol Infect. Nov. 2007. Vol. 13(11). pp: 1048-1057.
8. Depitre C, Delmeé M, Avesani V, L'Hardon R, Roels A, Popoff M and Cornier G. Serogroup F strains of *Clostridium difficile* produce toxin B but not toxin A. J Med Microbiol. Jun. 1993. Vol. 38(6). pp: 434-441.
9. Drudy D, Harney N, Fanning S, Hannan M and Kyne L. Emergence and control of fluoroquinolone-resistant, toxin A-negative, toxin B-positive *Clostridium difficile*. Infect Control Hosp Epidemiol. Aug. 2007. Vol. 28(8). pp: 932-940.
10. von Eichel-Streiber C, Boquet P, Souerborn M and Thelestam M. Large clostridial cytotoxins—a family of glycosyltransferases modifying small GTP-binding proteins. Trends Microbiol. Oct. 1996. Vol. 4(10). pp: 375-382.
11. Alex B, Ryder, Ying Huang, Haijing Li, Min Zheng, Xiaobo Wang, Charles W. Stratton, Xiao Xu and Yi-Wei Tang. Assessment of *Clostridium difficile* infections by quantitative detection of TcdB toxin by use of a real-time cell analysis system. Journal of Clinical Microbiology. Nov. 2010. pp: 4129-4134.

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MANUFACTURER'S AUTHORISATION LETTER

To whom it may concern,

EKF Diagnostics Holdings plc is the owner of Stanbio Laboratory LP, located at 1261 North Maine Street, Boerne, Texas 78006, USA, a Business that has been a manufacturer and supplier of clinical diagnostic products since 1960.

Stanbio's production department is regulated by the US Food and Drug Administration (US FDA) to comply with all Good Manufacturing Procedures (GMP). Stanbio's Quality System is registered to ISO: 13485:2003.

Stanbio's products are marketed worldwide through professional medical distributors who provide technical training and support to our clinical laboratory clientele. In Lithuania, Stanbio has appointed Interlux Ltd, whose head office is located at Avieciu 16, Lt-08418, Vilnius, Lithuania as their Authorized Distributor for Stat Site meters and test cards, and Stanbio labelled rapid test products.

Interlux Ltd is therefore authorised to import and distribute these products as well as to participate in public & private tenders.

This letter is in effect from 15th October 2014 until 31st December 2015.

For and on behalf of Manufacturer or Producer

Deputy Finance Director
EKF Diagnostics Holdings plc

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EC DECLARATION OF CONFORMITY

Directive 98/79/EC of the European Parliament and of the Council of 27 October 1998
on *in vitro* diagnostic medical devices

Manufacturer: Stanbio Laboratory
1261 North Main Street
Boerne, Texas 78006
Tel.: +1 830 249-0772

Product Name: HemaScreen®

Ref. Number: 1290

Intended Use: Hema-Screen is a guaiac slide test for the qualitative detection of fecal occult blood. It is a useful aid in the diagnosis of a number of gastrointestinal disorders and is recommended for use in:

- 1) Routine physical examinations
 - 2) Routine hospital testing
 - 3) Mass screening for colorectal cancer
- For professional use only.

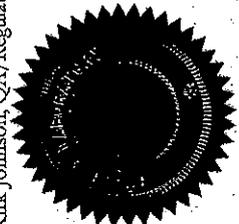
The product described above (and the manufacturing site which produces it, FDA registration number 1616487) is subject to the jurisdiction of the U.S. Food and Drug Administration (FDA). Additionally, the manufacturing site operates under a documented Quality Management System. The operations of the Quality System conforms with requirements promulgated in ISO 13485:2003 (KEMA certified, certificate # 560284-001) and FDA Quality System Regulation, CFR 820.

It is certified that the above product may be legally exported from the United States of America at this time. The manufacturing plant in which the product is produced is subject to periodic inspections. The last such inspection showed that the plant, at that time, appeared to be in substantial compliance with quality management requirements for the product.

It is furthermore certified that the above product is manufactured in accordance with the essential requirements of the EC Directive 98/79/EC (Annex I and Annex II) and harmonized standards, ISO 14971:2000 (risk analysis), and ISO 13485:2003 (quality system model for the quality assurance in design/development, production installation and servicing).

The EC declaration of conformity for HemaScreen® prepared by Kirk Johnson, QA/Regulatory Affairs Manager for Stanbio Laboratory.

Kirk Johnson
QA/Regulatory Affairs Manager
Stanbio Laboratory

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Hema-Screen® Occult Blood Test

INTENDED USE

Hema-Screen is a guaiac slide test for the qualitative detection of fecal occult blood. It is a useful aid in the diagnosis of a number of gastrointestinal disorders and is recommended for use in:

1. Routine physical examinations
2. Routine hospital testing
3. Mass screening for colorectal cancer

SUMMARY

The detection of occult blood is critical to many gastrointestinal diseases. The presence of occult blood in fecal material may indicate gastrointestinal pathology such as hemorrhoids, diverticulitis, fissures, colitis or colorectal cancer. Fortunately, these conditions may be detected with several diagnostic methodologies available which include testing of stools for occult blood, complete physical examination with digital examination, and proctosigmoidoscopy. Air contrast barium enema and fiberoptic colonoscopy also contribute significantly to the diagnosis of colonic lesions. Unfortunately, only a small percentage of bowel and rectal cancers are found on digital examination and patients with no symptoms of bowel disease do not readily present themselves for procedures such as proctosigmoidoscopy and barium enema.

Hema-Screen is a simple, aesthetic, inexpensive test designed for use in the collection and preparation of stool specimens. It overcomes the instability of guaiac solution and the hypersensitivity of benzidine and ortho-tolidine.¹⁰ If a positive result is obtained with the test, a follow-up with additional diagnostic tests, as soon as possible, is essential. As with any occult blood test, results with Hema-Screen cannot be considered conclusive evidence of the presence or absence of gastrointestinal bleeding or pathology. The test is not intended as a replacement for other diagnostic procedures such as proctosigmoidoscopy examination, barium enema, and X-ray studies.

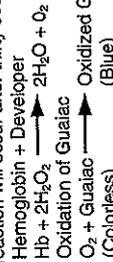
PRINCIPLE

Hema-Screen is composed of guaiac impregnated paper enclosed in a cardboard frame which permits sample application to one side, and development

and interpretation on the reverse side. The process involves placing two specimens, collected from three successive evacuations, onto the guaiac paper.

Hema-Screen, like all guaiac paper tests for occult blood, is based on the oxidation of phenolic compounds present in the guaiac (i.e. guaiaconic acids) to quinones resulting in production of the blue color.¹ Because of its similarity to the prosthetic group of peroxidase, the hematin portion of the hemoglobin molecule can function in a pseudoenzymatic manner, catalyzing the oxidation of guaiac.

When a fecal specimen containing occult blood is applied to the test paper, contact is made between hemoglobin and the guaiac. A pseudoperoxidase reaction will occur upon the addition of the developer solution, with a blue chromagen formed proportionally to the concentration of hemoglobins. The color reaction will occur after thirty seconds.

The Hema-Screen kits include on-slide monitors which provide a quality control system for each test.

REAGENTS

1. Hema-Screen Slides with On-Slide Monitors
Reactive Ingredients: Hema-Screen Slides are made of quality-controlled paper impregnated with guaiac resin. The Positive Monitor contains an impregnated substance which will turn blue if product is functioning properly. The Negative Monitor consists of guaiac impregnated paper.

WARNING: FOR IN-VITRO DIAGNOSTIC USE.
Preparation for Use: The slides are supplied ready to use.

Storage and Stability: This product should be stored at room temperature (15-30°C) and is stable until the expiration date indicated on each slide. Do not use after expiration date. Slides should be protected from heat, humidity, fluorescent light, U.V. radiation, excessive air flow, or volatile chemicals (e.g. iodine or bleach).
Do not refrigerate or freeze.

Signs of Deterioration: Discoloration of the

normally light tan paper may occur if exposed to sunlight, fluorescent or ultraviolet light. Failure of the control system to react as expected may be indicative of deterioration of the developer or the slide, and test results should be regarded as invalid.

2. Hema-Screen Developer

Reactive Ingredients: Hema-Screen Developer contains ~6% hydrogen peroxide and denatured ethanol.

WARNING: FOR IN-VITRO DIAGNOSTIC USE. DANGER: FLAMMABLE. NEVER PIPELITE BY MOUTH. VAPOR HARMFUL. DO NOT INGEST OR PLACE IN EYES. May cause blindness, or be fatal if swallowed. Keep away from heat, sparks or open flame. Avoid contact with eyes or skin. Should contact occur, flush the affected area with water and get immediate medical attention.

Preparation for Use: Hema-Screen Developer is ready for use as packaged.

Storage and Stability: Hema-Screen Developer should be stored tightly capped at 15 to 30°C protected from heat and light. Under these conditions, the developer will remain stable until the expiration date indicated on the bottle. Do not use after the expiration date. Do not substitute reagents from other manufacturers.

Signs of Deterioration: Failure of the on-slide Monitors to react as expected may be indicative of deterioration of the developer or the slide, and the test results should be regarded as invalid.

SPECIMEN COLLECTION AND HANDLING

Patient Preparation:

A. It is recommended that the patient be placed on a high residue diet starting 2 days before and continuing through the test period.

DIET MAY INCLUDE:

1. Meats: Only small amounts of well-cooked chicken, turkey and tuna.
2. Vegetables: Generous amounts of both raw and cooked vegetables including lettuce, corn, spinach, carrots and celery. Avoid raw vegetables with high peroxidase activity such as those listed below.
3. Fruits: Plenty of fruits, especially prunes and apples.
4. Cereals: Bran and bran-containing cereals.
5. Moderate amounts of peanuts and popcorn daily. If any of the above foods are known to cause discomfort, the patient is instructed to consult his/her physician.

TO BE AVOIDED:

1. Meat: Diet should not include any red or rare meat.
2. Raw fruits and vegetables containing high peroxidase activity:
Cauliflower Red radishes
Turnip Cantaloupe Broccoli Horseradish
Parsnip

B. Alternately, the special diet may be omitted initially with dietary restrictions imposed upon the retesting of all positive results. However, because gastrointestinal lesions may bleed intermittently and blood in feces is not distributed uniformly, all patients with positive tests regardless of diet, should have follow-up diagnostic procedures done.

C. Other factors which affect the test:

1. Medications: For 7 days prior to and during the testing, do not ingest aspirin[®] or other anti-inflammatory medicines. For 2 days prior to and during testing, do not use rectal medicines, tonics or vitamin preparations which contain Vitamin C (ascorbic acid) in excess of 250 mg per day.
2. Bleeding hemorrhoids or open cuts on hands.
3. Collection of specimen during menstrual cycle.
4. Improper specimen collection
5. Other diseases of the gastrointestinal tract such as colitis, gastritis, diverticulitis and bleeding ulcers.

Specimen Handling: Using the applicators provided, obtain a small sample of the stool from the toilet bowl. It is very important that the stool specimen be applied as a very thin smear to the HemaScreen Slides. Obtain a second sample of the stool, from a different location, in the same manner. Apply a very thin smear to the slide. Allow the smears to air dry. The slide smears may be prepared and developed immediately or stored up to 12 days prior to development. Care should be taken so that anything coming into contact with the specimen is free of blood. Because of the nonhomogeneity of the stool, it is recommended that the test be performed on three (3) consecutive evacuations or as close together as possible.^{2,3} Patient specimens and all materials in contact with them should be handled as potentially infectious and should be disposed of using proper precautions.

Return the completed slide to your physician or laboratory as instructed. If the slide is returned by mail, use the foil-back envelope provided. DO NOT

use a standard paper envelope, as they are not approved by U.S. Postal Regulations.

Interfering Substances: Ingestion of ascorbic acid (Vitamin C) in high doses has been shown to cause false negative results, and intake should be discontinued 2 days prior to, and during, the test period.⁴

Peroxidase from fruit and vegetables can cause false positive results.^{5,6} Elimination of red meat from the diet during the test period eliminates the source of hemoglobin which can cause false positives. Oral medications (such as aspirin, indomethacin, reserpine, phenylbutazone, corticosteroids, etc.) and heavy alcohol consumption may cause irritation or bleeding of the gastrointestinal tract and should be discontinued for 7 days prior to and during the test period.

PROCEDURE

Materials Provided: The following materials are provided for performance of fecal occult blood tests:

- Hema-Screen Slides with On-Slide Monitors
- Hema-Screen Developer
- Specimen Applicators
- Hema-Screen Mailing Envelopes (Patient Pack Only)

Kit combinations have been designed to meet individual laboratory demands. Please see the back page of this package insert for kit listings.

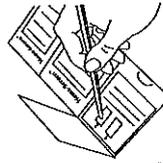
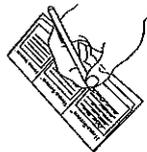
METHOD

Hema-Screen Slide

- Supply all information listed on the front flap of the Hema-Screen Slide.
- Open the front flap.
- Using the applicator sticks provided, collect a small amount of stool specimen from the toilet, on one end of the applicator. Apply a **very thin smear** in Box 1.

- Reuse applicator to obtain a second sample from a different part of the stool specimen. Apply a **very thin smear** inside Box 2.

- Flush stool and discard stick in waste container. (On subsequent bowel movements, repeat above steps using additional slides.)
- Allow the specimen to air dry, then close the cover.
- Apply two (2) drops of Hema-Screen Developer

False negative results may be obtained since most bleeding occurs intermittently. Hema-Screen tests are designed as a preliminary screen and are not intended to replace other diagnostic procedures such as proctosigmoidoscopy, barium enema or X-ray studies. Hema-Screen will detect only hemoglobin released upon hemolysis of the red cell. Should whole blood be applied to the test paper, it is necessary to hemolyze the red cells by the addition of a drop of water before adding the developer. Refer to "Interfering Substances" for a further list of limiting substances.

EXPECTED RESULTS

The guaiac paper tests detect occult blood but they are not diagnostic for disease. Positive occult blood tests may be obtained for reasons which range from red meat in the diet, diverticulitis, hemorrhoids, colitis to colorectal cancer. Patients who have a positive test should verify that they have followed a proper diet prior to specimen collection, and should immediately consult a physician who can perform definitive tests to determine the cause of bleeding. Patients experiencing symptoms such as persistent diarrhea or constipation, abdominal pain, visible bleeding, etc., should consult a physician.

Hema-Screen will detect 10 mg of hemoglobin per gram of homogenized fecal material.¹⁴ Guaiac impregnated paper has been extensively studied.¹⁵⁻¹⁸ Clinical studies illustrate that the guaiac impregnated slide tests yield a positive result of 3-5% in screening programs. The false positive rate was between 1-2% during controlled conditions (diet, medical supervision, etc.).

REFERENCES

- Kratochvil, J.F., et al., Isolation and Characterization of alpha Guaiaconic Acid and the Nature of Guaiacum Blue, *Phytochem*, 10:2529-2531, 1971.
- Greggor, D.H., Detection of Silent Colon Cancer in Routine Examination, *Cancer*, 19:330-337, 1968.
- Asymptomatic Colon Cancer, *Cancer*, 28:131-134, 1971.

- Jaffe, R.M., False-Negative Stool Blood Tests Caused by Ingestion of Ascorbic Acid (Vitamin C), *Ann Int Med*, 83:824, 1975.
- Winaver, S.J., et al., Current status of Fecal Blood Testing in Screening for Colorectal Cancer, *CA-A-Cancer J for Clinicians*, 32(2):100-113, 1982.
- Hagihara, P.F., Proceedings of 68th Annual Clinical Congress, Am College Surgeons, 55-57, 1982.
- Bassett, M.L. and Goulston, K.J., False Positive and Negative Hemoccult Reaction on a Normal Diet and Effect on Diet Restriction, *Aust NZ J Med*, 10:1-4, 1980.
- Winaver, S.J., and Fleisher, M., Sensitivity and Specificity of the Fecal Occult Blood Test for Colorectal Neoplasia, *Gastroenterology*, 82:986-991, 1982.
- Grossman, M.I., et al., Fecal Blood Loss Produced by Oral and Intravenous Administration of Various Salicylates, *Gastroenterology*, 40:383-388, 1961.
- Irons, G.V. and Kirsner, J.B., Routine Chemical tests of the Stool for Occult Blood: An Evaluation, *Am J of Med Sci*, 249:247-260, 1965.
- Ostrow, J.D., et al., Sensitivity and Reproducibility of Chemical Tests for Fecal Occult Blood with an Emphasis on False-Positive Reactions, *Amer J Dig Dis*, 18:930, 1973.
- Greggor, D.H., A Progress Report - Detection of Colorectal Cancer Using Guaiac Slides, *Cancer*, 22(6):360, 1972.
- Hastings, J.B., Mass Screening for Colorectal Cancer, *Amer J Surg*, 127:288, 1974.
- Ebaugh, Jr., F.G., Quantitative Measurement of Gastrointestinal Blood Loss, *J of Laboratory and Clinical Medicine*, 53:77, 1959.

PRODUCTS AVAILABLE

Cat. No.

1290-100

Hema-Screen Lab Pack

(Includes: Hema-Screen Single

Slides, 100; Hema-Screen Developer,

2 x 15 mL; Applicators, 100)

Shading indicates that text has been modified, added or deleted.

For Technical Service call:
800-531-5535
(830) 249-0772
<http://www.stانبio.com>

Marketed by:

Stانبio Laboratory
1261 North Main Street
Boerne, Texas 78006

Last Revision: 4/06(4)
Procedure No: 1290

STANBIO
LABORATORY

Pub 143
4/06(4)

to the back side of boxes 1 and 2.

- Read results after 30 seconds and within 2 minutes.

- Record the results; any trace of blue color, within or on the outer rim of the specimen, is positive for occult blood.

On-Slide Monitors

Note: The procedure for developing the sample test must be completed, interpreted and recorded before proceeding with the development of Monitors.

- To develop On-Slide Monitors, place one or two drops of Hema-Screen Developer between the Positive and Negative Monitor boxes.
- Read the results after 30 seconds and within 2 minutes.
- The Positive Monitor should turn blue, but the Negative Monitor should not have any trace of blue.

Stability of End Product: The color reaction is not permanent. Fading may occur after approximately 2 minutes.

QUALITY CONTROL

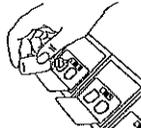
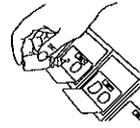
On-Slide Monitors are provided on each Hema-Screen Slide. This specially treated area provides assurance that the guaiac-impregnated paper and the Hema-Screen Developer are reacting according to product specifications. The Positive Monitor is an impregnated substance in a base carrier and will turn blue within 30 seconds after application of Hema-Screen Developer if the test system is reacting according to product specifications. The Negative Monitor consists of guaiac impregnated paper and will not turn blue upon addition of Hema-Screen Developer.

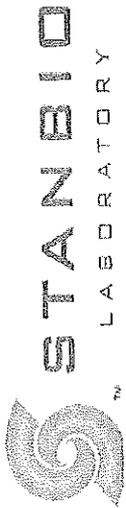
INTERPRETATION OF RESULTS

Any trace of blue color within the specimen application area is a positive for occult blood, if the Monitors react properly. Remember always to develop the test, interpret and record results before developing the On-Slide Monitors. Interpretation of the test should not be done by one who is color blind.

LIMITATIONS

Results obtained with Hema-Screen cannot be considered conclusive evidence of the presence or absence of gastrointestinal bleeding or pathology.

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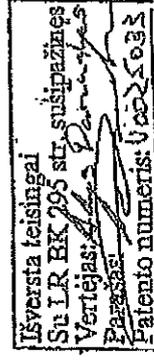
237

Artron Laboratories Inc.

3938 North Fraser Way, Burnaby, BC
Canada V5J 5H6

www.artronlab.com
Tel: 604-415-9757
Fax: 604-415-9795

Authorization Letter

This authorization letter confirms that Artron Laboratories Inc. located at 3938 North Fraser Way, Burnaby, BC, Canada V5J 5H6 officially grants "Distribution Rights" for all CE certified rapid test kits manufactured and provided by Artron Laboratories Inc. to Interlux Co. Ltd. located at Avieciu Str. 16, Vilnius, LT-08418 Lithuania - for distributorship in the territory of Lithuania.

This authorization will take effect on June 8, 2015 and terminate on June 8, 2017 or if Artron Laboratories Inc. should see fit otherwise, may have the option to cancel this agreement earlier at anytime with a 30-day notice.

Sincerely,

A handwritten signature in black ink, appearing to read "Olga Mineeva".

Olga Mineeva
Business Development Manager
Artron Laboratories Inc.

Artron Laboratories Inc.

3938 North Fraser Way, Burnaby, BC
Canada V5J 5H6

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VILNIUS, LIETUVA 2015-06-09
(Vieta/data)

(parašas)

| | | | |
|---|---|-----------------|--------------|
|  | Declaration of Conformity IVDD 98/79/EC | Document ID | DOC-502 |
| | | Current Version | 00 |
| | | Effective Date | May 08, 2009 |
| | | Page | 1/1 |

Declaration of Conformity

| | |
|------------------------------------|---|
| Manufacturer | Artron Laboratories Inc. 3938 North Fraser Way, Burnaby, BC Canada V5J 5H6 |
| European Representative | Wellkang Tech Consulting Suite B, 29 Harley Street London England, United Kingdom W1G 9QR |
| Product Designation | Artron Fecal Occult Blood (FOB) |
| EDMA Code | 12.70.03.90 |
| Catalogue No. | A05-02-413 (Strip) A05-02-422 (Cassette) |
| Classification | Others, Self-Declaration IVD MD |
| Conformity Assessment Route | Annex III Applied (IVD 98/79/EC) |

The undersigned hereby declares, under the sole responsibility of the manufacturer, that the medical device as specified above conforms with the essential requirements listed in the Annex I of the European in vitro Medical Device Directive 98/79/EC (IVD).

| | |
|----------------------------|--|
| Standard Applied | List of (Harmonized) standards for which documented evidence for compliance can be provided |
| | Quality Assurance (ISO13485:2003+AC:2007) Certified by TUV Rheinland LGA Products GmbH- TillystraBe 2 - 90431 Ntirnberg Certificate Number SX 60028646 0001 |
| Start of CE marking | May 09, 2009 |

Date of Issue October 03, 2010

On the behalf of
Artron Laboratories Inc.

Signature

 Steven Song
 Regulatory Affairs Manager

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Mes, VITREX Medical A/S, Vasekaer 6-8, DK-2730 Herlev, Danija
VITREX MEDICAL A/S
Vitrex Medical A/S. Suinteresuotiems asmenims
Vitrex Medical A/S. To whom it may concern