en	Self-Filling <i>safePICO</i> Samplers Operating instructions	4-5
bg	Самонапълващи се устройства за вземане на проби <i>safePICO</i> Инструкции за работа	6-7
cs	<i>safePICO</i> arteriální odběrová stříkačky Návod k použití	8-9
da	Selvfyldende <i>safePICO</i> samplers Brugsanvisning	10-11
de	Selbstfüllende <i>safePICO</i> Probennehmer Gebrauchsanweisung	12-13
el	Αυτοπληρούμενοι δειγματολήπτες <i>safePICO</i> Οδηγίες λειτουργίας	14-15
es	Extractores de muestras <i>safePICO</i> de autollenado Instrucciones de uso	16-17
et	Isetäituvad verevõtuvahendid <i>safePICO</i> Kasutusjuhised	18-19
fi	Itsetäyttyvät <i>safePICO</i>-näytteenottoruiskut Käyttöohjeet	20-21
fr	Seringues à remplissage automatique <i>safePICO</i> Mode d'emploi	22-23
hr	Samopunjivi uređaji za uzorkovanje <i>safePICO</i> Upute za rad	24-25
hu	Önfeltöltő <i>safePICO</i> mintavevők Használati utasítások	26-27
it	Siringhe <i>safePICO</i> a riempimento automatico Istruzioni per l'uso	28-29
ja	動脈血サンプラー <i>safePICO</i> 取扱説明書	30-31
lt	Prisipildantieji <i>safePICO</i> mėginių ėmikliai Naudojimo instrukcijos	32-33
lv	Pašuzpildošie <i>safePICO</i> asiņu savācēji Lietošanas pamācība	34-35
nl	Zelfvullende <i>safePICO</i> bloedafname spuit Gebruiksaanwijzing	36-37
no	Selvfyllende <i>safePICO</i>-prøvetakere Bruksanvisning	38-39
pl	Samonapełniające aspiratory <i>safePICO</i> Instrukcja obsługi	40-41
pt	Dispositivo de colheita por auto-enchimento <i>safePICO</i> Instruções de utilização	42-43
ro	Dispozitive de recoltare <i>safePICO</i> cu umplere automată Instrucțiuni de utilizare	44-45

ru	Самозаполняющиеся сэмплеры <i>safePICO</i> Руководство по эксплуатации	46-47
sk	<i>safePICO</i> arteriálna odberová striekačka Návod na použitie	48-49
sr	Samo-punjive posude za uzorkovanje <i>safePICO</i> Uputstva za upotrebu	50-51
sv	Självpåfyllande <i>safePICO</i>-sprutor Bruksanvisning	52-53
zh	自充盈式<i>safePICO</i>动脉采样器 操作指南	54-55

Intended use

The *safePICO* self-fill sampler is an arterial blood sampler for pH, blood gas, oximetry, electrolyte, and metabolite analysis.

Product description

The self-filling *safePICO* sampler is preheparinized, electrolyte balanced, arterial blood sampler. It is intended for the aspiration of sample volumes in the range of 0.7-1.5 mL. It is prebarcoded ensuring correct sample identification, it includes a vented tip cap which simplifies removal of air bubbles and a small metal ball, allowing automated mixing when the *safePICO* sampler is placed in the FLEXQ module or the *safePICO* mixer.

Not for injection. Aspiration only. Rx only. Single use.

WARNING – Risk of infection and erroneous results

Reuse of single use devices may lead to infection of patients and erroneous results.

Sampling from arterial line

The procedures described below apply to *safePICO* sampler kits without a needle.

WARNING – Risk of infection

Inspect sealed sterile package before opening. Discard sampler if seal is broken or package is otherwise damaged as contents may not be sterile and may cause infection in the patient.

Preparations

Prepare the A-line for sampling according to your standard procedure. Preset the plunger to the desired volume by holding on to the syringe barrel finger grips and pulling the plunger (**Fig. 1**).

WARNING – Risk of infection

Blood sampling should be performed by authorized personnel only. To avoid the risk of infection, always handle blood and the collection devices with care and avoid direct contact with the sample by using medical gloves.

Instructions for use

Connect the *safePICO* sampler to the stopcock and allow the arterial pressure to fill the sampler. Let the blood come in contact with the vent material to seal the sample. Close the stopcock and remove the sampler (**Fig. 2**).

WARNING – Risk of erroneous results

To measure ctHb, a sample volume above 1 mL is required. Smaller sample volumes may cause erroneous results.

Sealing the sampler

Holding on to the syringe barrel, firmly place the *safeTIPCAP* on the syringe luer tip. Tap the syringe to move the bubbles to the top of the sampler (**Fig. 3**) and gently press the plunger to expel any air from the sample. Press the plunger gently until the blood comes in contact with the vent material inside the *safeTIPCAP* or until resistance indicates that the *safeTIPCAP* is sealed immediately (**Fig. 4**). Mix the sample thoroughly by inverting the sampler several times and rolling it between the palms of your hands.

WARNING – Risk of infection

Do not point the syringe at people when expelling the air. Pressing the plunger too hard may cause sudden ejection of blood from the syringe. Failure to follow these procedures may result in exposure to potentially infectious blood.

Storage

Storage should be avoided whenever possible or at least kept to a minimum. If it is not possible to analyze the sample immediately, store it at room temperature and analyze it within 30 minutes after collection. Samples with expected high pO_2 values or for special studies like shunt studies should be analyzed immediately or within 5 minutes.

Analyzers

Analyzers prepared for *safeTIPCAP* (see list overleaf)

ABL800 FLEX analyzers with FLEXQ module

Place the *safePICO* sampler in the FLEXQ module for automatic mixing and sample introduction (**Fig. 5**).

Analyzers without FLEXQ module

Prior to analysis, thoroughly mix the sample again by inverting the sampler several times and rolling it between the palms of your hands. Holding the *safeTIPCAP*, place the sampler in the syringe inlet. After aspiration, remove the sampler by twisting the *safeTIPCAP* off the inlet.

WARNING – Risk of infection

Small quantities of blood may spill when the *safePICO* sampler is removed from the FLEXQ module or the syringe inlet. When removing the *safePICO* sampler, make sure to wear medical gloves and to handle the sampler as potentially infectious material. Failure to follow these procedures may result in exposure to potentially infectious blood.

Analyzers not prepared for *safeTIPCAP*

Prior to analysis, thoroughly mix the sample again by inverting the sampler several times and rolling it between the palms of your hands. Remove the *safeTIPCAP* by gently twisting it and analyze the sample immediately.

WARNING – Risk of erroneous results

Discard the used samples and blood remains as infectious waste to avoid exposing others to the risk of infection ¹⁾.

¹⁾ CLSI document GP5-A2, Clinical Laboratory Waste Management.

Specifications

Arterial blood sampler for pH, blood gas, oximetry, electrolyte and metabolite analysis. Preheparinized with approx. 60 IU dry electrolyte-balanced heparin (Porcine intestinal mucosa).

Contents: *safe*PICO vented arterial sampler (956-610)
*safe*TIPCAP

Materials: Polypropylene syringe containing a heparinized disk. Electrolyte-balanced lithium/sodium heparin. Components contain no dry natural rubber (latex).

Sample volume: 0.7-1.5 mL

Sterilization: By ethylene oxide

Analyzers prepared
for *safe*TIPCAP:

ABL800 FLEX analyzers with FLEXQ module
ABL800 FLEX analyzers SW version 5.1 and higher
ABL700 Series analyzers SW version 5.24 and higher

Rx only

CAUTION:

Federal (USA) law restricts this device to sale by or on the order of a licensed practitioner.

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