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### Intended use

The *safePICO70* self-fill sampler is an arterial blood sampler for pH, blood gas, oximetry, electrolyte, and metabolite analysis.

### Product description

The self-filling *safePICO70* sampler is a preheparinized, electrolyte-balanced, arterial blood sampler. It is intended for the aspiration of sample volumes in the range of 0.3-1.5 mL. It has a needle and a shield device to protect the user.

Not for injection. Aspiration only. Rx only. Single use.

#### **WARNING – Risk of infection and erroneous results**

Reuse of single use devices may lead to infection of patients and erroneous results.

### Sampling by arterial puncture

The procedures described below apply to *safePICO70* sampler kits with a needle shield device.

#### **WARNING – Risk of infection**

Inspect sealed sterile package before opening. Discard sampler if seal is broken or package is otherwise damaged as contents may not be sterile and may cause infection in the patient.

### Preparation

Prepare the puncture site according to your standard procedure. Preset the plunger to the desired volume by holding on to the syringe barrel finger grips and pulling the plunger (**Fig. 1**).

Expose the needle shaft by holding on to the needle shield device with one hand and pulling the needle sheath straight off without twisting with the other hand (**Fig. 2**).

#### **WARNING – Risk of infection**

Blood sampling should be performed by authorized personnel only. To avoid the risk of infection, always handle blood and the collection devices with care and avoid direct contact with the sample by using medical gloves.

### Instructions for use

Perform the puncture according to your standard procedure (**Fig. 3**). Allow the arterial pressure to fill the sampler to the predetermined volume and let the blood come in contact with the vent material in the plunger to seal the sample. Terminate the puncture according to your standard procedure.

#### **WARNING – Risk of erroneous results**

To measure ctHb, a sample volume above 1 mL is required. Smaller sample volumes may cause erroneous results.

### Needle disposal

Still holding the sampler in one hand, place your thumb on the thumb plate on the needle shield device. Without pressing the plate down, push gently to slide the protective shield over the exposed needle until a click is heard (**Fig. 4**). The click signifies that the protective shield is safely locked.

#### **WARNING – Risk of infection**

Do not attempt to override or defeat the locking mechanism as this may cause needle injuries or blood spillage.

Remove the needle and the needle shield device by holding on to the syringe barrel finger grips while twisting the needle shield device with the other hand (**Fig. 5**). Discard the needle shield device and needle in a sharps container.

#### **WARNING – Risk of infection**

During removal of the needle shield device do not move the plunger. Leakage of fluid may occur when disconnecting the components.

### Sealing the sampler

Should air bubbles occur in the sampler, they must be expelled immediately. Seal the sampler with the tip cap. Immediately mix the sample thoroughly by inverting the sampler several times and then rolling it between the palms (**Fig. 6**).

#### **WARNING – Risk of infection**

Do not point the syringe at people when expelling the air. Pressing the plunger too hard may cause sudden ejection of blood from the syringe. Failure to follow these procedures may result in exposure to potentially infectious blood.

### Storage

Storage should be avoided whenever possible or at least kept to a minimum. If it is not possible to analyze the sample immediately, store at room temperature and analyze it within 30 minutes after collection. Samples with expected high  $pO_2$  values or for special studies like shunt studies should be analyzed immediately or within 5 minutes.

For more information please refer to the relevant operator's manual.

Prior to analysis, thoroughly mix the sample again by inverting the sampler several times and then rolling it between the palms. More extensive mixing is required if hemoglobin or hematocrit is to be measured. Expel a few drops of blood into a piece of gauze and analyze the sample immediately (**Fig. 7**).

#### **WARNING – Risk of infection**

Discard the used samples and blood remains as infectious waste to avoid exposing others to the risk of infection <sup>1)</sup>.

<sup>1)</sup> CLSI document GP5-A2, Clinical Laboratory Waste Management.

## Specifications

Arterial blood sampler for pH, blood gas, oximetry, electrolyte and metabolite analysis. Preheparinized with 60 IU dry electrolyte-balanced heparin (porcine intestinal mucosa).

Contents:	safePICO70 vented arterial sampler (956-634) Tip cap Needle shield device
Materials:	Polypropylene syringe containing a heparinized disk. Electrolyte-balanced lithium/sodium heparin. Components contain no dry natural rubber (latex).
Sample volume:	0.3-1.5 mL
Sterilization:	By ethylene oxide
USA Patent No.:	US 6,183,445
European Patent No.:	EP 957 957
German Patent No.:	DE 69630619.0

## Rx only

### CAUTION:

Federal (USA) law restricts this device to sale by or on the order of a licensed practitioner.

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