# D115 – Ref

989-468. 200707E

<b>MEMBRANE BOX for E115 Reference Electrode</b>	<b>en</b>
<p><b>Contents:</b> 6 premembraned electrode jackets each filled with electrolyte solution.</p> <p><b>Electrolyte:</b> Contains organic and inorganic salts, preservative, acid, and buffer. For In Vitro Diagnostic Use.</p> <p><i>WARNING/CAUTION: Irritating to eyes, respiratory system and skin (R36/37/38). In case of contact with eyes, rinse immediately with plenty of water and seek medical assistance (S26).</i></p> <p><b>Storage:</b> Store at 2-32<span> </span>°C (36-90<span> </span>°F).</p> <p><b>Procedure:</b> Remove the reference electrode from the electrode jacket. Discard the used electrode jacket. Check that no used O-ring remains on the electrode; if so, remove the used O-ring before remembraning the electrode.</p> <ol style="list-style-type: none"><li>Turn the lid of the Membrane Box to a sealed electrode jacket and remove the foil.</li> <li>Press the electrode firmly through the seal into the electrode jacket until a click is heard.</li> <li>Remove the membraned electrode from the Membrane Box.</li> <li>Rinse the membraned electrode with distilled water and dry with lint-free tissue. Dry the contact.</li> <li>Place the electrode in the measuring chamber. Press the amplifier housing down to fasten the electrode in the measuring chamber.</li></ol> <p>Radiometer Ltd., Crawley ☎ 01293 517599</p> <p>Radiometer America Inc., Westlake OH 44145 ☎ 800-736-0600</p> <p>FOSS Ireland Ltd., Dublin ☎ 1 250 1100</p> <p>Radiometer Pacific Pty. Ltd., Blackburn, Victoria 3130 ☎ 3 9259 2222</p>	

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