SPACELABS
HEALTHCARE

OnTrak 90227

Ambulatorinis

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2.1.

NAUDOJIMO
INSTRUKCIJA

MD

070-2736-05 A red. | www.spacelabshealthcare.com

2.1.14.

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- **Incremental Hours** — Reading values are calculated starting with the first good reading and all other good readings for the next 59 minutes and 59 seconds. For example, if the first reading is at 9:42, all readings from 9:42:00 to 10:41:59 are grouped in Hour 1. This process continues for each hour, for the entire procedure.
- **Relative Hours** — Reading values are calculated starting with the hour of the first good reading (the minute time is :31). For example, if the first good reading is 9:42, the start time of the first hourly tabular reading will be 9:31, and the end time (not shown) is 10:30. The second hourly time will be 10:31, and the end time is 11:30. This continues for each hour, for the entire procedure.

8.4.14 Graphs

This icon displays the raw data graph.

8.4.14.1 Activity Graph

Where recorded, the activity graph reflects movement and volatility of activity by the patient based on accelerometer data from the recorder.

8.4.15 Tables

The raw data tables are accessed through this icon.

The tables will show the number of readings(#) and the time of the reading together with Systolic, Diastolic, MAP (Mean Arterial Pressure), Pulse Pressure, Heart Rate, Event Code (any errors during a recording will be shown here), Edit Status (this shows whether a reading is included in the statistical analysis).

There is also an **Exclude** button which can be ticked to exclude readings from the statistical analysis.

Diary Activity- this is a free text field or make use of the drop down menu with pre-populated list of common activities.

8.4.16 Callipers

The Calliper icon will show the user any defined callipers readings.

To insert a calliper: click on the **Graph** icon

1. Click the desired first reading and drag the mouse over to the desired last reading- release the mouse button. A window will appear click on **Insert Callipers** for selected area.
2. The Inserted calliper may have a comment added through the **Calliper** icon.

8.4.17 Summary

The **ABP Report Summary** displays a summary for the entire recording and for each of the statistical periods.

The information provided is for the patient's systolic, diastolic, MAP, pulse pressure, and

2.1.19.

P.

heart rate. The low, high, and mean rates for the prescribed periods are also recorded. This screen also indicates the percentage of time that the readings in each of the periods were higher than the statistical boundaries set in the Statistics Icon.

2.1.19.

Dipping is an indication of the change in blood pressure between day (or wake) and night (or sleep). It is the percentage change for the mean systolic and diastolic readings for the day and night. If there is no day or no night period, the dipping values cannot be calculated and, therefore, will not be available and will be indicated with a hyphen (-).

Also shown is AASI- Ambulatory Arterial Stiffening Index and MSI- Morning Surge Index.

8.4.18 Exit the ABP Review

To exit the ABP Review screen click on the Cancel icon.

8.4.19 Import / Export Data:

From the ABP test screen, select Click Import/Export and select from

- Export to SNTL
- Export to DICOM
- Export to ABP
- Export to ART

View Section 7 for more information on data import and export.

8.5 Protocols

8.5.1 Protocols

This allows programming of the ABP unit in order to vary the inflation intervals. These modified Inflation Protocols may be saved for easy access during the initialization process.

1. To add a new inflation protocol, from the ABP screen Click on **Other actions**, then select **Protocol**, then **Add protocol**
2. Name the Protocol and select options under **Recorder**, then Add the Interval timings as required.
3. Once completed click **Save**.
4. Existing Protocols can be edited by clicking on **Menu** next to required Protocol and selecting **Edit Protocol**.

A protocol can be deleted by selecting the protocol from the displayed list and clicking the **Delete** button on the menu on the left of the screen.

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