

NACL

Diluent NaCl 9 %

cobas®• Indicates **cobas c** systems on which reagents can be used**Order information**

Diluent NaCl 9 % (50 mL)

Cat. No. **04489357** 190

System-ID 07 6869 3

Roche/Hitachi **cobas c** systems**cobas c** 311 **cobas c** 501/502

*The value encoded in the instrument settings is 39.2 mL to account for the dead volume of the bottles.

English**System information**For **cobas c** 311/501/502 analyzers:**NACL:** ACN 951**Intended use**Diluent NaCl 9 % is used as a sample diluent in conjunction with assay reagents on **cobas c** systems.**Summary**

Dilution of samples is necessary when analyte concentrations exceed the measuring range of the respective **cobas c** test method. In addition, general predilution of samples is specified for some tests. For dilution of human body liquids, physiological sodium chloride solution (0.9 % NaCl in water) is recommended. The present diluent is concentrated and will be diluted on the instrument by a factor of 10.

Reagents - working solutions

9 % NaCl

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Safety data sheet available for professional user on request.

Disposal of all waste material should be in accordance with local guidelines.

Reagent handling

Ready for use.

Storage and stability*Diluent NaCl 9 %*

Shelf life at 2-8 °C:

See expiration date on
cobas c pack label.

On-board in use and refrigerated on the analyzer:

12 weeks

Materials provided

See "Reagents - working solutions" section for reagents

Materials required (but not provided)

See "Order information" section.

Roche/Hitachi **cobas c** analyzers and assay reagents.

See appropriate package insert and operator's manual for additional required materials.

Assay

Use Diluent NaCl 9 % as specified in the respective instructions for use of the system reagents.

FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.
Other brand or product names are trademarks of their respective holders.
Significant additions or changes are indicated by a change bar in the margin.

© 2010, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336

