

REF 07126999 190

→ 4 x 1.0 mL

English

Intended use

SCC CalSet is used for calibrating the quantitative Elecsys SCC assay on the Elecsys and **cobas e** immunoassay analyzers.

Summary

SCC CalSet consists of lyophilized human serum matrix with added SCC protein (recombinant from *E. coli*) in two concentration ranges.

The CalSet can be used with all reagent lots.

Reagents - working solutions

- SCC Cal1: 2 bottles, each for 1.0 mL of calibrator 1
- SCC Cal2: 2 bottles, each for 1.0 mL of calibrator 2

SCC (recombinant from *E. coli*) in two concentration ranges (< 1.5 ng/mL and approximately 20 ng/mL) in a human serum matrix; preservative.

The exact lot-specific calibrator values are encoded in the barcode as well as printed on the enclosed (or electronically available) calibrator barcode sheet.

Calibrator values

Traceability: The Elecsys SCC assay has been standardized against the ARCHITECT SCC method from Abbott Diagnostics.

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

2-methyl-2H-isothiazol-3-one hydrochloride

EUH 208 May produce an allergic reaction.

Product safety labeling primarily follows EU GHS guidance.

All human material should be considered potentially infectious. All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods applied were FDA-approved or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

However, as no testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.^{1,2}

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Handling

Carefully dissolve the contents of one bottle by adding exactly 1.0 mL of distilled or deionized water and allow to stand closed for 15 minutes to reconstitute. Mix carefully, avoiding foam formation.

Transfer the reconstituted calibrators into the supplied empty labeled snap-cap bottles.

Elecsys 2010 and **cobas e** 411 analyzers: The reconstituted calibrators should only be left on the analyzers during calibration at 20-25 °C. After use, close the bottles as soon as possible and store upright at 2-8 °C. Due to possible evaporation effects, not more than 3 calibration procedures per bottle set should be performed.

If necessary, freeze in aliquots; see section on MODULAR ANALYTICS E170, **cobas e** 601 and **cobas e** 602 analyzers.

MODULAR ANALYTICS E170, **cobas e** 601 and **cobas e** 602 analyzers: Unless the entire volume is necessary for calibration on the analyzers, transfer aliquots of the reconstituted calibrators into empty snap-cap bottles (CalSet Vials). Attach the supplied labels to these additional bottles. Store the aliquots at -20 °C for later use.

Perform **only one** calibration procedure per aliquot.

SCC reactive determinants are shed naturally in skin particles, saliva and other body fluids and can easily be distributed in dust or aerosols, e.g. as a result of sneezing. Contamination of the samples, the CalSet Vials or the

Elecsys consumables with SCC may cause falsely elevated SCC results. Gloves should be used throughout the test procedure when handling reagents, samples, CalSet Vials etc. A face mask is also recommended.

Please note: Both the vial labels, and the additional labels (if available) contain 2 different barcodes. The barcode between the yellow markers is for **cobas** 8000 systems only. If using a **cobas** 8000 system, please turn the vial cap 180° into the correct position so the barcode can be read by the system. Place the vial on the instrument as usual.

Storage and stability

Store at 2-8 °C.

The lyophilized calibrators are stable up to the stated expiration date.

Stability of the reconstituted calibrators:	
either at -20 °C	28 days (freeze only once)
or at 2-8 °C	28 days
or at 20-25 °C	8 hours

Store calibrators **upright** in order to prevent the calibrator solution from adhering to the snap-cap.

Materials provided

- SCC CalSet, barcode card, calibrator barcode sheet, 4 empty labeled snap-cap bottles, 2 x 6 bottle labels

Materials required (but not provided)

- REF 11776576322, CalSet Vials, 2 x 56 empty snap-cap bottles
- Elecsys 2010, MODULAR ANALYTICS E170 or **cobas e** immunoassay analyzers and Elecsys SCC assay reagents
- Distilled or deionized water

See the assay Method Sheet and the operator's manual for additionally required materials.

Assay

Place the reconstituted calibrators (in the system-compatible bottles with barcoded labels) in the sample zone.

Read in all the information necessary for calibrating the assay.

Ensure the calibrators are at 20-25 °C prior to measurement.

References

- 1 Occupational Safety and Health Standards: bloodborne pathogens. (29 CFR Part 1910.1030). Fed. Register.
- 2 Directive 2000/54/EC of the European Parliament and Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets, the product information and the Method Sheets of all necessary components (if available in your country).

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.



SCC antigen used in the Roche SCC products is licensed by Fujirebio Diagnostics, Inc.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard.

	Contents of kit
	Analyzers/Instruments on which reagents can be used
	Reagent
	Calibrator
	Volume after reconstitution or mixing
	Global Trade Item Number

SCC CalSet

cobas[®]

COBAS, COBAS E and ELECSYS are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2015, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

