- (GB)** Instructions for use/Technical description
STERILIT Power Systems oil spray GB600
- (USA)** STERILIT Power Systems-Ölspray GB600
- (D)** Gebrauchsanweisung/Technische Beschreibung
STERILIT Power Systems-Ölspray GB600
- (F)** Mode d'emploi/Description technique
Spray d'huile STERILIT Power Systems GB600
- (E)** Instrucciones de manejo/Descripción técnica
Aceite en espray STERILIT Power Systems GB600
- (I)** Istruzioni per l'uso/Descrizione tecnica
Olio spray STERILIT Power Systems GB600
- (P)** Instruções de utilização/Descrição técnica
Óleo pulverizador STERILIT Power Systems GB600
- (NL)** Gebruiksaanwijzing/Technische beschrijving
STERILIT Power Systems-oliespray GB600
- (S)** Bruksanvisning/Teknisk beskrivning
STERILIT Power Systems-oljespray GB600
- (RUS)** Инструкция по применению/Техническое описание
Масляный спрей STERILIT Power Systems GB600
- (CZ)** Návod k použití/Technický popis
Olejový sprej STERILIT Power Systems GB600
- (PL)** Instrukcja użytkowania/Opis techniczny
Spray olejowy STERILIT Power Systems GB600
- (SK)** Návod na používanie/Technický opis
Olejový sprej STERILIT Power Systems GB600
- (TR)** Kullanım Kılavuzu/Teknik açıklama
STERILIT Power Systems yağ spreyi GB600

B | BRAUN

SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

TA-Nr. 012081 08/15 V6 Änd.-Nr. 51366

CE – DIR 93/42/EEC

Aesculap®

STERILIT Power Systems oil spray GB600

Symbols on product and packages

	Extremely flammable aerosol
300 ml	Nominal filling quantity
e	Conformity mark for finished packaging ordinance
405	Full capacity volume
3	Conformity mark for aerosol packaging
	Use by
	Limited quantity (LQ)
	Store this way up

Intended use

The Sterilit maintenance oil is validated for steam sterilization procedures according to EN ISO 17665. Sterilit is applied prior to sterilization.

Properties

- Steam-permeable oil film
- No impairment of the sterilization effect from the lubrication film
- Lubricating effect and corrosion protection
- Silicone free

Safe handling and preparation

DANGER

Extremely flammable aerosol.
Pressurised container: May burst if heated.

- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not spray on an open flame or other ignition source.
- Do not pierce or burn, even after use.
- Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.
- Keep out of reach of children.
- Only put empty cans in the recyclables collection.

WARNING

Frostbite due to evaporative cold!

- Fold motor/hand piece and adapter into a towel, cellulose or similar.

WARNING

Hazardous for aquatic organisms, can cause long-term harmful effects in water!

- Do not release into the environment.
- Observe special instructions/safety data sheet.

- Before using Sterilit: Reprocess medical devices according to instructions for use.
- Apply prior to sterilization only.
- Ensure sufficient ventilation.
- Store in a cool, dry place, away from light.

Application

- Observe the instructions for use of the corresponding motor/handpiece.
- Apply a suitable adapter onto the spray can.
- Apply the adapter onto the connector on handpiece/motor.
- Hold the spray can upright.
- During the functional check before sterilization, spray through the handpiece/motor for approx. 2 seconds.
- Wipe off excess oil with a lint-free wipe.

Adapter overview

STERILIT Power Systems adapter	for hand piece/motor
GB600840	 Acculan 3Ti Acculan 4
GB600880	 Acculan 4 GA344 ELAN 4 GA844
GB600860	 ELAN 4 electro
GB600850	 ELAN 4 air
GB600870	 microspeed uni XS
GB600820	 microspeed uni HiLAN HiLAN XS Hi-Line Hi-Line XS Hi-Line XXS
GB600830	 miniLine
GB600810	 microLine

Technical Service

WARNING

Risk of injury and/or malfunction!

- Do not modify the product.

- For service and repairs, please contact your national B. Braun/Aesculap agency. Modifications carried out on medical technical equipment may result in loss of warranty rights and applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 (7461) 95-1602
Fax: +49 (7461) 16-5621
E-Mail: ats@aesculap.de

Or in the US:

Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042
Aesculap Repair Hotline

Phone: +1 (800) 214-3392
Fax: +1 (314) 895-4420

Other service addresses can be obtained from the address indicated above.

Disposal

- Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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