

ECO-D

EcoTergent

Order information

REF	CONTENT	Analyzer(s) on which cobas c pack(s) can be used
08063354190*	EcoTergent (40 mL)	System-ID 2901 001 cobas c 303, cobas c 503
08063354214*	EcoTergent (40 mL)	System-ID 2901 001 cobas c 303, cobas c 503

* Some kits shown may not be available in all countries.

English

System information

ECO-D: ACN 29010

Intended use

EcoTergent is an additive to the reaction bath to reduce surface tension on **cobas c** systems.

Summary

EcoTergent is added to the reaction bath. It acts as a surfactant to minimize the formation of bubbles that could potentially interfere with the photometer readings.

Reagents - working solutions

Detergent

Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Danger

H314 Causes severe skin burns and eye damage.

H410 Very toxic to aquatic life with long lasting effects.

Prevention:

P273 Avoid release to the environment.

P280 Wear protective gloves/ protective clothing/ eye protection/ face protection/ hearing protection.

Response:

P303 + P361 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.

P304 + P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER/ doctor.

P305 + P351 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/ doctor.

P391 Collect spillage.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590

Reagent handling

Ready for use

Storage and stability

Store protected from light.

Shelf life at 15-25 °C:

See expiration date on **cobas c** pack label.

On-board in use and refrigerated on the analyzer:

4 weeks

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Assay

Use EcoTergent as specified in the respective instructions for use of the system reagents.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see dialog. Roche.com for definition of symbols used):

CONTENT	Contents of kit
➔	Volume for reconstitution
GTIN	Global Trade Item Number

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2022, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

+800 5505 6606

