04912349001V16

CleanCell M

cobas®

REF 04880293190*

REF 04880293214*

* Some kits shown may not be available in all countries.

For USA: Elecsys CleanCell M

English

Reagent handling: Ready for use. Avoid foam formation.

Storage and stability: on the **cobas e** analyzers: 21 days.

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

Warning

H290 May be corrosive to metals.

H315 Causes skin irritation.

H319 Causes serious eye irritation.

H412 Harmful to aquatic life with long lasting effects.

Prevention:

P264 Wash skin thoroughly after handling.

P273 Avoid release to the environment.

P280 Wear protective gloves/ eye protection/ face protection.

Response:

P337 + P313 If eye irritation persists: Get medical advice/attention.

P390 Absorb spillage to prevent material damage.

Disposal:

P501 Dispose of contents/container to an approved waste disposal plant.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

For US users: Warning. Bottles contain potassium hydroxide solution; corrosive. In case of contact, flush areas with copious amounts of water. Get immediate medical attention for eyes, or if ingested.

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FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications
 stated in the labeling when used in accordance with such labeling and will be
 free from defects in material and workmanship until the expiration date printed
 on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER
 WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED
 WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR
 PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR
 INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS, COBAS E and ELECSYS are trademarks of Roche.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2022, Roche Diagnostics

Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

+800 5505 6606

Distribution in USA by:
 Roche Diagnostics, Indianapolis, IN
 US Customer Technical Support 1-800-428-2336

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4282336 / AB: +800 5505 6606 / RU: +8 800