Zircate®

Cleaning and Polishing Material- Prophy Paste

Instructions For Use

Caution: This is a medical device. For dental use only.

USA: Rx only.

1. PRODUCT DESCRIPTION

Zircate® Cleaning and Polishing Material - Prophy Paste is used in adults and children for tooth polishing in conjunction with dental prophylaxis procedures. The abrasive filler, Zirconium Silicate is used to effectively clean and polish tooth structure with minimal abrasion.

1.1. Indications

Zircate Prophy Paste is indicated for use to effectively clean and polish tooth structure with minimal abrasion.

1.2 Contraindications

Zircate Prophy Paste is contraindicated for use with patients who have a known hypersensitivity to sorbitol.

1.3. Delivery Forms

Zircate is available in 170g collapsible laminate tubes

1.4. Composition

Zircate Cleaning and Polishing Material- Prophy Paste is composed of zirconium silicate in selected and controlled particle sizes in an aqueous spearmint flavored base.

Zircate® Cleaning and Polishing Material - Prophy Paste consists of zirconium silicate, tin oxide, DI water, glycerin, sorbitol, thickeners and spearmint oil

2 GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

1. Zircate Prophy Paste may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

Wear suitable protective clothing, eyewear, and gloves. Protective eyewear is recommended for patients.

Avoid Eye Contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.

Avoid Skin Contact to prevent irritation and possible allergic response. If contact with skin occurs immediately remove material with cotton and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.

Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the procedure is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2. Zircate Prophy Paste should not be used with patients who have a known hypersensitivity to any of the components.

2.2 Precautions

- This product is intended to be used as specifically outlined in the Instructions for Use. Any use of this product inconsistent with the Instructions for Use is at the discretion and sole responsibility of the practitioner.
- Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
- The tubes cannot be reprocessed. To prevent tubes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the tube is handled offside the dental unit with clean/disinfected gloves. During treatment clinicians with patient contact should not handle tubes.
- Zircate Prophy Paste is intended to be used with a rubber prophylaxis cup run at SLOW SPEED. High speeds may cause uncontrolled splashing of material (See Step-by-step Directions for Use).
- The Zircate Prophy Paste container should be tightly closed immediately after use.

Interactions:

- None known.

2.3 Adverse Reactions

• Product may irritate the eyes and skin. Eye contact: irritation and possible corneal damage. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin. Mucous Membranes: inflammation, edema, sloughing. See Warnings.

2.4 Storage Conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

Zircate material containers should be stored with the original cap only. Keep out of direct sunlight and store in a well-ventilated place at temperatures between 10°C/50°F and 25°C/77°F. Allow material to reach room temperature prior to use.

Protect from moisture. Do not freeze. Do not use after expiration date.

3. STEP-BY-STEP INSTRUCTIONS

1. Dispense Zircate Prophy Paste into dappen dish or other suitable container. A saliva ejector may be helpful during the actual prophylaxis.
2. Fill rubber prophylaxis cup or brush with paste. Before starting handpiece, carry paste to field of operation and deposit a small amount on each tooth surface. It is necessary that fresh paste be provided for each surface in order to obtain maximum cleaning effectiveness.

To Reduce Risk of Material Splashing-Eye Irritation

Wear protective eyewear.

Provide protective eyewear for patients.

Use only slow speed handpiece.

3. Run rubber cup at SLOW SPEED over each tooth surface for approximately 10 seconds. Make sure that the edge of the cup goes below the free margin of the gingival into the gingival sulcus.

Continue procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished.

4. A rubber cup or soft bristle brush may be used on occlusal surfaces.

TECHNIQUE HINT

Use unwaxed dental floss to draw excess Zircate Prophy Paste through interproximal areas.

4. HYGIENE AND DISPOSAL

Cross-contamination

To reduce risk of infection

- Tubes cannot be reprocessed. Dispose of contaminated tubes in accordance with local regulations.
 - When not in use, all supplies and instruments should be in covered storage, such as drawers and cabinets, and away from potential contamination.
 - During treatment clinicians with patient contact should not handle the tubes.
 - To prevent the tubes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the tubes are handled with clean/disinfected gloves, outside of the patient treatment operatory.
 - Dispensing materials with clean/disinfected gloves or hands in a separate room, bringing only what will be used into the operatory is strongly recommended.
 - Do not reuse tubes if contaminated.
- 4.1 To prevent tubes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended. The use of protective barriers is an additional precautionary measure against gross debris but not against all contamination. The barrier is designed for single use and must be disposed of after each use in accordance with local regulations.
 - 4.2. Disposal
Dispose of in accordance with local regulations.

5. LOT NUMBER, EXPIRATION DATE AND CORRESPONDENCE

1. Do not use after expiration date. ISO standard is used: "YYYY/MM" or "YYYY-MM-DD."
2. The following numbers should be quoted in all correspondences:
 - Reorder number
 - Lot number
 - Expiration date
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.

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Dentsply De Trey GmbH