


## BM-100C Transcutaneous Jaundice Detector

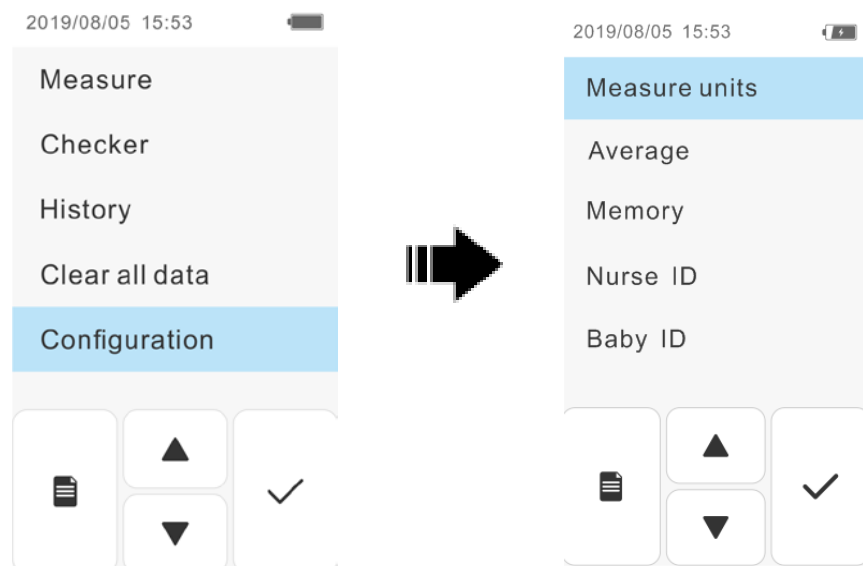

### Set up the equipment

- 1 Press the power switch for more than 5 seconds to enter the main interface.
- 2 Select "Configuration" on the menu screen. Or press ▲ or ▼ button to move the highlighted cursor over "Configuration", then press ✓ button to enter the Configuration setting screen, display the setting screen.
- 3 Select the items you want to change in sequence on the setting screen, and then set them separately:

**Measure units:** Set the unit displayed during measurement.


**Average:** Set the number of times when the average measurement is carried out, such as "3 times".

**Memory:** Set whether the measured data is saved or not.

**Nurse ID:** Select the input method for the nurse ID. Select Touch input or Barcode input None (Not enter nurse ID).

**Baby ID:** Select the input method for the baby ID. Select Touch input or Barcode input. After setting, press ✓ button to confirm.

## BM-100C Transcutaneous Jaundice Detector

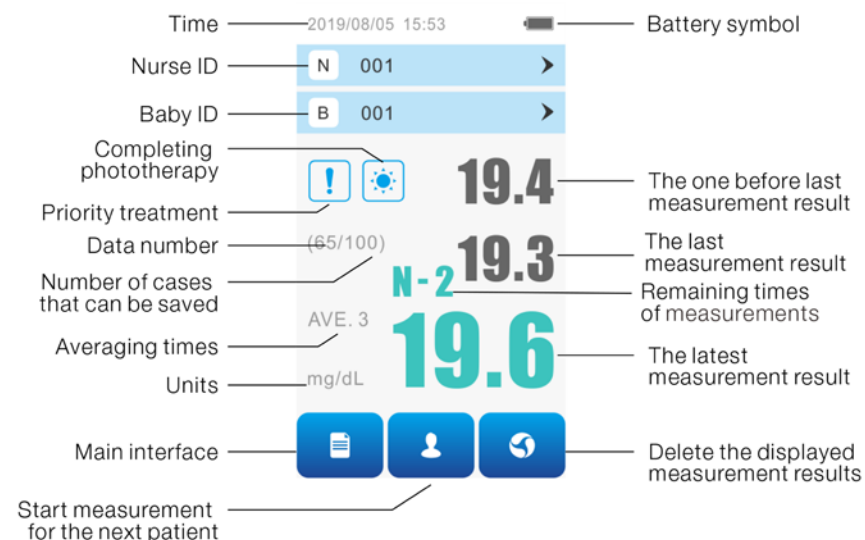

### The measurement

- ① Select "Configuration" on the menu screen to enter the measurement screen.
- ② Clean the optical probe with medical alcohol.
- ③ Make sure the "Ready" indicator green light lights up, indicating that the device is ready for measurement.
- ④ Place the optical probe vertically on the measuring point (sternum or forehead) and gently press until it glows. The measured values will be displayed on the display screen. Read the measured value.
- ⑤ If the average number of measurements is set to multiple times, repeat the above ③ ④ operation at the measurement point number. After multiple measurements are completed, the interface displays the average measurement value.

**Note:** If the idle time of the device is more than 1 min, there will be no display on the screen. Please touch the display screen to wake up. If the power is OFF, press and hold the power switch for more than 5 seconds to set the power to ON.

EDITION/REVISION A-0  
DATE OF ISSUE 2021/09