<b>egemen</b> INTERNATIONAL	<b>INSTRUCTIONS FOR USE</b>	<b>CE</b> 1014
<b>SPINAL NEEDLE</b> <b>(QUINCKE / PENCIL POINT / ATRAUMATIC-SPECIAL GRINDED)</b>		

1. Used for spinal anesthesia.
2. Open pack using sterile technique.
3. Remove plastic tip protector and examine needle body and tip in order to confirm its integrity.
4. Remove stylet of Spinal needle then place it again to confirm smooth movement.
5. Prepare puncture area for the procedure according to standard aseptic technique.

**NOTE:** Based on the physician's clinical training, local anesthesia may be necessary for needles with larger size than 25G.

**⚠** The use of Guide Needle is highly recommended with 25G, 26G, 27G and 29G Spinal needles.

6. Determine a puncture area (point) according to accepted procedure and advance until you reach subarachnoid space. You can pull back stylet and monitor CSF flow in order to confirm if needle tip is inside subarachnoid space.
7. After confirming that needle tip is inside subarachnoid space, remove stylet and inject anesthetic drug.
8. After completing Spinal anesthesia application, remove Spinal Needle and dispose it according to medical waste regulations.

#### WARNINGS:

- This device is to be used only for professional purpose and by trained professionals.
- Do not use if package is torn or damaged. These are not sterile.
- Store in cool, dry and dust free environments.
- After use dispose device according to medical disposal regulations.
- Do not apply any techniques other than the approved procedure.

The instructions in this manual and all other Egemen International® (TMT Tıbbi Medikal Malz. San. Tic. Ltd. Şti) sources are for guidance purpose only. The proper use of this device should be based on clinical training of the physician.

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TMT TIBBİ MEDİKAL MALZ. SAN. TİC. LTD. ŞTİ.