

Chemicals for processing telescopes, fiberscopes, and endoscopic instruments

Manual cleaning and disinfection/ultrasonic cleaning*

The following KARL STORZ instruments and telescopes are **not** suitable for complete **immersion** in liquids:

- HAMOU® telescopes with eyepiece drive mechanism
- Magnifying lenses with drive mechanism
- For motors and their handles, observe the respective instructions for use

The following KARL STORZ instruments are **not** suitable for **ultrasonic cleaning**:

- Rigid telescopes
- Flexible and semiflexible fiberscopes
- Other instruments with optical components made of glass, with chips, or with electronic components



We maintain a separate list of substances approved for flexible videoscopes.



Please follow the instructions "Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments." They provide a detailed description of cleaning, disinfection, and sterilization procedures.



To minimize stress on materials, avoid repeated switching between processing procedures.



Damage to materials:

Cleaning the outer sheath of flexible fiberscopes with alcohol or ammonium chlorides will damage the material, particularly at the hose nozzle.



Please note that when preparing and using the solutions, **the chemical manufacturers' instructions on concentration and contact time must be strictly followed.**


Excessively long immersion and incorrect concentrations may cause damage. Please contact the chemical manufacturer regarding microbiological effectiveness.



Country-specific laws and regulations must be observed.



During all cleaning and disinfection procedures, observe the occupational safety guidelines of the employer's liability insurance association (Berufsgenossenschaft) or equivalent organization.

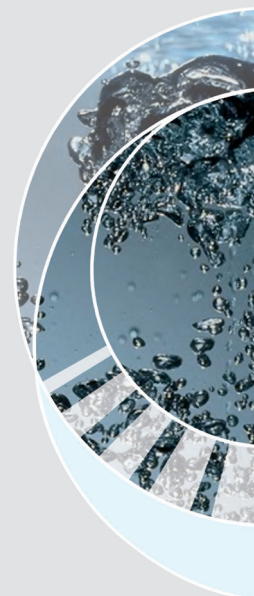

EndoFax®
Endo @ Mail®


**KARL STORZ
GmbH & Co. KG**


Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany


Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

* For detailed information on processing, please refer to the respective instructions for use.

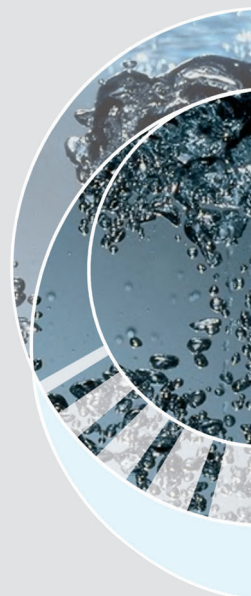

 Using SODIUM HYDROXIDE SOLUTION may lead to surface alterations in aluminum parts (even if coated), plastics, and soldered joints, and it may reduce the service life of the instruments. For flexible endoscopes, only immerse the examination sheath, but not the housing/handle.

 Gigasept FF from the chemical manufacturer Schülke & Mayr GmbH at a concentration of 8% and a contact time of 60 minutes is not approved for fiberscopes.

 **Note**
 **Caution**
 **Warning**

KARL STORZ has tested and approved the following agents regarding material compatibility:

- Suitable chemical!
- Unsuitable chemical!

Manufacturer	Trade name	Cleaning agent	Disinfectant	Suitable for ultrasonic baths	Rigid telescopes	Instruments	Fiberscopes
Various	NaOH (1 mol/l) 1 h sodium hydroxide solution	-	•	-	•	•	•
Acto GmbH	Actosed Endo	•	•	•	•	•	-
Acto GmbH	Actosed Endo Terra	-	•	-	•	•	•
Acto GmbH	Actosed Enzym	•	-	•	•	•	•
Acto GmbH	Actosed Forte	•	•	•	•	•	•
Acto GmbH	Actosed Forte N	-	•	•	•	•	•
Acto GmbH	Actosed PA Powder	-	•	-	•	•	•
Advanced Sterilization Products*	Cidex OPA	-	•	-	•	•	•
Advanced Sterilization Products*	Cidezyme	•	-	•	•	•	•
Advanced Sterilization Products*	Enzol	•	-	•	•	•	•
Advanced Sterilization Products*	NU Cidex	-	•	-	•	•	•
Akadia – Chemie	Akadent	-	•	-	•	•	•
Alkapharm	Peralkan	-	•	-	•	•	•
Alpro Dental-Produkte GmbH	BIB forte	•	•	•	•	•	•
Anios	Aniosyme P.L.A.	•	•	•	•	•	•
Anios	Anioxyde 1000	-	•	-	•	•	•
Anios	Octanius basique	•	•	-	•	•	•
Anios	OPASTER	-	•	-	•	•	-
Anios	Steranios 2%	-	•	-	•	•	•

EndoFax[®]
Endo @ Mail[®]


**KARL STORZ
GmbH & Co. KG**

Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
2/10

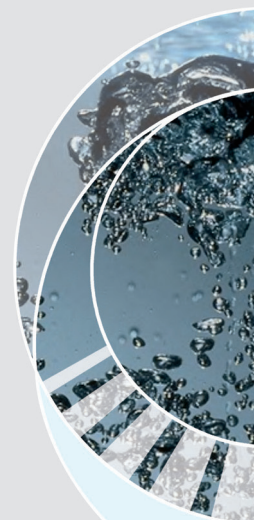

EndoFax[®]
Endo @ Mail[®]

**KARL STORZ
GmbH & Co. KG**


Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
3/10

Manufacturer	Trade name	Cleaning agent	Disinfectant	Suitable for ultra-sonic baths	Rigid telescopes	Instruments	Fiberscopes
Antiseptica chem. pharm. Prod. GmbH	Descogen I	•	•	-	•	•	•
Antiseptica chem. pharm. Prod. GmbH	Kombi Instru.-Desinfektion N	•	•	-	•	•	•
Antiseptica chem. pharm. Prod. GmbH	Triacid N	•	•	•	•	•	-
B. Braun Medical AG	Helipur H plus N	-	•	•	•	•	•
B. Braun Medical AG	Helizyme	•	-	-	•	•	•
B. Braun Medical AG	Helix ultra	-	•	-	•	•	•
B. Braun Medical AG	Stabimed	•	•	•	•	•	•
Bandelin electronic GmbH	Stammopur DR	•	•	•	•	•	•
Bandelin electronic GmbH	Stammopur DR 8	•	•	•	•	•	•
Bandelin electronic GmbH	Stammopur GR	•	-	•	•	•	•
Bandelin electronic GmbH	Stammopur R	•	-	•	•	•	•
Bochemie a.s.	Chirosan	-	•	-	•	•	•
Bochemie a.s.	Chirosan plus	•	•	-	•	•	•
Bochemie a.s.	Discleen enzyme	•	-	•	•	•	•
Bochemie a.s.	Discleen Endo AF	-	•	-	•	•	•
Bochemie a.s.	Discleen Endo PAA	•	•	-	•	•	•
Bochemie a.s.	Discleen Extra	•	•	-	•	•	•
Bode Chemie GmbH	Bodedex forte	•	-	•	•	•	•
Bode Chemie GmbH	Bomix plus	•	•	-	•	•	•
Bode Chemie GmbH	Dismoclean twin basic/enzyme	•	-	-	•	•	-
Bode Chemie GmbH	Korsolex med AF	-	•	-	•	•	-
Bode Chemie GmbH	Korsolex basic	-	•	•	•	•	•
Bode Chemie GmbH	Korsolex extra	-	•	-	•	•	•
Bode Chemie GmbH	Korsolex PAA	-	•	-	•	•	•
Borer Chemie AG	Deconex ENDOMATIC	-	•	-	•	•	•
Borer Chemie AG	Deconex 36 Intensiv	•	-	•	•	•	•
Borer Chemie AG	Deconex 50 FF	•	•	-	•	•	•
Borer Chemie AG	Deconex 53 Instrument	•	•	•	•	•	-
Borer Chemie AG	Deconex 53 Plus	-	•	-	•	•	•
Borer Chemie AG	Deconex HLD PA/PA 20	-	•	-	•	•	•
Borer Chemie AG	Deconex Power Zyme	•	-	•	•	•	•
Borer Chemie AG	Deconex 54 Sporicide	-	•	-	•	•	•
Borer Chemie AG	Deconex 53 INSTRUMENT	-	•	•	•	•	•
DuPont	PeraSafe	-	•	-	•	•	•

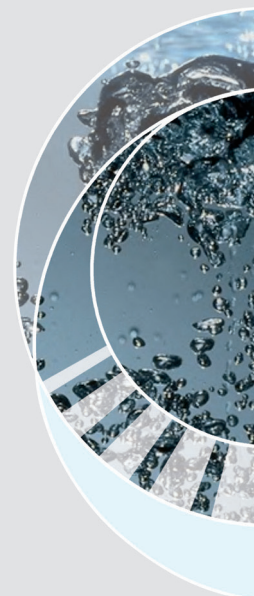

EndoFax[®]
Endo @ Mail[®]

**KARL STORZ
GmbH & Co. KG**


Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
4/10

Manufacturer	Trade name	Cleaning agent	Disinfectant	Suitable for ultrasonic baths	Rigid telescopes	Instruments	Fiberscopes
DuPont	Rely+On Perasafe	•	•	-	•	•	•
Deppe, Dr. rer. nat.; Laboratorium	EndoStar	-	•	-	•	•	•
Deppe, Dr. rer. nat.; Laboratorium	InstruPlus	•	•	•	•	•	•
Deppe, Dr. rer. nat.; Laboratorium	InstruPlus N	-	•	•	•	•	•
Deppe, Dr. rer. nat.; Laboratorium	InstruStar	•	•	•	•	•	-
Deppe, Dr. rer. nat.; Laboratorium	InstruZym	•	-	•	•	•	•
Detrox	Detro Forte	-	•	-	•	•	•
Detrox	Detro Plus	-	•	-	•	•	•
Detrox	Detro Plus OPA	-	•	-	•	•	•
Detrox	Detro OPA	-	•	-	•	•	•
Ecolab GmbH & Co. OHG	Sekusept aktiv	•	•	-	•	•	•
Ecolab GmbH & Co. OHG	Sekusept Cleaner	•	-	•	•	•	•
Ecolab GmbH & Co. OHG	Sekusept easy	-	•	-	•	•	•
Ecolab GmbH & Co. OHG	Sekusept extra N	•	•	•	•	•	•
Ecolab GmbH & Co. OHG	Sekusept forte	-	•	•	•	•	•
Ecolab GmbH & Co. OHG	Sekusept MultiEnzyme	•	-	•	•	•	•
Ecolab GmbH & Co. OHG	Sekusept Plus	•	•	•	•	•	•
Ecolab GmbH & Co. OHG	Sekusept Pulver Clas- sic	•	•	•	•	•	•
Esteer Pharma GmbH	Ultraseptin Aktiv	•	•	•	•	•	•
Esteer Pharma GmbH	Ultraseptin Classic	•	•	•	•	•	•
Esteer Pharma GmbH	Ultradesmit AF	•	•	•	•	•	•
Esteer Pharma GmbH	Ultraseptin PAA	•	•	-	•	•	•
Fresenius AG	Afid	•	•	-	•	•	•
Fresenius AG	Afid plus	-	•	-	•	•	•
Fresenius AG	Sprocid FF	•	•	-	•	•	•
Heck dental GmbH	Ventisept M Plus Neu	•	•	-	•	•	•
Helvemed	Instrument Forte SVP	•	•	•	•	•	•
HIMED GmbH	EndoSeptol FF	-	•	-	•	•	•
HIMED GmbH	InstruCid AF	-	•	-	•	•	-
Holifa Poska Sp. zo.o	Polsept Holifa	•	•	-	•	•	•
Holifa Poska Sp. zo.o	Polsept Plus	•	•	-	•	•	•

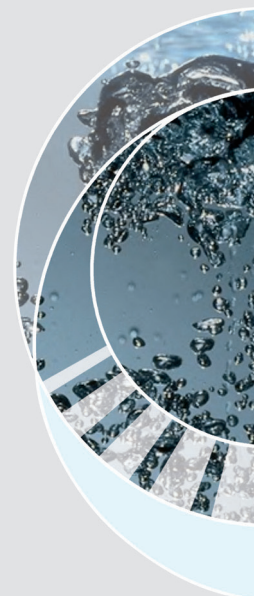

EndoFax®
Endo @ Mail®

**KARL STORZ
GmbH & Co. KG**


Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
5/10

Manufacturer	Trade name	Cleaning agent	Disinfectant	Suitable for ultra-sonic baths	Rigid telescopes	Instruments	Fiberscopes
Indeba Industira E Comercio LTDA	Letahdeido	-	•	-	•	•	•
Laboratorios INIBSA	INSTRUNET F.A. CONCEN.	•	•	•	•	•	•
Jose Collado S.A.	Darodor 4000 Liquido	•	-	-	•	•	•
Jose Collado S.A.	Darodor 9000	-	•	-	•	•	•
Jose Collado S.A.	Darodor Sinaldehyd 2000	-	•	•	-	•	•
Lonza AG	ID-50	-	•	•	-	•	•
Lysoform Dr. Rosemann GmbH	Aldasan 2000	•	•	•	-	•	•
Lysoform Dr. Rosemann GmbH	Almyrol	•	•	•	-	•	•
Lysoform Dr. Rosemann GmbH	Desoform	•	•	•	•	•	•
Lysoform Dr. Rosemann GmbH	Lysoformin 3000	•	•	-	•	•	•
MediChem International	MediZyme	•	-	-	•	•	•
MediChem International	MedDis	-	•	-	•	•	•
MediChem International	PERASCOPE 3 DAY Sterilant	-	•	-	•	•	•
Merz Hygiene GmbH	Mucadont-Zymaktiv	•	-	•	•	•	•
Merz Hygiene GmbH	Edisonite Super	•	-	•	•	•	•
Merz Hygiene GmbH	Mucadont-IS	-	•	•	•	•	•
Merz Hygiene GmbH	Mucocit-T neu	•	•	•	•	•	•
METASYS Mediz-intechnik GmbH	ID 50	-	•	-	•	•	•
METASYS Mediz-intechnik GmbH	Green & Clean ID	•	•	•	•	•	-
Minntech International	Adaspor	-	•	-	•	•	•
Necmkimya	BIORAD GDA 2%	-	•	-	•	•	-
Necmkimya	BIORAD Rayd Guanide	•	•	•	•	•	-
Nova-Praxis-Hygiene	NOVA Biguades AF	•	•	•	•	•	•
Orochemie	A 20	•	-	•	•	•	-
Promagent AB	Wavacide	-	•	-	•	•	•
Rheosol Wachendorff-Chemie GmbH	RHEOSEPT-ID plus	•	•	•	•	•	•
Rheosol Wachendorff-Chemie GmbH	RHEOSEP-ID aktiv Granulat	•	•	•	•	•	•
Schumacher, Dr.; GmbH	Descoton Forte	-	•	-	•	•	•
Schumacher, Dr.; GmbH	Descoton 2% GDA	-	•	-	•	•	•

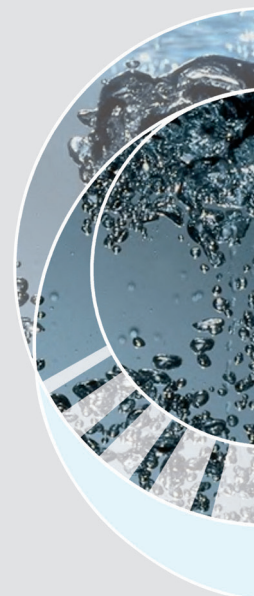

EndoFax[®]
Endo @ Mail[®]


**KARL STORZ
GmbH & Co. KG**

Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
6/10

Manufacturer	Trade name	Cleaning agent	Disinfectant	Suitable for ultrasonic baths	Rigid telescopes	Instruments	Fiberscopes
Schumacher, Dr.; GmbH	Descoton Extra	•	•	-	•	•	•
Schumacher, Dr.; GmbH	Multizyme Forte	•	-	•	•	•	•
Schumacher, Dr.; GmbH	Perfektan Endo	-	•	•	•	•	•
Schumacher, Dr.; GmbH	Perfektan TB	•	•	•	•	•	-
Schumacher, Dr.; GmbH	Plurazyme	•	-	•	•	•	•
Schülke & Mayr GmbH	Gigasept AF (forte)	•	•	-	•	•	•
Schülke & Mayr GmbH	Gigasept FF (new) 	•	•	-	•	•	•
Schülke & Mayr GmbH	Gigasept Instru AF	•	•	•	•	•	-
Schülke & Mayr GmbH	Gigasept Med forte	•	•	-	•	•	•
Schülke & Mayr GmbH	Gigasept PAA concentrate	-	•	-	•	•	•
Schülke & Mayr GmbH	Gigazyme	•	-	•	•	•	•
Schülke & Mayr GmbH	Lysetol V	-	•	-	•	•	•
SIRMAXO CHEMICALS PVT.LTD.	Sanizyme	•	-	•	•	•	•
SIRMAXO CHEMICALS PVT.LTD.	Sanidex OPA	-	•	-	•	•	•
SIRMAXO CHEMICALS PVT.LTD.	Saniscope	•	-	-	•	•	•
STERIS Corporation	EnzyCare 2	•	-	•	•	•	•
STERIS Corporation	Resert TM HLD XL	-	•	-	•	•	•
Tristel Solutions Ltd.	Tristel Sporicidal Wipes	-	•	-	•	-	•
Weigert, Dr.; GmbH & Co.	neodisher endo CLEAN	•	-	-	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher LM 2	•	-	•	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher Septo MED	•	•	•	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher mediclean	•	-	-	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher mediclean forte	•	-	•	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher medizym	•	-	•	•	•	•

Manufacturer	Trade name	Cleaning agent	Disinfectant	Suitable for ultra-sonic baths	Rigid telescopes	Instruments	Fiberscopes
Weigert, Dr.; GmbH & Co.	neodisher SeptoClean	•	-	-	•	•	-
Weigert, Dr.; GmbH & Co.	neodisher Septo PAC	-	•	-	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher SeptoPreClean	•	•	-	•	•	•
Whiteley Industries PTY.Ltd	Aidal Plus	-	•	-	•	•	•
Whiteley Industries PTY.Ltd	Matrix	•	-	-	•	•	•

* = Johnson & Johnson

If agents other than those listed above have been used, any arising damage is not covered by KARL STORZ.

EndoFax[®]
Endo @ Mail[®]

**KARL STORZ
GmbH & Co. KG**

Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
7/10



Cleaning and disinfection in washer-disinfectors*

The following KARL STORZ instruments and telescopes are **not** suitable for complete **processing in a washer-disinfector**:

- HAMOU® telescopes with eyepiece drive mechanism
- Magnifying lenses with drive mechanism
- Prismatic light deflectors
- For motors and their handles, observe the respective instructions for use

Select cleaning and disinfection procedures in consultation with the manufacturers of the machine and the chemicals. Only use special processes that have been validated for this purpose. Please contact the manufacturer of the processing chemicals regarding microbiological effectiveness. Thermal disinfection is preferable.



Please follow the instructions "Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments." They provide a detailed description of cleaning, disinfection, and sterilization procedures.




To minimize stress on materials, avoid repeated switching between processing procedures.



During all cleaning and disinfection procedures on contaminated instruments, observe the occupational safety guidelines of the employer's liability insurance association (Berufsgenossenschaft) or equivalent organization.

* For detailed information on processing, please refer to the respective instructions for use.

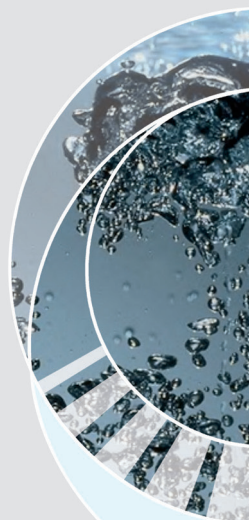

EndoFax®
Endo @ Mail®

**KARL STORZ
GmbH & Co. KG**

Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
8/10

KARL STORZ has approved the following agents for cleaning and disinfection in washer-disinfectors in terms of material compatibility:

- Suitable chemical!
- Unsuitable chemical!

Manufacturer	Trade name	Cleaning agent	Disinfectant	Rigid telescopes	Instruments	Fiberscopes
BHT Hygiene Technik GmbH	BHT Scope Cleaner	•	-	-	-	•
BHT Hygiene Technik GmbH	BHT Scope Disinfectant	-	•	-	-	•
Bode Chemie GmbH	Dismoclean 24 Vario	•	-	•	•	-
Bode Chemie GmbH	Dismoclean 28 Alka one	•	-	•	•	-
Bode Chemie GmbH	Dismoclean twin basic/enzyme	•	-	•	•	-
Bode Chemie GmbH	Korsolex-Endo-Cleaner	•	-	-	•	•
Bode Chemie GmbH	Korsolex-Endo-Disinfectant	-	•	-	-	•
Borer Chemie AG	Deconex 23 Neutrazym	•	-	•	•	•
Borer Chemie AG	Deconex 28 Alka One	•	-	•	•	-
Borer Chemie AG	Deconex 28 Alka One x	•	-	•	•	-
Borer Chemie AG	Deconex Alkamild	•	-	•	•	-
Borer Chemie AG	Deconex Endomatic	-	•	•	•	•
Borer Chemie AG	Deconex Power Zyme	•	-	•	•	•
Borer Chemie AG	Deconex twin system	•	-	•	•	•
B. Braun Medical AG	Helimatic Cleaner alkaline	•	-	•	•	-
B. Braun Medical AG	Helimatic Cleaner enzymatic	•	-	•	•	•
B. Braun Medical AG	Helimatic Disinfectant	-	•	•	•	•
Deppe, Dr. rer. nat.; Laboratorium	Endomat Plus	-	•	-	-	•
Deppe, Dr. rer. nat.; Laboratorium	Labomat E	•	-	-	-	•
Deppe, Dr. rer. nat.; Laboratorium	Endomat Plus Viruguard	-	•	•	•	•
DiverseyLever	Suma Tox E	•	-	•	•	•
Ecolab GmbH & Co. OHG	Sekumatic FD	-	•	•	•	-
Ecolab GmbH & Co. OHG	Sekumatic FR	•	-	•	•	-
Ecolab GmbH & Co. OHG	Sekumatic FRE	•	-	•	•	-
Ecolab GmbH & Co. OHG	Sekumatic MultiClean	•	-	•	•	-
IMS	Adapspor Ready to Use	-	•	•	•	•
Medisafe UK, Ltd.	3E-Zyme/HS - Zyme	•	-	•	•	-
Merz Consumer Care GmbH	Mucapur AF	•	-	•	•	-
Merz Consumer Care GmbH	Mucapur ED	-	•	•	•	•



EndoFax[®]
Endo @ Mail[®]

**KARL STORZ
GmbH & Co. KG**

Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany


Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
9/10

Manufacturer	Trade name	Cleaning agent	Disinfectant	Rigid telescopes	Instruments	Fiberscopes
Merz Consumer Care GmbH	Mucapur ER	•	-	•	•	•
Merz Consumer Care GmbH	Mucapur ED Plus	-	•	•	•	•
Merz Consumer Care GmbH	Mucapur ER Plus	•	-	•	•	•
Ruhof Corporation	Endozime AW	•	-	•	•	•
Schumacher, Dr.; GmbH	Thermoton Endo	-	•	•	•	•
Schumacher, Dr.; GmbH	Thermoton NR	•	-	•	•	•
Schumacher, Dr.; GmbH	Thermoton Desinfektant	-	•	-	-	•
Schülke & Mayr GmbH	Thermosept alka clean forte	•	-	•	•	-
Schülke & Mayr GmbH	Thermosept DK	-	•	•	•	-
Schülke & Mayr GmbH	Thermosept ED	-	•	-	-	•
Schülke & Mayr GmbH	Thermosept ER	•	-	-	-	•
Schülke & Mayr GmbH	Thermosept OXY	•	-	-	•	-
Schülke & Mayr GmbH	Thermosept PAA	-	•	-	-	•
Schülke & Mayr GmbH	Thermosept RKN-zym	•	-	•	•	-
Schülke & Mayr GmbH	Thermosept RKF forte	•	-	•	•	-
Weigert, Dr.; GmbH & Co.	neodisher endo CLEAN	•	-	-	-	•
Weigert, Dr.; GmbH & Co.	neodisher endo Sept GA	-	•	-	-	•
Weigert, Dr.; GmbH & Co.	neodisher DuoClean	•	-	•	•	-
Weigert, Dr.; GmbH & Co.	neodisher FA	•	-	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher FA forte	•	-	•	•	-
Weigert, Dr.; GmbH & Co.	neodisher Oxivario	•	-	•	•	-
Weigert, Dr.; GmbH & Co.	neodisher mediclean	•	-	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher mediclean forte	•	-	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher medizym	•	-	•	•	•
Weigert, Dr.; GmbH & Co.	neodisher SeptoClean	•	•	•	•	-
Weigert, Dr.; GmbH & Co.	neodisher Septo DA	-	•	-	•	-
Weigert, Dr.; GmbH & Co.	neodisher Septo DN	-	•	-	•	•

If agents other than those listed above have been used, any arising damage is not covered by KARL STORZ.

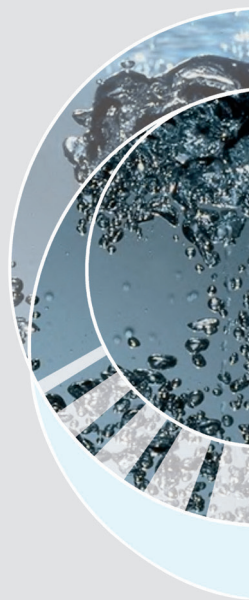

EndoFax[®]
Endo @ Mail[®]

**KARL STORZ
GmbH & Co. KG**

Mittelstraße 8
78532 Tuttlingen/Germany
Postfach 230
78503 Tuttlingen/Germany

Phone: +49 (0)7461 708-0
Fax: +49 (0)7461 708-105

E-Mail: info@karlstorz.de
Web: www.karlstorz.com

HYG 10 05/2012-E
10/10