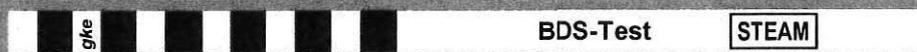
Colour reference chart

for all *gke* Bowie-Dick-Simulation (BDS) Tests

(SV = 121°C, 15 min / 132°C, 3.7 min / 134°C, 3.5 min / 137°C, 3.2 min)

PASS RESULTS

A successful Bowie-Dick Simulation (BDS) Test proves rapid and even steam penetration. The result does not guarantee that all subsequent sterilization cycles achieve the same process conditions. Therefore, a *gke* Batch or Process Monitoring System (BMS/PMS) is recommended for all production cycles.


all colour segments are black

→ sufficient steam penetration


FAIL RESULTS

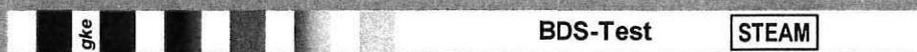If the first Bowie-Dick Simulation Test fails – repeat the test. If the test continues to fail the sterilizer must be checked and, if necessary, repaired. Fail conditions may be due to various reasons (see note below). In that case sufficient steam penetration to all inner surfaces of the load may not be achieved.

colour segments are black at one end and yellow-brownish at the other end

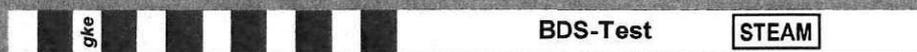→ insufficient steam penetration and air removal

all colour segments are brownish, but not black

→ temperature achieved, but no steam penetration and air removal

all colour segments are yellow

→ insufficient temperature and no steam penetration and air removal

Note

A failure of the BDS Test is not a conclusive proof that the fault is necessarily associated to the sterilizer itself (insufficient air removal or leakages), but may well be due to external causes such as unpurged steam pipes at the start of the process, non-condensable gases in the external steam supply or changing temperature of the cooling water. Therefore, BDS Tests made in subsequent cycles may show different results.

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Declaration of Conformity

We, Pharmalabel BV, hereby declare that the following product is used with rotary heat sealer and for daily testing which are conform to EN ISO 11607, part 2.

This product do not contain any Latex materials.

Product Description : Seal test

Technical data : medical paper (80 gr/m²), one sided satin with coating

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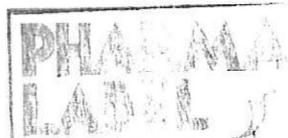
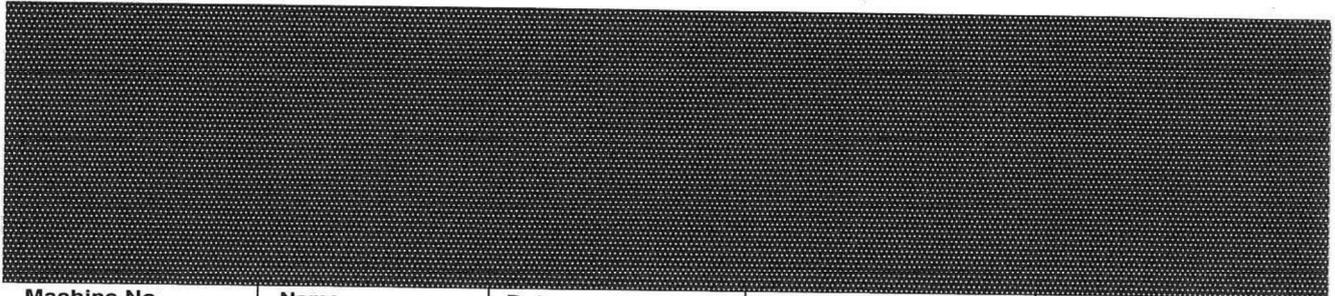
Put this test sheet into your pouch and pass it through the sealer once.
The seal must be darker than the environment throughout.

LOT	518936
	2021-07

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91555 Feuchtwangen - Germany

SRN (Single Registration Number): DE-MF-000006373

declare under sole responsibility, that our products

Wrapping material interleaved

non-woven polypropylene SMMS blue 60g / non-woven green 67g – articles see annex
VP-code/quality **375_01**

Basic UDI-DI 40094453FITL375DA

are sterile barrier systems, and therefore accessories to medical devices intended by their manufacturer to be used terminally sterilized. As such, the products named above are registered as medical devices of **class I** according to annex VIII, chapter 3, rule 1 of the **Regulation (EU) 2017/745** and meet the general safety and performance requirements according to annex I of the **Regulation (EU) 2017/745**.

The conformity assessment procedure is carried out according to article 52 (7) of **Regulation (EU) 2017/745** by issuing the EU declaration of conformity referred to in Article 19 after drawing up the technical documentation set out in Annexes II and III.

This Declaration of Conformity will remain effective until 2 May 2027.

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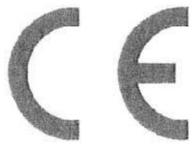
Thomas Lechner
Head of Management Systems VP-Group

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Technical data is submitted to the usual variations of operating procedures. Therefore it is not compulsory and does not exempt from validation and operating tests.

Tolerances which differ from this product specification including the shipping container and the labelling system have to be confirmed - after negotiation - in our article description.

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The packaging material complies with DIN EN ISO 11607 part 1 and DIN EN 868 part 2 sections 4.2.1 and 4.2.2.3.

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Wrapping material interleaved non-woven polypropylene

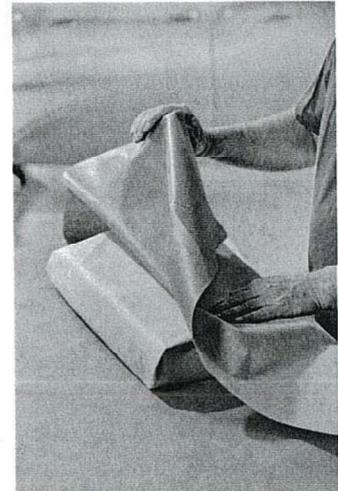
SMMS 60g / non-woven green 67g - ISO 11607, EN 868, CE

SMMS 60g is an extraordinarily tear proof and resilient non-woven material made of polypropylene fibres which is perfectly suitable for heavy duty applications. It is a four layered SMMS-material, which is made up of two layers of „spunbond-fibres“ and two layers of „meltblown-fibres“.

Non-woven 67g of green colour is a heavy duty material made from a mixture of plastic and pulp fibres. It is characterized by good drapability and an over-average mechanical strength. The fibre mixture supports complete drying of condensate after steam sterilisation.

Within this interleaved combination blue and green sheets are packed together. There is no need to combine from two different packs. The different colours of inner and outer layers make detection of damages of the final pack easier. This specific selection combines unsurpassed strength of the SMMS with the easy drying features of the 67g mixed fibres. Non-woven green 67g should be used as inner packaging, SMMS 60g as outer packaging.

Please follow the packing methods described in the DIN 58953 part 7 or other relevant international standards.

Interleaved non-woven polypropylene SMMS 60g / non-woven green 67g

Article-No.	Size in cm	Pack (pcs.)
3FITL375114	100 x 100	160
3FITL375118	120 x 120	120
3FITL375126	140 x 140	80

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