TENSMED S82

CE
0476

INSTRUCTIONS FOR USE

14. September 2020 | 1727751-51

www.enraf-nonius.com

PARTNER FOR LIFE

DEAR CUSTOMER

**THANK YOU FOR CHOOSING THIS PRODUCT. WE REMAIN AT YOUR ENTIRE
DISPOSAL FOR ANY ASSISTANCE OR ADVICE YOU MAY NEED**

The device has been manufactured in compliance with applicable technical standards and has been certified, in compliance with Directive 93/42/EEC as amended by directive 2007/47 on medical devices, by the Notified Body Kiwa Cermet Italia, Via Cadriano 23, 40057 Granarolo Dell'Emilia (BO) Italy (n. 0476), in order to ensure product safety.

Table of contents

TECHNICAL FEATURES.....	5
Device	5
Use conditions	5
Technical features of the currents NMES and TENS:.....	5
EQUIPMENT	6
INTENDED USE.....	6
CONNECTIONS	7
Accessories	7
LABELLING AND SYMBOLS	7
WARNINGS AND CONTRAINDICATIONS.....	10
Mandatory behavior	10
Warnings before use	11
Warnings during the use	11
SIDE EFFECTS AND CONTRAINDICATIONS	12
Contraindications	12
MAINTENANCE AND CLEANING.....	13
Maintenance and cleaning of the device	13
Accessories Use and storage of the electrodes and the cables.....	13
INSTRUCTIONS FOR USE	15
ELECTROSTIMULATION.....	18
Electrostimulation advantages.....	18
Mechanism of the muscular contraction	18
TENS.....	20
Incontinence programs.....	21
Warnings.....	21
Maintenance.....	21
PROGRAM LIST	22
TREATMENT DESCRIPTIONS	25
WARRANTY	27
FREQUENTLY ASKED QUESTIONS	28

TECHNICAL FEATURES

Device

Size:	130 X 80 X 20 mm
Weight:	220 g.
Case:	in ABS
Protection level:	IP 22
Storage and transportation temperature:	from -10°C to 45°C
Max. relative humidity:	30% - 75%
Medical device classification class:	Ila

The values indicate the limits allowed if the product or its accessories are not in the original package.

Made in PRC.

Charger

Tipo:	 Switching Power Adaptor
Model:	DK7-065-0200-EU
Input:	100-240Vac - 50-60Hz - 0,06 A
Output:	5,8 Vdc - 200 mA
Polarity:	
Battery pack:	Ni-MH AAA 4,8V 800 mAh
Made in PRC.	

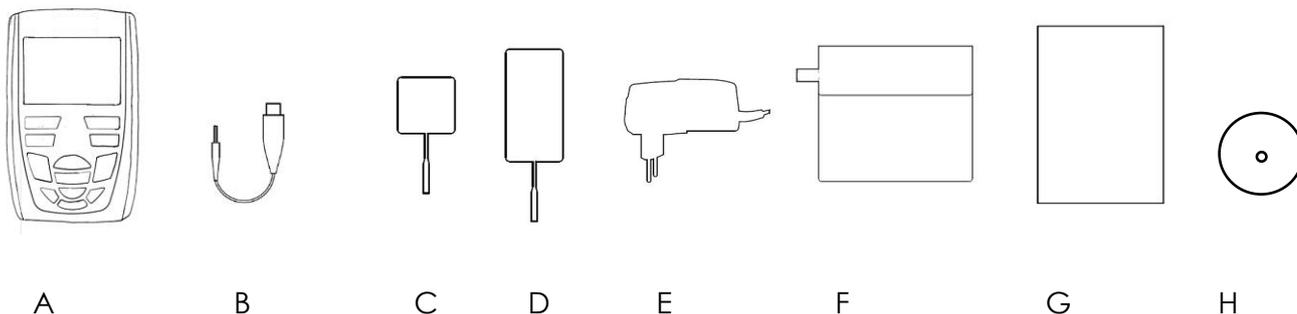
Use conditions

Temperature:	from 0°C to 35°C
Max. relative humidity:	from 15% to 93%
Atmospheric pressure:	from 700 hPa to 1060 hPa

Technical features of the currents NMES and TENS:

Channels available:	channel 1-2 2P
Constant current:	Yes
Intensity:	0-100 mA p.p. per channel
Wave form:	Rectangular, biphasic, symmetric, compensated
Working frequency:	1-150 Hz
Recovery frequency:	1-150 Hz
Pulse width (single phase):	50-400 μ s (total pulse time: 100-800 μ s)
Working time:	from 1 to 30 seconds <small>Type text here</small>
Recovery time:	from 0 to 1 minute
Frequency modulation range:	continuous variation from 1 to 150 Hz
Min. modulation time:	3 seconds
Pulse width modulation range:	continuous variation from 50 to 400 μ s
Min. Time:	1 minute
Max. Time:	100 minutes

EQUIPMENT

The electrostimulator is supplied complete of cables and electrodes to use: therefore, opening the package, it is necessary to check that the basic equipment is complete. If some elements should be missing, contact immediately the authorized retailer where you purchased the product. Control carefully the integrity of the device and its electrodes.

- A. Device
- B. 2 colored electrode connection cables (for NMES and TENS treatments)
- C. 1 bag containing 4 reusable self-adhesive electrodes (50 x 50 mm)
(It is suggested to use these electrodes for small areas such as upper limbs, calves, cervical areas...)
- D. 1 bag containing 4 reusable self-adhesive electrodes (50 x 90 mm).
(It is suggested to use these electrodes for big areas such as thighs, abdomen and glutei...).
- E. Charger (see technical features)
- F. Carrying bag
- G. User manual in English (printed)
- H. CD with user manual in English, German, French, Spanish, Italian

The device can be used with some optional accessories. For purchase information, contact your Enraf-Nonius dealer or visit www.enraf-nonius.com.

INTENDED USE

The after sales service is guaranteed for 5 years. We suggest having a check of the device every 2 years for the maintenance and to ensure the safety. The numbers of treatments depends on the battery charge.

The electrostimulators are designed to be used in the following operating environments:

- domestic environment;
- general pain treatments;
- beauty and sport purposes.

The Tensmed S82 is intended to be used, and shall only be used, by or under the supervision of professional users in physiotherapy, rehabilitation and adjacent areas.

It is important to read the following instructions carefully before using your Tensmed S82. The manufacturer cannot be held responsible for the results of using this device for any purposes other than described in these operating instructions.

Should the use of this device lead to an undesirable event such as death or serious injury to the user, the manufacturer and the competent authority of the Member State MUST be informed without delay!

CONNECTIONS

Accessories

Cable connections

One or two cables may be used depending on whether you have chosen to use one or two channels of the unit.

To connect the cable connector to the device, plug it into the appropriate inlet in the upper part of the unit.

The inlets are situated exactly under the correspondent channel (left or right). Insert the cable at the channel you want to use.

Electrode application

Take the electrodes from the original package; all new electrodes have a seal on the package. Be sure that the device is off. To start, connect the two cable plugs to the electrodes, then disconnect the electrodes from their position and apply them on the skin. To place the electrodes correctly, see the pictures included in this manual.

After the use, place the electrodes in their original position again.

ATTENTION: do not unplug the electrodes if the unit is working.

LABELLING AND SYMBOLS

	It refers to the manufacturer name and address
	Warning
	This symbol on your device indicates that it complies with the directives on medical devices (93/42/CEE 47/2007CEE). The number of the notified unit is 0476.
	It indicates that this is a II class device.

	It indicates that this device has type BF parts.
	WEEE symbol (Waste of Electrical and Electronic Equipment). Recycling symbol. The WEEE symbol used for this product indicates that the device may not be disposed of as a household product. Properly dispose of the product to help protect the environment. For more information on recycling this product, contact the local competent department, the household waste management company or the store in which the product was purchased.
	It indicates the optimal temperatures for the storage and transportation of the product.
	It indicates that the product has been produced respecting the directive 2011/65/EEC.
	It informs the operator that before using the device he must read the manual.
IP22	It indicates the water protection degree
	It informs the operator of a compulsory conduct
	It refers to the pressure of the environment where the device is stored
	It refers to the humidity of the environment where the device and its accessories are stored.
	It refers to the manufacturing date
	Use indoor

Device

Model: TENSMED S82

Electrostimulator Type: A3R
 Max Power: 4.8Vdc 150mA
 Output: 100mA (1000ohm)
 Battery Pack: 800mA 4.8Vdc

Power Input: $\sqrt{2}$ [®]
 Model: DK7-065-0200-EU
 Input: 100-240Vac 50-60Hz 0.06A
 Output: 5.8Vdc 200mA

 A3RWWYYXXXX

  0476

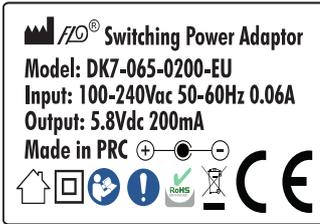
DOMINO s.r.l. - Via Vittorio Veneto, 52 31013 Codognè (TV) - IT
 Electrostimulatore - Electrostimulator - Electroestimulador
 Electrostimulateur - Elektostimulator

Distributed by
 Enraf-Nonius B.V.
 Vareseweg 127
 3047 AT Rotterdam
 The Netherlands


TENSMED S82
 REF 1727.901

Charger

Electrodes


	It refers to the production lot
	It indicates the product code
	It refers to product certification and indicates that complies with directive 93/42/CEE + 47/2007CEE.
	It indicates the storage temperature of the electrodes
	It refers to the expiry date

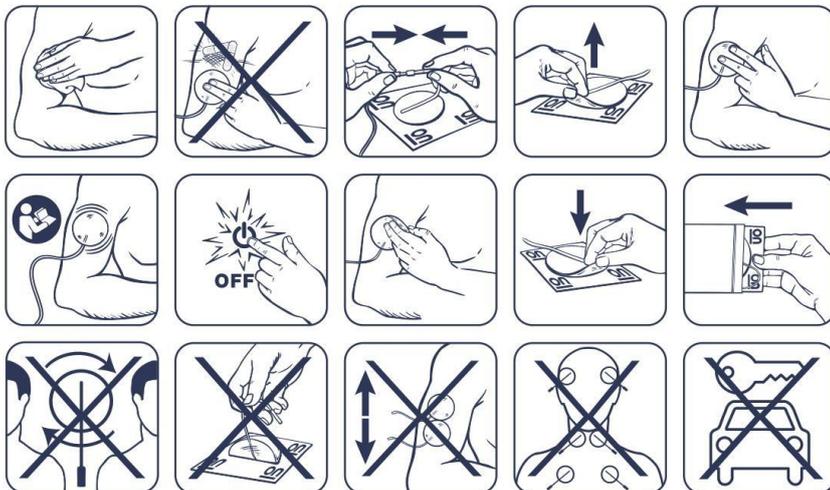
- Clean and degrease the skin.
- Do not apply the electrode on wounds or injured skin.
- Connect the cable connector to the electrode connector.
- Remove the electrode.
- Apply on the skin.
- Start the program.
- At the end, put the electrode back in the package.
- Electrodes are for personal use.

- Do not remove the electrode by grabbing the connector.
- Electrodes should not touch each other.
- Do not apply the electrodes on the temples, the neck and in a transthoracic way.
- Do not leave the electrodes in the car.

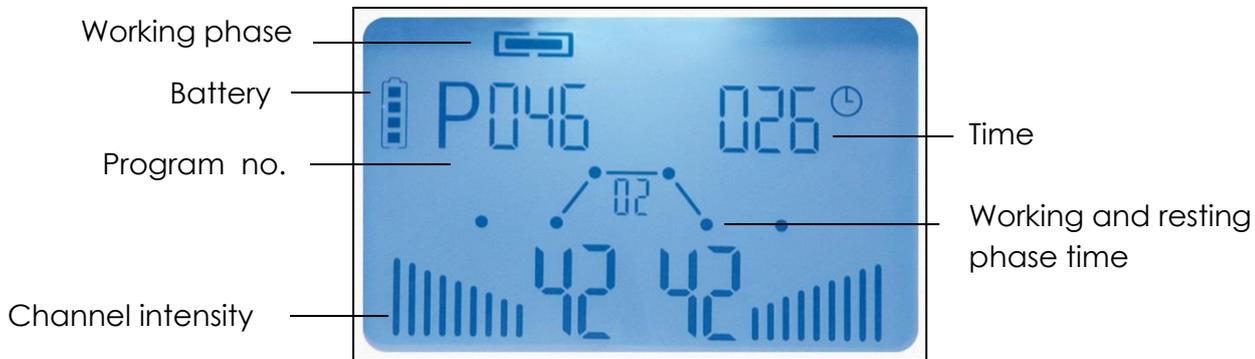Display

PANEL/KEYBOARD

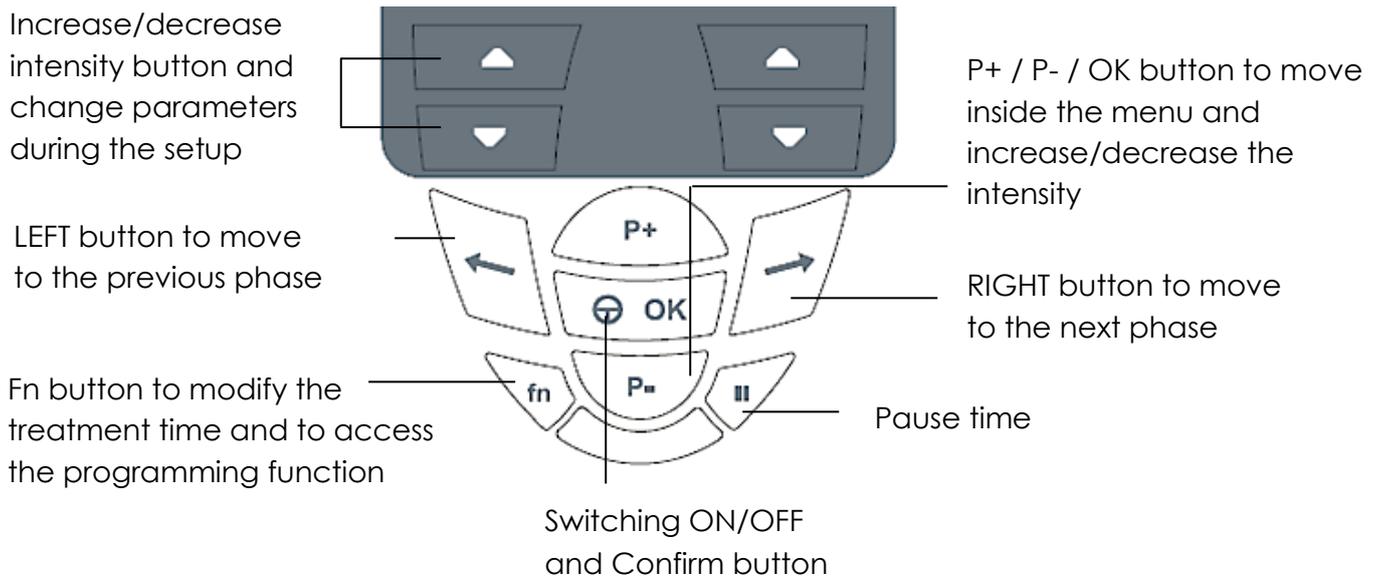WARNINGS AND CONTRAINDICATIONS

Mandatory behavior

For safety reasons, the device must be used in the prescribed manner and within the limits of use explained in this manual.

The manufacturer declines any responsibility due to a different use from what it is indicated in this manual.

No part of this manual (texts and photos) may be reproduced by electronic or mechanical manner without the prior written authorization of the manufacturer.

It is suggested not to carry out treatments in presence of skin wounds.

If the package, the cable or the connector of the power supply unit show signs of wear or damage, replace it instantly.

The unit should be connected to the mains by its power supply unit. Before starting the treatment, make sure that the power system specifications comply with the directives in force within your country. Make sure that the power supply unit will be in a comfortable position and that it will be easy to be removed.

Warnings before use

We suggest avoiding the use of the device together with other electronic devices, especially with those used to maintain vital functions. For a correct use of the Electromedical device, make reference to the tables attached. If it is necessary to use the device near or together with other devices, pay attention to its working.

- It is recommended to read carefully the entire operating manual before using the device; keep carefully this operating manual;
- The device can emit current value above 10mA_{rms};
- Before each use always check the integrity of the unit. This is a fundamental requirement for carrying out therapies, do not use the unit if either the buttons or cables are defective or malfunctioning;
- It must be used only by people over 18 who are able to understand and take action;
- It should not be used for purposes other than neuromuscular stimulation as described in this manual;
- It must be used following the indications and under the physician or physiotherapist's control;
- It must be used with the supplied electrodes intended for transcutaneous neuromuscular stimulation;
- It must be kept out of the reach of children;
- ECG monitoring devices may not operate properly when electrostimulation is working;
- Do not use the device in trans-thoracic modality because it may cause cardiac arrhythmia, imposing its frequency over the heart one. Do not stimulate the pectoral and dorsal muscles simultaneously;
- If there is any health problem, it must be used only after seeing a doctor;
- A simultaneous connection of the patient to a high frequency electrosurgery device can cause burns where the electrodes are placed and the electrostimulator can result damaged;
- After turning on the device, make sure that in the display the software version and the model of the device appear, it means that the device is working and it is ready to be used. If not, or in the display all the segments do not appear, turn it off and on again. If the problem persists, contact the service center and do not use the device;
- An unexpected switching-off of the device means that the battery has run-down. Charge it according to the instructions in the paragraph: how to charge the batteries.

Warnings during the use

While using the electrostimulator some warnings should be followed:

- In case of damaged cables, they must be replaced with original parts and no longer used;
- Use only the electrodes supplied by the company;
- The device must be kept out of the reach of any pet that could damage it and contaminate it with parasites;
- Pay particular attention when the current density for every electrode is above 2mA/cm² (effective value);

- The cables of the electrostimulation should not be wrapped around people's neck to avoid any risk of strangulation and suffocation;
- The mobile and fixed radio communications devices could influence the functioning of the electromedical device: see the tables attached to this manual. Preventative measures to take using the device for incontinence treatments.
- Patients with extra-urethral incontinence should not be treated with the electrostimulator;
- Patients suffering excessive incontinence due to evacuation problems should not be treated with the electrostimulator;
- Patients with severe urinary retention to the upper urinary passages should not be treated with the stimulator;
- Patients with total peripheral denervation of the pelvic floor should not be treated with the stimulator;
- Patients suffering of a total/subtotal prolapse of the uterus/vagina should be stimulated with extreme care;
- Patients with infections to the urinary passages should be treated for these symptoms before starting the stimulation treatment;
- Before removing or touching the probe it is necessary to turn off the stimulator or to regulate the intensity of both the channels on 0,0 mA;
- The treatment is a personalized medical prescription: do not borrow the stimulator to other persons.

SIDE EFFECTS AND CONTRAINDICATIONS

Isolated incidents of skin irritation may occur in subjects with high epidermal sensitiveness. In case of an allergic reaction to the electrode gel, suspend the treatment and contact a specialist. If during the treatment signs of tachycardia and extrasystole appear, suspend the treatment and contact your physician.

Contraindications

Do not use the device in the following cases:

- Stimulation of the front part of the neck (carotid sinus);
- Pacemaker weavers;
- Patients with tumor diseases (see your oncologist);
- Stimulation of the brain region;
- Pains whose etiology is unknown;
- Sores and dermatological diseases;
- Severe traumas;
- Stimulation on recent scars;
- Pregnancy;
- It is strictly forbidden to use the electrostimulator in the ocular area;
- Near body areas with metallic implants or infratissue metals (prostheses, osteosynthetic devices, coils, screws, orthopedic plates), when using monophasic current, interferential, or continuous current, ionophoresis.

It is recommended to use the device with caution in people with capillary fragility, as an excessive stimulation could cause capillary ruptures.

MAINTENANCE AND CLEANING

Maintenance and cleaning of the device

- In case of real or alleged malfunction, do not tamper with the device or try to repair it yourself. Do not intervene on the device and do not open it. Only specialized and authorized centers can repair it;
- Avoid violent impacts that may cause damage and malfunctions to the device even if undetectable immediately;
- Use this device in a dry environment and in an open space (not wrapped in any materials);
- Clean the device and accessories only with disinfectant with sodium hypochlorite or quaternary ammonium salt diluted with distilled water equal to 0,2-0,3%.
- After cleaning/disinfecting it, dry the device and its accessories with a clean cloth;
- Always use the device and its accessories with clean hands;
- It is recommended to use the device in a clean room, to avoid the contamination of the device with dust and dirt;
- It is recommended to use the device in a ventilated space, with regular air change;
- It is recommended to clean/disinfect the parts after every use.

Accessories

Use and storage of the electrodes and the cables.

After using the multi-purpose single patient and/or single-use electrodes, they must be stored using their plastic film and placed in the plastic bag.

Avoid that the electrodes touch each other or that they overlie one over the other.

Once the package has been opened, the electrodes can be used for 25-30 applications.

The electrode must be always used with clean hands and they must be replaced if they are not perfectly in contact with the skin.

If using non self-adhesive electrodes it is suggested to clean the surface with proper cleansers that respect the requirements described in the manual.

The electrodes must be stored in their bag and in an environment that respects the requirements described in the manual.

After the end of a treatment, unplug the cables from the connectors and clean them carefully with proper cleansers that respect the requirements described in the manual.

After cleaning and drying them, they must be folded up and placed in the plastic bags supplied with the cables.

Battery: how to charge the batteries

The device is supplied with a set of rechargeable nickel-metal hydrate batteries (4,8V 800mAh).

When in the display the battery to be charged icon appears, we suggest recharging the device, as after the appearance of this symbol the electrostimulator autonomy is limited and could not allow the execution of a complete program.

To charge the batteries, turn off the electrostimulator and disconnect the electrodes, then connect the electrostimulator to the charger provided by inserting the plug in the appropriate inlet.

We suggest charging the device for 8-12 hours to obtain the greater duration of the battery pack. A possible increase in the temperature in the battery area during the charge is normal. Do not use a different charger from the one provided with the device. To replace the electro-stimulator batteries, contact an authorized service center.

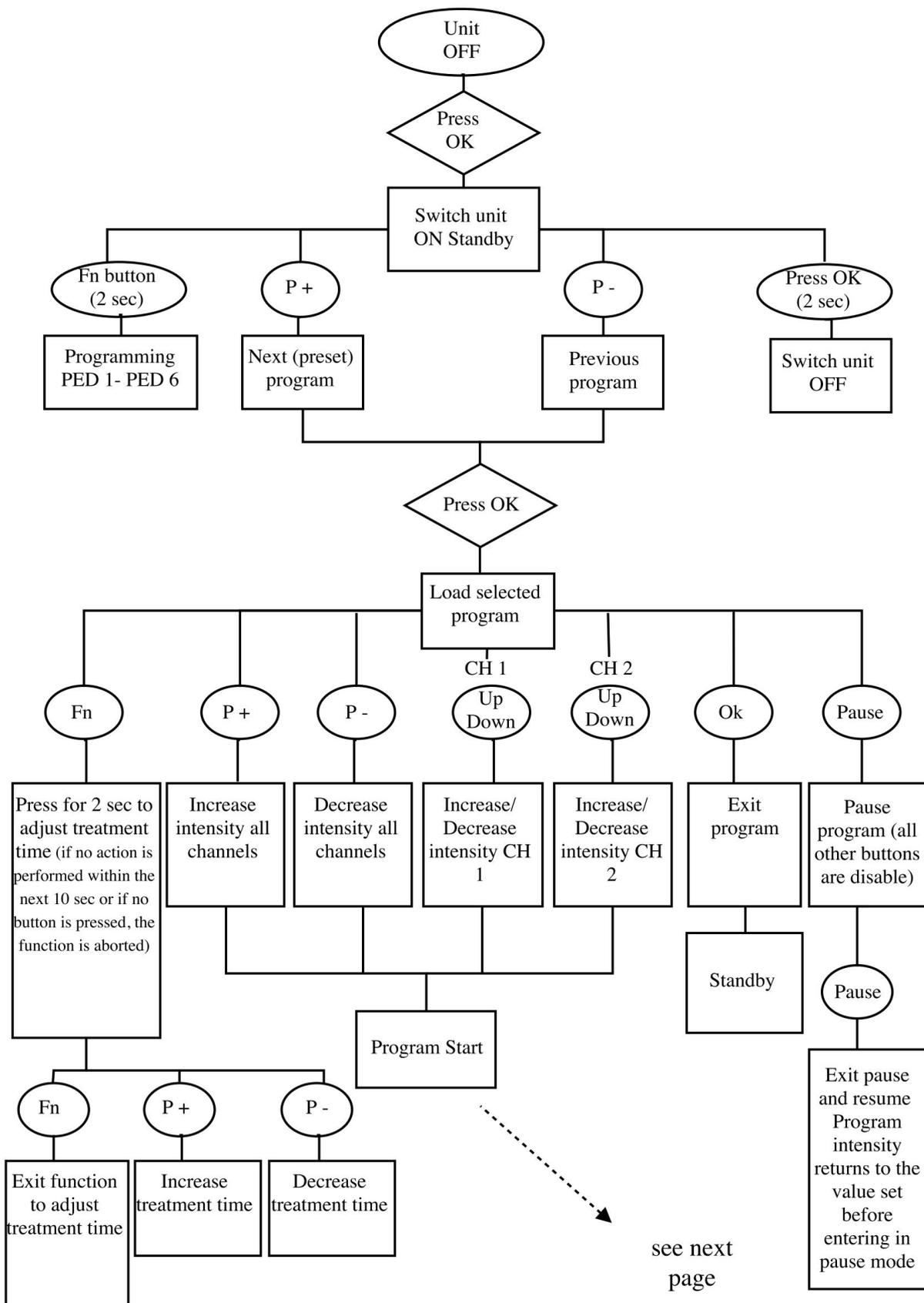
Disposal of the device

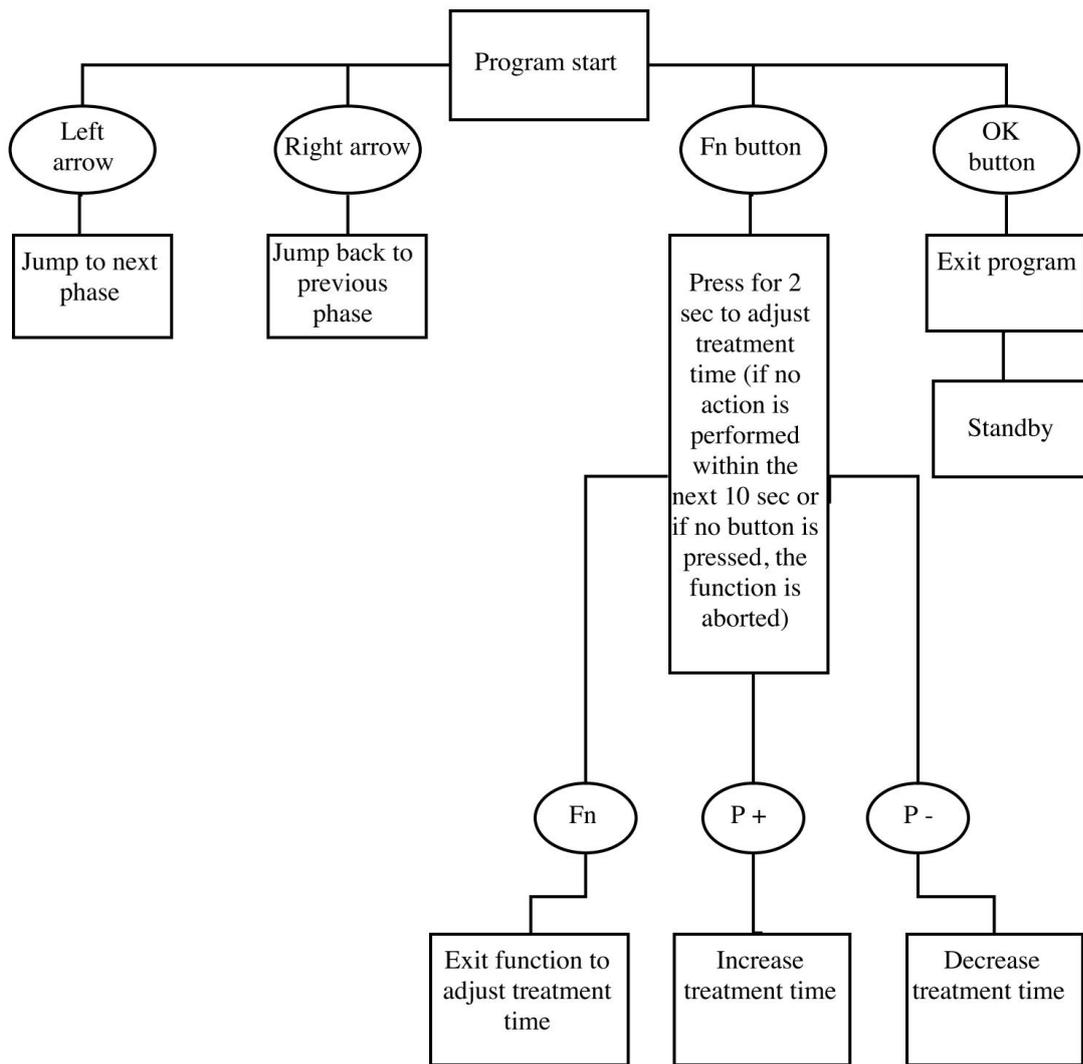
Do not throw the device or part of it in the fire, but dispose of the product in the specialized centers and respecting the directives in force within your country. When the product has to be disposed of, the user can give it back to the retailer when purchasing a new unit.

A correct separate waste collection or following what above mentioned contribute to avoid possible negative effects on environment and health and promote the reuse and/or recycle of materials of which the device is composed. The illegal disposal of the product by the user involves the application of the administrative fines according to the current regulations.

INSTRUCTIONS FOR USE

Programming

By pressing the Fn button for 2 seconds in the starting screen, you will have the possibility to program the device through 6 ways (PED1 – PED6).

To move from a parameter to the next one press OK.

PED1:

It is possible to set the values of frequency and pulse width.

Frequency: from 1 to 150 Hz in steps of 1 Hz.

Pulse width: from 50 to 250 μ s in steps of 10 μ s.

Default values: 100 Hz / 200 μ s.

Treatment time can be set from 5 to 100 min. in steps of 5 min. (default value: 20 min.)

PED2:

It is possible to regulate the pulse width with a frequency set at 100 Hz (fixed).

Pulse width: from 50 to 250 μ s in steps of 10 μ s.

Default value: 200 μ s.

Treatment time can be set from 5 to 100 min. in steps of 5 min. (default value: 20 min.)

PED3:

It is possible to set the frequency values.

Frequency: from 1 to 150 Hz in steps of 1 Hz.

Default value: 100 Hz

The pulse width is automatically modulated from 100 μ s to 250 μ s and can not be set.

Treatment time can be set from 5 to 100 min. in steps of 5 min. (default value: 20 min.)

PED4:

It is possible to set the values of frequency and pulse width.

Frequency: from 1 to 150 Hz in steps of 1 Hz.

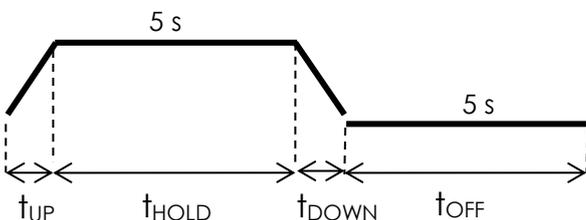
Pulse width: from 50 to 320 μ s in steps of 10 μ s.

Default values: 30 Hz / 250 μ s.

Treatment time can be set from 5 to 100 min. in steps of 5 min. (default value: 20 min.)

Work-rest program: $t_{UP} = 1$ s; $t_{HOLD} = 5$ s; $t_{DOWN} = 1$ s; $t_{OFF} = 5$ s.

The work-rest program is fixed and can not be modified.

PED5:

It is possible to set the frequency values.

Frequency: from 1 to 150 Hz in steps of 1 Hz.

Default value: 30 Hz

The pulse width is automatically modulated from 100 μ s to 320 μ s and can not be set.

Treatment time can be set from 5 to 100 min. in steps of 5 min. (default value: 20 min.)

PED6:

It is possible to set the values of t_{HOLD} e t_{OFF} .

From 1 to 30 s in steps of 1 s.

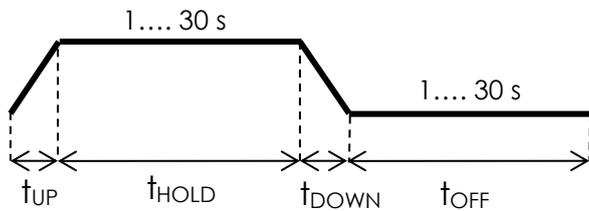
Time values for t_{UP} and t_{DOWN} are fixed (at 1 s) and can not be modified).

It is possible to set the frequency values from 1 to 150 Hz in steps of 1 Hz.

Value of the pulse width set on 300 μ s (fixed).

Default values: t_{HOLD} = 2 s; t_{OFF} = 10s; 30 Hz.

Treatment time can be set from 5 to 100 min. in steps of 5 min. (default value: 20 min.)

ELECTROSTIMULATION

Electrostimulation advantages

The electro stimulation does not want to replace physical activity but it must be considered as an integrating treatment.

According to the aims it will be useful:

- To the sportsmen/sportswomen, in order to integrate the normal training and to increase performances
- To who wants to intervene on some imperfections
- To who suffers from some pathologies
- To who has suffered from a trauma or in the rehabilitation.

The electro stimulation is a technique that provokes a muscular contraction completely similar to the volunteer one, using electrical TENS impulse. There are two different ways of use:

1. Muscular stimulation (ideal for the development of the force and for aesthetic treatments)
2. The stimulation to nervous ends (ideal for the treatments against the pain).

Types of muscle

The muscle can be subdivided in three different types: streaked or voluntary muscle; cardiac muscle and smooth or involuntary muscle.

The voluntary muscle includes the skeletal muscle that gives movement to the skeleton and the cutaneous muscle that acts on facial expressions.

The cardiac muscle and the smooth muscle are not voluntarily controlled. The most part of the muscles of the human body belongs to the category of streaked or voluntary muscles, with approximately 200 muscles for every side of the body (approximately 400 in total).

Skeletal muscles are the target of the NMES.

Mechanism of the muscular contraction

The skeletal muscle exercises its functions through the mechanism of the contraction.

When the muscular contraction happens, the movement of the articulations is produced and consequently the movement of the skeleton, too.

The muscle contracted in the following way: when a person decides to make a movement, an alteration of the electric potential in the motor center of the brain and an electric impulse is transmitted to the muscle that has to move.

The electrical impulse is transmitted as a change of potential (voltage); an electrical impulse crosses the motor nerve to the muscle that must be stimulated. After the reception of the impulse, Ca ions are freed by the reservoirs causing the approach of actin and myosin. The result is that the distance between the bands Z diminishes causing therefore the muscular contraction.

The energy demanded for the contraction is supplied by the supply of sugars and fats stored in the human body. In other words, the electrical stimulation is not a directed energy resource but it works as an instrument that triggers the muscular contraction. The same type of mechanism is activated when the muscular contraction is produced by the NMES.

They assume in other words the same role of a natural impulse transmitted by the motor nervous system. Normally the muscle relaxes and returns to its original state them at the end of the contraction.

Isotonic and isometric contractions

The isotonic contraction reveals when, in a movement of our body, the interested muscles produce a state of constant tension. The isometric contraction reveals when, in a movement of our body, the interested muscles produce a state of constant tension. Instead, when the joint heads of a physical segment are blocked and the musculature produces a tension, we talk about an isometric contraction.

In the event of the electro stimulation an isometric contraction is preferred because it fosters to obtain a more effective control.

The distribution of the different types of fibers in the muscle

The relationship between the two main categories (type I and the type II) can vary in sensitive way. There are muscular groups that are typically constituted by fibers of type I, like the soleus, and there are muscles that have only fibers of type II like the orbicular muscle. But in the most part of the cases we have a simultaneous presence of various types of fibers.

The studies lead on the distribution of fibers in the muscle have put in evidence the strait relationship that exists between the motoneuron (tonic or phasic) and the functional features of fibers from it innervate and have demonstrated as to a specific motor activity (and sports in particular) can determine a functional adaptation of the fibers and a modification of the metabolic features of the same ones.

Motor unit	Contraction type	Fiber type	Stimulation frequencies
Tonic	ST slow contraction	I a	10 - 50 Hz
Phasic	FTa fast contraction	II a	50 - 70 Hz
Phasic	FTb fast contraction	II b	80 - 120 Hz

In order to make a tissue pass from the phase of rest to the one of excitation, by means of an induced electrical stimulus (impulse of the electrostimulation), some conditions are necessary:

- the current amount
- the duration of the stimulus that must be adapted to the corporeal district that is wanted to be stimulated.

From this consideration, for the excitation of a tissue, the relationship between the duration and the intensity of the stimulus is particularly important and not only the value of the peak of the intensity. This relationship varies according to the muscular district.

Stimulation intensity

The current intensity necessary to obtain muscular contraction is personal and depends on the position of the electrodes, the underlying adipose tissue layer, sweating, the presence of hair on the area to be treated, etc.. For these reasons, the same current intensity can generate different feelings from person to person, from day to day, and from the right side to the left side of the body. During different sessions, it will be necessary to regulate the intensity in order to obtain the same level of contraction because of the accommodation phenomena. The current intensities recommended in the different phases are proposed as indicative values, and each person should modify these levels according to his/her personal needs.

- Moderate intensity: the muscle does not tire, not even during prolonged treatments. The contraction induced is tolerable and pleasant. This is the first level on the graphic representation of intensity.
- Intermediate intensity: the muscle is visibly contracted but the stimulation does not cause the joints to move. This is the second level on the graphic representation of intensity.
- Elevated intensity: the muscle is contracted substantially. The muscular contraction will cause the extension or bending of the limb if this is not blocked. This is the third level on the graphic representation of intensity.
- Maximum intensity: the muscle is contracted maximally. This is an intense treatment that should be performed only after having executing different applications at lower

Moderate		from 10 mA to 20 mA
Intermediate		from 20 mA to 30 mA
High		above 30 mA
Maximal		on the verge of the tolerance limit, always under the threshold of pain.

In the descriptions of the treatments, the best levels of intensities are recommended.

NOTE: The recommended levels of current are only indicative.

TENS

Transcutaneous Electrical Nerve Stimulation (TENS) is a selective stimulation of the large fibers of the peripheral nerves favoring the closing of the gate entrance for the pain pulses and increasing the release of endorphin substances, reducing in this way the pain intensity. Therefore with TENS we want to treat the severe and chronic pain due to the main musculoskeletal pains.

The pain decrease following to the TENS currents application is due to these factors:

- a. Gate control theory
- b. Endorphin secretion
- c. Different sedative effects in relation with the frequency

Gate theory

If the electric signals that lead to the brain information about pain are stopped, also the pain perception is eliminated. If, for instance, we hit our head into an object the first thing we do is massaging the area affected by the trauma. In this way we stimulate the receptors related to touch and pressure. TENS in continuous mode and in frequency modulation can be used to generate signals similar to the ones of touch and pressure. If their intensity is enough, their priority is so high that it prevails on the pain signals. Once the priority is gained, the gate related to the sensory signals is opened and the pain one is closed, impeding in this way the passage of these signals to the brain.

Endorphin secretion

When a nervous signal proceeds from the pain area to the brain, it spreads through a chain of connections joined together called synapse. The synapse can be seen as the space between the end of a nerve and the start of another. When an electric signal reaches the end of a nerve, it produces some substances called neurotransmitters that pass through the synapse and activate the start of the next nerve. This process repeats for all the length needed to the signal to reach the brain. The opioids involved in the pain reduction have the task to insinuate in the synapse space and impede the neurotransmitter propagation. In this way a chemical block of the pain signals occurs. The endorphins are opioids naturally produced by the body to tackle the pain and they can act both on the marrow and on the brain, in this way they are effective analgesics. Tens can increase the natural production of endorphins and, thereby, they act decreasing the pain perception.

Incontinence programs

The urology programs require the use of specific endovaginal and endorectal electrode probes, certified according to the Directive for Medical Devices 93/42/EEC. These are bipolar probes with a 2-mm female adapter which attaches to 2-mm male cables.

Warnings

Urological electrostimulation is a medical application, which must be carried out under medical supervision.

Use

To correctly use the probe electrode, follow the instructions provided by the manufacturer and given by the physician.

Maintenance

For cleaning, sterilization and disinfection, refer to the manufacturer's instructions.

Suggestions

In case of deterioration of the probe electrode, replace it immediately.

PROGRAM LIST

* Because of the presence of clinical programs, this product is a medical device. Therefore it is certified by the Kiwa Cermet Italia Body n. 0476 according to the 93/42/EEC EU directive for medical devices. The certification covers clinical applications.

CE0476 does not refer to non-medical treatments.

Name	Type	Tensmed S82
DEMO	SPORT	P 1
Motor point pen	SPORT	P 2
Warm-up lower limbs	SPORT	P 3
Pre-competition warm-up lower limbs	SPORT	P 4
Maximum strength lower limbs	SPORT	P 5
Endurance strength lower limbs	SPORT	P 6
Explosive strength lower limbs	SPORT	P 7
Reactivity lower limbs	SPORT	P 8
Capillarization lower limbs	SPORT	P 9
Active recovery lower limbs	SPORT	P 10
Muscle cool down lower limbs	SPORT	P 11
Decontracting lower limbs	SPORT	P 12
Leg anti-cellulite lipolysis	BEAUTY	P 13
Leg drainage	BEAUTY	P 14
Leg firming	BEAUTY	P 15
Leg toning	BEAUTY	P 16
Leg sculpting	BEAUTY	P 17
Abdom. area post-pregnancy drainage	BEAUTY	P 18
Abdom. area post-pregnancy lipolysis	BEAUTY	P 19
Abdominal area post-pregnancy toning	BEAUTY	P 20
Conventional antalgic Tens	TENS*	P 21
Endorphinic Tens	TENS*	P 22
Rotator cuff tendinitis	TENS*	P 23
Muscle pain	TENS*	P 24
Knee osteoarthritis	TENS*	P 25
Post-surgery pain	TENS*	P 26
Muscle injury	TENS*	P 27
Cervical pain	TENS*	P 28
Trapezius pain	TENS*	P 29
Sciatica	TENS*	P 30

Name	Type	Tensmed S82
Lumbago	TENS*	P 31
Epicondylitis	TENS*	P 32
Scapulo-humeral periarthritis (shoulder)	TENS*	P 33
Carpal tunnel	TENS*	P 34
Osteoarthritis	TENS*	P 35
Menstrual pain	TENS*	P 36
Ankle prevention	SPORT-PREVENTION	P 37
Knee prevention	SPORT-PREVENTION	P 38
Quadriceps muscle prevention	SPORT-PREVENTION	P 39
Lower limb reathletization	SPORT-PREVENTION	P 40
Shoulder reathletization	SPORT-PREVENTION	P 41
Shoulder prevention	SPORT-PREVENTION	P 42
Elbow prevention	SPORT-PREVENTION	P 43
Upper limb reathletization	SPORT-PREVENTION	P 44
Ankle muscle tone recovery	SPORT-PREVENTION	P 45
Vastus medialis reinforcement	REHAB*	P 46
Swollen ankles	REHAB*	P 47
Hemiplegia upper limbs	REHAB*	P 48
Hemiplegia lower limbs	REHAB*	P 49
Recovery after ACL surgery	REHAB*	P 50
Functional recovery lower limbs	REHAB*	P 51
Quadriceps atrophy with prosthesis	REHAB*	P 52
Shoulder subluxation prevention	REHAB*	P 53
Muscle spasms	REHAB*	P 54
Multiple sclerosis - Leg flexor extensor mm.	REHAB*	P 55
Multiple sclerosis - Ankle flexor extensor mm.	REHAB*	P 56
Mixed incontinence	INCONTINENCE*	P 57
Stress incontinence	INCONTINENCE*	P 58
Urge incontinence	INCONTINENCE*	P 59
	TOTAL	59
	SPORT	12
	BEAUTY	8
	TENS	16
	PREVENTION	9
	REHAB	11
	INCONTINENCE	3

GENERAL NOTES ON ELECTRODE POSITIONING

The correct electrode positioning and the correct choice of their size are critical aspects for the electrostimulation to be effective.

To choose the size of the electrodes and as for their positioning it is necessary to refer to the images at the end of this manual.

NOTE For all the programs that cause an important muscle contraction (such as, for example, strength, hypertrophy, toning and firming programs ...) it is important to position the electrode on the muscle **motor point**, that is the most sensitive point to stimulation.

If the electrode is not positioned exactly on the motor point, the contraction could be small and/or annoying. In this case it is necessary for the positive electrode to be moved a few millimeters up to feel an effective and comfortable contraction.

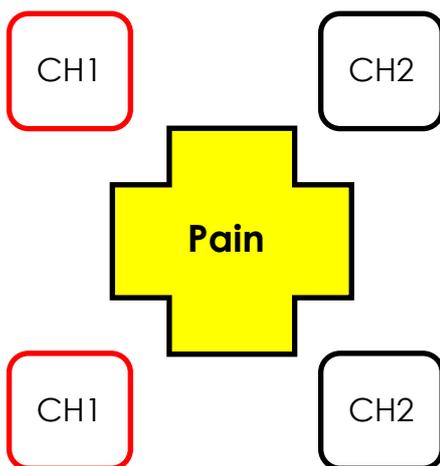
The body position during the stimulation

The body position during the electrostimulation session depends on the body part involved and on the program type that is being carried out. During the treatment execution with high intensities, we suggest blocking the limbs in order to work in isometry. For instance, if you want to treat the quadriceps with a strength program, we suggest carrying out the treatment while sitting with the foot blocked, in order to avoid an involuntary leg extension during the contraction phase.

For all the programs that do not imply high intensity (massages, decontracting, drainage programs) the body position is not important, as long as it is comfortable.

Electrode positioning for TENS programs

In the following pages of this manual you can find the images with the correct electrode positioning for tens treatments. If the localization of your pain is not included in the images represented, you can position the electrodes by forming a "square" on the painful area. Here's an example.

TREATMENT DESCRIPTIONS

Following is a short description of the programs provided by the electrostimulator.

SPORT

WARM-UP: Increases muscular temperature for better training preparation. Increases the blood flow to and metabolism of the muscle. Recommended before training sessions or competitions. Intensity: low or intermediate.

MAXIMUM STRENGTH: Improves maximal force capability and increases muscular mass. Allows training with maximum load and reduced trauma risks compared to classical training. Recommended for activities requiring great physical strength. Also useful during the early preparation period as it trains the muscles to receive significant stress.

Intensity: maximum, at pain threshold limit. During active Rest, adjust the intensity to obtain comfortable contractions.

ENDURANCE STRENGTH: Improves the ability to display a high level of force for an extended period of time and to resist toxin accumulation, thus delaying muscular fatigue. The program involves a long series of lengthy contractions, alternated with brief active rest. Recommended for sports requiring intense muscular work for an extended period of time.

Intensity: intermediate or maximum. Active Rest should induce muscular vibrations.

EXPLOSIVE STRENGTH: The Explosive Force program increases the ability to rapidly achieve maximum levels of force. The objective is to recruit the maximum number of muscle fibers in the shortest time possible. It is recommended to begin with a Maximal Force program to prepare the muscle for the Explosive Force program. Recommended in all sports requiring maximal force under natural load conditions, such as sprints, ball sports, and alpine skiing.

Intensity: maximum muscular contraction at pain threshold limit. Active Rest should induce muscular vibrations.

DECONTRACTING: Enables effective muscle relaxation thanks to the pleasant contractions that increase blood circulation. These programs can be used whenever muscular relaxation is needed. Intensity: low to intermediate.

CAPILLARIZATION: Increases the blood flow to the muscle and improves muscle resistance and recovery. An increased arterial capacity keeps the primary and secondary capillary network active, thus improving the tissue oxygenation system. This delays fatigue during very intense work. Capillarization programs are particularly recommended for sports requiring resistance force and endurance.

Intensity: intermediate.

POST-COMPETITION/TRAINING ACTIVE RECOVERY: Improves recovery from muscular fatigue after training or competitions and enables easier training resumption. The stimulation varies from muscular vibrations to light contractions. This allows greater circulation and reduced muscular hardening. It also alleviates pain thanks to the release of endorphins. This program is valid for all sports as it improves and speeds up the recovery of maximum muscular capacity. This program is recommended in the hours immediately after training or competitions.

Intensity: low to intermediate.

BEAUTY

DRAINAGE: The coordinated stimulation obtained by using all channels of the electrostimulator produces an in-depth muscular massage effect. The massage activates venous and lymphatic

systems by means of a mechanical effect. The purpose is to improve blood circulation and enable the toxin elimination. Intensity: low to inter- mediate.

LIPOLYSIS: The low frequency of this program positively affects the metabolism and increases the elimination of toxins and excess fluid. This is a method for reshaping critical areas, modifying the metabolism of lipocytes. Improves blood circulation. Helps to eliminate the orange peel skin. Intensity: low to intermediate.

VASCULARIZATION: Enables effective muscle relaxation thanks to the pleasant contractions that increase blood circulation. These programs can be used whenever muscular relaxation is needed. Intensity: intermediate.

FIRMING: This program is suitable for sedentary individuals with flaccid muscles that require reactivation of muscular tone. It prepares the muscles for more intense stimulations. It can be performed every day on large muscle groups such as thighs, abdominal and dorsal muscles. Intensity: intermediate to high.

TONING: The toning program allows to develop and maintain muscular tone. This type of treatment may be combined with aerobic activity and is suitable for users of any level. The program is especially directed at all those who, mindful of their own physical condition, wish to integrate their own regular training. The program can be used up to three times a week. Intensity: intermediate to high.

SCULPTING: The sculpting program is a merely aesthetic treatment and is ideal for those who wish to better define their already toned muscle system. The work performed is very intense, so slight muscle soreness may be experienced the day after the treatment. However, this effect may be avoided by performing stretching exercises before and after the session or a winding- down program. The program is suitable for those who wish to achieve muscular definition in those areas where traditional muscular development exercises are insufficient. Intensity: inter- mediate to high.

MASS BUILDING: A specific program for those who wish to increase muscle mass and volume. Intensity: maximum muscular contraction at pain threshold limit.

A muscle warm-up program should be carried out before performing any of the following programs: Firming, Toning, Sculpting and Mass Development.

TENS

Uses mechanisms of the nervous system to alleviate both acute and chronic pain. Impulses travel along the nerves to block the pain propagation paths. This analgesic effect occurs during program execution and can last for a few hours. However, its duration may vary. The TENS programs also improves circulation. Place the electrodes on the painful area. These programs can be used in any situation requiring pain relief.

Caution! TENS programs alleviate the pain but do not eliminate the cause. If the pain persists, seek medical advice.

TENS (Antalgic-Endorphinic): Allows pain relief and effective muscle relaxation. TENS programs use mechanisms of the nervous system to alleviate both acute and chronic pain. Impulses travel along the nerves to block the pain propagation paths. Muscular relaxation improves blood circulation. It can be used in any situation requiring both pain relief and muscle relaxation. *Caution!* TENS programs alleviate the pain but do not eliminate the cause. If the pain persists, seek medical advice.

Intensity: ANALGIC TENS: tingling but not painful sensation. Should not induce muscular contractions. ENDORPHINIC TENS: visible muscular vibrations.

TENS (Cervical pains, Lumbago, Periarthritis, Epicondylitis, Sciatica, ...): Uses mechanisms of the nervous system to alleviate both acute and chronic pain. Impulses travel along the nerves to block the pain propagation paths. This program combines stimulation with light contractions that stimulate the release of endorphins.

Caution! TENS programs alleviate the pain but do not eliminate the cause. If the pain persists, seek medical advice.

Intensity: tingling but not painful sensation.

REHAB

VASTUS MEDIALIS REINFORCEMENT AND RECOVERY AFTER ACL SURGERY: Resumption of physical activity after a period of interruption. After a trauma or other interruptions of physical activity, muscles may lose their tone and mass. In these cases, normal muscular capacity must be regained as soon as possible. The program enables training of specific muscles concerned.

It can be used in any situation requiring activity resumption.

Intensity: low to intermediate.

PREVENTION PROGRAMS: The stimulation is aimed at neuromuscular facilitation, which especially improves the mobilization of the injured articulation. Prepares the limb to resume its normal functionality. It is recommended to use this program during the first phases of the recovery of muscular trophism.

Intensity: intermediate.

RE-ATHLETIZATION: Increases the diameter and capacity of muscle fibers.

This program is recommended after using the Atrophy program for a period, as soon as the muscle shows a slight recovery of volume and tone.

Intensity: Intermediate.

HEMIPLEGIA: If the patient suffers from a perception problem with decreased attention (semi-inattention), integrating the treatment with training of this ability may be extremely useful. The treatment must not be merely passive, but the patient can take on an active role in the training. The program can be used for the facilitation and rehabilitation of motor faculties as well as a method to reduce spasticity.

Intensity: depending on the muscle district to be stimulated.

INCONTINENCE: Programs recommended for the stimulation of perianal muscles.

These programs are recommended in case of urological pain or incontinence.

Programs to be used with probes in compliance with the law.

WARRANTY

The device is guaranteed to the first user for a period of twenty-four (24) months from the date of purchase against defects in materials or of the manufacturing (if the user uses the device for professional purposes: warranty is twelve, 12, months), provided that it is used properly and maintained under normal operating conditions.

Warranty coverage is limited in the following cases:

- Six (6) months for supplied accessories such as batteries, chargers, power supplies, cables, etc.;
- No warranty for extendable accessories and materials such as, for example, electrodes, etc.

The warranty is valid and enforceable in the country where the product was purchased. In the event that the product is purchased in any country of the European Community, the warranty is still valid in all its countries.

In order to take advantage of the warranty service, the user must comply with the following warranty clauses:

1. The products, and all accessories, to be repaired must be sent by and at the expenses of the customer in their original packages.
2. The product's warranty is subject to the production of a fiscal document (fiscal receipt, receipted bill or sale invoice), attesting the product's purchase date.
3. The repair work shall have no effect on the original warranty expiry date and shall neither renew nor extend it.
4. If, during the repair work, no defects are found, the costs related to inspection times shall in any case be charged.
5. The warranty becomes void where the fault has been caused by: impacts, falls, erroneous or improper use of the product, use of non-original power supply unit or charger, accidental events, alteration, replacement/detachment of the warranty seals and/or tampering with the product. The warranty does not cover damages caused during transportation when unsuitable packages are used.
6. The warranty does not cover the inability to use the product, other incidental or consequent costs or other expenses incurred by the purchaser.

The features and dimensions reported in this manual are not binding.

The manufacturer reserves the right to make changes without previous notice.

FREQUENTLY ASKED QUESTIONS

Why, when I increase the current intensity above 1 mA, it returns immediately to 0 mA?

Before starting any program, check that the cables are connected and that the electrodes are placed on the area to be treated. In addition, be sure that the electrodes are not worn, because this could inhibit their ability to conduct current.

What kind of electrodes should be used for electrostimulation?

Use self-adhesive electrodes, which are practical and improve the quality of stimulation. If used with care, they will last for 25-30 applications. The electrodes should be replaced when they have no longer a good connection with the skin.

Where should the electrodes be placed?

In the back of this manual, there is a comprehensive electrode placement guide (it is not necessary to respect the polarities indicated). You can follow these instructions. To verify the correct placement of the electrodes, use the special Find Motor Point Pen program or follow this empirical method: place the electrodes as indicated in the pictures in the back of this manual; start the stimulation; with your hand, move the electrode by sliding it along the muscle without removing it from the skin. You will notice a change in contraction according to the different positions. Once you locate the point where the stimulation is greatest, decrease the channel intensity to zero (0,0 mA), replace the electrode and increase gradually the intensity.

Use of Y cables. This permits to use more electrodes on the same channel.

This permits, for instance, to stimulate vastus medialis and vastus lateralis of the quadriceps with one single channel. They are not recommended for medical applications.

Does the power decrease using Y cables?

The power intensity for each channel does not vary. However, when Y cables are used to split one single channel in two, the current is distributed on a wider muscle area, therefore contraction will be less pronounced. Increase the intensity to obtain the same contraction level.

Can electrostimulation hurt me?

It is very unlikely that electrostimulation damages muscles. In this case it is important to increase intensity gradually while observing the muscle behavior and avoiding to keep the limb completely outstretched. In case of doubt, please contact a specialist.

Is it possible to use the electrostimulator during menstruation cycle?

Electrostimulation may interfere in some way with menstruation, causing anticipation, delay, accentuation or reduction of the cycle; however, these effects are subjective and highly variable. It is recommended to avoid treatments in the abdominal zone during menstruation cycle and immediately before or after it.

Is it possible to use the electrostimulator during lactation?

Until now, no collateral effects regarding lactation have been observed. Nonetheless, during lactation, it is recommended not to stimulate the thoracic region.

Are dermatological diseases (e.g. psoriasis, urticaria) contraindications for electrostimulation?

Yes, do not treat areas affected by dermatological diseases.

When are the first results visible?

Aesthetic results of electrostimulation are always subjective. For Toning program, with a regular program of 3-4 sessions per week, a noticeable result may be observed after 15 days. For Lipolysis and Drainage programs, 40 days of treatment are necessary. Results are obtained more quickly if treatments are combined with good physical activity and a correct life style.

How many sessions can I weekly perform?

For fitness and aesthetics programs, the number of sessions depends on the type of treatment: 3-4 sessions per week on alternate days are suggested for toning; daily treatments are permitted for Lipolysis and Drainage programs.

ELECTRODE PLACEMENT FOR NMES TREATMENTS

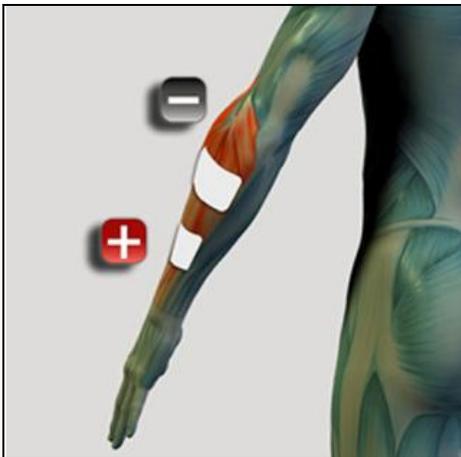
Biceps brachii muscle


Triceps brachii muscle


Flexor carpi muscle


Extensor carpi muscle

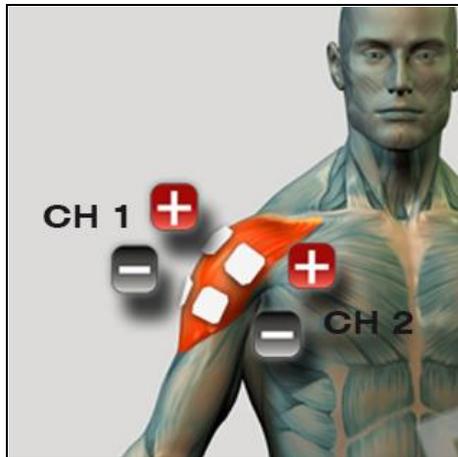
Deltoid muscle

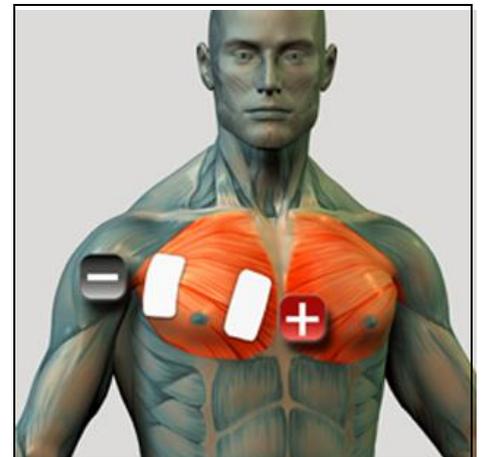
Pectoral muscle

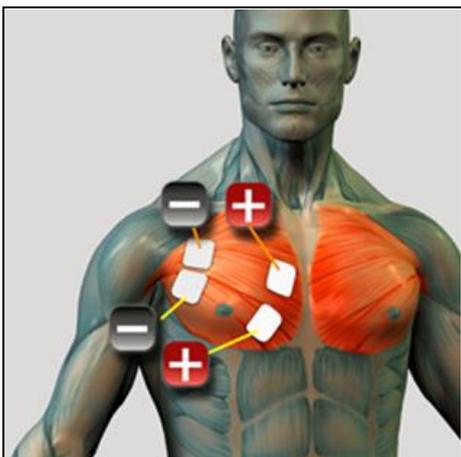Pectoral muscle

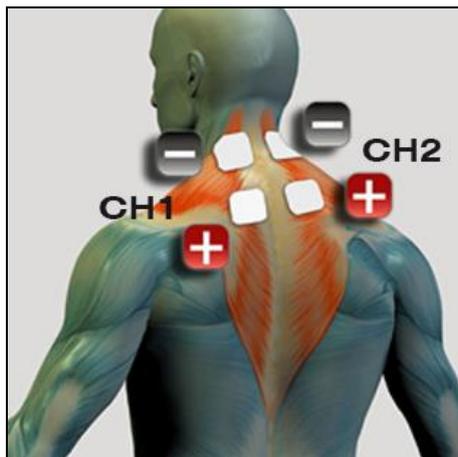
Trapezius muscle

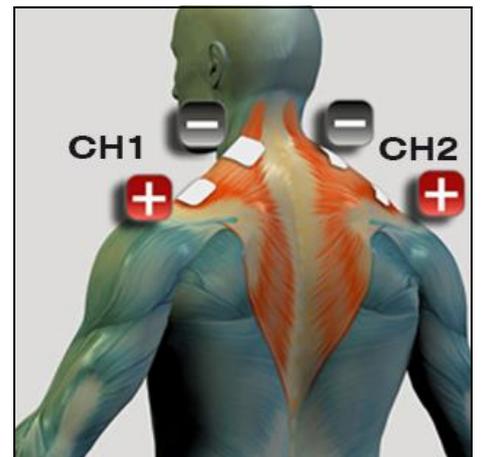
Trapezius muscle

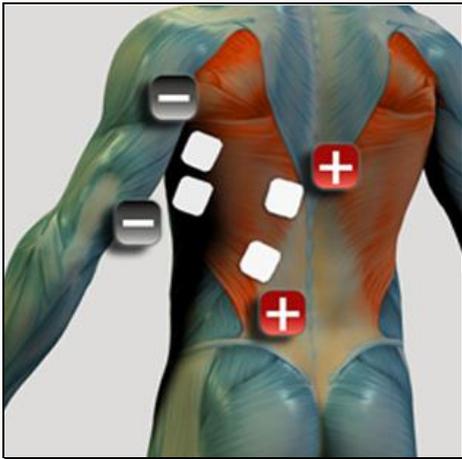
ELECTRODE PLACEMENT FOR NMES TREATMENTS

Latissimus dorsi muscle

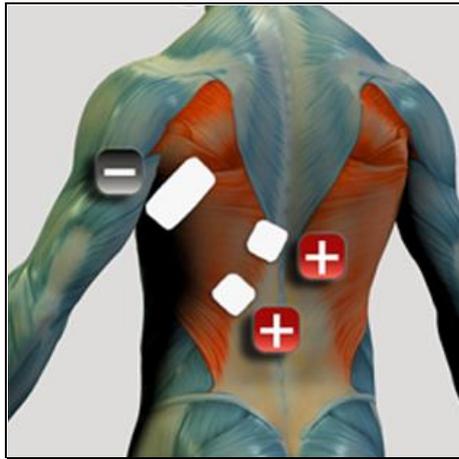
Latissimus dorsi muscle

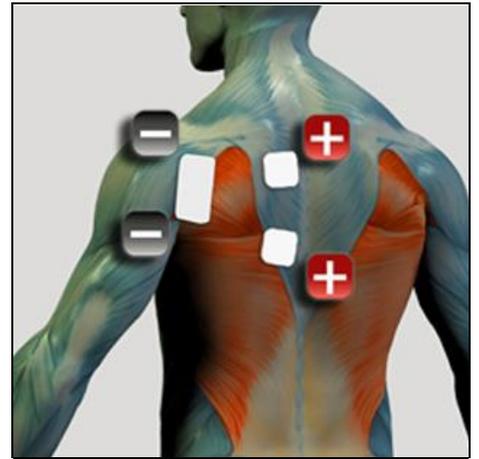
Infraspinatus muscle

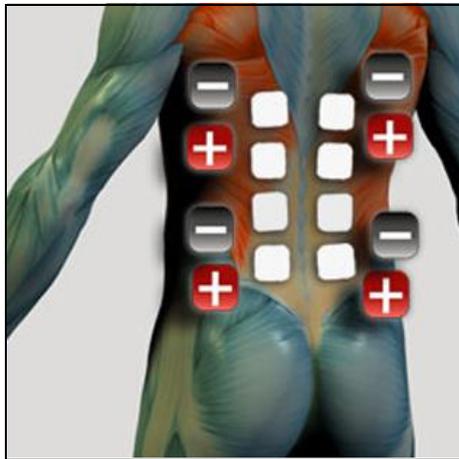
Lumbar muscles


Lumbar/Dorsal muscles

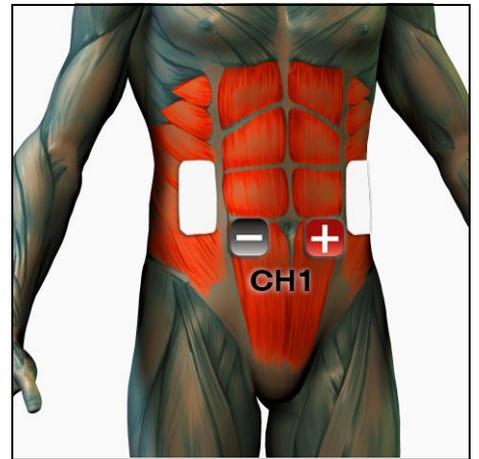
Abdominals

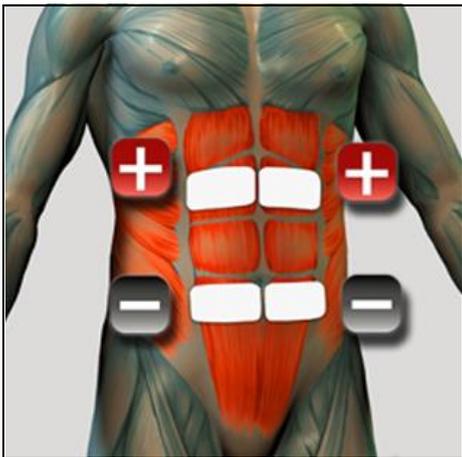Abdominals

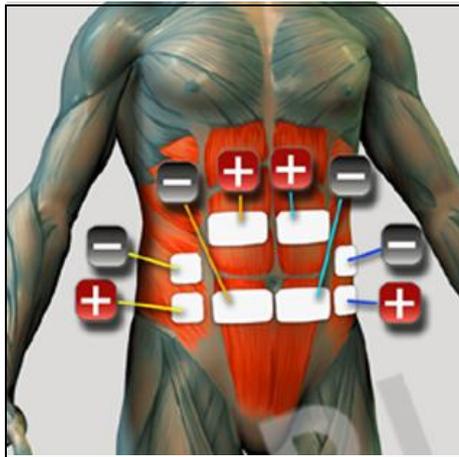
Abdominals

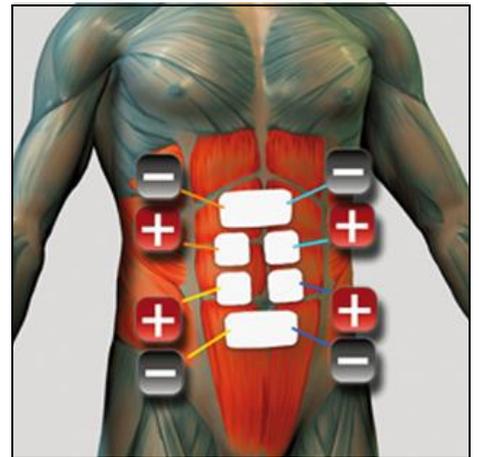
Rectus abdominis muscle

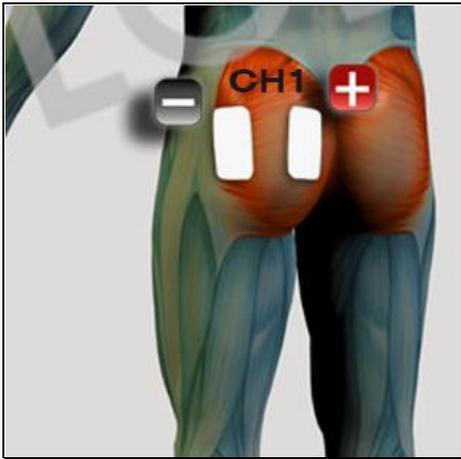ELECTRODE PLACEMENT FOR NMES TREATMENTS


Gluteus

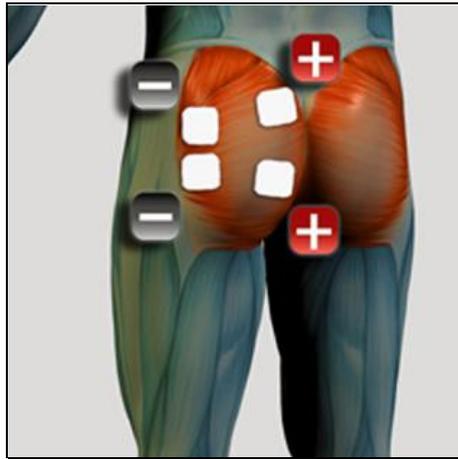
Gluteus

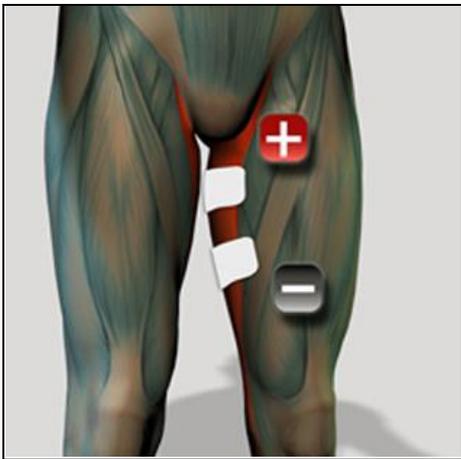
Biceps femoris muscle


Adductors

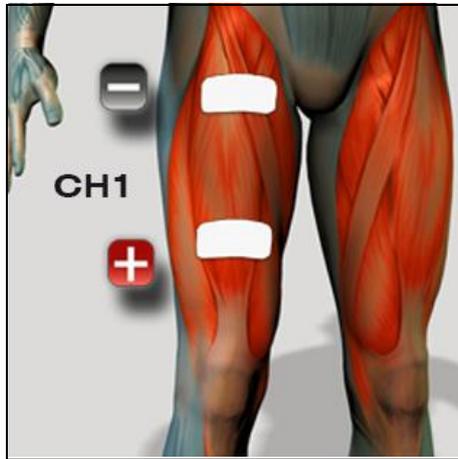
Rectus femoris muscle

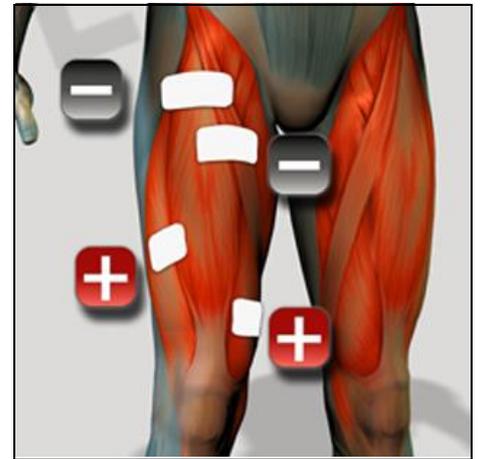
Quadriceps

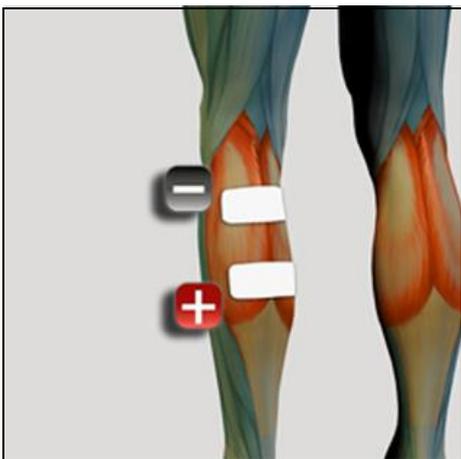Gastrocnemius muscle

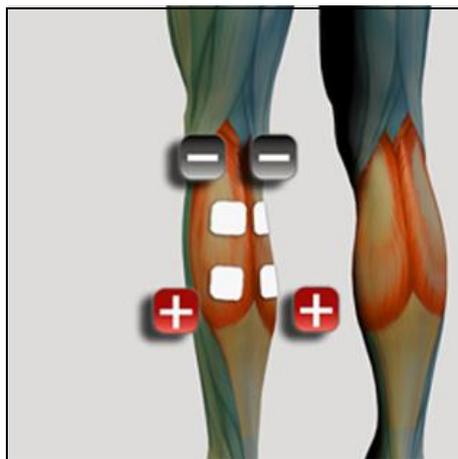
Gastrocnemius muscle

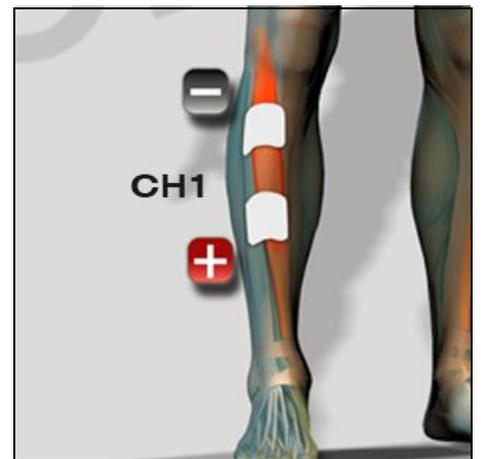
Tibialis anterior muscle

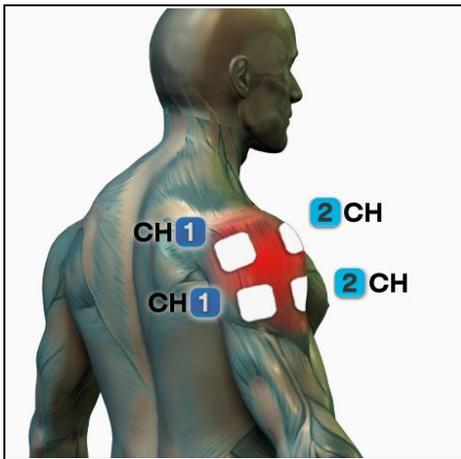
ELECTRODE PLACEMENT FOR TENS TREATMENTS


Shoulder pain

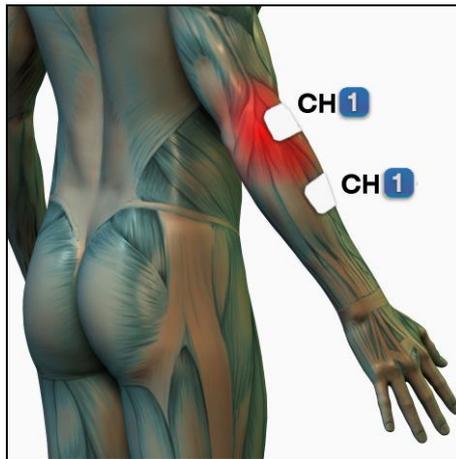
Elbow pain

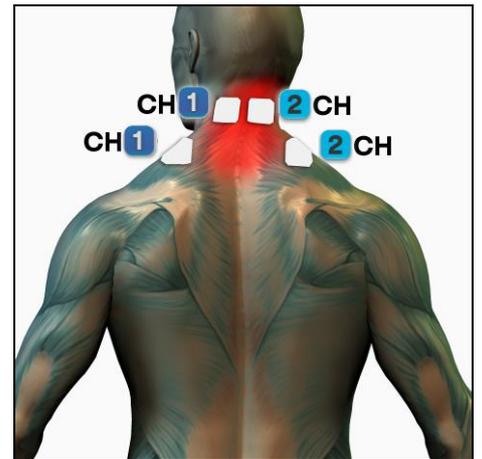
Cervical pain

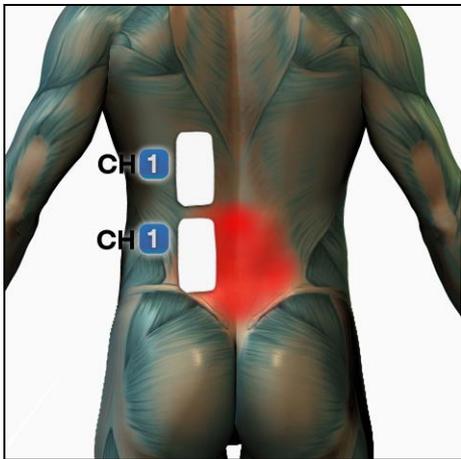
Lumbar pain

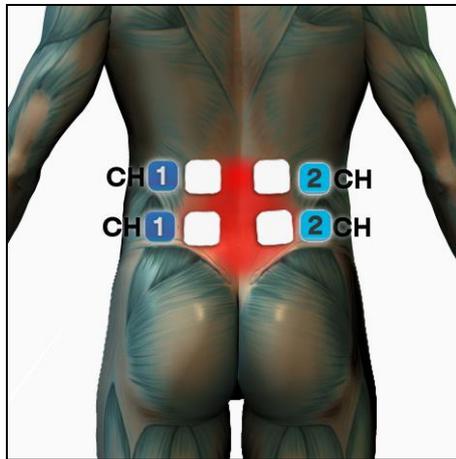
Lumbar pain

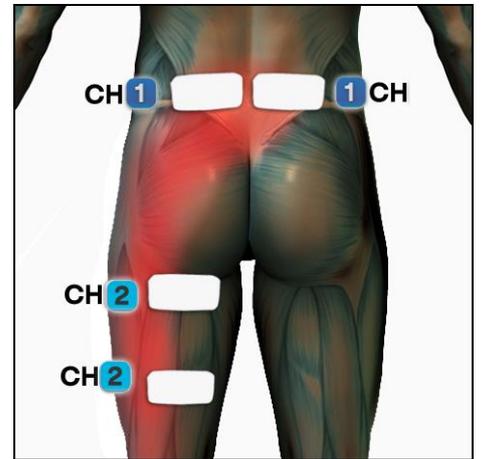Sciatica

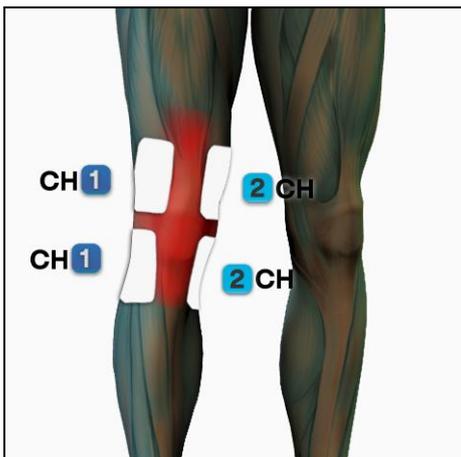Knee pain

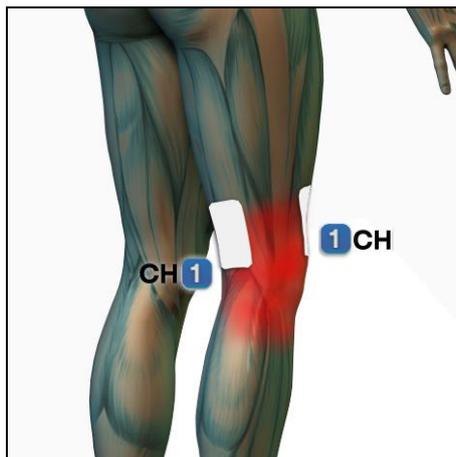Knee pain

Knee pain

EMC ACCOMPANYING DOCUMENTS

Essential performance

PERFORMANCE	CONDITION	RISK	ACCEPTED EVENT
Electrostimulation.	External disturbance (Burst).	Display information no longer readable.	The machine must stop the stimulation.
			The machine must maintain the stimulation and accept the commands.
	Lack of internal power supply.	Interruption of the treatment.	The machine must signal the battery exhaustion and the interruption of the treatment.
	Lack of external power supply.	Interruption of the treatment.	The machine must not work when connected to the mains.
	Detachment of an electrode.	Unpleasant stimulation or painful electric shock in case of reconnection of the electrode.	The machine must monitor the current constantly on each active channel. In case the detected current is below a certain threshold, the machine must rest the current of the channel.
Setting of a current that is too high.	Dangerous stimulation.	The machine must always display with the same scale the current level set. The set level must stay on display.	
		The machine that does not detect the contact of the electrode must reset the current of the channel and indicate the resetting on the display.	
Loading of the programs from the memory.	Error in the data from the memory.	Execution of an incorrect program.	The machine must check the correctness of the data of the programs. In case an error is detected, the device must restart. The user can try a new execution of the program. If the error persists, the restart will be done again avoiding the relaunch of the program in which the errors are detected.
Battery charge.	Battery overheating.	Damaging of the device, display information no longer readable, explosion, fire.	The device must limit the battery charging current to avoid overheating.
			The device must be provided with fuses at the connector input for the battery charger and into the battery pack to prevent dangerous current.

In compliance with:

EN 60601-1: 2006 + A1: 2013

EN 60601-1-2: 2015

EN 60601-2-10: 2015

EN 60601-1-11: 2015

Warning: radiofrequency communication devices (including accessories like antennas or antenna cables) must be used at least 3 meters away from every part (including cables and accessories) of the electrostimulation device. Otherwise the performance of the electrostimulation device can be affected.

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Small in size,
great in performance

TENSMED S82

The TENSMED S82 is the ultimate choice for the professional treatment of muscle stimulation and pain relief.

With this device you can treat chronic pain, patients recovering from a traumatic condition, immobilisation, and improve posture and stabilisation of the knee joint using TENS (Transcutaneous Electrical Nerve Stimulation). TENS treatments are safe, have no side-effects and can be an excellent alternative for pain medication.

The TENSMED S82 is a 2-channel electrostimulator with a clearly legible LCD screen. **10p**

65 Programs **3p**

- 12 Sport
- 8 Fitness, Beauty & Wellness
- 16 Pain (TENS)
- 9 Prevention
- 11 Rehabilitation
- 3 Incontinence
- 6 Free programs

PRO
PROGRAMMABLE

PRESET

FAST
MENU

SyS
Synchro Stim

Special features

PROGRAMMABLE	6 free memory positions to create customized programs
PRESET PROGRAMS	59 specific programs for functional rehabilitation and pain
FAST MENU	makes navigation faster
SYNCHRO-STIM	increase the intensity of the 2 channels simultaneously using only one button.
BACKLIT DISPLAY	the clearly legible LCD display contributes to optimal usability and ease-of-operation
BATTERY OPERATED	

TENSMED S82 programs

Rehabilitation

Muscle stimulation programs are particularly indicated for the functional recovery after surgical articulation interventions and after immobilization time. Programs included for Vastus Medialis Reinforcement and Recovery after ACL Surgery, Prevention Programs, Re-Athletization and Hemiplegia.

Pain

Eliminate pain with the special pulses of the TENS currents that, stimulating your body in a soft way, enables you to recover your health, wellness and a renovated life enjoyment. Programs included for Acute and Chronic Pain, Cervical pain, Lumbago, Periarthritis, Epicondylitis, Sciatica and soft tissue injuries.

Incontinence

Specific programs for the treatment of urological disorders such as stress, urge and mixed incontinence.

TENSMED S82 programs

Sport

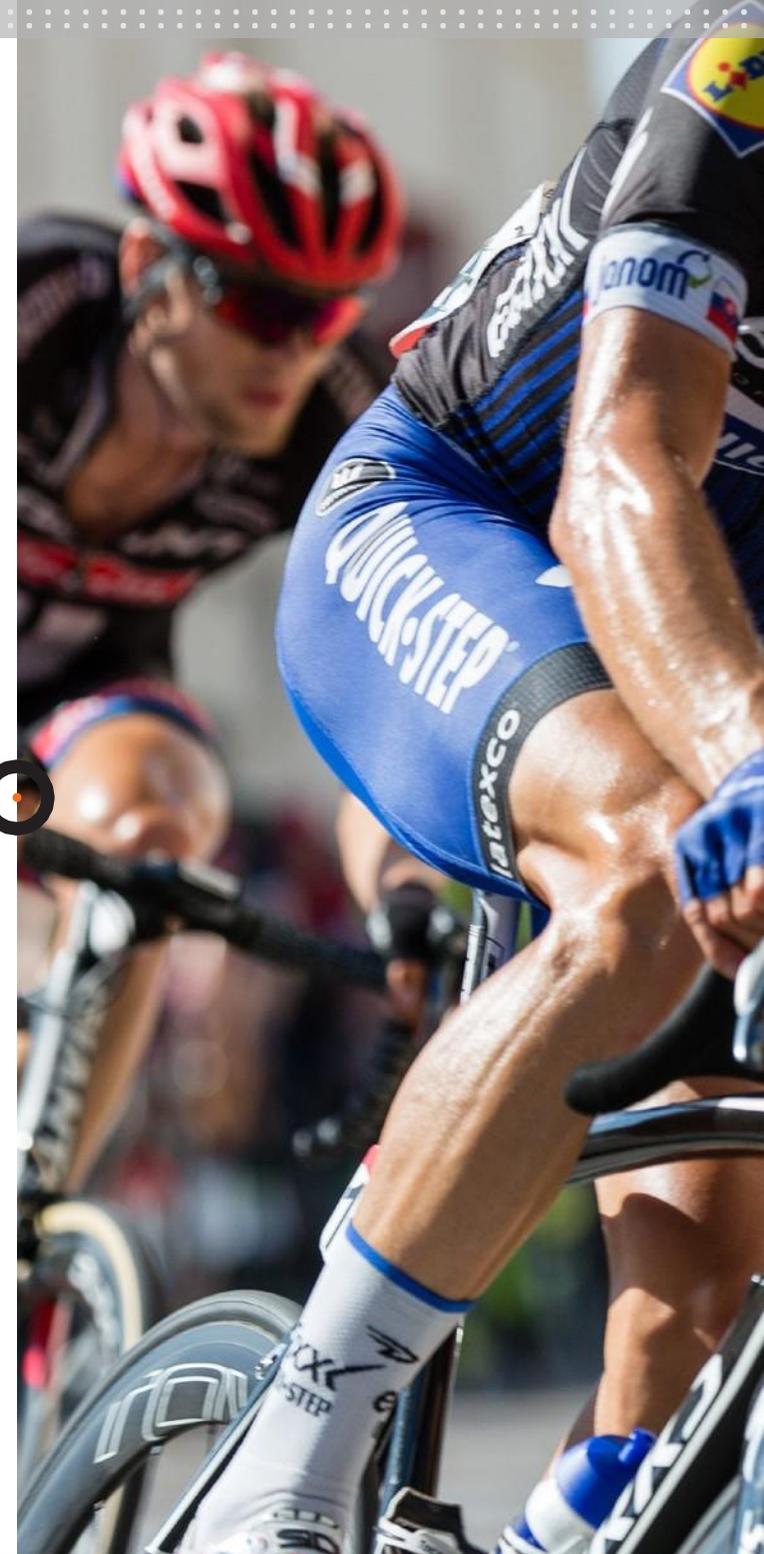
The TENSMED S82 is programmed in order to selectively stimulate fast and slow fibers to train different muscle qualities. Includes programs for Warm-up, Maximum strength, Endurance strength, Explosive strength, De-contracting (muscle relaxation), Capillarization and Post-competition/training Active Recovery.

Fitness, Beauty & Wellness

Body beauty is the expression of a wellness state and a form balance that we have to value and defend from the attacks of time and sedentary life. With the TENSMED S82 you can care for your body and defend its vitality and beauty using the electrostimulation beauty treatments. Ideal for Drainage, Lipolysis, Vascularization, Firming, Toning, Sculpting and Mass Building.

Prevention

Programs to reinforce main articulations (such as shoulder, knee and hip) and to prevent traumas and injuries.

TENSMED S82

ORDERING DATA

1727901 Tensmed S82

Standard accessories

Carrying bag

2 Cables for electrode connection

4 Self-adhesive square electrodes (50x50mm)

4 Self-adhesive rectangular electrodes (50x90mm)

Charger

User manual

TENSMED S82

TECHNICAL SPECIFICATIONS

Numbers of channels:	2 non-independent channels
Intensity:	0-100 mA per channel
Wave form:	symmetrical biphasic pulse, 100% compensated
Constant current:	up to a resistance of 1000 ohm
Frequency:	1 – 150 Hz
Pulse width (single phase):	50-400 μ s 7p
Total pulse width	100-800 μ s
Number of pre-set programs:	59
Numbers of personal programs:	6
Stimulation forms:	Conventional / continuous, Burst, Frequency Modulation, Pulse Width Modulation, Intermittent
Timer:	max. 100 min
Power supply:	Mains or battery
Battery:	Rechargeable battery (NI-MH / 800mAh / 4.8V)
Charger:	Input: 100-240VAC, 50-60Hz, 0,32 max. Output: 6,5V ----200mA
Dimensions:	130x80x20 mm
Weight with battery :	220 g
Weight without battery*	170 g
Backlit display	

This is a medical device.

It complies with all applicable medical regulations. For more details please consult the instructions for use (downloadable via www.enraf-nonus.com).

0476

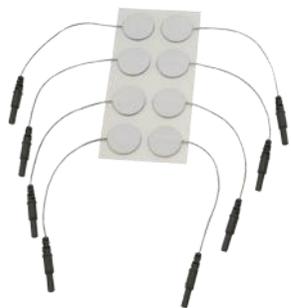
TENSMED S82

ACCESSORIES

Adhesive electrodes

- 3444222 EN-Trode Ø 2,2 cm, 2 mm female, 10 sheets of 8 (also for EMG)
- 3444056 EN-Trode Ø 3,2 cm, 2 mm female, 10 sheets of 4
- 3444135 EN-Trode Ø 5,0 cm, 2 mm female, 10 sheets of 4
- 3444057 EN-Trode 5x5 cm, 2 mm female, 10 sheets of 4
- 3444058 EN-Trode 5x9 cm, 2 mm female, 10 sheets of 4


3444222

3444056

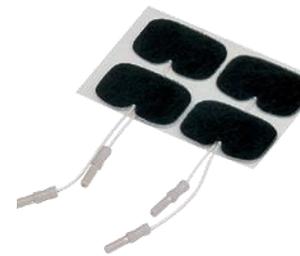3444135

3444057

3444058

TENSMED S82

ACCESSORIES

Rubber electrodes

- 3444128 Rubber electrodes 4x6 cm, 2 mm female, set of 2.
- 3444129 Rubber electrodes 6x8 cm, 2 mm female, set of 2
- 3444130 Rubber electrodes 8x12 cm, 2 mm female, set of 2
- 3444380 Silicon electrode 5x5 cm TENS, 2 mm female

3444128

3444129

3444130

3444380

TENSMED S82

ACCESSORIES

Moist pads for rubber electrodes

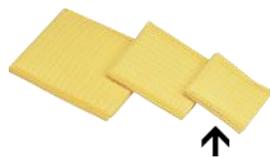
- 1460273 Moist pads for rubber electrodes 4x6 cm, set of 4
- 1460266 Moist pads for rubber electrodes 6x8 cm, set of 4
- 1460275 Moist pads for rubber electrodes 8x12 cm, set of 4

Point electrodes


- 3444180 Point electrode (pen model), 5 mm Ø, 2 mm female, incl 10 conductive rubber caps

Adapters


- 2523524 Adapter plug, 2 mm female, 4 mm male, red
- 2523523 Adapter plug, 2 mm female, 4 mm male, black

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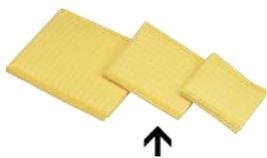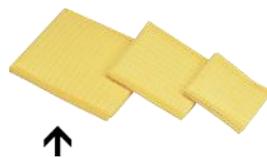Complies with the requirements of 93/42/CEE annex II except p.to 4, acknowledged in Italy with D.L. n° 46 dated February 24 1997, and updated by 2007/47/CEE, acknowledged in Italy with D.L. n° 37 dated January 25 2010 as per Certificate n° MED 23006 issued by notified body KIWA CERMET ITALIA n° 0476.

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Copyright: **Enraf-Nonius B.V.**
Domino S.r.l.