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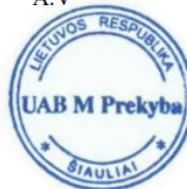
DATA SHEET  
SCANGLISS 2 LITE FOLDABLE

REVISION N° 2  
  
DATE 09/2021

Product Name  
Transfer Board Scangliss 2 Lite Foldable.

Product	Code	Dimensions (cm)	Weight (Kg)
Maxi	03.04.9130	180 x 50	2,30
Midi 170	03.04.9135	170 x 50	2,20
Midi 150	03.04.9150	150 x 50	2,00
Midi 125	03.04.9140	125 x 50	1,60

Use <sup>2.</sup> skirta perkelti pacientams

The Transfer Systems are intended for facilitating the manual handling operations of patients through thrust, traction, sliding maneuvers. They have been designed for use in hospitals, nursing homes and healthcare facilities and for home care.

### Description

The transfer board SCANGLISS 2 LITE FOLDABLE allows one or more operators to transfer a patient from one surface to another with a difference in height up to 15 cm and in distance up to 20 cm. <sup>2.</sup> skirta perkelti pacientams tarp skirtingų paviršių  
The transfer board is equipped with two Padded handles for transport, it is radiolucent. High frequency welding that make the transfer board totally waterproof. It is radiolucent. Maximum Capacity 250 Kg. <sup>8.</sup> maksimali apkrova

Materials	
Handles	Nylon
Base	Etaflon <sup>3.</sup> eteflonas
Base Coating	Nylon <sup>3.</sup> nailonas
Ultra Low Friction Sheet <sup>5.</sup> ultra žemo sukibimo audinys	Nylon RIP-STOP
Sheet Maintenance and Cleaning	
 <sup>7.</sup> tinkamas plauti	Water Temperature not Above 60°C or 140°F
	Do not Iron
	Do not Tumble Dry
	Do not Dryclean

Frame Maintenance and Cleaning	
Disinfection is possible with 5% diluted disinfectants even with alcohol base <sup>6.</sup> tinkamas dezinfekuoti	
Storage and Disposal	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	12.31.03.003
CND Code	Y123003
RDM Code	1532910/R

## DECLARATION OF CONFORMITY

Regulation 2017/745 on MEDICAL DEVICES of 5 April 2017

Demarta Virginio S.a.S. declares under its sole responsibility that the products made and traded by Demarta Virginio S.a.S. and belonging to the group of:

BASIC UDI-DI TRANSFER BOARDS: 8052141080304AGP

Article	Code	UDI-DI / EAN
Scangliss 2 Plus Maxi Foldable 180x50 cm	03.04.9000	08052141086667
Scangliss 2 Plus Maxi Rigid 180x50 cm	03.04.9001	08052141086674
Scangliss 2 Plus Midi 150 Foldable 150x50 cm	03.04.9004	08052141086681
Scangliss 2 Plus Midi 150 Rigid 150x50 cm	03.04.9005	08052141086698
Scangliss 2 Plus Midi 125 Foldable 125x50 cm	03.04.9002	08052141086704
Scangliss 2 Plus Midi 125 Rigid 125x50 cm	03.04.9003	08052141086711
Scangliss 2 Plus Mini 90x50 cm	03.04.9010	08052141086728
Scangliss 2 Lite Maxi Foldable 180x50 cm	03.04.9130	08052141086766
Scangliss 2 Lite Maxi Rigid 180x50 cm	03.04.9131	08052141086803
Scangliss 2 Lite Midi 170 Foldable 170x50 cm	03.04.9135	08056697942491
Scangliss 2 Lite Midi 170 Rigid 170x50 cm	03.04.9136	08056697942507
Scangliss 2 Lite Midi 150 Foldable 150x50 cm	03.04.9150	08052141086841
Scangliss 2 Lite Midi 150 Rigid 150x50 cm	03.04.9151	08052141086889
Scangliss 2 Lite Midi 125 Foldable 125x50 cm	03.04.9140	08052141086773
Scangliss 2 Lite Midi 125 Rigid 125x50 cm	03.04.9141	08052141086780
Scangliss 2 Lite Mini 90x50 cm	03.04.9160	08052141086797
SUPER ROLL WITH FOUR ROLLS cm. 37x62	03.04.35105	08052141086902

SUPER ROLL WITH FOUR ROLLS cm. 37x62	03.04.35105	08052141086902
SUPER ROLL WITH FIVE ROLLS - cm. 43x62	03.04.35106	08052141086957
PATIENT SLIDER - large cm.183x65 kg.4	03.04.PSL	08052141086919
PATIENT SLIDER - standard cm.183x56 kg.3.9	03.04.PSS	08052141086926
PATIENT SLIDER - narrow cm.183x46 kg.3.5	03.04.PSN	08052141086933

comply with the applicable provisions of the 2017/745 Regulation on Medical Devices of 5 April 2017.

Demarta Virginio S.a.S. guarantees and declares under its sole responsibility what follows:

- The devices satisfy the requirements of general safety and performance requested by the Annex IV of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
- The devices **ARE NOT MEASURING INSTRUMENTS**.
- The devices **ARE NOT MADE FOR CLINICAL TESTS**.
- The devices are packed in **NON-STERILE BOX**.
- The devices belong to **class I** in accordance with the provisions of Annex VIII of the above mentioned regulation.

Pollone, 26/05/2021

Certificate LT23/0000007

The management system of

## M Prekyba, UAB

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has been assessed and certified as meeting the requirements of  
**ISO 14001:2015**

For the following activities  
Sale of medical furniture.

# SGS

This certificate is valid from 06 February 2023 until 05 February 2026 and remains valid subject to satisfactory surveillance audits.

Issue 1. Cert# February 2023.

Authorized by  
SGS Société Générale de Surveillance SA  
Technoparkstrasse 1, 8005, Zurich, Switzerland  
t +41 (0)44 445-16-80 - www.sgs.com

This document is an authentic electronic certificate for Client' business purposes use only. Printed version of the electronic certificate are permitted and will be considered as a copy. This document is issued by the Company subject to SGS General Conditions of certification services available on [Terms and Conditions](#) | SGS. Attention is drawn to the limitation of liability, indemnification and jurisdictional clauses contained therein. This document is copyright protected and any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful.

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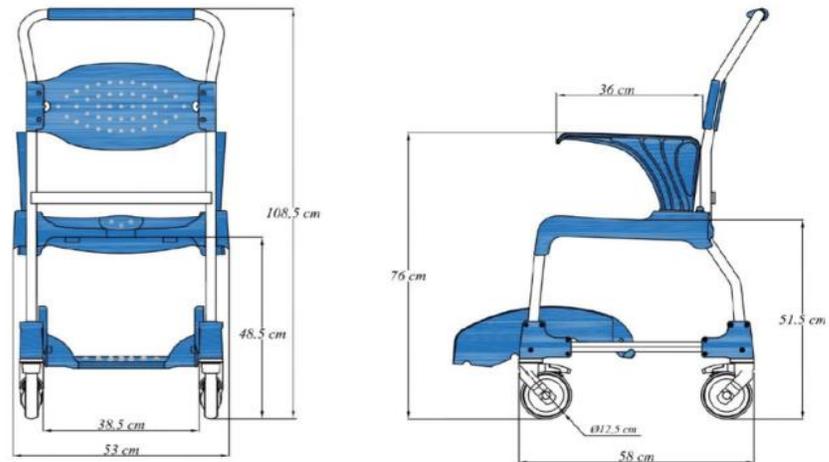
MAXIMUM LOAD	150KG
WEIGHT	15KG.
WHEELS	Ø125 with brake
OUTER FRAME, epoxy-polyester paint (According to UNE 10305-5 standard)	Steel tube Ø25x1,5mm Steel plattine 30x5mm.
ARMREST	Removable
FOOTREST	Removable and adjustable in 5 positions
COVER SEAT	Removable and folding
Stainless steel screws	
Backrest, armrests, seat, footrest, seat cover, commode, in polypropylene	
Handle and wheel supports in polypropylene and fiberglas	

*\*For reasons of improving our products, TecniMoem Care reserves the right to change technical specifications and/or images of our products without prior notice.*

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**tecniemoem**  
Research+Development+Innovation

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
## MOEM bath commode chair

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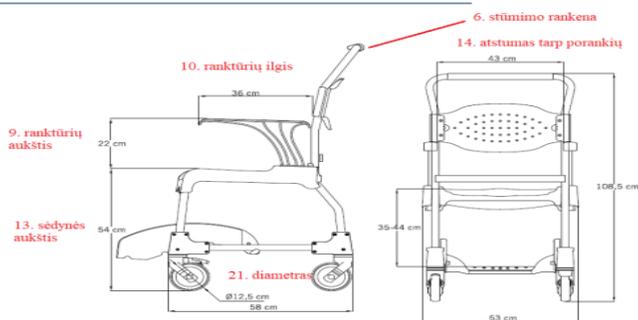With MOEM bath commode chair will reduce people's movements facilitating the cleanliness

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0

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### 11.5 Removing the footrests

- Remove or fold up the seat. Then, raise the footrest by approx. 100 mm and turn outwards by 90 degrees.
- Now you can push the footrest all the way up and pull it out

### 11.6 Handling of the toilet bucket

- From the back of the chair, slide the toilet bucket under the seat. To remove, slightly lift the bucket and then pull it out backwards.
- In the version with a 2-section toilet seat, you can alternatively insert or remove the toilet bucket from above by lifting the seat.

### 11.7 Removing the backrest cushion

- For cleaning purposes or for replacement the backrest cushion can be removed from the arresting grid at the backrest of the chair by pressing against it with your hand from the back.

## 12 Troubleshooting

Fault	Potential cause	Remedy
Armrest cannot be pulled up	Deposits in the locking device (locking bolt)	Cleaning, checking regularly
Locking bolt cannot be pulled out	Deposits, tension spring jammed or defective	Cleaning, Replace the locking bolt

Fault	Potential cause	Remedy
Footrest cannot be folded down	Deposits in the joint, Bracing	Cleaning, Check screw connection
Wheel lock does not brake	General defect	see maintenance!
Turning the wheel is stiff	Deposits	Cleaning
Wheel does not turn	General defect	see maintenance!

Fault	Potential cause	Remedy
Uneven running of the wheel	Severe uneven tread wear	see maintenance!
Chair frame too unstable	Screw connection too loose	Screw connections: Tighten frame, wheel inserts
Wheel insert bent/distorted	Hitting an obstacle	see maintenance!

## 13 Care and maintenance

### 13.1 Cleaning and care

The product can be cleaned using a mild conventional cleaning and disinfecting agent. After cleaning, wipe the product with a clean, moist cloth and then wipe it dry with a dry cloth.

- Do not use scouring agents or cleaning agents with aggressive ingredients (acids, thinners), as well as hard cleaning objects (brushes), as otherwise the surfaces may be damaged or deposits may form on the materials.

Keep the product clean and clean it regularly. Special attention must be paid to hygiene in the sensitive areas.

- The product is suitable for manual cleaning and disinfection. Please observe the reprocessing instructions.

### 13.2 Maintenance

For safety reasons, the product must be professionally maintained at least once a year.

- Check product for visible damage
  - (Cracks, breakage, loose parts)
- Check stability before use

#### Maintenance/inspection scope

#### Maintenance/inspection scope

- Maintenance should include the following inspection work and should be documented.
- All bolted connections for tightness (especially the wheel inserts and the frame)
- Function of the armrests and locking bolts
- Function of the footrest (replace defective footrests)
- Running characteristics and tread of the wheels (replace defective wheels)

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**Name:** CHAIR FOR SHOWER. Mod.: MOEM, MOEM SELF-PROPELLED

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**Intended Use:** To support the patient to facilitate the hygiene of people with movement disabilities, as well as their transport. It is intended to be used by adults in the shower, sink or on the toilet.

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**Specific Harmonized standards:** EN 12183:2014. Manual wheelchairs - Requirements and test methods.

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**Certified Quality System according with ISO 9001:2015.** Quality management systems. Requirements.

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**Certified Quality System according with ISO 13485:2016.** Medical devices. Quality management systems. Requirements for regulatory purposes..

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