




VASCULAR ACCESS
Adult Central Venous Catheters



multicath UP range
The Ultimate Protection



Value Life

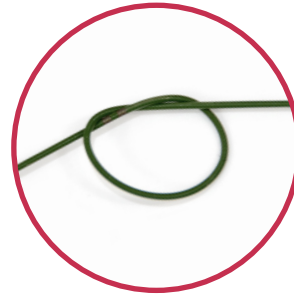

Time / cost saving

- Nitinol & Teflon coated J guidewire to avoid kinking




Infusion line management

- 2 to 7 lumen catheter to optimise your infusion process
- High flow UP for rapid, safe volume replacement therapy
- High pressure to enable a flow rate of 5ml/s at a viscosity of 11.8mPas under 150psi/10bar



Thrombosis & arrhythmia limitation

- combocard for precise tip placement
- 4 catheter lengths adapted to your patient and the chosen puncture site: 12.5cm, 16cm, 20cm, and 30cm

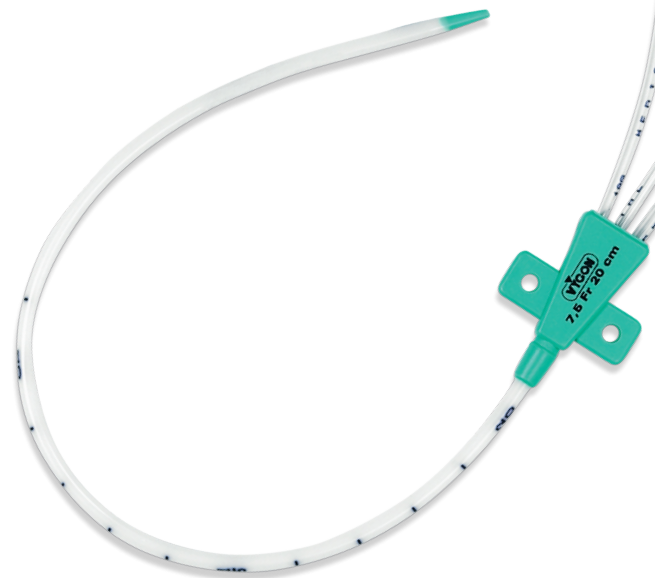

Infection risk management

- multicath UP with polyurethane for low risk patient
- multicath **expert** UP technology for middle risk patient
- multistar UP technology for high risk patient




User safety

- seldisafe to prevent needlestick injuries
- Blood Less System (BLS) to limit blood exposure
- Safety Scalpel to avoid sharp injuries




“Choose the right catheter that your patient needs”

3 different material proposals to manage risk patient from low to high.

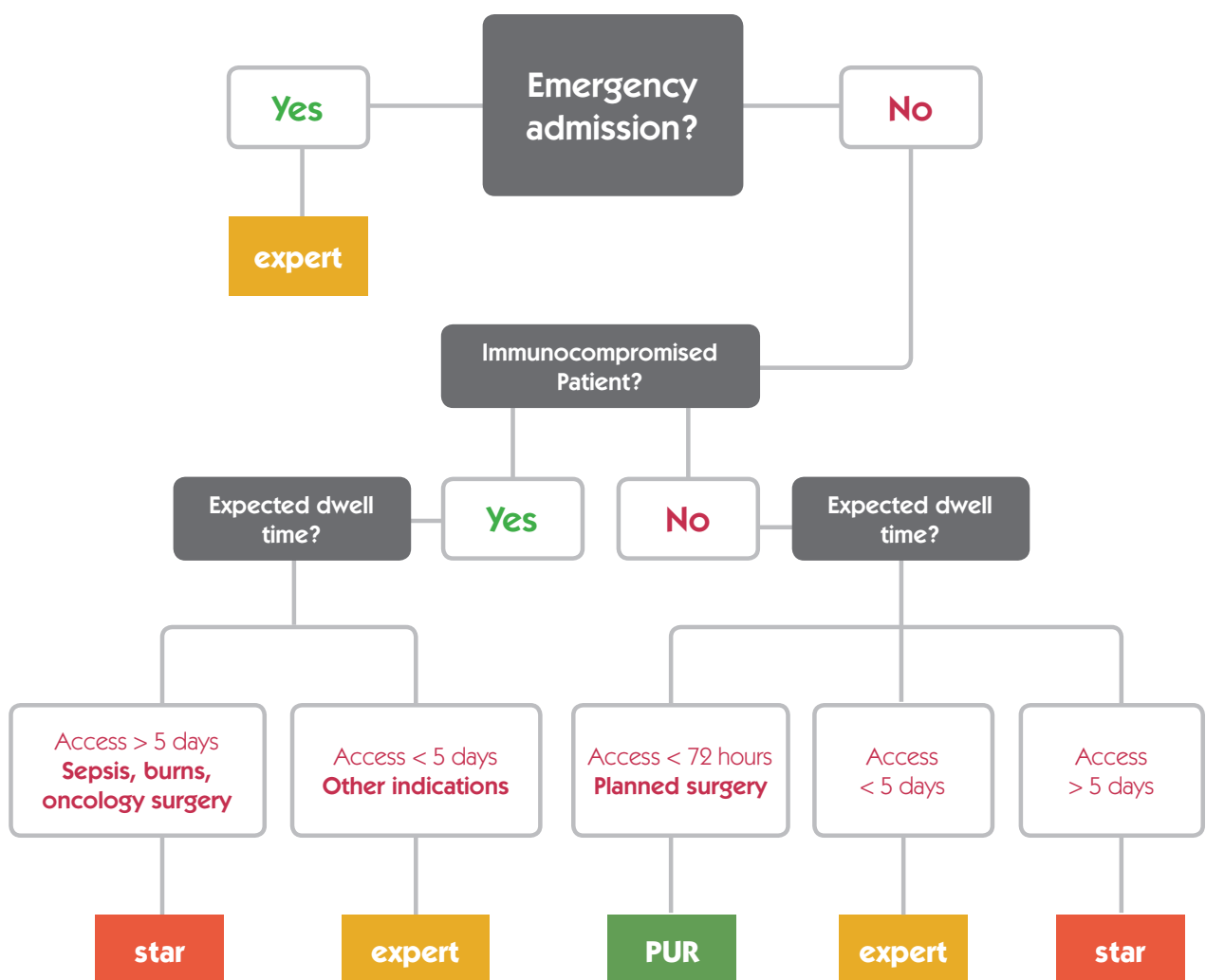

multicath expert UP technology for middle risk patient

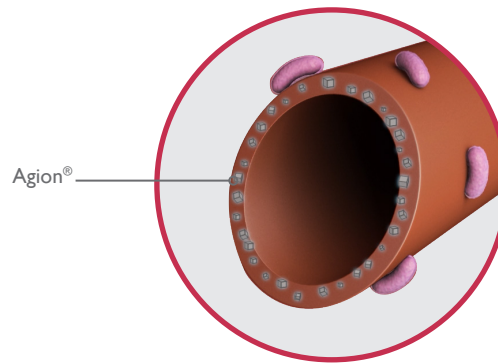


The main component of the **expert UP** technology is the **patented silver compound Agion®** incorporated into the catheter material.

The Agion® has a bacteriostatic activity. It is able to inhibit the growth or reproduction of gram+ and gram- bacteria, and fungi thanks to silver ions.


Silver ions are continuously released out of the catheter when it comes into contact with blood, body fluids or infusion solutions.

Microbial growth on the inner and outer surface of the catheter is suppressed, minimising the onset of catheter related sepsis. Due to the vast active substance content of silver ions, the antimicrobial effect unfolds over the entire life of the catheter.


Bacteriostatic activity

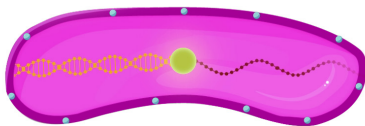

Cell wall damage
and loss of function

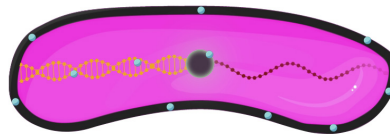

Inhibition of the reproductive
process

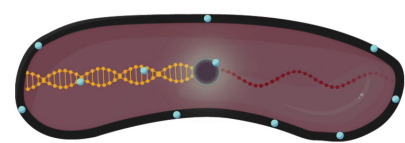


Interruption of cell
metabolism

Clinical studies


“This study indicates that the silver zeolite-impregnated catheter is superior to non-impregnated catheter in reducing the rate of CVC colonisation.” (1)

The study involved 246 CVC insertions of which 122 were “multicath expert CVC” and 124 were “control CVC”.

Overall incidence of catheter tip colonisation:

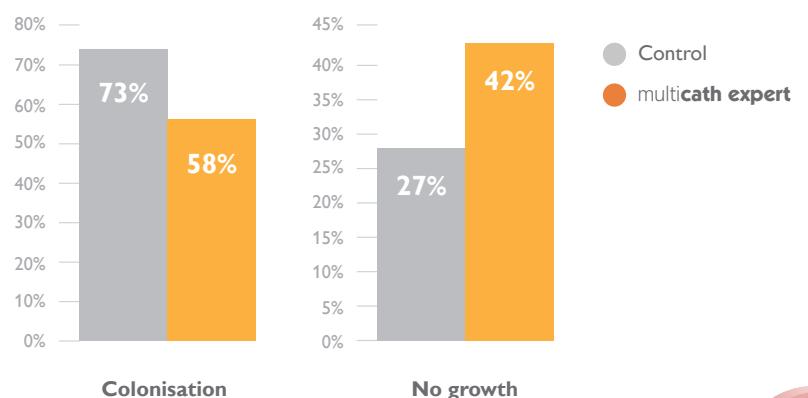

multistar UP technology for high risk patient

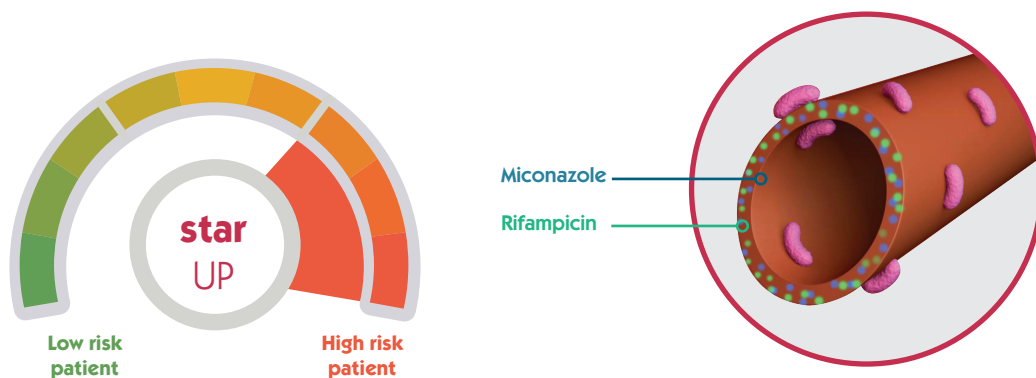

The **star UP** technology is the innovative combination of two active substances, Rifampicin and Miconazole, chosen for their synergic properties:


- Overlapping spectrum of efficacy on gram+ and gram- bacteria and fungi; (2,3)
- Multiple level of action on the bacteria;
- Low risk of bacterial resistance development. (3)

Thanks to the very high concentration in small quantities of the two active substances, the **star UP** technology ensures a strong local action at the inner and outer surfaces of the catheter. The induction of resistant micro-organisms is therefore improbable. (3)

Combined antimicrobial effect


Miconazole

- Synthetic antifungal with low toxicity
- Mechanical action: damages cell wall and inhibits peroxide degradation

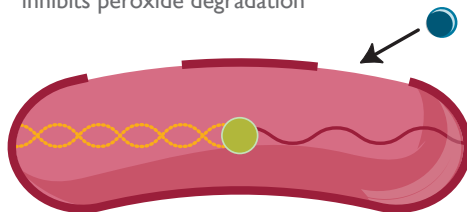


Rifampicin

- High efficacy against fast growing and resting bacteria in biofilm
- Mechanical action: inhibits RNA synthesis

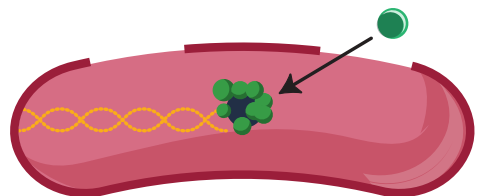

Clinical studies

The clinical efficacy of the star UP technology has been demonstrated in two controlled clinical trials involving a total of 648 individuals and a retrospective cohort study. (4,5,6)

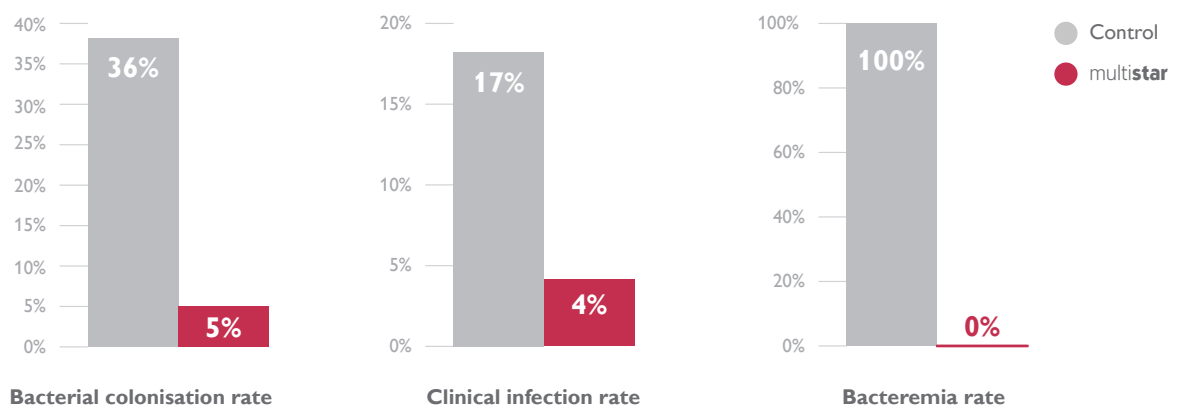

User safety

Every year, many needlestick injuries (including sharp injuries) occur all over the world...

Estimated Occurrence of total and Injection/IV-related NSI (7) in 6 different countries:

Country	Total annual number of NSI	Annual number of infection/IV-related NSI
United States	384,000	123,000
UK	100,00	38,000
Germany	700,000	168,000
France	18,720a	6,552a
Italy	28,200	18,900
Spain	21,815	14,388

IV: Intravenous / NSI: Needlestick Injuries / a: Nurses only

...leading for an important part to infections from three major viruses:

- Hepatitis B virus - 6 to 30 % of the needlestick injuries; (8)
- Hepatitis C virus - 1.8 %; (8)
- Human Immunodeficiency Virus "HIV" – 0.3 %.(8)

Therefore the UP range integrates three exclusive safety components for user Ultimate Protection:

seldisafe: Safety needle for Seldinger technique

- Prevention of needlestick injuries and blood exposure;
- Intuitive passive system with safe mode clearly apparent;



Safety Scalpel

- Sharp injuries avoidance;
- Active safety system.

BLS: Innovative system on seldisafe


- Blood spillage and exposure reduction;
- Guidewire insertion optimisation.



Thrombosis & arrhythmia limitation

The misplacement of central venous catheter induces several mechanical complications. They occur with an incidence from 6.2% to 11.8% in internal jugular and subclavian vein accesses. (9)

(9,10,11,12,13)

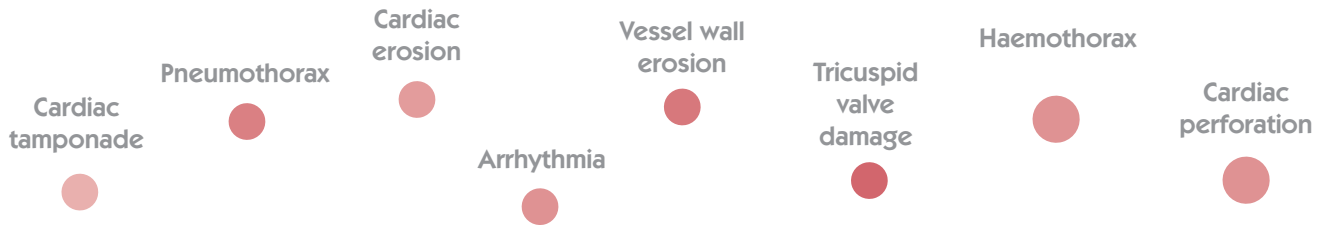

These complications result from erosion through vessels, atrial or ventricular chambers by the tip of the catheter. (11,14,15)

To protect your patient from complications and raise up the multicath UP placement to the state of the art, the new UP range offers:

- 4 different lengths: **12.5cm**, **16cm**, **20cm**, and **30cm**;
- **combcath**: ECG cable integrated with the 16cm, 20cm, and 30cm catheters to reach the Ultimate Placement.

“HOW DOES IT WORK?”

Connect the combcath on the guidewire and to the ECG and simply observe the P wave form during the placement of the catheter tip.



“The closer to the right atrium the catheter tip is, the higher the P wave form is.”




Upper segment of the Superior Vena Cava (SVC): The P wave has similar dimensions to normal. (16)




Lower segment of the SVC: The height of the P wave is approximately half of the peak that it will reach at the cavo-atrial junction. (16)



Cavo-atrial junction: The P wave is at its peak amplitude and no negative segments are seen in the standard P wave. (16)

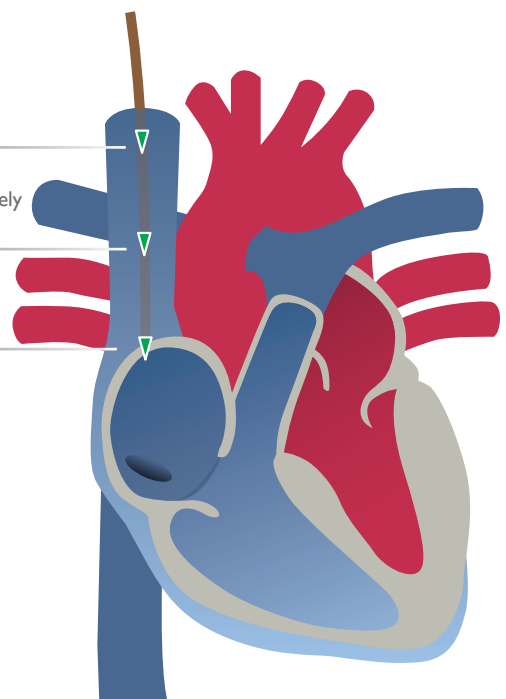

Infusion line management

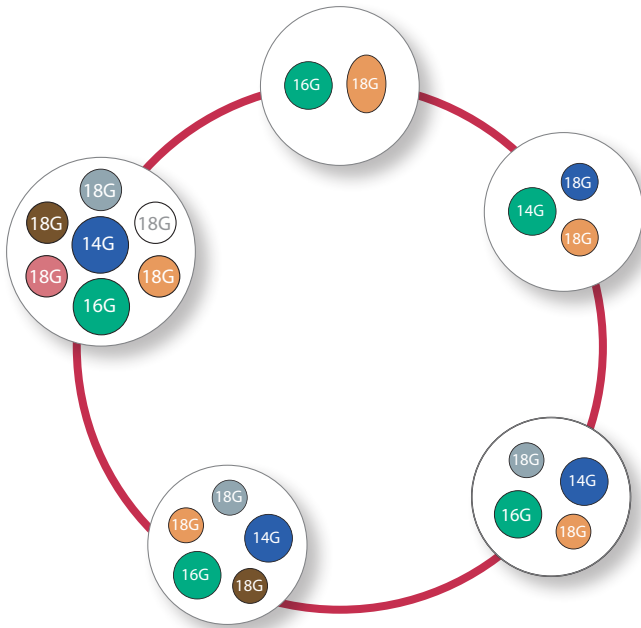


2 to 7 lumen

The management of the infusion line is a real challenge in the Intensive Care Units.

To optimise your infusion processes, the multi**cath** UP range presents a large choice from 2 to 7 lumen catheters, for simultaneous infusion of incompatible solutions.

High flow UP

High flow UP have been designed to achieve rapid, safe volume replacement and allow intravenous therapy, infusion and blood sampling. The multi**cath** HF UP and multi**cath expert** HF UP exist in 3 and 5 lumen with a flow rate up to 400ml/min.

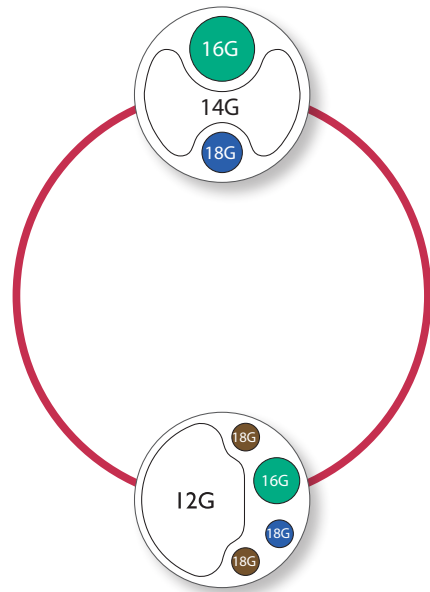

High pressure

The HF UP catheters present a high pressure distal line (marked «HP») to enable a flowrate of 5ml/s at a viscosity of 11.8mPas under 150psi/10bar.


Time / cost saving

“Prevent guidewire kinking for a Unique Puncture.”



Nitinol and Teflon coated J guidewire

The J guidewire combines two technologies to avoid kinking. The Nitinol core has shape memory and the Teflon coating has a great sliding property.

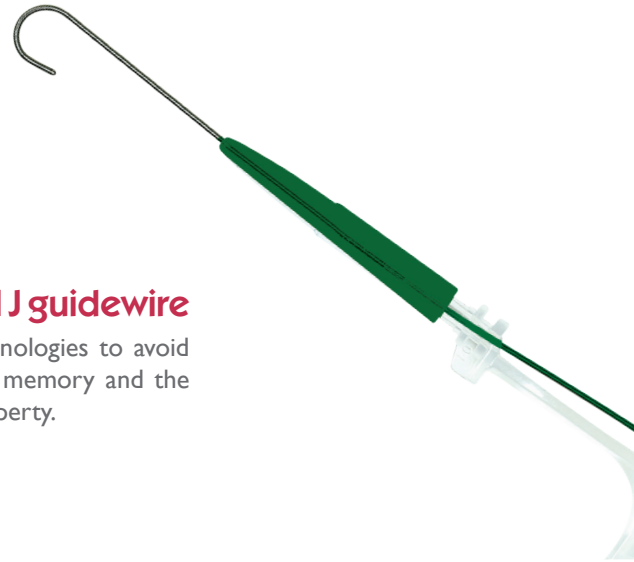

VYSET

Customization with VYSET® enables you to create a pack specific to your requirements and standardizing insertion procedures and ensuring maximum barrier precautions, therefore reducing the associated risk of infection.



Probe cover

Ultrasound gel



Sponge stick



Peelable drape



Pre-filled syringe with saline

13.

	multicath UP	multicath expert UP	multistar UP	Catheter information									Guidewire		Unit of sales
				Length cm	Size Fr (Ø mm)	Priming volume ml (Flow rate ml/min)							Length cm	Ø mm	
						D.	M. 1	M. 2	M. 3	M. 4	M. 5	P.			
2 Lumen	157.072	8157.072	6157.072	12.5	7.5 (2.5)		-	-	-	-	-		60	0.88	5/box*
	157.172	8157.172	6157.172	16	7.5 (2.5)	0.56 (50)	-	-	-	-	-	0.56 (52)	60	0.88	5/box*
	157.272	8157.272	6157.272	20	7.5 (2.5)	0.60 (43)	-	-	-	-	-	0.60 (45)	60	0.88	5/box*
	157.372	8157.372	6157.372	30	7.5 (2.5)	0.65 (35)	-	-	-	-	-	0.65 (37)	70	0.88	5/box*
3 Lumen	155.072	8155.072	6155.072	12.5	7.5 (2.7)			-	-	-	-		60	0.88	5/box*
	155.172	8155.172	6155.172	16	7.5 (2.7)	0.30 (63)	0.33 (34)	-	-	-	-	0.36 (38)	60	0.88	5/box*
	155.272	8155.272	6155.272	20	7.5 (2.7)	0.45 (60)	0.34 (30)	-	-	-	-	0.36 (30)	60	0.88	5/box*
	155.372	8155.372	6155.372	30	7.5 (2.7)	0.49 (50)	0.4 (18)	-	-	-	-	0.50 (20)	70	0.88	5/box*
4 Lumen	158.072	8158.072	6158.072	12.5	8.5 (2.8)	0.51 (60)	0.65 (107)	0.44 (18)	-	-	-	0.44 (19)	60	0.88	5/box*
	158.172	8158.172	6158.172	16	8.5 (2.8)	0.61 (49)	0.74 (93)	0.56 (13)	-	-	-	0.52 (18)	60	0.88	5/box*
	158.272	8158.272	6158.272	20	8.5 (2.8)	0.66 (47)	0.82 (69)	0.6 (11)	-	-	-	0.58 (14)	60	0.88	5/box*
	158.372	8158.372	6158.372	30	8.5 (2.8)	0.71 (34)	1 (64)	0.68 (9)	-	-	-	0.63 (8)	70	0.88	5/box*
5 Lumen	159.072	8159.072	6159.072	12.5	9.5 (3.15)	0.43 (65)	0.55 (110)	0.36 (18)	0.36 (19)	-	-	0.42 (20)	60	0.88	5/box*
	159.172	8159.172	6159.172	16	9.5 (3.15)	0.43 (57)	0.57 (84)	0.36 (15)	0.36 (16)	-	-	0.42 (18)	60	0.88	5/box*
	159.272	8159.272	6159.272	20	9.5 (3.15)	0.47 (55)	0.60 (80)	0.39 (15)	0.38 (12)	-	-	0.43 (17)	60	0.88	5/box*
	159.372	8159.372	6159.372	30	9.5 (3.15)	0.51 (38)	0.65 (68)	0.40 (7.8)	0.42 (8.4)	-	-	0.45 (9.4)	70	0.88	5/box*
7 Lumen	170.072	8170.072	6170.072	12.5	11.5 (3.9)	0.49 (60)	0.58 (110)	0.50 (18)	0.50 (18)	0.50 (18)	0.50 (21)	0.50 (21)	60	0.88	5/box*
	170.172	8170.172	6170.172	16	11.5 (3.9)	0.61 (60)	0.62 (110)	0.43 (15)	0.50 (18)	0.50 (18)	0.50 (18)	0.52 (18)	60	0.88	5/box*
	170.272	8170.272	6170.272	20	11.5 (3.9)	0.61 (50)	0.62 (94)	0.49 (12)	0.50 (12)	0.50 (14)	0.50 (14)	0.56 (14)	60	0.88	5/box*
	170.372	8170.372	6170.372	30	11.5 (3.9)	0.64 (38)	0.80 (77)	0.52 (8)	0.6 (8)	0.61 (10)	0.61 (10)	0.61 (10)	70	0.88	5/box*

*multicath UP are supplied in boxes of 10 units.

	multicath HF UP	multicath expert HF UP	Catheter information							Guidewire		Unit of sales
			Length cm	Size Fr (Ø mm)	Priming volume ml (Flow rate ml/min)				Length cm	Ø mm		
					D.	M. 1	M. 2	M. 3			p.	
3 Lumen	155.912	8155.912	12.5	9 (3)	0.50 (55)	0.50 (16)	-	-	1.50 (200)	60	0.88	10/box
	155.916	8155.916	16	9 (3)	0.55 (50)	0.55 (14)	-	-	1.60 (180)	60	0.88	10/box
	155.920	8155.920	20	9 (3)	0.65 (45)	0.65 (12)	-	-	1.8 (160)	60	0.88	10/box
	155.930	8155.930	30	9 (3)			-	-		70	0.88	10/box
5 Lumen	159.912	8159.912	12.5	12 (4)	0.60 (50)	0.50 (16)	0.50 (16.5)	0.55 (17)	1.60 (400)	60	0.88	10/box
	159.916	8159.916	16	12 (4)	0.65 (45)	0.55 (14)	0.55 (14.5)	0.60 (15)	1.80 (350)	60	0.88	10/box
	159.920	8159.920	20	12 (4)	0.70 (40)	0.60 (12)	0.60 (12.5)	0.65 (13)	2.0 (300)	60	0.88	10/box
	159.930	8159.930	30	12 (4)						70	0.88	10/box

Product designation	Ø (G)	Length (mm)	Article code	Unit of sales
seldisafe	18	70	219.131	20/box

Product designation	Article code	Unit of sales
combcard	9168.202	10/box

Literature

- 1- Khare J MD, et al, Reduction of catheter-related colonisation by the use of a silver zeolite-impregnated central vascular catheter in adult critical care. *Infect.* 2007 Feb;54.
- 2- Schierolz JM. et al. Antimicrobial Central Venous Catheters in Oncology Efficacy of Rifampicin-Miconazole-releasing catheter. *Anticancer research* 30: 1353-1358. 2010.
- 3- Rump et al. Pharmacokinetics of antimicrobial agents rifampicin and miconazole released from a loaded central venous catheter. *Journal of Hospital Infection.* 53; 129-135, 2003.
- 4- Yücel N, Lefering R, Maegele M, Max M, Rossaint R, Koch A, Schwarz R, Korenkov M, Beuth J, Bach A, Schierholz J, Pulverer G, Neugebauer EA. Reduced colonization and infection with miconazole-rifampicin modified central venous catheters: a randomized controlled clinical trial. *JAC* 54(6):1109-15, 2004.
- 5- Lorente L, Lecuona M, Ramos MJ, Jiménez A, Mora ML, Sierra A. The use of rifampicin-miconazole-impregnated catheters reduces the incidence of femoral and jugular catheter-related bacteremia. „Clinical infectious diseases: an official publication of the Infectious Diseases Society of America.“ *Clin Infect Dis.* 1; 47(9): 1171-5, 2008.
- 6- Lorente L, Lecuona M, Ramos MJ, Jiménez A, Mora ML, Sierra A. Rifampicin-miconazole-impregnated catheters save cost in jugular venous sites with tracheostomy. *Eur J Clin Microbiol Infect Dis.* 2011 Dec 22, FI: 2,631.
- 7- Biomedecine International (2010), Needlestick Injuries: Incidence and Cost in the US, UK, Germany, France, Italy and Spain, p44.
- 8- NHS Greater Glasgow and Clyde (2017), Management of Needlestick & Similar Injuries Policy, p7.
- 9- McGee DC, Gould MK. Preventing Complications of Central Venous catheterization. *N Engl J Med.* 2003;348: 1123–33.
- 10- Kasten GW, Owens E, Kennedy D. Ventricular tachycardia resulting from central venous catheter tip migration due to arm position changes: Report of two cases. *Anesthesiology.* 1985;62:185–7
- 11- Czepizak CA, O'Callaghan JM, Venus B. Evaluation of Formulas for Optimal Positioning of Central Venous Catheters. *Chest.* 1995; 107:1662–4.
- 12- Joshi Anish M, Bhosale Guruprasad P, Parikh Geeta P, Shah Veena R. Optimal positioning of rightsided internal jugular venous catheters: Comparison of intra-atrial electrocardiography versus Peres' formula. *Indian J Crit Care Med.* 2008; 12:10–4.
- 13- Fletcher SJ, Bodenham AR. Safe placement of central venous catheters: Where should the tip of the catheter lie. *Br J Anaesth.* 2000; 85:188–91.
- 14- Sivasubramaniam S, Hiremath M. Central venous catheters: Do we need to review practice on positioning? *JICS.* 2008;9: 228–31.
- 15- Stonelake PA, Bodenham AR. The carina as a radiological landmark for central venous catheter tip position. *Br J Anaesth.* 2006; 96:335–40.
- 16- Y.L. Argoti-Velasco, O. Carrillo-Torres, R.A. Sandoval-Mendoza, W.G. Paez-Amaya, X.Y. CAhuantzi-Caballero, Proper electrocardiography-guided placement of a central venous catheter, 2016.
- 17- Pronovost P, et al. An Intervention to Decrease Catheter-Related Bloodstream Infections in the ICU. *N Engl J Med* 2006;355:2725-32.
- 18- Pratt R.J., et al. (2007) epic2: National evidence-based guidelines for preventing healthcare-associated infections in NHS hospital in England. *Journal of Hospital Infection.* 65S, S1-64.
- 19- 2011 guidelines for the Prevention of Intravascular Catheter-Related Infections (Centre for Disease Control)

CRITICAL CARE

For further information, please contact: questions@vygon.com

The specifications shown in this leaflet are for information only and are not, under any circumstances, of a contractual nature.

Vygon – 5, rue Adeline • 95440 ECOUEN • FRANCE
Reception: +33 (0)1.39.92.63.63 – Service clients France: +33 (0)1.39.92.63.81
Export customer service: +33 (0)1.39.92.64.15
Fax.: +33 (0)1.39.92.64.44 • www.vygon.com



www.vygon.com