

SPECIALIST MEDICAL DEVICES

esvimed

ECG electrodes instruction for use



1. Shave the skin if necessary




2. Degrease the skin



3. Plug the wire into the sensor



4. Remove the pad cover

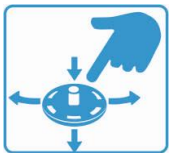

5. Press the electrode pad firmly to the skin



6. Start the examination



7. Dispose of the pad after examination

ESVICO Sp.z o.o.

84-214 Bożepole Wielkie, ul. Chmieleniecka 15, Poland
NIP: 588-237-42-30, REGON: 221198703, KRS: 0000420978

artur.ostrowski@esvico.pl, tel.: +48 517 704 596

krzysztof.rutkowski@esvico.pl, tel.: +48 512 272 628

www.esvimed.pl