



## 2.1. Intended use

The C7-C8-C9 series stools are designed to ensure that the dentist or the assistant is properly supported during normal dental practice.



### WARNING!

- The maximum allowable load that the stool can bear is 135 Kg. Do not exceed this value.
- If the rear wheels are obstructed while moving back, the stool may overturn.
- The device is not intended to be used in rooms subject to the risk of explosion.

### 2.1.1. Reference standard

Reference standards: the C7-C8-C9 series dentist's stools are devices designed in compliance with the ISO 7493 standard.

### 2.1.2. Environmental conditions

The C7-C8-C9 series dentist's stools must be used in environments with the following characteristics:


- Temperature between 10 and 40°C;
- Relative humidity between 30 and 75%.

## 2.2. Identification

The identification plate is positioned underneath the seat.

Data given on plate:

- Manufacturer name.
- Stool model.
- Serial number.
- Month and year of manufacture.

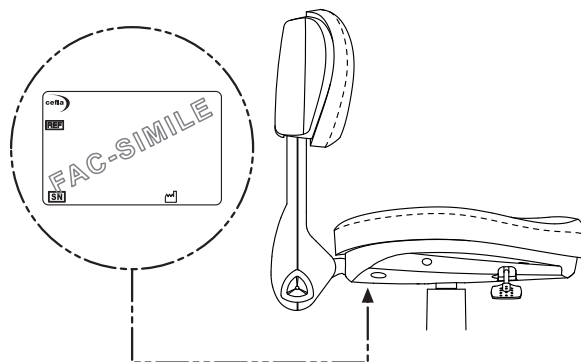

## 2.3. Technical information

Dentist's stool model:	C7	C8	C9
Base wheels outside diameter (mm):	605	605	605
Seat height from the floor (mm):	430	445	600
Seat vertical adjustment travel (mm):	125	125	200
Backrest vertical adjustment travel (mm):	65	-	-
Seat dimensions (mm):	460x420	ø407	365x350
Backrest adjustment angle:	+11° / -9°	-	-
Gross weight (kg):	18	18	18
Net weight (kg):	16	16	16
Packaging dimensions (mm):	650x700x590(h)		