# LAB-ACCURATE GLUCOSE AND LACTATE MEASUREMENT IN ONE TEST

Easy to use

Results in 20-45 seconds

Up to 160 results an hour

Low operating cost

4.1 dalis, poz. 7

C-LINE	
GP+	• Up to 5 samples
CLINIC	• Up to 20 samples
S-LINE	
LAB+	• Up to 63 samples

EASY TO USE

4.1 dalis, poz.3

ACCURATE AND RELIABLE

4.1 dalis, poz.10

4.1 dalis, poz.8

MAXIMUM EFFICIENCY AND CONVENIENCE

4.1 dalis, poz.11

EFFICIENT DATA HANDLING

- Multi-lingual touch screen display with step by step instructions
- Just 20 µl blood, plasma or serum sample required when used with EKF pre-filled 'Safe-Lock' reaction cups (1000 µl)
- Results in 20-45 seconds. Up to 160 results an hour
- Automatically detects presence of reaction cups
- Chip sensors easily exchangeable, no electrode cleaning required

- Enzymatic-amperometric method using chip-sensor technology
- Measuring range: Glucose 0.5–50 mmol/L; lactate 0.5–40 mmol/L
- Imprecision: CV ≤1.5 % (12 mmol/L)
- Pre-filled 'Safe-Lock' reaction cups eliminate manual dilution
- Innovative needle and exchanger design eliminates cross contamination
- 3 calibration modes

4.1 dalis, poz.6


- Dual channel option: Glucose and/or lactate results from one sample
- Long sensor life: Glucose 60 days / 7,500 tests; lactate 50 days / 6,000 tests
- Sample stable for up to 5 days (refrigeration recommended)
- STAT position

4.1 dalis, poz.9

- PC, network and printer connectivity via RS232C interfaces
- Stores up to 1.000 results
- Integrated Barcode reader (optional)

Biosen S-Line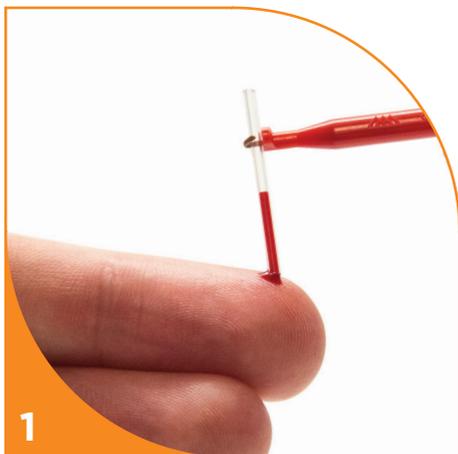

1 Collect 20 µl blood sample in capillary.

2 Place capillary into cup and invert until mixed.

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**EKF-diagnostic GmbH**
Atsisakymas: EKF-diagnostic GmbH pasilieka teisę keisti produkto specifikaciją be išankstinio įspėjimo.