

## Carrying Case for BeneHeart D60&D50 Series Defibrillator/Monitor

### Instructions for Use

#### Statement

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this product and this manual. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

This manual provides the instructions necessary to use and clean the product in accordance with its function and intended use. Observance of this manual is a prerequisite for proper performance and correct operation.


**mindray** and **BeneHeart** are the registered trademarks or trademarks.

© 2023 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

#### Product Introduction

This carrying case is intended to carry and store BeneHeart D60/D50A/D50/D50C defibrillator/monitor (hereinafter called defibrillator/monitor) and accessories. Applicable to the equipment with and without the paddles tray, two types of carrying cases are provided. The two types are exactly the same in structure and functions, only different in the direction of opening right-side pouch.

The carrying case applicable to the equipment configured with the paddles tray is taken as an example in this manual. The following figure shows all parts of the carrying case.

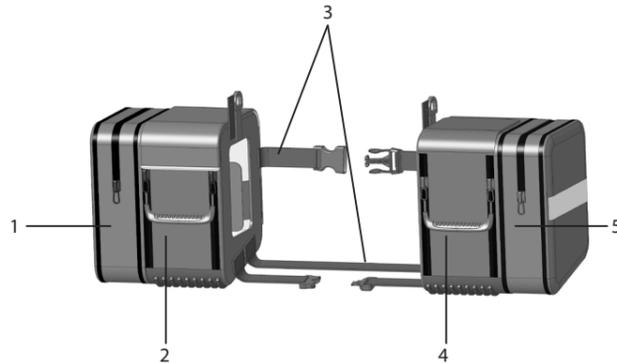


No.	Name	Storage content	Quantity
1	Left expansion pouch	<input type="checkbox"/> CPR sensor, CPR sensor cable, CPR adhesive tap <input type="checkbox"/> Ultrasound probe	One set for either type of accessories
2	Left-side pouch	<input type="checkbox"/> ECG accessories <input type="checkbox"/> SpO <sub>2</sub> accessories <input type="checkbox"/> NIBP accessories <input type="checkbox"/> CO <sub>2</sub> accessories	One set for each type of accessories
3	Straps	Fixing defibrillator/monitor	One
4	Right-side pouch	<input type="checkbox"/> Temp accessories <input type="checkbox"/> IBP accessories <input type="checkbox"/> Paddles cable or pads cable	One set for each type of accessories
5	Right expansion pouch	<input type="checkbox"/> Thermometer <input type="checkbox"/> IBP accessories <input type="checkbox"/> Temp accessories	One set for each type of accessories

#### Precaution


- You should periodically check electrode pads stored in the carrying case. Bended pads may shorten the service life or even make pads unusable.
- The carrying case and accessory pouches are not waterproof. Keep the defibrillator/monitor and accessories from liquid during transport and storage.
- Do not wash or dry this carrying case by machine.

#### Attaching the Defibrillator/Monitor

- Disconnect the external power supply and remove all accessories from the defibrillator/monitor.
- Remove hooks from the defibrillator/monitor.
- Align the defibrillator/monitor with the case bottom, and place it into the carrying case. Ensure that the display is facing outside.

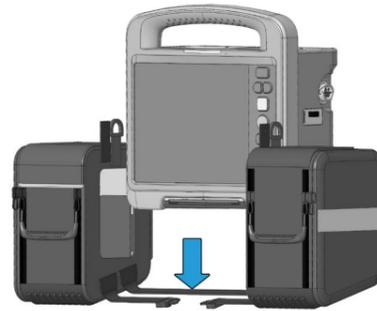

- Use the back strap to fix the defibrillator/monitor, and then fasten buckles.
- Use two screws to fix the case on back of defibrillator/monitor handle.

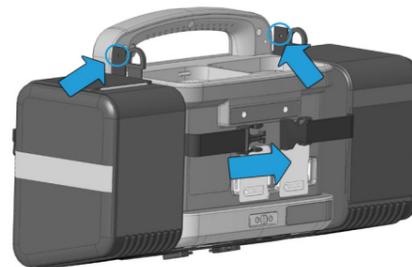

- Place the defibrillator/monitor on the worktable with face up.
- Use the bottom strap to fix the defibrillator/monitor, and then fasten buckles.


The following figure shows the final assembly.



#### Attaching the Transport Dock

When the transport is needed, you should connect the carrying case and the transport dock. To attach the transport dock, follow this procedure:

- Use four screws to fix the fixing plate on bottom of the defibrillator/monitor.

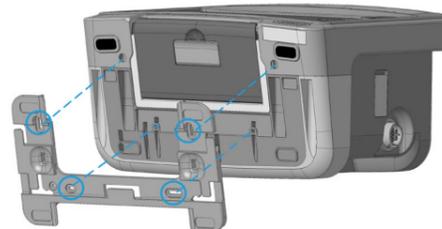

- Connect the carrying case and defibrillator/monitor as described in step 1 to 6 of "Attaching the Defibrillator/Monitor".
- Align with slots of the fixing plate, use the bottom strap to fix the defibrillator/monitor, and then fasten buckles.



- Place the defibrillator/monitor on top of the transport dock until it clicks into place.




The following figure shows the final assembly.



#### Placing Accessories

- Open the pouch.
  - To open the left-side pouch, lift the flap outward or pull the zip down.
  - To open the right-side pouch, lift the flap inward or pull the zip down.
  - To open the expansion pouch, pull the zip along the edge of left- or right-side pouch.

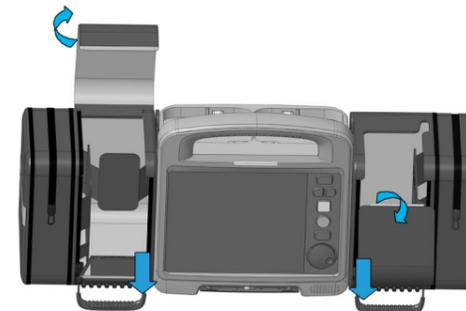


- Place accessories into corresponding pouches as described in the column "Storage content" of "Attaching the Defibrillator/Monitor".
- Close the pouch.


#### Placing External Paddles

When external paddles are not in use, you should place them in the paddles tray and secure the paddles cable carefully. To do so, follow this procedure:

- Hold handles of both paddles with hands, and align them with metal parts of the paddles tray.
- Press down on the external paddles until you hear a click.

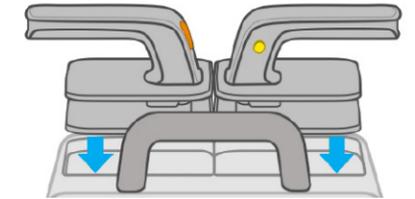

- Lift the flap outward to open the right-side pouch.
- Place the paddles cable into the right-side pouch.
- Pull the cable plug outside the pouch with one hand, and close the pouch with the other hand.

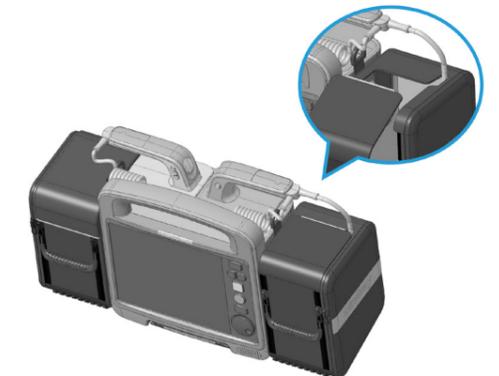

#### Cleaning/Disinfection

You should clean and disinfect the carrying case as required in your hospital's servicing schedule.

To wipe or wash the carrying case, you can use a soft cloth dampened with water or cleaner. For stubborn stains, you can use a soft brush dampened with fabric detergents.

After cleaning and disinfection, you should dry the carrying case in a ventilated and cool place.

#### Company Contact

www.mindray.com

Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Mindray Building, Keji 12th Road South, High-tech industrial park, Nanshan, Shenzhen 518057, P.R.China

Tel: +86 755 81888998

Fax: +86 755 26582680

E-mail: service@mindray.com