

# Monoprotect® INIBSA

## Dental needles

Choose a high quality dental needle that provides a precise injection.



Smooth injections to comfort your patients.

# Monoprotect® INIBSA

## Dental needles



**Stainless steel**  
cannulas with  
silicon-coated tip.



**Triple beveled**  
to facilitate tissue penetration  
and reduce tissue injuries.



**Bevel indicator**  
printed on the cone that  
allows a proper needle positioning.



**Latex free.**  
Reduces the risk of allergic  
reactions or skin irritations.



**Sterility guarantee.**  
Welded point that joins  
safety cap and protector.



**Color-coded.**  
Needles individually packed  
in boxes of 100 pieces.



**Quick identification.**  
Production batch, expiry  
date and size printed directly  
on the needle cap.



**Manufactured in EU.**  
Metric thread.

DESCRIPTION	CODE	SIZE	CANNULA DIAMETER (mm)	NEEDLE LENGTH (mm)
● MONOPROTECT® INIBSA 30G 12MM EXTRASHORT	<b>5283</b>	30G	0,3	12
● MONOPROTECT® INIBSA 30G 16MM EXTRASHORT	<b>5284</b>	30G	0,3	16
● MONOPROTECT® INIBSA 30G 21MM MEDIUM	<b>5285</b>	30G	0,3	21
● MONOPROTECT® INIBSA 30G 25MM MEDIUM	<b>5286</b>	30G	0,3	25
● MONOPROTECT® INIBSA 27G 36MM LONG	<b>5282</b>	27G	0,4	36

### Instructions for use:

#### Positioning the needle on the syringe.

- 1) Break the guarantee seal.
- 2) Pull while twisting to remove the cap.
- 3) Screw the threaded base onto the syringe, taking care not to remove the protector.
- 4) Remove the protector with a slight twisting motion just before anaesthesia.

#### After use:

- 1) Carefully remove the needle from the syringe to avoid accidental puncture.
- 2) Dispose of the used needle immediately in a suitable container conform to regulations, without bending it, break it or handle it in any other way.



**INIBSA DENTAL S.L.U.**  
Ctra. Sabadell a Granollers km 14,5  
08185 Lliçà de Vall - Barcelona (Spain)  
Tel. +34 93 860 95 00 Fax +34 938 437 389  
email: [export@inibsa.com](mailto:export@inibsa.com)

**TECNOFAR SPA**  
Via al Piano 54 A, 23020  
Gordona (SO) - Italy  
[www.tecnofar.it](http://www.tecnofar.it)



follow us



[www.inibsa.com](http://www.inibsa.com)