Elbacare

Lying surface 90 cm x 200 cm
with wooden side rails

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Elbacare

Lying surface 90 cm x 200 cm
with 2-part aluminum side rail

Elbacare

Lying surface 90 cm x 200 cm
with lower headboard and footboard
with 2-part aluminum side rail

Elbacare

Lying surface 90 cm x 200 cm
with 3-part steel side rail

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6.4 Operation of the side rails

6.4.1 The non-split wooden side rail

1. To use the side rails, lift the upper side rail (a) until it engages in the highest position.
2. To lower the side rail (a), lift the upper side rail and simultaneously press the release button (b) for the side rail lock and release the side rail.

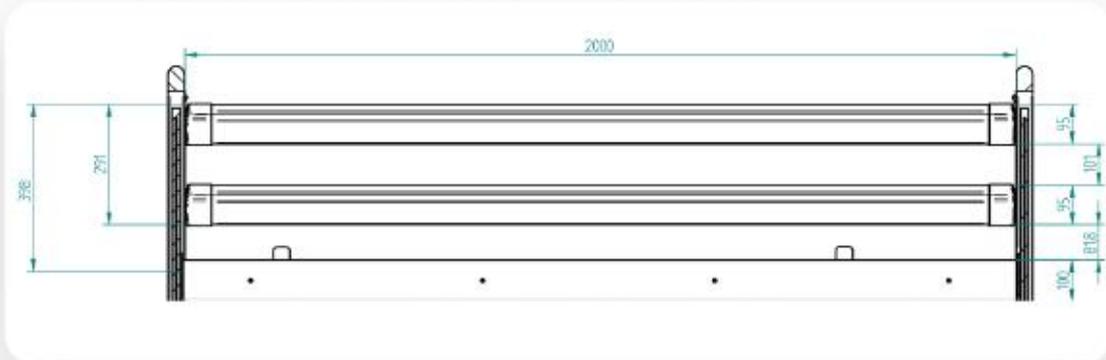
 When the side rail is raised, always ensure that it is securely engaged!

 The side rails are only intended to prevent people from falling out of bed. Do not climb or lean over them under any circumstances!

 Use the following overview to check the use of the correct side rail and the permissible positions or distances of the side rail variants.

Non-split side rail, divided in two with 95mm wide uprights (item no. 274 or 01681)

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6.2 Handset with locking function

The electric bed functions can be operated via the handset. All functions can be locked with the nurse key.

- a. Backrest adjustment up/down electric stepless variable 0°-70°
- b. Thigh adjustment up/down electric stepless variable 0°-30°
- c. Lying surface up/down electric stepless 380-800 mm
- d. Trendelenburg position electrically stepless with separate locking function
- e. Anti-Trendelenburg position electrically stepless with separate locking function
- f. Comfort seating position, back adjustment, thigh adjustment and total lying surface. Head-up at the push of a button
- g. Height adjustment for low position, electrically stepless 240-380 mm with separate locking function (Note chapter 6.7.6)
- h. key illumination
- i. Hand switch hook
- j. Lock for activating/deactivating the handset functions
- k. Nurse key

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