RUBENS Bedside cabinets

1.0. Product description

1.1. Purpose and range of application

RUBENS bedside cabinets are great addition to nursing beds. They are intended to alleviate and compensate injuries and disabilities for older, frail and handicapped people. They allow patients to store their private things on them and simplify eating meals, reading books, newspapers or using a computer. They are easy to clean and high quality workmanship.

RUBENS bedside cabinets are made of furniture board finished with PVC veneer. They can be located on both sides of the nursing bed, due to its construction, without loss of functionality - doors and drawer can be opened from both sides, what is very useful. Cabinets stand on four castors - two of them can be locked. Private things patient can store on the open shelf or inside the closed space, which is split on two levels. Only difference between RUBENS 1 and RUBENS 2 is that patient or user can swivel and adjust height of table tray in RUBENS 2.

Our products meet the varying needs of patients and professional caregivers. In order to meet customer expectations we care about attractive design, as well as matching furniture systems in every place of use. That is why we offer our customers wide choice of decors, although our standard is dark beech décor.

RUBENS bedside cabinets described in this document compliant with the requirement of the EEC Directive on medical devices 93/42/EEC dated 14 June 1993 with subsequent amendments.

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Parameters	Bedside cabinet model	
	RUBENS 1	RUBENS 2
Height [cm]	86,0	86,0
Depth [cm]	45,0	45,0
Width [cm]	53,0	53,0
Width with bedside table tray [cm]	118,5	123,0
Height adjustment range of the bedside table tray [cm]	---	71,0 – 99,0
Castors diameter [cm]	Ø5	Ø5
Safe working load of the bedside table tray [kg]	6,0	6,0
Weight [kg]	32,4	38,6

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
8.2. Cleaning and care

All components of the bedside cabinet are made from the best quality material. The surface of steel tubes is coated with a durable polyester powder layer. All wooden or wood-based parts are protected with non-hazardous varnish. The bedside cabinet components can be easily cleaned and maintained according to the effective hygienic rules with various cleaning and disinfecting means. Adhering to the described below rules of the furniture maintenance ensures its long life and keeps its nice appearance.

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7.4. Open tray out - RUBENS 2

Firstly, push handle down (9) and pull it right up (10). It releases locking mechanism – then bedside table tray can be opened outwards and placed parallel to the floor (11).

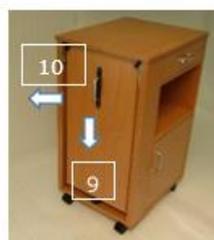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
7.5. Height adjustment and swiveling of the bedside table tray - RUBENS 2

Warning

⚠ Before adjusting the height of bedside table tray, make sure there are not any obstacles, such as furniture, window sill etc.

The biggest advantage of the RUBENS 2 is that there is possibility to adjust height and angle level of bedside table tray.

Pull two black levers (12) to release gas spring. It allows to adjust height in demanded range (release levers when the height is good enough for patient). The adjustment range is between 71 and 99 cm (13).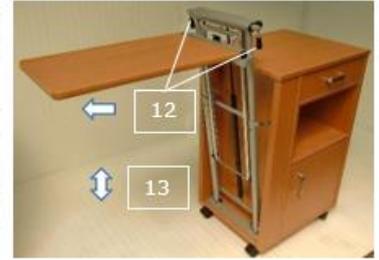

Press button (14) and swivel bedside table tray - it clicks into required position after releasing that button.

There are three positions available to achieve for bedside table tray.


Warnings

- ⚠ Do not load bedside table tray before locking it in the proper position and during pushing it closer to the nursing bed.
- ⚠ Do not put any drinks or meals on bedside table tray when it is swiveled.
- ⚠ Safe working load of the bedside table tray is 6 kg and it cannot be exceeded. Do not lean on the table - it can cause damage of adjustment mechanism and lead to serious injuries!
- ⚠ Close tray down before relocating the bedside cabinet.

1.3.1. Spalva suderinama su lovos spalva (bukas)

Standard colors:

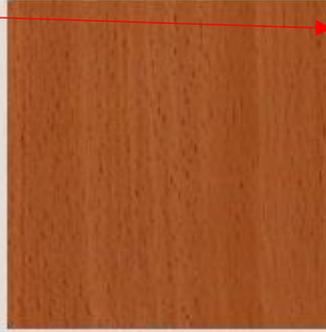C1 Dark beech

C2 Light beech

Non-standard colours*:

C5 Maple

C6 Acacia

C7 Light oak

C8 Nut

C153 Sonoma oak