

TEST STRIP USE INSTRUCTIONS**Product Preparation:**

The product to be tested should be prepared in accordance with its label and user manual.

Testing Steps:

The test strip should be immersed in the prepared solution. The indicator area of the strip should be completely immersed in the solution and held still for a few seconds.

The strip should be removed from the solution and excess liquid safely shaken off.

The strip should be placed in contrast to the color scale on the box.

The colors on the color scale represent the reaction occurring on the strip. The reaction on the strip should be compared to the closest color on the color scale.

Result Evaluation:

The reaction on the strip must be matched to the appropriate color on the color scale. This mapping provides an indication of the state of the solution. If the result is not satisfactory, the solution must be prepared again.

Notes:

Read the instructions for use carefully and act in accordance with the instructions.

Before evaluating test results, ensure that the product was prepared correctly and the test was performed properly.

Each strip is for single use.

KAF GRUP SAĞLIK HİZMETLERİ
İNŞAAT SANAYİ VE TİC. LTD.ŞTİ.
Atakent Mh. 221.Sk. No:3A Rota Office
A Blok D.83 K.Çekmece / İSTANBUL
Tel: 0212 471 42 00 Fax: 0212 471 42 01
Halkalı V.D.: 486 053 3864

Kaf Grup Sağ. Hizm. İnş. San.ve Tic. Ltd. Şti

Merkez: Atakent mah. 221. Sk No:3A Rota Office K. 14 D. 83 K.Çekmece / İSTANBUL

Fabrika: Hadımköy mah, Denizkızı sok. No:4/5 Arnavutköy / İSTANBUL

Tel: +90 0 212 471 4 200 Fax: +90 212 471 4 201

www.kafgrup.com