WANCARE

## ONESEPT PA POWDER

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2250  
ppm

Peracetic  
Acid

It is a concen-  
trated powder for  
professional use

1

Solution  
can be used  
during 1  
working day

KAFGRUP  
SAĞLIK HİZMETLERİ

Ataşehir Mh. 221. Sk. No:3A Rota Office A Blok D.83  
K.Çekmece/İSTANBUL  
T: +90 212 471 4 200 (pbx) F: +90 212 471 4 201  
www.kafgrup.com / info@kafgrup.com

	<b>OneSept PA POWDER High Level Disinfectant For Medical Instruments And Endoscopes Instruction For Use</b>			
	<b>Document Number</b>	<b>Release date</b>	<b>Revision No</b>	<b>Revision date</b>
	TD.05.124	10.06.2019	06	24.05.2021

**ONESEPT PA POWDER**  
**HIGH LEVEL DISINFECTANT FOR MEDICAL**  
**INSTRUMENTS AND ENDOSCOPES**

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	<b>OneSept PA POWDER High Level Disinfectant For Medical Instruments And Endoscopes Instruction For Use</b>			
	<b>Document Number</b>	<b>Release date</b>	<b>Revision No</b>	<b>Revision date</b>
	TD.05.124	10.06.2019	06	24.05.2021

### General features

It is used to provide high level disinfection of all kinds of invasive surgical instruments, dental surgical instruments, surgical laparoscopy instruments, flexible and rigid endoscopes. It is a concentrated powder product. Our OneSept PA Powder products are offered for sale in powder form as 2 kg, 2.5 kg and 5 kg.

**Package content of our products;** OneSept PA POWDER-Transfer spoon\*/Product efficacy Control strip

\*The transfer spoon accessory is supplied with our powder product, it is used to transport the product while turning the product into solution, and it does not have any measurement function.

### Chemical Composition

Chemical Composition	Contents
Active ingredient	Peracetic Acid (a solution diluted with 2% water) 2250 ppm
Excipients	*Sodium Percarbonate *TAED *Citric Acid Anhydrate *Sodium carbonate *C10 Alcohol Ethoxylate *Ethylenediaminetetraacetic acid *Benzotriazole *Triple enzyme complex (Protease, amylase, lipase)

### Purpose of usage:

It is used to provide high level disinfection of interventional surgical instruments, dental surgical instruments, surgical laparoscopy instruments, endoscopes and heat sensitive instruments. It does not cause damage such as rusting, corrosion, softening of the endoscope covers, discoloration. It is anticorrosive. It is suitable for endoscope washing machines and manual use.

	<b>OneSept PA POWDER High Level Disinfectant For Medical Instruments And Endoscopes Instruction For Use</b>			
	<b>Document Number</b>	<b>Release date</b>	<b>Revision No</b>	<b>Revision date</b>
	TD.05.124	10.06.2019	06	24.05.2021

### Effect Mechanism

The mechanism of action of our products as disinfectants is bactericidal, mycobactericidal, virucidal, sporocidal and fungicidal. It denatures the proteins and enzymes of microorganisms contained in the medical device; It increases cell wall permeability by cleaving sulfhydryl(-SH) and sulfur (SS) bonds.

Products	Shelf Life	Tub Life Time	Usable surface materials	Microorganisms and Effect times
<b>OneSept PA POWDER</b>	2 years	12 hours	<ul style="list-style-type: none"> <li>- Glass</li> <li>- Ceramic</li> <li>- Silicone</li> <li>- Plastic</li> <li>- PVC</li> <li>- Acrylic</li> <li>- polymer</li> <li>- Polycarbonate</li> <li>- Polystyrene</li> <li>- HDPE</li> <li>- Stainless steel</li> <li>- Porcelain</li> </ul>	<p><i>Bacillus subtilis</i> (EN 17126 and EN 13704) - 10 minutes</p> <p><i>Staphylococcus aureus</i> (EN 13727) - 10 minutes</p> <p><i>Pseudomonas aeruginosa</i> (EN 13727) - 10 minutes</p> <p><i>Enterococcus hirae</i> (EN 13727) - 10 minutes</p> <p><i>Mycobacterium avium</i> (EN 14348) - 10 minutes</p> <p><i>Mycobacterium terrae</i> (EN 14348) - 10 minutes</p> <p><i>Candida albicans</i> (EN 13624) - 10 minutes</p> <p><i>Aspergillus niger</i> (EN 13624) - 10 minutes</p> <p><i>Staphylococcus aureus</i> (EN 14651) - 10 minutes</p> <p><i>Pseudomonas aeruginosa</i> (EN 14651) - 10 minutes</p> <p><i>Enterococcus hirae</i> (EN 14651) - 10 minutes</p> <p><i>Mycobacterium avium</i> (EN 14563) - 10 minutes</p> <p><i>Mycobacterium terrae</i> (EN 14563) - 10 minutes</p> <p><i>Candida albicans</i> (EN 14562) - 10 minutes</p> <p><i>Aspergillus niger</i> (EN 14562) - 10 minutes</p> <p>Poliovirus Type 1 (EN 14476) -10 minutes</p> <p>Human Adenovirus Type 5 (EN 14476) -10 minutes</p> <p>Murine Norovirus (EN 14476) -10 minutes</p>

\*\* When our products were subjected to corrosion test according to TS 6773 standard, it was observed that they did not show any corrosion effect.

### Usage

OneSept PA POWDER (eg 100 gr for 5 L) solution is prepared with the transfer spoon of our product.

 SAĞLIK HİZMETLERİ	<b>OneSept PA POWDER High Level Disinfectant For Medical Instruments And Endoscopes Instruction For Use</b>			
	<b>Document Number</b>	<b>Release date</b>	<b>Revision No</b>	<b>Revision date</b>
	TD.05.124	10.06.2019	06	24.05.2021

- Disassembled medical instruments are immersed in the container/container filled with the product.
- It is kept for 10 minutes for the duration of the activity.
- Medical instruments are removed from the Container/Container.
- It is rinsed with distilled water.

#### Usage Concentration

It is a concentrated powder product and used by preparing a 2% solution. eg; 100 gr is used for 5 liters.

#### INDICATIONS-CONTRAINDICATIONS,SIDE EFFECTS, WARNINGS AND PRECAUTIONS

##### Indications

It contains parasetic acid and is indicated for high-level disinfection of the surfaces of invasive medical devices, Endoscopes, surgical-medical and heat-sensitive equipment.

*It is indicated for Bacillus subtilis, Staphylococcus aureus, Pseudomonas aeruginosa, Enterococcus hirae, Mycobacterium avium, Mycobacterium terrae, Candida albicans, Aspergillus niger, Polyvirus type 1 virus, Human adenovirus type 5 virus, Murine norovirus microorganisms.*

##### Contraindications

- It should not be used directly (without pre-cleaning) on surfaces and instruments containing organic compounds (blood, tissue, urine, etc.).
- It should not be used together with other chemicals as it may react with chemicals.
- It should not be used by people who are sensitive to paracetic acid.

##### Side effects

Our products have no known side effects.

#### WARNINGS and PRECAUTIONS

- Be sure to read the instructions for use before using the product.
- Do not contact with eyes.
- Use protective clothing (gloves and eye/face protection) equipment.

	<b>OneSept PA POWDER High Level Disinfectant For Medical Instruments And Endoscopes Instruction For Use</b>			
	<b>Document Number</b>	<b>Release date</b>	<b>Revision No</b>	<b>Revision date</b>
	TD.05.124	10.06.2019	06	24.05.2021

- IF SWALLOWED: Rinse mouth, do not induce vomiting.
- In case of prolonged contact with skin, wash with plenty of soap and water. Get medical attention if skin irritation occurs.
- Do not reuse the packaging of the finished product .
- **IF IN EYES:** Rinse cautiously with water for several minutes. In case of eye irritation, consult a doctor.
- Avoid spreading to the environment.
- Store in original container
- Dispose of waste and used packaging in accordance with official regulations.

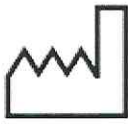
**Storage and Shipping Conditions:**

- Store at 0°C -25°C in its original packaging in a cool and dry place.
- The product shelf life is 2 years from the date of manufacture.
- Use the opened product within 6 (six) months.
- Keep out of reach of children.

**Sterilization**

Our products are used non-sterile.

- **Symbols Used on the Label and Their Meanings**

	Notified Body Number		Expiration date
	Production date		Non-sterile

**OneSept PA POWDER High Level Disinfectant For Medical Instruments  
And Endoscopes Instruction For Use**

**Document Number**

TD.05.124

**Release date**

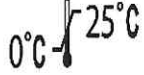
10.06.2019

**Revision No**

06

**Revision date**

24.05.2021

	Read the Instructions for Use.		Protect from sunlight.
	Store in dry place.		Do not use damaged package.
	Manufacturer Information		Do not use damaged package.
	Barcode number		Referance number
	Lot number		Caution (Warnings)

	<b>OneSept PA POWDER High Level Disinfectant For Medical Instruments And Endoscopes Instruction For Use</b>			
	<b>Document Number</b>	<b>Release date</b>	<b>Revision No</b>	<b>Revision date</b>
	TD.05.124	10.06.2019	06	24.05.2021

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**„KAF GRUP SAĞLIK HİZMETLERİ SAN. VE TİC. LTD. ŞTİ.“**