COULTER DxH Cleaner

REF 628022, 628023

TABLE OF CONTENTS

English	2
Français (FR)	4
Deutsch (DE)	6
Italiano (IT)	8
Español (ES)	10
Português Portugal (PT-PT)	12
Dansk (DA)	14
Svenska (SV)	16
Norsk (NO)	18
Ελληνικά (EL)	20
日本語 (JP)	23
Lietuviškai (LT)	25
Magyar (HU)	27
Polski (PL)	29
Čeština (CZ)	31
Slovák (SK)	33
한국어 (KO)	35
Türkçe (TR)	37
Русский (RU)	39
Hrvatski (HR)	42
Български (BG)	44
中文 (ZH-TW)	46
Română (RO)	48
Србија (SR)	50
Українська (UK)	52
Português Brasil (PT-BR)	54
Nederlands (NL)	56
Tiếng Việt (VI)	58
REFERENCES	60

COULTER DxH Cleaner

REF 628022, 628023

For *In Vitro* Diagnostic Use
Rx Only in the U.S.A.

INTENDED USE

A cleaning agent for use on UniCel DxH Coulter Cellular Analysis Systems for components that come in contact with blood samples.

INTENDED USER

The device is intended to be used by a laboratory professional.

SUMMARY

The COULTER DxH Cleaner is an azide-free, formaldehyde-free, biodegradable cleaner which contains a proteolytic enzyme that aids in the removal of protein buildup.

PRINCIPLE

Proteins that flow through apertures may accumulate, and narrow or block aperture openings. The cleaning reagent dissolves proteins at apertures as well as at aspiration and dispensing components of the systems.¹

REACTIVE INGREDIENTS

A solution of a proteolytic enzyme (0.1-~0.3% by wt.)

NON-REACTIVE INGREDIENTS

Buffer, stabilizers, preservatives, surfactant

WARNING AND PRECAUTIONS

- **Inhalation:** If product is inhaled, move exposed individual to fresh air. If individual is not breathing, begin artificial respiration by trained personnel and obtain medical attention immediately.
- **Eye Contact:** If product enters eyes, rinse eyes gently with water for 15 minutes or longer, making sure that the eyelid is held open. If pain or irritation occurs, obtain medical advice/attention.
- **Skin Contact:** In case of skin contact, rinse with water as a precaution.
- **Ingestion:** If product is ingested, rinse mouth with water. If irritation or discomfort occurs, obtain medical attention immediately.

GHS HAZARD CLASSIFICATION		
COULTER DxH Cleaner	DANGER	May cause allergy or asthma symptoms or breathing difficulties if inhaled.
		Avoid breathing vapours.
		Wear protective gloves, protective clothing and eye/face protection.
		In case of inadequate ventilation, wear respiratory protection.
		IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
		IF INHALED: Remove person to fresh air and keep at rest in a position comfortable for breathing.
		If experiencing respiratory symptoms: Call a POISON CENTER or doctor/physician.
		Causes serious eye irritation.
		Subtilisin 0.1-~0.3%
		2-Phenoxyethanol <1%
		Ethoxylated lauryl alcohol <1%

STORAGE, STABILITY, AND DISPOSAL

- Store COULTER DxH Cleaner at 2-25°C.
- Do not use product past expiration date.
- Use product at temperatures stated in the instrument online help and/or product manuals.
- Opened containers are stable for 3 months when stored as recommended.
- Dispose of waste product, unused product, and contaminated packaging in compliance with federal, state, and local regulations.

PREPARATION

COULTER DxH Cleaner is ready to use. Replace the reagent container as directed in your instrument online help and/or product manuals.

IMPORTANT: If product has been partially or completely frozen, allow product to warm to room temperature. Mix product by gentle inversion prior to placement on the instrument. Install and prime the reagent as directed in your instrument online help and/or product manuals.

PRODUCT AVAILABILITY

COULTER DxH Cleaner - 1 x 5 L

REF

628022

COULTER DxH Cleaner - 1 x 10 L

REF

628023

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

ADDITIONAL INFORMATION

Glossary of Symbols is available at beckmancoulter.com/techdocs by searching for part number C07163.

May be covered by one or more pat. - see www.beckmancoulter.com/patents

For additional information or if damaged product is received, call Beckman Coulter Customer Service at 800-526-7694 (USA or Canada) or contact your local Beckman Coulter Representative.

SDS

Safety Data Sheet is available at beckmancoulter.com/techdocs

NOTICE TO USER

For a patient/user/third party in the European Union and in countries with identical regulatory regime (*Regulation 2017/746/EU on In vitro Diagnostic Medical Devices*); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

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- Updated copyright date
Phiên bản AB, 11/2022
Beckman Coulter Ireland Inc.
Beckman Coulter, Inc., 250 S. Kraemer Blvd., Brea, CA 92821 U.S.A.