# COULTER LATRON CP-X Control

**REF** 628024

## TABLE OF CONTENTS

English . . . . .	2
Français (FR) . . . . .	4
Deutsche (DE) . . . . .	6
Italiano (IT) . . . . .	8
Español (ES) . . . . .	10
Português Portugal (PT-PT) . . . . .	12
Dansk (DK) . . . . .	14
Svenska (SE) . . . . .	16
Norsk (NO) . . . . .	18
Ελληνικά (GR) . . . . .	20
日本語 (JP) . . . . .	22
Lietuvių (LT) . . . . .	24
Magyar (HU) . . . . .	26
Polski (PL) . . . . .	28
Čeština (CZ) . . . . .	30
Slovák (SK) . . . . .	32
한국어 (KR) . . . . .	34
Türkçe (TR) . . . . .	36
Русский (RU) . . . . .	38
Hrvatski (HR) . . . . .	40
Български (BG) . . . . .	42
中文 (台灣) (ZH-TW) . . . . .	44
Română (RO) . . . . .	46
Srpski (Latinski) (SP) . . . . .	48
Українська (UA) . . . . .	50
Português Brasil (PT-BR) . . . . .	52
Nederlands (NL) . . . . .	54
Tiếng Việt (VN) . . . . .	56

---

# COULTER LATRON CP-X Control

**REF** 628024

**For *In Vitro* Diagnostic Use  
Rx Only in the U.S.A.**

## INTENDED USE

COULTER LATRON CP-X Control is intended for use on UniCel DxH Coulter Cellular Analysis Systems that use VCSn technology, in conjunction with specific COULTER reagents. It is used to monitor the volume, conductivity and light scatter parameter measurements.

## SUMMARY

Coulter Cellular Analysis Systems with VCSn technology require a stable quality control material of uniform size with appropriate light scattering characteristics. The control monitors the stability of the electrical processing and the fluidic flow rate systems used to measure the volume, conductivity and light scatter.

## PRINCIPLE

The COULTER LATRON CP-X Control is supplied as a ready-to-use suspension of latex particles. These particles pass through the flow cell and produce characteristic electrical signals that are measured as volume, conductivity and various angles of light scatter and light loss, for quality control of Coulter Cellular Analysis Systems with VCSn technology.

## REAGENTS

COULTER LATRON CP-X Control: A suspension of polystyrene particles in a buffered medium containing a surfactant.

## WARNING AND PRECAUTIONS

1. Product contains <0.1% Sodium Azide. Sodium Azide preservative may form explosive compounds in metal drain lines. See National Institute for Occupational Safety and Health Bulletin: Explosive Azide Hazards (8/16/76).
2. May cause sensitization by skin contact.
3. Avoid contact with skin.
4. Wear suitable gloves.
5. After contact with skin, wash immediately with plenty of water.

GHS HAZARD CLASSIFICATION		
COULTER LATRON CP-X Control	<b>WARNING</b> 	May cause an allergic skin reaction.
		Wear protective gloves, protective clothing and eye/face protection.
		If skin irritation or rash occurs: Get medical advice/attention.
		Take off contaminated clothing and wash it before use.
		Reaction Mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC# 247-500-7] and 2-methyl-4-isothiazolin-3-one [EC# 220-239-6](3:1) <0.05%

## STORAGE, STABILITY, AND DISPOSAL

When stored at 2-30°C, unopened tubes of COULTER LATRON CP-X Control are stable until the expiration date indicated on the label. Opened tubes are stable for 30 days when stored at recommended temperatures.

Bring tube to AMBIENT TEMPERATURE before use.

Keep tubes tightly capped when not in use.

### DO NOT FREEZE.

Dispose of waste product, unused product and contaminated packaging in compliance with federal, state and local regulations.

## EVIDENCE OF DETERIORATION

Inability to recover expected results may indicate product instability or deterioration due to improper storage, handling or contamination.

Product frozen and thawed or exposed to temperatures outside the recommended temperature range might not recover indicated values.

Tubes of product that become contaminated with debris might exhibit more than one population resulting in an increase in Coefficients of Variation.

## MATERIALS PROVIDED

- LATRON CP-X Control - 8 x 4 mL
- Table of EXPECTED RESULTS containing Assigned Values and EXPECTED RANGES

## MATERIALS REQUIRED, BUT NOT PROVIDED

- Sufficient COULTER reagents to complete analysis
- System HELP or Instructions for Use

## INSTRUMENT OPERATION

Refer to your Systems HELP or Instructions for Use for proper instrument operating instructions.

## INSTRUCTIONS FOR USE

**NOTE:** Bring COULTER LATRON CP-X Control to AMBIENT TEMPERATURE before use.

1. Prepare your instrument: Follow the instructions for Setting Up Controls in System HELP or the Instructions for Use when a new lot of controls is received.
2. Perform Daily Checks according to System HELP or the Instructions for Use.
3. Gently mix a room temperature tube of COULTER LATRON CP-X Control by inversion five (5) to eight (8) times. Avoid foaming.
4. Run the control using Cassette Presentation on the instrument.
5. Aspirate the COULTER LATRON CP-X Control.
6. Examine the Mean/Mode Channel and Coefficient of Variation results for the volume, conductivity and various light scatter measurements. Compare the results to the TABLE OF EXPECTED RESULTS.

**NOTE:** If a result exceeds the assay limits, refer to your System HELP or Instructions for Use for suggested actions. Follow your laboratory's procedure for handling results that exceed limits.

## ASSIGNED VALUES AND EXPECTED RESULTS

Refer to TABLE OF EXPECTED RESULTS.

**ASSIGNED VALUES** are determined on validated systems using specific COULTER reagents. The values are confirmed by multiple analysis of the control.

**EXPECTED RANGES** include variation between lots and between instruments, and represent 95% confidence limits for well maintained instrument systems.

## PRODUCT AVAILABILITY

COULTER LATRON CP-X Control

**REF**

628024 - 8 x 4 mL

## TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

For additional information or if damaged product is received, call Beckman Coulter Customer Service at 800-526-7694 (USA or Canada) or contact your local Beckman Coulter Representative.

## ADDITIONAL INFORMATION

Glossary of Symbols is available at [techdocs.beckmancoulter.com](http://techdocs.beckmancoulter.com) (PN C07163)

May be covered by one or more pat. - see [www.beckmancoulter.com/patents](http://www.beckmancoulter.com/patents)

**SDS**

Safety Data Sheet is available at [beckmancoulter.com/techdocs](http://beckmancoulter.com/techdocs)

## REVISION HISTORY

Revision AK, 01/2018

Changes were made to:

- Added new Serbian language
- Added new Vietnamese language

Revision AL, 09/2018

Changes were made to:

- INTENDED USE
- Removed European Hazard Classification
- Added new Patent Information Statement
- Added new Dutch, Traditional Chinese, and Ukrainian languages

Revision AM, 05/2023

- Updated EC REP Address
- Updated copyright date

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Beckman Coulter Ireland Inc.
Beckman Coulter, Inc., 250 S. Kraemer Blvd., Brea, CA 92821 U.S.A.