

FITTING*

1 Select cannula size

Select appropriate cannula size; A clear gap must be visible around each prong. The sizing chart on the following page should only be used as a guide.

2 Prepare skin

Prepare the patient's skin according to hospital protocol.

3 Connect to gas source

Connect the cannula to the gas source.
TIP: Place hand close to nasal prongs to ensure that there is gas flow exiting the prongs.

4 Remove the first F&P Wigglepad™ 2 tabs

Remove the first backing tabs from the F&P Wigglepads™ 2 and avoid touching the adhesive.

5 Insert cannula


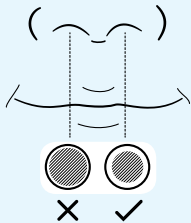



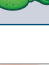
Insert the cannula into the nares. Ensure the cannula bridge rests close to the nose without touching the septum. DO NOT stretch the cannula during application. Stick the F&P Wigglepads™ 2 to the patient's cheeks.

6 Secure cannula

Remove the second backing tabs and stick the F&P Wigglepads™ 2 onto the cheeks.

* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.

1 SELECT CANNULA SIZE

| SIZE RANGE | | ENSURE GAP AROUND PRONGS* |
|------------|---|---|
| XS |  |  |
| S |  | |
| M |  | |
| L |  | |
| XL |  | |
| | | OJR410 |
| | | OJR412 |
| | | OJR414 |
| | | OJR416 |
| | | OJR418 |

43.



REMOVE CANNULA

1 Place fingertip

Place fingertip on the outside edge of the F&P Wigglepads™ 2 and gently peel the cannula away from the Wigglepads 2.



F&P WIGGLEPADS™ 2 REMOVAL AND REPLACEMENT

1 Lift the edge

Lift the edge of the F&P Wigglepads™ 2. Use a damp cloth to wipe the patient's skin and the underside of the Wigglepads 2 while gently peeling away from the patient's face.

2 Adhere






Adhere the replacement F&P Wigglepads™ 2 to the cannula, remove the first backing tabs and stick the Wigglepads 2 onto the patient's cheeks.

3 Remove

Remove the second backing tabs and stick onto the patient's cheeks.



SIZING INFORMATION

| WEIGHT GUIDE (KG) | | | | | | | | | | SPARES | | | | | |
|--|------|---|-----|---|-----|---|-----|---|--|--------|----|----|----|----|--|
| | 500g | 1 | 1.5 | 2 | 2.5 | 3 | 3.5 | 4 | | | | | | | |
|  OJR410 XS | | | | | | | | | F&P Wigglepads™ 2 WJR110 | | | | | | |
|  OJR412 S | | | | | | | | | | | | | | | |
| WEIGHT GUIDE (KG) | | | | | | | | | | SPARES | | | | | |
| | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9 | 10 | 15 | 20 | 25 | 30 | |
|  OJR414 M | | | | | | | | | | | | | | | F&P Wigglepads™ 2 WJR112 |
|  OJR416 L | | | | | | | | | | | | | | | |
|  OJR418 XL | | | | | | | | | | | | | | | |
| <div><div></div> Expected to fit patient <div></div> May fit patient</div> | | | | | | | | | | | | | | | |

NOTE:

1. Prongs must not create a seal in the nares.
A clear gap must be visible around each prong.
2. Patient weight should only be used as a guide.



To view the
Optiflow™ Junior 2
inservice video
scan this QR code

* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.