



- GB** Instructions for use/Technical description  
STERILIT Power Systems oil spray GB600
- USA** STERILIT Power Systems oil spray GB600
- D** Gebrauchsanweisung/Technische Beschreibung  
STERILIT Power Systems-Ölspray GB600
- F** Mode d'emploi/Description technique  
Spray d'huile STERILIT Power Systems GB600
- E** Instrucciones de manejo/Descripción técnica  
Aceite en espray STERILIT Power Systems GB600
- I** Istruzioni per l'uso/Descrizione tecnica  
Olio spray STERILIT Power Systems GB600
- P** Instruções de utilização/Descrição técnica  
Óleo pulverizador STERILIT Power Systems GB600
- NL** Gebruiksaanwijzing/Technische beschrijving  
STERILIT Power Systems-oliespray GB600
- S** Bruksanvisning/Teknisk beskrivning  
STERILIT Power Systems-oljespray GB600
- RUS** Инструкция по применению/Техническое описание  
Масляный спрей STERILIT Power Systems GB600
- CZ** Návod k použití/Technický popis  
Olejový sprej STERILIT Power Systems GB600
- PL** Instrukcja użytkowania/Opis techniczny  
Spray olejowy STERILIT Power Systems GB600
- SK** Návod na používanie/Technický opis  
Olejový sprej STERILIT Power Systems GB600
- TR** Kullanım Kılavuzu/Teknik açıklama  
STERILIT Power Systems yağ spreyi GB600

**B | BRAUN**  
SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | [www.aesculap.com](http://www.aesculap.com)

Aesculap – a B. Braun company

TA-Nr. 012081 08/15 V6 Änd.-Nr. 51366

**CE** – DIR 93/42/EEC

**Symbols on product and packages**

	Extremely flammable aerosol
300 ml	Nominal filling quantity
e	Conformity mark for finished packaging ordinance
405	Full capacity volume
3	Conformity mark for aerosol packaging
	Use by
	Limited quantity (LQ)
	Store this way up

**Intended use**

The Sterilit maintenance oil is validated for steam sterilization procedures according to EN ISO 17665. Sterilit is applied prior to sterilization.

**Properties**

- Steam-permeable oil film
- No impairment of the sterilization effect from the lubrication film
- Lubricating effect and corrosion protection
- Silicone free

**Safe handling and preparation**



DANGER

**Extremely flammable aerosol.**  
Pressurised container: May burst if heated.

- ▶ Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- ▶ Do not spray on an open flame or other ignition source.
- ▶ Do not pierce or burn, even after use.
- ▶ Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.
- ▶ Keep out of reach of children.
- ▶ Only put empty cans in the recyclables collection.



WARNING

**Frostbite due to evaporative cold!**

- ▶ Fold motor/hand piece and adapter into a towel, cellulose or similar.



WARNING

**Hazardous for aquatic organisms, can cause long-term harmful effects in water!**

- ▶ Do not release into the environment.
- ▶ Observe special instructions/safety data sheet.

- ▶ Before using Sterilit: Reprocess medical devices according to instructions for use.
- ▶ Apply prior to sterilization only.
- ▶ Ensure sufficient ventilation.
- ▶ Store in a cool, dry place, away from light.

**Application**

- ▶ Observe the instructions for use of the corresponding motor/handpiece.
- ▶ Apply a suitable adapter onto the spray can.
- ▶ Apply the adapter onto the connector on handpiece/motor.
- ▶ Hold the spray can upright.
- ▶ During the functional check before sterilization, spray through the handpiece/motor for approx. 2 seconds.
- ▶ Wipe off excess oil with a lint-free wipe.

**Adapter overview**

STERILIT Power Systems adapter	for hand piece/motor
GB600840 	Acculan 3Ti Acculan 4
GB600880 	Acculan 4 GA344 ELAN 4 GA844
GB600860 	ELAN 4 electro
GB600850 	ELAN 4 air
GB600870 	microspeed uni XS
GB600820 	microspeed uni HiLAN HiLAN XS Hi-Line Hi-Line XS Hi-Line XXS
GB600830 	miniLine
GB600810 	microLine

**Technical Service**



WARNING

**Risk of injury and/or malfunction!**

- ▶ Do not modify the product.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency. Modifications carried out on medical technical equipment may result in loss of warranty rights and applicable licenses.

**Service addresses**

Aesculap Technischer Service  
Am Aesculap-Platz  
78532 Tuttlingen / Germany  
Phone: +49 (7461) 95-1602  
Fax: +49 (7461) 16-5621  
E-Mail: ats@aesculap.de

Or in the US:  
Attn. Aesculap Technical Services  
615 Lambert Pointe Drive  
Hazelwood  
MO, 63042  
Aesculap Repair Hotline  
Phone: +1 (800) 214-3392  
Fax: +1 (314) 895-4420

Other service addresses can be obtained from the address indicated above.

**Disposal**

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

**Distributor in the US/Contact in Canada for product information and complaints**

Aesculap Inc.  
3773 Corporate Parkway  
Center Valley, PA, 18034,  
USA