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### Inhalt

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## GB Instructions for use

### Contents

Closed wound drainage system for connection to Redyrob® Comp

- 600 ml secretion vessel, PVC-free
- Vacuum regulator
- Quantitative vacuum gauge.

### Materials used

ABS, NR, PUR, PA, PE, PETG, Luran, Terluran, Borex, glycol.

### Indications

Removal of blood and secretions from wounds and body cavities as haemoma and seroma prophylaxis.

Vacuum drainage ensures better adaptation and stabilisation of the wound surfaces and reduces the risk of infection. To supplement Redyrob® Comp, in the case of a full primary bottle and/or exhausted vacuum and continued requirement for vacuum drainage.

### Contraindications

Do not use the vacuum drainage system in direct contact with brain, liver,

spleen and other parenchymal organs. Infection of the vacuum drainage system in spongy bone wounds may result in excessive blood loss.

### Risks

Known risks of wound drainage, in particular risk of infection.

### Warnings

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient. Do not use if packaging is damaged. Do not resterilise.

### Duration of application

General statement not possible as drainage duration is guided by clinical observation. If the suction bottle is full or contains air, vacuum drainage is no longer possible without the addition of a Redyrob® Transplus system.

### Directions

#### Use aseptic techniques.

1. Turn controller of the filled primary Redyrob® Comp system to "0".
2. Remove the red combi stopper or secretion bag from Luer Lock of the Redyrob® Comp. CAUTION: Do not connect anything to the red combi stopper! The combi stopper prevents wound secretion flow.
3. Use sterile techniques to remove Redyrob® Transplus from the packaging. Check that quantitative vacuum gauge shows vacuum in vessel. Connect Redyrob® Transplus to Redyrob® Comp.
4. Set desired vacuum on the Redyrob® Transplus.

#### Functions of control positions 0-3:

**Position 0:** The inlet is closed. Thus, wound secretion cannot flow away. This may cause tamponade of the wound or the formation of haematomas.

**Position 1:** Low vacuum is applied to the wound.

**Position 2:** Medium vacuum is applied to the wound.

**Position 3:** High vacuum is applied to the wound. In positions 1-3 wound secretion is sucked into the Redyrob® Transplus vessel.

5. If the vacuum gauge shows "low" and vacuum drainage is still needed, connect another Redyrob® Transplus.
6. Connect Redyrob® Transplus to the Luer Lock below the nonreturn valve. Regulate the vacuum using the secondary Redyrob® Transplus bottle.

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