


Enterprise 9000X

3 PD 25. 3 PD 19.4.

Attendant Control Panel (ACP)

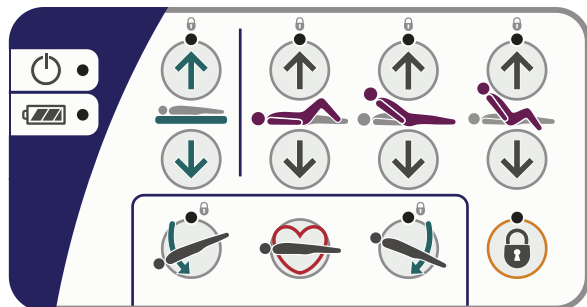

3 PD 23.



Power On indicator



Battery Status LED indicator



fully charged



charging



discharged



CPR button - flattens the mattress platform.



Lower head end
(Trendelenburg position).



Lower foot end
(reverse Trendelenburg position).



Function Lock button - press the button, then press selected Attendant Control Panel button(s) to lock/unlock function. Yellow light = locked




Auto-Chair up button - press and hold for chair position.




Auto-Chair down button - return mattress platform to flat and level position.

Caregiver Controls

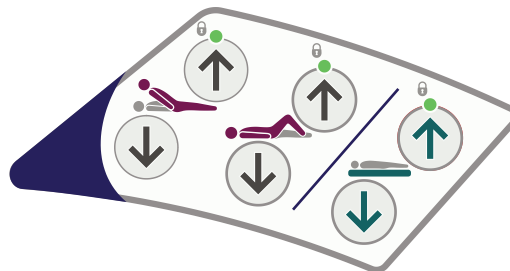


Patient Controls

3 PD 24.

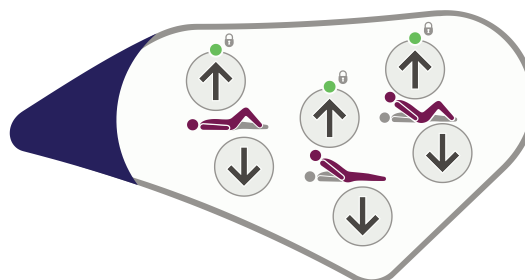

Patient Handset

3 PD 22.

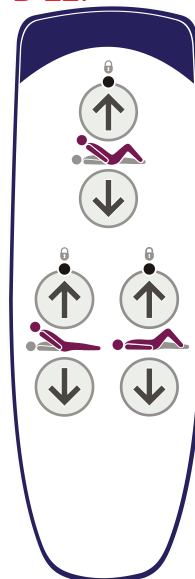

Raise/lower the mattress platform.



Raise/lower the thigh section.



Raise/lower the backrest.




Bio-Contour up button
(patient control only) - raise backrest and thigh section.



Bio-Contour down button
(patient control only) - return to flat position.

Patient Weighing and VariZone Movement Detection Controls

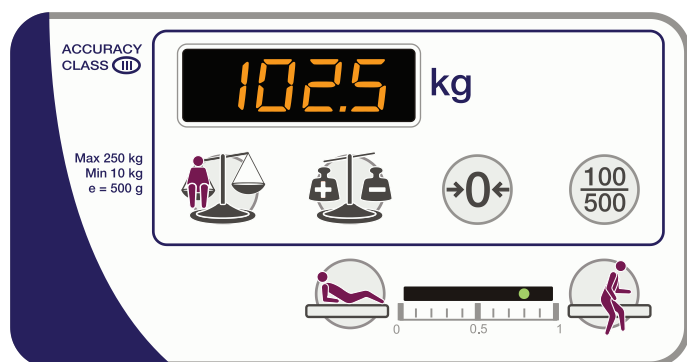

Weigh - calculate and display patient weight.



Auto Compensation - allows items to be added or removed without affecting indicated patient weight.




Zero - reset weighing system.



Select Weight Display - rounds the patient weight to nearest 100 g or 500 g.



In bed - activate/deactivate patient movement detection and increase the sensitivity of the system.

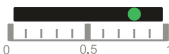

Patient movement detection threshold display - an indicator shows the current system status and the selected sensitivity of patient movement detection.



Egress - activate/deactivate patient movement detection and decrease the sensitivity of the system.

