

Ecoflac[®] plus

Benefits that pay off



Ecoflac® plus

An intelligent system for your hospital is one that offers you flexibility and lets you adapt readily to changing needs.

In our current times, not only functionality, but economy, too, is demanded.

Benefits that pay off

Fluid Administration

The hospital value chain

Follow Ecoflac® plus on its way from delivery through the various options for use all the way to proper disposal at your hospital and you will see all its benefits ...

1. Uncomplicated delivery and transport

2. Space-saving storage

3. Versatile use in the pharmacy and on the ward

4. Ecological and proper disposal



Performance ...

Ecoflac® plus combines the benefits of a bag with those of a bottle, generating a new performance class that adds safety to your daily routines.

This new generation of infusion containers features excellent handling properties that make it virtually error-resistant.

The material is compatible with a maximum number of pharmaceuticals.

The form and function of Ecoflac® plus are perfectly adapted to meet the modern needs of hospital routine.

Your plus

- Error resistant
- Compatible with pharmaceuticals

... without compromise

Forget about the hassle and expense of special requirements.

Ecoflac® plus offers you one product for all applications!



Fluid Administration

Delivery ...

Form and material offer clear advantages

Ecoflac® plus is hard to knock off its feet. Standing upright or laying on its side, the container remains stationary. You even benefit before it is delivered because Ecoflac® plus is virtually break-proof.

Temperature resistant

During transport and storage, in winter, summer, spring and fall, Ecoflac® plus resists them all. The material is not affected by extreme temperatures.

... and storage

Fluid Administration

Ecoflac® plus saves space: in the stockroom, in the pharmacy and on the ward.



The infusion containers can be stored upright or horizontally – they are easy to stack.




Thanks to its ergonomic shape, the outer carton on Ecoflac® plus is also much smaller and lighter. That spares material resources and saves your strength.



At the pharmacy ...

What really makes a difference: the twin cap

The twin cap on Ecoflac® plus is doubly practical. The two separate resealing puncture sites are kept bacteria free by separate sealant foils.

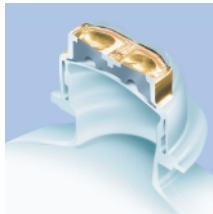

The minute you remove the protective foil, you can pierce a hole right away without extra disinfection.

The ports are flexible to use.

For example, the pharmacy can prepare an infusion solution with the prescribed drugs through the first port. The second is protected by the covering and remains bacteria-free – it can be used later on the ward.

Uncomplicated drug transfer

Ecoflac® plus leaves you a lot of elbow room for adding drugs.

They can be readily mixed into Ecoflac® plus regardless of whether you use a needle, the B. Braun Mini-Spike® or the Ecoflac® mix transfer set.



Injection volumes

80 ml 90 ml 150 ml 220 ml

As a matter of fact, Ecoflac® plus is completely free of latex and PVC.



... and on the ward

Fluid Administration

An all-round-talent in every regard

1. Stays put

Ecoflac® plus stays firmly in place. It can't roll away, no matter where and how you place it. And, if by accident, the container should be dropped, there are no glass splinters – an invaluable advantage, particularly when working with cytostatics.



2. Safe and easy application

After preparing the infusion solution at the pharmacy, the second port is still bacteria-free and can now be used for the infusion – unvented, just like a bag.



3. Pressure infusion

With its light and ergonomic shape, Ecoflac® plus is also suitable for routine pressure infusions. Because it is flexible, you can always hold it securely and firmly in your hand. All air needs to be expelled prior to use for pressure infusion.



Even disposal is easy.



Additional system options ...


Fluid Administration

As part of the Ecoflac® plus assortment, we offer a wide range of accessories for easier handling in everyday routine. These include:



Ecoflac® Mix transfer cap

Ecopin offers a tamper-proof closure of the injection port

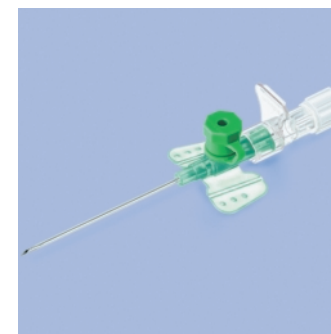

A comprehensive safety port-folio for intravenous access



Mini-Spike® for multi-dose containers



Intrafix® infusion sets

... improve safety

Fluid Administration

The ConComp database supplies easy access to information concerning the stability of drugs in various combinations of:

- Active ingredients
- Diluents
- Container-materials

All data are supplied as abstracts taken from several hundreds of original publications and internal research.

These data are not restricted to B. Braun containers but include all relevant products used in clinical routines worldwide.

– ConComp®, a unique information source –



Do you have an access code yet?
If not, please contact your B. Braun representative.

B | BRAUN
SHARING EXPERTISE

B. Braun Melsungen AG
P.O.Box 1120
D-34209 Melsungen
Tel +49(0) 56 61 71-0
www.bbraun.com