


Balionas, skirtas aortos
okliuzijai ir aortos protezų
modeliavimui.

Reliant™


Stent Graft Balloon Catheter

EXPAND POSSIBILITIES




The versatile, reliable and conformable solution
that extends procedural possibilities

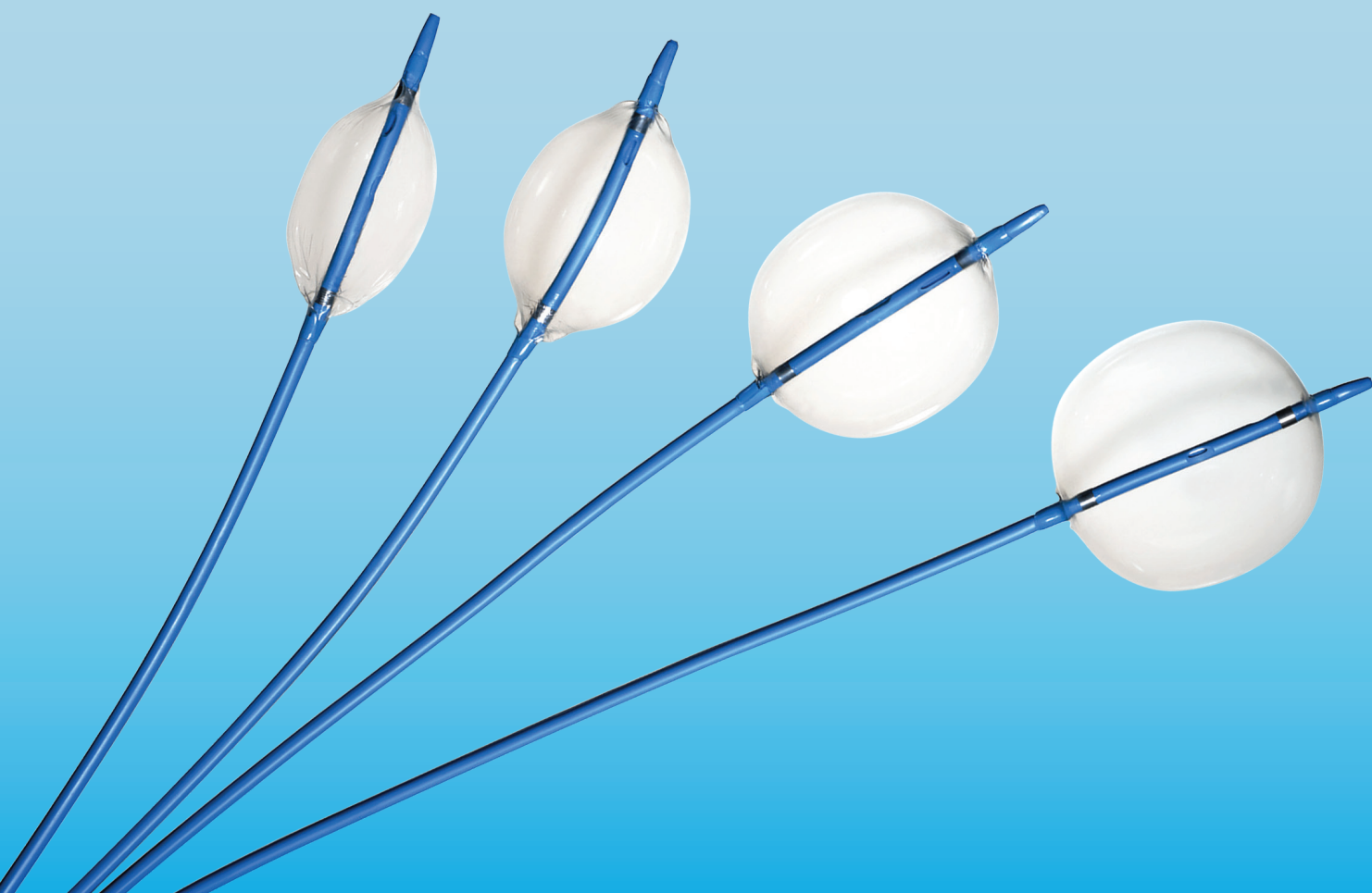

Medtronic
Further, Together

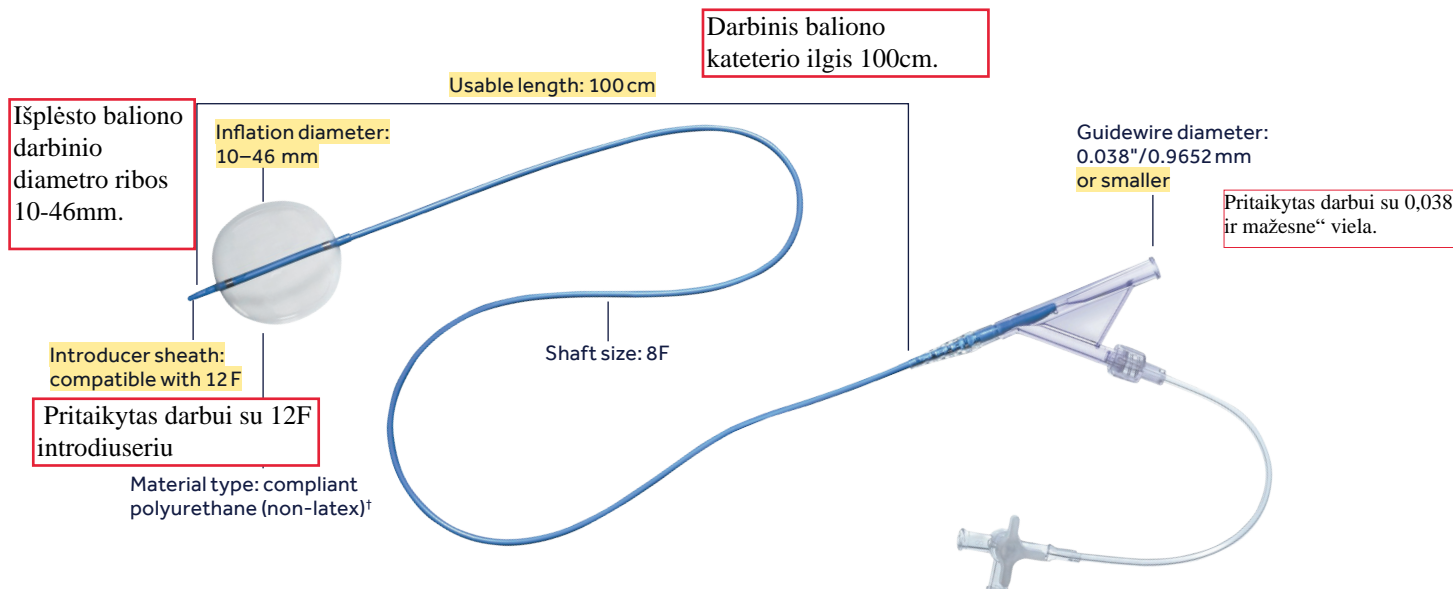

MULTIPLE TREATMENTS IN A SINGLE BALLOON FOR THORACIC, ABDOMINAL AND ILIAC THERAPIES

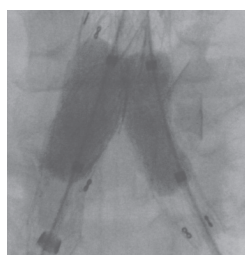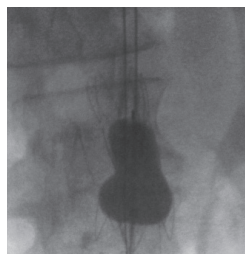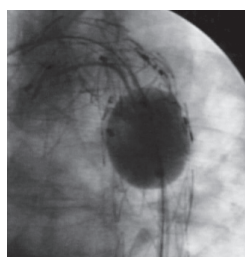

Versatile Design

- Widest range of inflation diameters in a single low profile balloon able to treat thoracic to iliacs (10 mm–46 mm)[‡]
- Compatible with 12F sheath

Reliable Performance

- Consistent inflation and deflation time
- Stable expansion with minimum balloon overhang to reduce risk of vessel trauma
- Dependable expansion even after multiple inflations and deflations[§]

Improved Conformability

- Expands kinks and smooths creases from the graft material
- Compliant balloon expansion

Balionas visiškai adaptatyvus (angl. Full compliant)

Clinical Uses Include:

- For use in thoracic, abdominal and iliac endograft procedures
- Endograft molding
- Apposition of endografts in seal zones
- Temporary aortic occlusion

Balionas, skirtas aortos okliuzijai ir aortos protezų modeliavimui.

BALLOON INFLATION TABLE
46 mm Balloon

Diameter (mm)	mL (cc)
10	3
20	9
30	19
40	41
46 [†]	60

CAUTION: This table is only a guide. Balloon expansion should be carefully monitored under fluoroscopy. Do not exceed maximum inflation diameter (46 mm). Rupture of balloon may occur.

PRODUCT ORDERING INFORMATION

Product Code	Inflation Diameter (mm)	Shaft Size (F)	Usable Length (cm)	Sheath Compatibility (F)
AB46	10–46	8	100	12

[†] Does not contain latex.

[‡] Reliant™ Stent Graft Balloon Catheter Instructions for Use, Cook Coda™ Balloon Catheter Instructions for Use, Gore Tri-Lobe™ Instructions for Use.

[§] Data on file Medtronic. Maximum of 20 inflations, deflations.

[‡] Maximum inflation diameter.

Important Information: www.medtronic.com/manuals.

Consult instructions for use at this website. Manuals can be viewed using a current version of any major internet browser. For best results, use Adobe Acrobat® Reader with the browser.

Medtronic

Invatec S.p.A.
Via Martiri della Libertà 7
25030 Roncadelle (BS)
Italy
Tel: +39.030.2589311

Medtronic
710 Medtronic Parkway NE
Minneapolis, MN 55432
USA
Tel: +1.763.514.4000

Medtronic International Trading Sàrl
Route du Molliou 31
CH-1131 Tolochenaz
Switzerland
Tel: +41.21.802.7000

Medtronic of Canada Ltd.
99 Hereford Street
Brampton, Ontario L6Y 0R3
Canada
Tel: +1.905.460.3800

Medtronic Latin America
3750 NW 87th Avenue, Suite 700
Miami, FL 33178
USA
Tel: +1.786.709.4200

Medtronic International Ltd.
49 Changi South Avenue 2
Singapore 486056
Tel: +65.6436.5000

Medtronic Australasia Pty Ltd.
97 Waterloo Road
North Ryde, NSW 2113
Australia
Tel: +61.29857.9000

Medtronic New Zealand
Unit N16, Mezzanine Level 5
Gloucester Park Road
Onehunga
Auckland
New Zealand

Medtronic Korea Co., Ltd.
5F, Sajo Building
1001 Daechi-dong, Kangnam-ku
Seoul, 135-280
Korea
Tel: +82.2.3404.3600

Medtronic META FZ-LLC
Office Park, Block D, 2nd Floor
PO Box 500638
Dubai Internet City | Dubai,
United Arab Emirates
Tel: +971.4.818.2666

medtronic.com/aortic