COULTER DxH Cell Lyse

REF 628019, 628018

TABLE OF CONTENTS

English	2
Français (FR)	4
Deutsch (DE)	6
Italiano (IT)	8
Español (ES)	10
Português Portugal (PT-PT)	12
Dansk (DA)	14
Svenska (SV)	16
Norsk (NO)	18
Ελληνικά (EL)	20
日本語 (JP)	22
Lietuviškai (LT)	24
Magyar (HU)	26
Polski (PL)	28
Čeština (CZ)	30
Slovenčina (SK)	32
한국어 (KO)	34
Türkçe (TR)	36
Русский (RU)	38
Hrvatski (HR)	40
Български (BG)	42
中文 (ZH-TW)	44
Română (RO)	46
Србија (SR)	48
Українська (UK)	50
Português Brasil (PT-BR)	52
Nederlands (NL)	54
Tiếng Việt (VI)	56
REFERENCES	58

COULTER DxH Cell Lyse

REF 628019, 628018

For *In Vitro* Diagnostic Use
Rx Only in the U.S.A.

INTENDED USE

Erythrocyte lytic agent for quantitative determination of hemoglobin, enumeration of NRBC, and counting and sizing leukocytes on UniCel DxH Coulter Cellular Analysis Systems. Use only with COULTER DxH Diluent.

INTENDED USER

This device is intended to be used by a laboratory professional.

SUMMARY

COULTER DxH Cell Lyse, a cyanide-free reagent, is used as a lytic agent to prepare a diluted whole-blood sample for the quantitative determination of hemoglobin and the enumeration of leukocytes and NRBC.

PRINCIPLE

COULTER DxH Cell Lyse is a cyanide-free reagent. Because the reagent composition does not contain cyanide ions, the oxyhemoglobin methodology is used.¹ In this method, surfactants are used to lyse the erythrocytes and release the hemoglobin. The released hemoglobin (oxyhemoglobin) reacts with an antioxidant and a stabilizing reagent, converting the hemoglobin into a stable hemochromogen, which is quantitatively measured at an absorbance wavelength.

The lytic reagent solubilizes the red blood cell membrane while simultaneously preserving the white blood cell and NRBC cellular morphology to be counted and sized.

REACTIVE INGREDIENTS

Quaternary Ammonium Salts5-80g/L

Sodium Sulfite1-5g/L

NON-REACTIVE INGREDIENTS

Stabilizers

WARNING AND PRECAUTIONS

- Irritating to eyes.
- In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- Do not inhale and/or ingest.
- DO NOT REUSE CONTAINER.

GHS HAZARD CLASSIFICATION		
COULTER DxH Cell Lyse	WARNING	H316: Causes mild skin irritation.
		H402: Harmful to aquatic life.
		P273: Avoid release to the environment.
		P332+P313: If skin irritation occurs: Get medical advice/attention.
		Tetradecyltrimethylammonium bromide <3% w/w
		Dodecyltrimethylammonium chloride 1-5% w/w

STORAGE, STABILITY, AND DISPOSAL

- Store COULTER DxH Cell Lyse at 2-40°C.
- Do not use product past expiration date.
- Use product at temperatures stated in the instrument product manuals and/or online help.
- Opened containers are stable for 60 days when stored as recommended.
- Dispose of waste product, unused product, and contaminated packaging in compliance with federal, state, and local regulations.

MATERIALS PROVIDED

COULTER DxH Cell Lyse

MATERIALS REQUIRED BUT NOT PROVIDED

- Sufficient COULTER reagents to complete analysis. For the reagents used on the system refer to the UniCel DxH Coulter Cellular Analysis Systems IFU.
- System HELP or Instructions for Use.

PREPARATION

COULTER DxH Cell Lyse is ready to use. Replace the reagent as directed in your instrument product manuals and/or online help.

IMPORTANT: If product has been partially or completely frozen, allow product to warm to room temperature. Mix product by gentle inversion prior to placement on the instrument. Install and prime the reagent as directed in your instrument product manuals and/or online help. Verify background counts are acceptable before analyzing patient samples.

PRODUCT AVAILABILITY

COULTER DxH Cell Lyse - 1 x 1 L

REF 628018

COULTER DxH Cell Lyse - 1 x 5 L

REF 628019

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

ADDITIONAL INFORMATION

Glossary of Symbols is available at beckmancoulter.com/techdocs by searching for part number C07163.

May be covered by one or more pat. - see www.beckmancoulter.com/patents

For additional information or if damaged product is received, call Beckman Coulter Customer Service at 800-526-7694 (USA or Canada) or contact your local Beckman Coulter Representative.

SDS Safety Data Sheet is available at beckmancoulter.com/techdocs

NOTICE TO USER

For a patient/user/third party in the European Union and in countries with identical regulatory regime (*Regulation 2017/746/EU on In vitro Diagnostic Medical Devices*); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Revision History

Revision AA, 03/2022

Initial release to include documentation updates for compliance to European In Vitro Diagnostic Regulations (IVDR) (2017/746). No changes to product or instructions for use.

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Beckman Coulter Ireland Inc.,
Beckman Coulter, Inc., 250 S. Kraemer Blvd., Brea, CA 92821 U.S.A.