# Edge™

## Extended Working Channel Catheter

<b>en</b>	Instructions for Use
<b>fr</b>	Mode d'emploi
<b>pt</b>	Instruções de uso
<b>de</b>	Gebruiksaanleitung
<b>es</b>	Instrucciones de uso
<b>it</b>	Istruzioni per l'uso
<b>nl</b>	Gebruiksaanwijzing
<b>sv</b>	Bruksanvisning
<b>ru</b>	Инструкция по применению
<b>zh</b>	使用说明
<b>cs</b>	Návod k použití
<b>da</b>	Brugsanvisning

<b>ko</b>	사용 방법
<b>no</b>	Bruksanvisning
<b>pl</b>	Instrukcja stosowania
<b>tr</b>	Kullanma Talimatı
<b>zh-t</b>	使用說明

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THE BACK OF COVER IS BLANK • DO NOT PRINT ON THIS PAGE

## Symbols on Product and Packaging

	Sterile using ethylene oxide
	Single use
	Federal law (USA) restricts this device to sale by or on the order of a physician
	Do not resterilize
	Do not use if package is opened or damaged
	Consult instructions for use
	Warning or Caution
	Keep dry
	Authorized representative in the European Community
	Catalog number

	Manufacturer
	Use by
	Batch code
	Date of manufacture
	Package quantity
	Pouch quantity
	CE mark. Designates that the labeled product is authorized for sale in European countries

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## Edge™ Extended Working Channel Tool Dimensions

### Indications for Use

This device is intended for use with the superDimension™ navigation system. The superDimension navigation system is: Indicated for displaying images of the tracheobronchial tree to aid the physician in guiding endoscopic tools or catheters in the pulmonary tract and to enable marker placement within soft lung tissue. It does not make a diagnosis and is not an endoscopic tool. Not for pediatric use.

### Description

The Edge extended working channel functions as a guiding sheath. It consists of a composite tube with a lumen. It is assembled with the Edge telescope and inserted into the working channel of a bronchoscope in conjunction with the Edge locatable guide. The Edge extended working channel is designed to guide an endoscopic tool or catheter to the target after retraction of the Edge locatable guide. The Edge extended working channel is radiopaque to allow fluoroscopic imaging.

The Edge extended working channel is available in straight and curved models. Be sure to select the correct shape using the Tools pull down menu. Consult the *superDimension Navigation System User Manual* for instructions.

The Edge telescope connects the Edge handle to the Edge bronchoscope adapter.

Edge Extended Working Channel	Inner diameter	2.0 mm
	Outer diameter	2.6 mm
	Overall length	1070 mm

### Compatible Biopsy Tool

Only biopsy tools that meet the specifications can be used with the superDimension navigation system. Compatible tools include: biopsy forceps, aspirating needles and cytology brushes. Consult with Covidien customer service before using additional tools.

Biopsy Tool	Minimum working length	1100 mm
	Maximum outer diameter	1.9 mm

**Warning**

The superDimension navigation system may only be used by a qualified bronchoscopist (referred to as the physician).

When using the extended working channel and the locatable guide, do not bend, twist, drop, or otherwise roughly handle them as damage can result. Damage to superDimension navigation system accessories can potentially result in tissue irritation or injury to the bronchial airways.

To avoid damage to the Edge extended working channel, potentially resulting in tissue irritation or injury to the bronchial airways, never use excessive force (1 kg force (10N)) during the insertion or extraction of the Edge locatable guide or endoscopic tools through the Edge extended working channel.

**Warning**

In case of difficulty during the insertion or extraction of the Edge locatable guide, endoscopic tools, or marker delivery tools through the Edge extended working channel, straighten the bronchoscope distal tip, unlock the Edge bronchoscope adapter, and gently pull out the Edge locatable guide, endoscopic tool, or marker delivery tools together with the Edge extended working channel. If that is not possible, pull the entire bronchoscope out along with whatever is inserted through its working channel. If damaged, replace the Edge extended working channel, Edge locatable guide, endoscopic tool, or marker delivery tool. Reinsert and navigate back to the target area as needed.

Never operate the Edge locatable guide without the Edge extended working channel attached. Before each use check the outer surface of the Edge locatable guide and Edge extended working channel to ensure there are no rough surfaces, sharp edges, or protrusions. Sharp edges on these accessories can cause bleeding and/or pneumothorax.

**Warning**

To avoid potential patient or user injury, endoscopic tools used in conjunction with the superDimension navigation system must comply with the requirements of standard IEC 60601-2-18, IEC/EN 60601-1, IEC/EN 60601-1-2, or their equivalent, and should only be used in accordance with their user manuals.

The Edge extended working channel and Edge telescope are sterilized by ethylene oxide. Do not resterilize.

This device has been validated and warranted for single-use only. Any re-use, reprocessing, or re-sterilization could result in device failure causing patient or user injury, illness, or death. Re-use, reprocessing, or re-sterilization could cause contamination of the device, which could result in patient infection.

To avoid potential infection, before opening any sterile package containing superDimension navigation system accessories check that the packaging is intact, sealed, and free from damage that could compromise the sterile barrier.

Do not use the Edge extended working channel or the Edge telescope if they appear to be damaged.

**Warning**

To maintain accurate CT-to-body registration during the procedure, it is recommended that the bronchial tree retain its natural shape as it appears in the pre-operative CT images. During navigation of the Edge locatable guide, position the distal tip of the bronchoscope so that it does not apply any pressure to the endobronchial walls.

Use of bronchoscopic tools that do not comply with the specifications contained in the *superDimension Navigation System User Manual* has the potential to cause damage to the Edge extended working channel.

**Caution**

There is no guarantee that instruments selected solely using the minimum instrument channel diameter described in the *superDimension Navigation System User Manual* will be compatible in combination.

To maintain accurate CT-to-body registration during the procedure, it is recommended that the bronchial tree retain its natural shape as it appears in the pre-operative CT images. To facilitate this, it is recommended that you mark registration points in anatomical locations where access by the Edge locatable guide will not require forceful maneuvering which may temporarily affect the shape of the bronchial tree.

To minimize error during the registration process, instruct the patient to hold his/her breath as long as possible during each registration point acquisition.

Any movement of the bronchoscope while the Edge locatable guide is being removed and the Edge extended working channel is locked in place will affect the position of the Edge extended working channel, which may cause it to move off target.

**Caution**

If the bronchoscope is unintentionally moved after removal of the Edge locatable guide, re-insert the Edge locatable guide and re-navigate to the target.

Check to make sure that the tip of the Edge locatable guide protrudes at least 3 mm from the end of the Edge extended working channel.

If the Edge locatable guide does not move smoothly through the working channel, there may be an obstruction. Check that the working channel is free of any obstruction and that the Edge locatable guide is free of any debris.

Any tool from the approved tool list that is intended for insertion into the Edge extended working channel must be marked for length before use.

Prolonged use or navigating in extreme tortuosity may damage or change the shape of the extended working channel curved catheter. In such circumstances it may be necessary to substitute a new extended working channel curved catheter in order to reach the target location.

## Inspection of Package Contents

Examine package carefully before opening to confirm its integrity and that the use by date has not passed. The device is provided in a sterile package. Do not use the Edge extended working channel with telescope if the package is damaged, opened, or if the use by date has passed. The Edge extended working channel and Edge telescope are sterilized by ethylene oxide. Do not resterilize.

## Instructions for Use

1. **Unpacking:** Open the package and remove the Edge extended working channel with telescope onto a clean surface in accordance with hospital or clinic protocol. Remove the tip protector from distal end of the extended working channel. Open any tools, such as biopsy forceps or brushes.
2. **Marking tool length:** Consult the *superDimension Navigation System User Manual* for instructions on marking the tool length.
3. **Inserting the locatable guide into the extended working channel:** Make sure the tip protector has been removed from the distal end of the Edge extended working channel and then gently insert the Edge locatable guide into the proximal end of the Edge extended working channel. If the Edge locatable guide does not move smoothly through the Edge extended working channel, check the Edge extended working channel for any obstruction.

Ensure that the white alignment tab clicks into the gray Edge extended working channel hub.

4. **Inserting a tool into the extended working channel:** After removing the Edge locatable guide, gently insert the tip of the tool into the Edge extended working channel. To avoid unintentional movement, insert the tool gradually using short steps. Push the tool through the Edge extended working channel until the tool limit mark is reached. Do not push the tool past the limit mark.
5. **Disposal:** When the procedure is complete, remove the tool from the Edge extended working channel. You may then remove the Edge extended working channel and dispose of it in accordance with hospital or clinic protocol.

## Further Information

Consult the *superDimension Navigation System User Manual* for directions for use. If further information is required, please contact Covidien.

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© 2015 Covidien.
Covidien Ireland Limited