


REPROCESSING & CARE

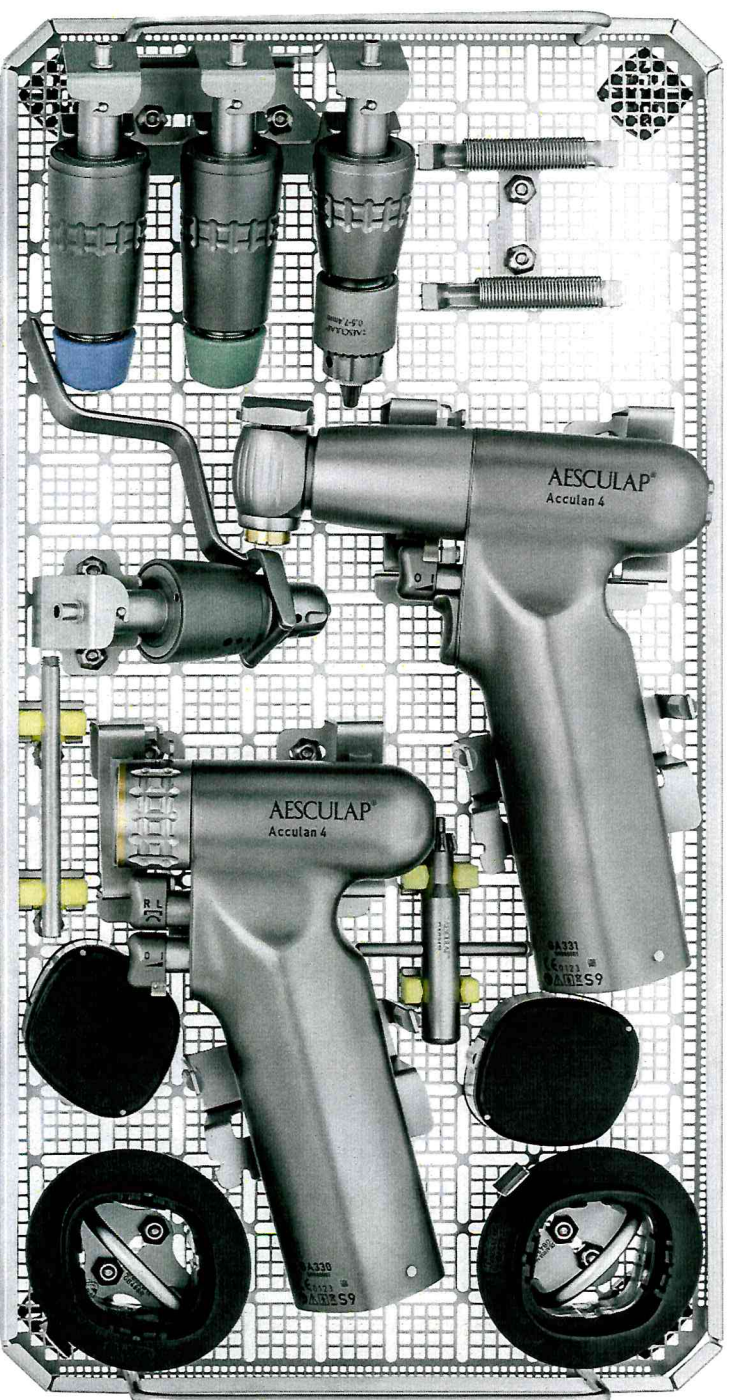

Reprocessing

Pre-Cleaning

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemistry/Note
I	Rinsing	RT (cold)	-	-	D-W	Unit visually clean
II	Brushes	RT (cold)	-	-	D-W	Unit visually clean

D-W: Drinking water
RT: Room temperature

- Phase I**
- Mobilize non-rigid components during cleaning.
 - Thoroughly clean the product under running water
 - Recip-saw only: use cleaning template to clean the slot of the saw blade coupling
- Insert cleaning template into the slot and carefully push out the debris through the slot or the inspection holes at least 5 times in both directions.
- Turn template 180° in order to do so.

- Phase II**
- Mobilize non-rigid components during cleaning.
 - Brush cannulation with cleaning brush TAO11944 and difficult to access surfaces with a suitable plastic cleaning brush for at least 1 min.

- After manual preparation, check visible surfaces for residue and repeat the pre-cleaning process as needed.

Note
For details on difficult to access surfaces, see Acculan pre-cleaning and care information TAO16000 (available in Aesculap Extranet at <https://extranet.bbraun.com>).

Mechanical Cleaning


Phase	Step	T [°C/°F]	t [min]	Water qual- ity	Chemistry/Note
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	<ul style="list-style-type: none"> ■ Concentrate, alkaline: <ul style="list-style-type: none"> - pH ~ 13 - <5 % anionic surfactant ■ 0.5 % working solution <ul style="list-style-type: none"> - pH ~ 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfection	90/194	5	FD-W	-
V	Drying	-	-	-	mind. 10 min at max. 120 °C

DW: Drinking water
FD-W: Fully desalinated water (demineralized, low microbiological contamination; drinking water quality at least)

*Recommended: Bbraun Helimatic Cleaner alkaline

- Insert the product in its proper position in the ECCOS fixation.
- protection guide GB436R only: Pull back locking sleeve, push cleaning template between sleeve and support foot and release locking sleeve
- Mount rinsing adapter on the product
- Connect the interior rinse device and connect to the rinse connector of the cleaning/disinfection device/rinse cart.
- Attach the Kirschner wire protection sleeve to a rinse hose.
- After automatic cleaning/disinfection, check visible surfaces for residues and repeat the cleaning/disinfection process as needed.

Care

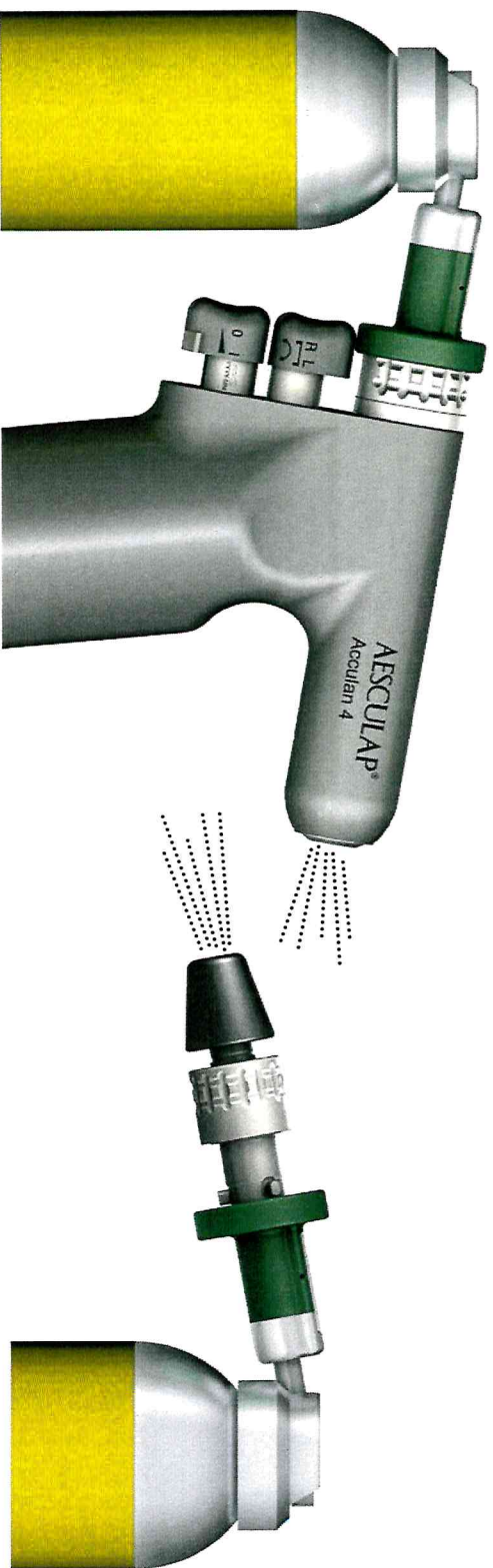

⌚ 2 Seconds
with GB600 STERILIT oil spray