

FITTING*

1 Select cannula size

Select appropriate cannula size; A clear gap must be visible around each prong. The sizing chart on the following page should only be used as a guide.

2 Prepare skin

Prepare the patient's skin according to hospital protocol.

3 Connect to gas source

Connect the cannula to the gas source.
TIP: Place hand close to nasal prongs to ensure that there is gas flow exiting the prongs.

4 Remove the first F&P Wigglepad™ 2 tabs

Remove the first backing tabs from the F&P Wigglepads™ 2 and avoid touching the adhesive.

5 Insert cannula


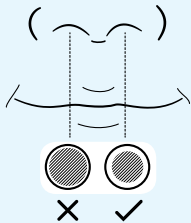



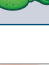
Insert the cannula into the nares. Ensure the cannula bridge rests close to the nose without touching the septum. DO NOT stretch the cannula during application. Stick the F&P Wigglepads™ 2 to the patient's cheeks.

6 Secure cannula

Remove the second backing tabs and stick the F&P Wigglepads™ 2 onto the cheeks.

** IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.*

1 SELECT CANNULA SIZE

SIZE RANGE			ENSURE GAP AROUND PRONGS*
XS		OJR410	
S		OJR412	
M		OJR414	
L		OJR416	
XL		OJR418	



REMOVE CANNULA

1 Place fingertip

Place fingertip on the outside edge of the F&P Wigglepads™ 2 and gently peel the cannula away from the Wigglepads 2.



F&P WIGGLEPADS™ 2 REMOVAL AND REPLACEMENT

1 Lift the edge

Lift the edge of the F&P Wigglepads™ 2. Use a damp cloth to wipe the patient's skin and the underside of the Wigglepads 2 while gently peeling away from the patient's face.



2 Adhere






Adhere the replacement F&P Wigglepads™ 2 to the cannula, remove the first backing tabs and stick the Wigglepads 2 onto the patient's cheeks.



3 Remove

Remove the second backing tabs and stick onto the patient's cheeks.

SIZING INFORMATION

		WEIGHT GUIDE (KG)								SPARES								
		500g	1	1.5	2	2.5	3	3.5	4									
	OJR410 XS	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>								F&P Wigglepads™ 2 WJR110 19.								
	OJR412 S	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>																
		WEIGHT GUIDE (KG)													SPARES			
		1	2	3	4	5	6	7	8	9	10	15	20	25	30			
	OJR414 M	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>													F&P Wigglepads™ 2 WJR112			
	OJR416 L	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>																
	OJR418 XL	<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>																
		<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>																
		<div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div><div></div></div>																

Expected to fit patient

May fit patient

Expected to fit patient May fit patient

NOTE:

1. Prongs must not create a seal in the nares. A clear gap must be visible around each prong.
2. Patient weight should only be used as a guide.



To view the
Optiflow™ Junior 2
inservice video
scan this QR code

* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.