



Ikalamas



Impact metal striking head on the back of the PushLock driver to insert the tak portion of the anchor into the hole and lock the sutures

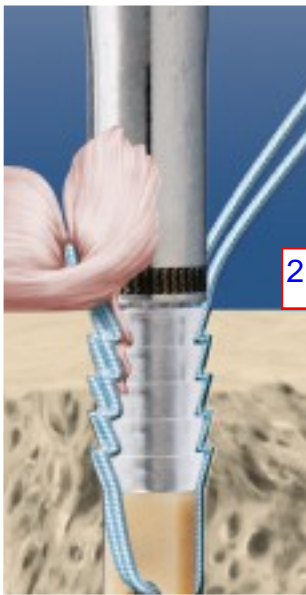


2.4p.d.

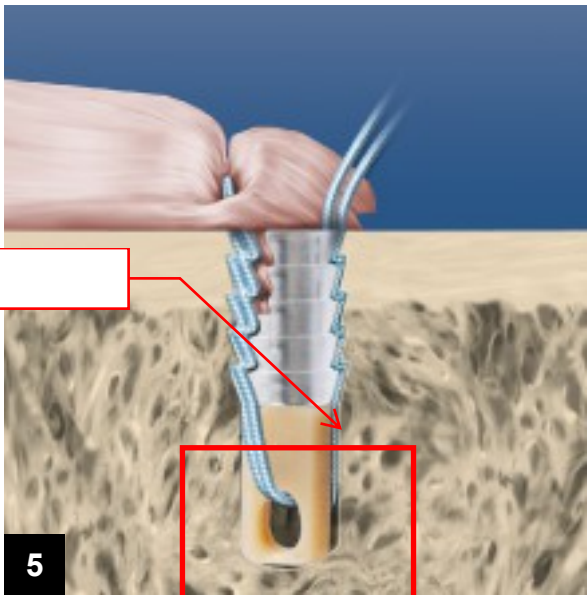

Rotate the handle six full turns counterclockwise to unthread from the eyelet. Remove the handle. Cut the sutures flush.

2.5.p.d. vidinis  
uzrakinimo kanale  
savybe

Pull-out Comparison\*

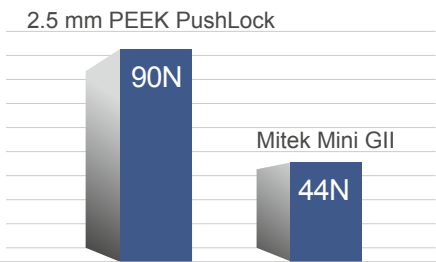

\*data on file



Actual Size

2.5 mm PushLock



2.5 mm diameter  
Bio-PushLock

2.5 mm diameter  
PEEK PushLock

