

PRODUCT FEATURES



- Specially designed air pistol for applications in the medical world
- Compatible with all Ergotrics devices
- Compatible with compressed air system in hospitals
- Safe and easy to use
- Reusable
- Standard available with NIST or DISS body.
- Shelf-life: 5 years

PRODUCT SPECIFICATIONS

Material specifications

Air pistol.....	6061 Alloy
Nose.....	6061 Alloy
Activator.....	Alu 6082
Bolt barb hose	SPI-MS 6 X 1/8" NPT MALE
Bolt NIST/DISS body	6061 Alloy

Certification

CE-marking	Class I (EU) MDR 25/03/2020
------------------	-----------------------------

Sterilisation

Sterile?	No
----------------	----

PACKAGING CHARACTERISTICS


Product nr.	dimensions
HMB-AAA-03 (NIST body)	22x8x6,5cm Weight: 0,7 kg
HMB-DAA-01 (DISS body)	22x8x6,5cm Weight: 0,7 kg

GENERAL INFORMATION

Intended use:

Inflation and deflation of compatible Ergotrics inflatable medical devices using medical compressed air (4-6 bar).

Use

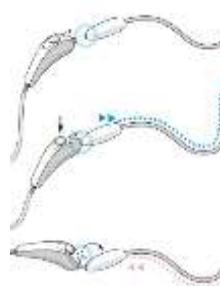

- Put the nose of the HMB against the inflation valve of the Ergotrics inflatable device.
- Push the activator to release medical air
- Release activator to stop the release of medical air
- Use the horn (cone) to deflate air
- Relief pressure from system when not in use.

CAUTIONS

- Do not use with any gas other than compressed medical air at 4 - 6 bar. Higher pressures may cause damage.
- Only for inflation and deflation of Ergotrics' inflatable medical devices labelled as 'compatible with HMB.'
- Do not use in case of visible damage.

MAINTENANCE INSTRUCTION

- Designed to last 5 years
- Check periodically the connection to the hose
- Clean with a soft cloth using Isopropyl alcohol or water and soap